

NEW!

Stimulite® Adjustable Contoured Cushion


Supracor's Stimulite Adjustable Contoured Cushion is the next generation of the popular Stimulite Contoured Cushion maintaining all of its features with the new ability to be customized for the user's changing physical and functional needs. The Stimulite Adjustable Contoured Cushion provides excellent positioning and superior pressure management. The multi-layered honeycomb is shaped into a subtle contour and specially engineered with soft and firm areas. A rear dish provides optimal pelvic positioning for improved posture and stability.



An **adjustment cavity** at the bottom of the cushion allows the user or caregiver to easily customize the cushion by selecting from multiple




Stimulite honeycomb inserts that vary in softness. The softer insert allows for more immersion as compared to the firmer inserts. Removing the inserts all together provides additional relief to users with bony protuberances. In addition to adjusting the immersion, skin protection, and positioning properties of the cushion, the bottom insert layer can also be used to address other positioning needs, such as pelvic obliquities.

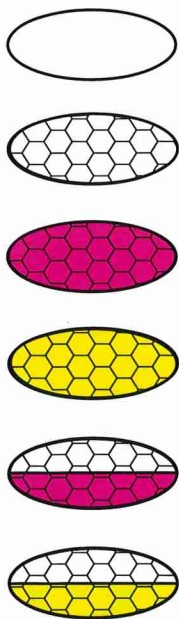

Insert	Engineered specifically for:
None	A need to eliminate as much pressure as possible from the ischial tuberosities or other bony prominences.
White	Hypotonicity or low tone, with accompanying high risk for skin breakdown. This hypotonicity can result in pelvic instability and subsequently poor sitting posture and positioning. Will offer increased pelvic engulfment.
Magenta	A need for less immersion (with maintenance of a moderate level of skin protection) as compared to incorporating no inserts as noted in row 1 of this table.
Gold	A need for a significantly stable base of support for the performance of transfers, but who presents with a reduced risk of skin breakdown (pressure injury) as compared to those who would be utilizing a white or magenta insert or no insert at all.
White on Magenta	A need for a significantly stable base of support for the performance of transfers, with an added layer of pressure relief.
White on Gold	A need for the highest level of support (horizontal stiffness) for stability (gold insert) with the highest degree of pressure relieving overlay (white insert), typically for a client who has not had a history of significant breakdown, as the suspension technique above is most helpful for this.


Deep Immersion with Positioning

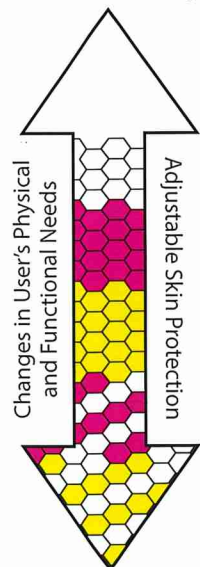

Immersion with Added Support

TECHNOLOGY

Stimulite cushions combat pressure injuries with their **complete approach to pressure management**. Pressure, shearing and maceration—softening of the skin due to heat and moisture—are the primary causes of pressure injuries. Stimulite cushions uniformly distribute pressure through their cellular matrix. The cells flex with movement to reduce shearing and are perforated to prevent heat build-up and evaporate moisture, keeping the user cool and dry. Stimulite cushions address the primary causes of pressure injuries by providing **Total Pressure Management®**.

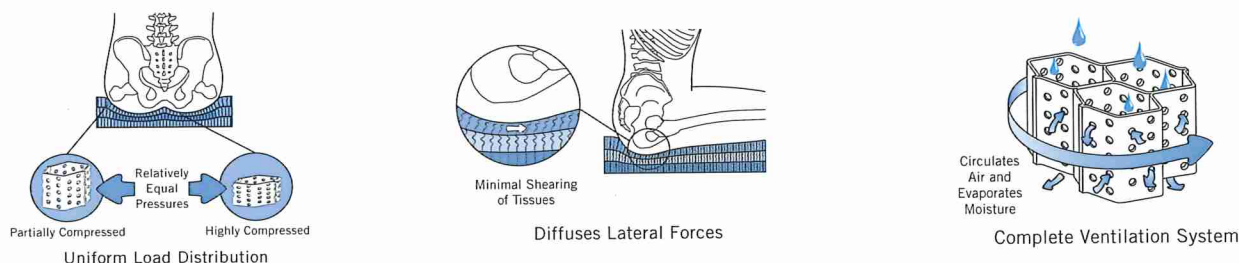

The Stimulite Adjustable Contoured Cushion is recommended for paraplegics, quadriplegics and other individuals requiring a higher level of positioning and pressure management. **A new adjustment cavity allows the user or caregiver to make adjustments to the cushion as needed on an ever-changing basis.** The Stimulite Adjustable Contoured Cushion is available in width and depth sizes ranging from 14 to 20 inches. Like all of Supracor's cushions, the Stimulite Adjustable Contoured Cushion is machine washable and dryer safe.



- Provides Total Pressure Management®
- Ability to Adjust Immersion
- Ability to Add Positioning Inserts
- Machine Washable and Dryer Safe
- Lightweight – just over 3 lbs.
- Anti-Microbial Honeycomb Structure
- Complete Ventilation System
- Reimbursable under HCPCS Code: E2624 or E2625



Supracor.

Discover How Our Cells
Work For Your Cells®

408.432.1616 www.supracor.com

Made in U.S.A. ©2020 Supracor, Inc. All rights reserved. Supracor's products are covered under one or more of the following U.S. and corresponding foreign patents, as well as pending applications filed in the U.S. and abroad: 5,110,653; 5,122,405; 5,137,769; 5,180,619; 5,203,607; 5,289,878; 5,375,305; 5,403,642; 5,427,566; 5,444,881; 5,496,610; 5,509,484; 5,534,343; 5,617,595; 5,701,621; 5,756,182; 5,840,397; 5,840,400; 6,415,583; 7,376,996. Supracor, Stimulite, Total Pressure Management and Discover How Our Cells Work For Your Cells are registered trademarks of Supracor, Inc.