



PRODUCT SHEET

XXL-Rehab Rollator King

Strong and functional rollator designed exclusively for bariatric users.

The Rollator King is intended for users weighing up to 325 kg. The rollator can be used outdoors and indoors in institutions, care centers and in private homes.



- Fits through doorways
- Also works as chair for resting
- Extremely strong construction
- Weight capacity up to 325 kg



Scan or click on the qr code and find the product on the Danish AssistData.



Cobi Rehab and Cobi Rehab's suppliers comply with UN conventions on child and forced labor and ILO 029, C132 and C182.

One of the strongest rollators on the market

The design of the Rollator King is functional and simple. Furthermore, it is designed by Danish occupational therapists and engineers.

A weight capacity up to 325 kg makes it one of the strongest rollators on the market.

Construction

The Rollator King comes with a large upholstered seat that provides the user with the possibility to rest and sit during activity.

The very strong and durable construction together with the solid wheels with steel forks ensures great safety and support for the user.

Safety back rest

When the user is resting on the seat, the back rest ensures that the weight of the user is evenly distributed on the seat. This way the risk of fall preventions is prevented significantly.

Cleaning and maintenance

The frame, seat, handle, and safety back rest of the rollator must be cleaned with water and a regular cleaning agent.

Bolts and nuts should be checked every three months. Tighten them if needed.

Disinfection

The rollator's frame, seat, handle, and safety backrest can also be disinfected with e.g. Wet Wipe 70% or a chlorine product 1000-1200 ppm (e.g. Wet Wipe Chlorine Disinfection).

Transport in motor vehicle

Rollator King can easily be folded if the rollator is to be transported in a motorized vehicle.

1. Grasp the front edge of the seat with one hand. Hold one of the handles with the other hand.
2. Lift up on the seat and fold the rollator.

The Rollator King fits through doorways and folds easily for transport in motor vehicles and storage.

The Rollator King provides the user with opportunities for do activities both indoors and outdoors.

The Rollator King is equipped with stable brakes, which can both lock as a parking brake or be used as a manual brake. The two types of brakes ensure control and safe maneuvering.

The rollator comes with a basket and the handles are fitted with reflectors.


It also makes the user more comfortable when sitting.

 Never use the rollator without the safety back rest.

Please contact Cobi Rehab for further guidance on maintenance of the Rollator King.

The upholstered seat and the safety back rest can be exchanged when switching to a new user.

The basket and the safety back rest can be removed to take up less space.

 The Rollator King must not be used as a seat in a motor vehicle.

Specifications	
Maximum user weight	325 kg
Total length	85 cm
Total width	71 cm
Width between handles	61 cm
Height adjustable in 2.5 cm intervals	77,5 - 99 cm
Seat width	51 cm
Seat depth	35,5 cm
Seat opening	58 cm
Seat height	50 cm
Weight	15,7 kg
Weight capacity in basket	5 kg
Foldable	Yes
Resting chair	Yes
Massive wheels	Yes
Frame	Powder-lacquered aluminium
Seat and back	Polyurethane
Wheels	8" massive
Expected product life	7 years
ISO classification	ISO 12 06 06 - 11
Risk class	I
UDI-DI	5740000102490
Basic UDI	5740000100007N8
User manuals and product sheets are available on the website and can be scaled up to large font size for the visually impaired.	

Model	Seat width	Seat opening	Item number
XXL-Rehab Rollator King, including basket	51 cm	58 cm	0172-050-000