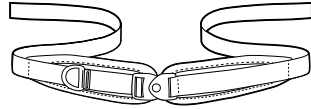
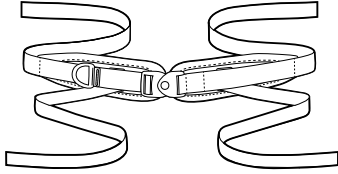
# Pelvic Support Belt

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ART NOS. GB2, HB2, HB4, LB2, LB4, & NB2

These instructions provide important information for the safe use and maintenance of all Bodypoint pelvic support belts. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

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**⚠️ WARNING!** As with any new seating support, this product may change the way a person sits. Users

must continue to practice regular pressure relief activities and skin integrity checks, not only where this product contacts the user, but also in primary pressure-bearing areas such as the sacrum, legs, and buttocks. If increased skin redness or irritation occurs, discontinue use and consult your physician or seating specialist. Failure to do so may result in serious injury, such as pressure ulcers.

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### PERIODIC SAFETY AND PERFORMANCE CHECKS:

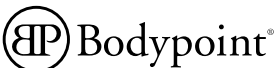
To ensure user safety, this product must be checked periodically for function and signs of wear. If the product does not function correctly or if significant wear is found in the buckles, mounting points, webbing, padding, or stitches, stop using it and contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons — health and safety depend on it!

**🧺🧼🧴🧻 CLEANING:** Machine wash, hot, 60°C (140°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

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
**WARRANTY:** This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA go to [www.bodypoint.com](http://www.bodypoint.com).

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

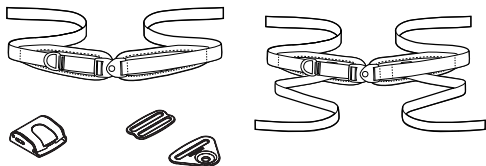
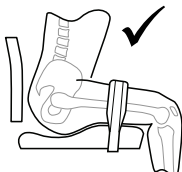
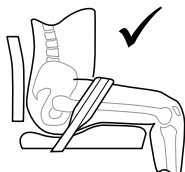
#### INSTALLATION: TWO-POINT PELVIC SUPPORT BELTS

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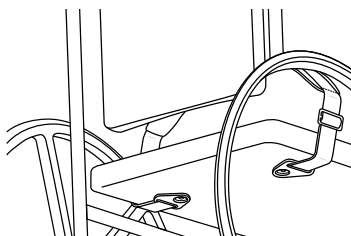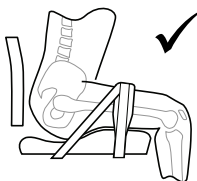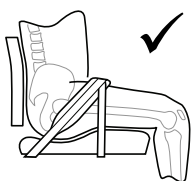#### INSTALLATION: FOUR-POINT PELVIC SUPPORT BELTS

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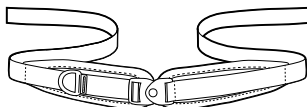
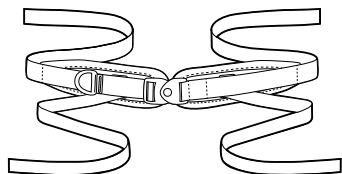
# Pelvic Support Belt

## INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. GB2, HB2, HB4, LB2, LB4, & NB2

**⚠️ WARNING!** Product should be installed and fitted by a qualified rehab technician.

For installation, this product should be mounted with one of the following:

- Flat-Mount end-fittings
- Cinch-Mount end-fittings
- Cam buckles (1" suitable for users up to 75kg, 1-1/2" and 2" suitable for all users)

Optional Bodypoint belt mounting hardware may also be used for mounting to wheelchairs and seating systems.

### INTENDED USE:

**⚠️ WARNING!** This product should only be used for positioning a person in a wheelchair or work chair. It is NOT intended for use as a transportation safety device, as a personal restraint device, or in any other application where its failure could result in injury. Misuse of this product is unauthorized and unsafe.

**⚠️ WARNING!** This pelvic support belt must be worn tightly fitted across the pelvis or thighs at all times. Have your seating specialist demonstrate its proper adjustment and use.

- A loose belt can allow the user to slip down and create a risk of strangulation.
- Accidental release of this pelvic support belt can allow the user to slip down or fall from the wheelchair.
- A user's inability to self-release can be hazardous if the user slips down or is trapped in the chair in an emergency.

If the user's physical or cognitive abilities prevent them from safely operating this belt, a caregiver must be present at all times during its use. Ensure that all caregivers know how to correctly tighten and unfasten the product.

**⚠️ WARNING!** As with any new seating support, this product may change the way a person sits. Users must continue to practice regular pressure relief activities and skin integrity checks, not only where this product contacts the user, but also in primary pressure-bearing areas such as the sacrum, legs, and

buttocks. If increased skin redness or irritation occurs, discontinue use and consult your physician or seating specialist. Failure to do so may result in serious injury, such as pressure ulcers.

**⚠️ WARNING!** If a serious incident occurs related to the use of this product, it should be reported to the manufacturer (Bodypoint, Inc.) and the local Competent Authority.

### PERIODIC SAFETY AND PERFORMANCE

**CHECKS:** To ensure user safety, this product must be checked periodically for function and signs of wear. If the product does not function correctly or if significant wear is found in the buckles, mounting points, webbing, padding, or stitches, stop using it and contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons — health and safety depend on it!

**CLEANING:** Machine wash, hot, 60°C (140°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

**SCRAP/DISPOSAL:** Product is made of materials which can be disposed of safely without special precautions at the end of its useful life.

### WARRANTY

#### Bodypoint full product range

Warranty Period: 1 year limited

Permobil Australia Pty Limited. ("Permobil") warrants that Permobility products to be free from defects in material and workmanship, for the period listed next to the relevant product, under proper use, care and service according to the user's manual and instructions accompanying the product, with the following specific exception: any supplied batteries are covered by a separate warranty provided by the battery manufacturer; The batteries are not warranted by Permobil. The express warranty set forth herein ("the Warranty") is subject to all terms set forth herein below.

#### COMMENCEMENT OF WARRANTY PERIOD

The Warranty shall begin on the date the product is first delivered to the customer, or forty (40) days from

Bodypoint Europe BV  
Kerkstraat 29  
7396 PG Terwolde  
The Netherlands

Medical  
Device

the date the product is shipped to the Dealer by Permobil, whichever comes first.

## REPAIR OR REPLACEMENT

For warranty service, customers should contact the authorised Dealer from whom the product was purchased. In the event of a defect in material or workmanship, the Dealer must obtain a return authorisation (RA) number from Permobil and the product must be shipped to a service centre designated by Permobil. Permobil will repair or, at Permobil's option, replace any product covered by the Warranty, subject to the terms set forth herein.

The liability of Permobil for a breach of a consumer guarantee or any warranty made under these Warranty terms for any products not of a kind ordinarily acquired for personal, domestic or household use is limited, to the extent permissible by law and at the option of Permobil in relation to the products supplied, to:

- a) the replacement of the product or the supply of an equivalent product;
- b) the repair of the product;
- c) the payment of the cost of replacing the product or of acquiring an equivalent product; or
- d) the payment of the costs of having the product repaired.

If you do not receive satisfactory warranty service, please contact Permobil Australia (Phone: 1300 845 483, Fax: +61 2 9012 0087, Email: info.au@permobil.com). Do not return products to Permobil without Permobil's prior written consent. Returns must be shipped with freight pre-paid. Permobil recommends that you insure the product for its full replacement cost when shipping. You will bear all the expenses of claiming the warranty. If your claim is successful then you may claim back any out of pocket expenses incurred.

If the product or part (as the case may be) is repaired or replaced during the Warranty Period, this Warranty remains based on the original date of purchase.

The benefits given to you in this Warranty are in addition to any other rights or remedies under a law in relation to the product to which the Warranty applies.

## AUSTRALIAN CONSUMER LAW

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

## DISCLAIMER AND LIMITATIONS OF REMEDIES

Subject to the Australian Consumer Law and any other applicable law, the Warranty is made in lieu of and supersedes all other warranties of merchantability or fitness of purpose.

The Warranty covers parts only. Labour, service calls, shipping costs and other charges incurred for the repair of the product are not included in the Warranty, unless specifically authorised by Permobil.

The Warranty is effective only if the product is purchased from an authorised Permobil Dealer. The Warranty is available only to the initial purchaser of the product from an authorised Permobil Dealer.

To the extent permitted by law, parts subject to "normal wear and tear", including but not limited to cushions, pads, upholstery, tires and tubes, brake pads, fuses, bulbs, and grips, are not covered in the Warranty except as it applies to defects in material or construction of those parts.

The Warranty will be void if the original Permobil serial number tag on the product has been removed, altered or defaced.

To the extent permitted by law, in no event shall Permobil be liable for any direct, indirect, incidental or consequential damages resulting from any defect in this product. These include but are not limited to any damages in the form of lost profits, loss of use, attorneys' fees, economic loss, personal injuries, or any other damages caused by circumstances beyond the control of Permobil. In no event shall damages for breach of any warranty exceed the original cost of the product.

The use or installation of non-Permobil issued equipment or parts, or, to the extent permitted by law, the repair or replacement of parts by anyone other than an authorised Permobil Dealer, may void the Warranty.

The Warranty is contingent upon the proper use, maintenance and care of the product. The Warranty will be void if the product has been used in a manner contradictory or in violation of the terms of the user's manual, warnings or instructions accompanying the product, including but not limited to exceeding the maximum weight limit of the product, or using the product with tie downs or locking, clamping, or other restraint systems. To the extent permitted by law, the Warranty may also be void if the product has been repaired or any part replaced by persons other than Permobil or an authorised Permobil Dealer.

The addition of equipment or features to the product that are not manufactured or recommended by Permobil could affect the intended function of the Permobil product, and therefore will void the Warranty.

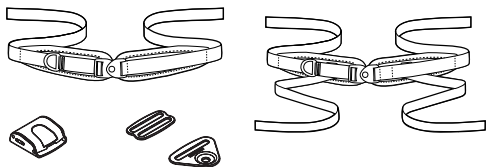
Permobil cannot guarantee the continued performance or safety of the product if it is subject to acts of God or Nature, including but not limited to terrorist activities, war, labour disputes, water, floods, wind, tsunamis, cyclones, tornados, earthquake or fire, or involved in automobile accidents or extraordinary impact events, such as dropping or crushing. Such extraordinary events to products covered by this Warranty may create considerable damage that may be imperceptible to an inspection. Any damages caused by these types of events are not covered by the Warranty.

## AMENDMENTS

No person is authorised to alter, extend or waive the Warranty of Permobil. An original document, countersigned by the party or parties who wish to make the alteration, extension or waiver of the Warranty, must be received by Permobil before any such amendment will be considered. The Warranty shall be extended as necessary to comply with state and federal laws.




## CONTACT INFORMATION

Permobil Australia Pty Ltd. • Unit 3A/6 Boundary Rd, Northmead, NSW 2152 AUSTRALIA  
 Phone: 1300 845 483 • Fax: +61 2 9012 0087 • Email: info.au@permobil.com • www.permobil.com.au  
 For more information on Bodypoint products, and a list of distributors outside the USA go to www.bodypoint.com.




**PACKAGE CONTENTS**

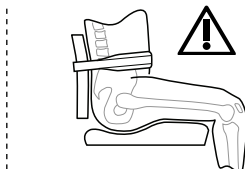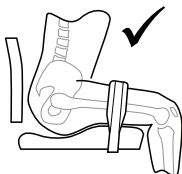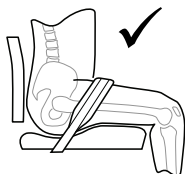- Two-Point or Four-Point Pelvic Support Belt
- Set of Cinch-Mount™ or Flat-Mount™ End-Fittings (Optional)
- FrameSaver™ Clamps (Optional)

**INSTALLATION: TWO-POINT PELVIC SUPPORT BELTS**

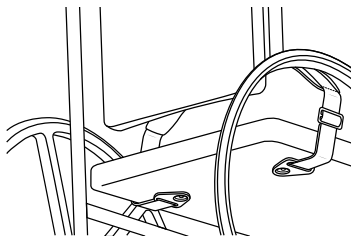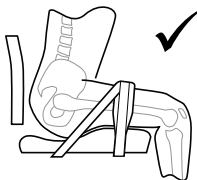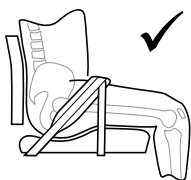1. Based on clinical goals, determine the most effective position and angle for the belt. Determine whether putting pull strap on left or right side is best for user.
2. Locate desired mounting points on the seating system or wheelchair frame.
3. Attach end-fittings to rigid seat base or wheelchair frame using optional Bodypoint belt mounting hardware, if needed.
4. Center the belt on the user, thread the strap ends through the end-fittings, and adjust to desired length.
5. Tighten the belt until it is snug.

**INSTALLATION: FOUR-POINT PELVIC SUPPORT BELTS**

1. Follow the installation instructions above to fit and install the belt to stabilize and position the user.
2. Attach the secondary straps to the wheelchair to prevent the belt from shifting.

**DIRECT MOUNTING OPTION**

End-fittings can be attached directly to solid seat surfaces. Mounting screws must have minimum pullout strength of 90kg/200lbs. (Not included in package.)

**ADJUSTMENT**

When the belt is properly adjusted and tightened, it should fit snug, with all slack removed, so that the user is secure.

Use the pull strap for daily adjustments, such as to accommodate changes in clothing. If additional length adjustment is necessary to achieve a good fit or the belt needs to be removed for cleaning, this can be done using the belt end-fittings.

**SAFETY CHECK**

When properly positioned in wheelchair, have user lean forward and side to side to check fit. Check for:

1. Normal operation of buckle and adjustment straps.
2. Comfort – Look for areas of irritation.
3. Position – If too high or too low, adjust mounting points.
4. Move chair through full range of motion, including tilt, folding, and rolling. Check for any interference and relocate mounting points as necessary.