### JAY®GS Cushion

Important Consumer Information

**NOTE:** This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

**SUPPLIER:** This manual must be given to the user of this product.

**USER:** Before using this product, read this entire manual and save for future reference.

**WARRANTY REGISTRATION:** To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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### ENGLISH

**6**

WARNINGS	6
INSTALLATION / ADJUSTMENT	7
MAINTENANCE/CLEANING	10
CLEANING CHART	11
REASSEMBLING	12
WARRANTY	13
OTHER JAY PRODUCTS	13

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**NOTE:**

Check all parts are free from shipping damage. In case of damage DO NOT use. Contact Sunrise Medical or your supplier for further instructions.

Sunrise Medical recommends that a clinician such as a doctor, therapist or rehab engineer experienced in seating and positioning be consulted to determine if a JAY GS Cushion is appropriate.

Cushions should only be installed by:

- an authorized Sunrise Medical supplier
- a qualified healthcare professional who has undergone appropriate training by either Sunrise Medical or their approved supplier.

**⚠ GENERAL WARNING**

- DO NOT install this equipment without first reading and understanding this manual. If you are unable to understand the Warnings, Cautions and Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.
- Hook and loop Velcro® should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the Velcro®.

**INTRODUCTION**

The JAY GS Growth and Positioning Cushion is designed to provide a high degree of comfortable postural support and to meet the changing needs of growing individuals.

To ensure proper use of the cushion, please read these instructions and save them for future reference.

**Maximum User Weight:**

- 150 kg for all sizes.

**⚠ WARNING**

- Installing a cushion on a wheelchair may affect the centre of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.
- Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.
- The JAY GS cushion is designed to help reduce the incidence of pressure ulcers. However, no cushion can completely eliminate sitting pressure or prevent pressure ulcers.

**⚠ WARNING**

- Do not leave the cushion outside overnight at temperatures below 40° F (5° C). Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.
- Avoid sharp objects or exposure to excessive heat or open flame.
- **OBSTRUCTIONS: DO NOT:** place any obstructions between the user and the cushion because it will reduce product effectiveness.
- **CUSHION AND COVER ORIENTATION:** Product must be used the correct way up. If the cover is not used correctly or if the cover is the wrong size, it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.

## FITTING THE GS CUSHION

Place the Cushion on the seat of the wheelchair.

### NOTE:

A solid base of support, like the Jay Adjustable Solid Seat or Solid Seat Insert is recommended.

If using an adjustable depth back on the wheelchair, like the JAY 3 Back, position the back of the Cushion inline with the front of the upright back posts.

If using a fixed depth back, like a JAY®J2 Back or Jay Care™ Back, notch the rear corners of the base to establish seat depth to within 1" (2.5 cm) from the back of the knees (see score lines on base).

This will allow the back of the Cushion to go between the posts.

Place the individual on the Cushion allowing the buttocks to sit in the rear, carved-out portion and the legs to rest comfortably on the raised front portion.

To achieve the lowest average sitting pressures possible, utilize the GS Pressure Relief Pad.

Additionally, the footrests should be adjusted low enough to allow the legs to rest firmly on the front of the Cushion.

The Cushion should come within 0 - 1" (0-2.5 cm) of the back of the knee.

Assess the client.

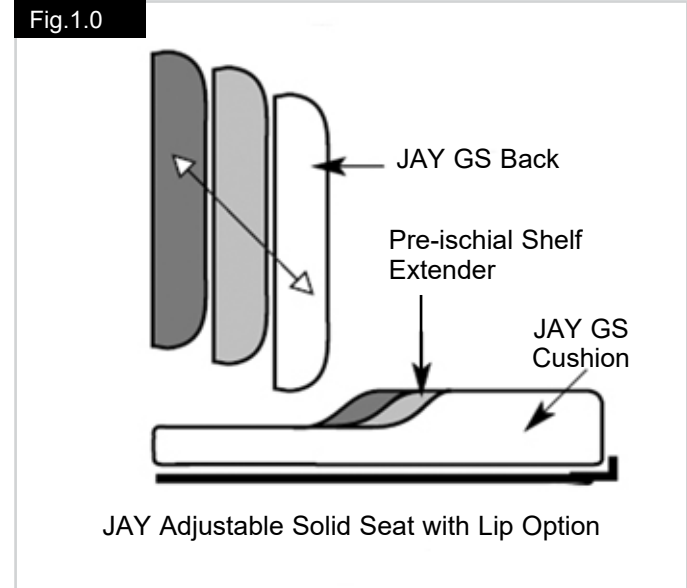
- Are the hips level?
- Are the legs in good position?
- Is the client comfortable?

If the answer to any of these questions is "no," see the Accessories section in this booklet.

## HOW THE JAY GS CUSHION GROWS

The JAY GS Cushion grows by maintaining the positional relationship between the user and the cushion.

The JAY GS Cushion shown with other components of the Growth and Positioning System.

As the client grows, the area under the buttocks remains roughly the same, while the area under the legs grows backward.

Using the Pre-ischial Shelf Extender, (Fig.1.0), will increase leg support and position the anti-thrust shelf where needed.

## GROWING THE CUSHION WITH A DEPTH ADJUSTABLE BACK FITTED

Begin by seating the client forward on the Cushion, with the Back adjusted forward of the upright posts. As the client grows, adjust the Back Support backwards towards the posts. This allows the user to be seated further back in the well and maintain proper seat depth.

Pre-Ischial Shelf Extenders can be added to position the anti-thrust shelf where needed and optimize leg support. The ischial tuberosities, or seat bones, should be located approximately 1" to 2" (2.5 to 5 cm) behind the antithrust shelf. One Shelf Extender is included with the cushion, others can be ordered as needed.

The Cushion "grows" by 1" (2.5 cm) every time a Shelf Extender is added.\*

When the Back has been adjusted as far back as possible, the Cushion can be moved forward for additional growth.

The front edge of the GS Cushion can extend up to 2" (5 cm), unsupported beyond the front of the seat base without affecting performance.

The use of a JAY Adjustable Solid Seat with Lip Option is highly recommended when "growing" the GS Cushion in this way. It allows you to easily relocate the cushion in its desired position if you remove it from the wheelchair.

## GROWING THE CUSHION WITH A FIXED BACK FITTED

Notch the rear corners of the base enough to allow the Cushion to slide between the upright posts of the wheelchair and to establish proper seat depth.

Proper seat depth equates to approximately one to two finger widths between the back of knee and front of cushion.

As the client grows, adjust the Cushion forward ensuring that the buttocks sit in the carved-out portion of the cushion.

Remember to maintain the desired distance between the back of the knee and the front of the Cushion.

When the Cushion is adjusted forward by 1" (2.5 cm), or more\*, add a Pre-ischial Shelf Extender to the cushion, (included), this will maintain the anti-thrust properties of the GS Cushion.

The front edge of the GS Cushion can extend up to 2" unsupported beyond the front of the seat base without affecting performance.

The use of a JAY Adjustable Solid Seat with lip option is highly recommended when "growing" the GS Cushion in this way. It allows you to easily relocate the cushion in its desired position, if you remove it from the wheelchair.

Growth in width of the Cushion is accomplished through the use of accessories. Lateral Pelvic Supports, (hip guides), can be adjusted inward 1" (2.5 cm) on either side and then moved out later for growth at the rear of the Cushion.

Lateral Thigh Supports (abductors) can be adjusted inward 1" on either side and then moved out later for growth at the front of the Cushion.

\* If using the GS Pressure Relief Pad, use of only one Shelf Extender is recommended.


### NOTE:

The JAY GS Cushion may be used with different backs depending on the level of disability. For moderately involved clients, try the JAY J3 Back. For more severely involved clients, try the JAY Fit Back. Growth in the seating system is possible with all of the above products when combined with the GS Cushion.

## JAY GS CUSHION ACCESSORY PARTS

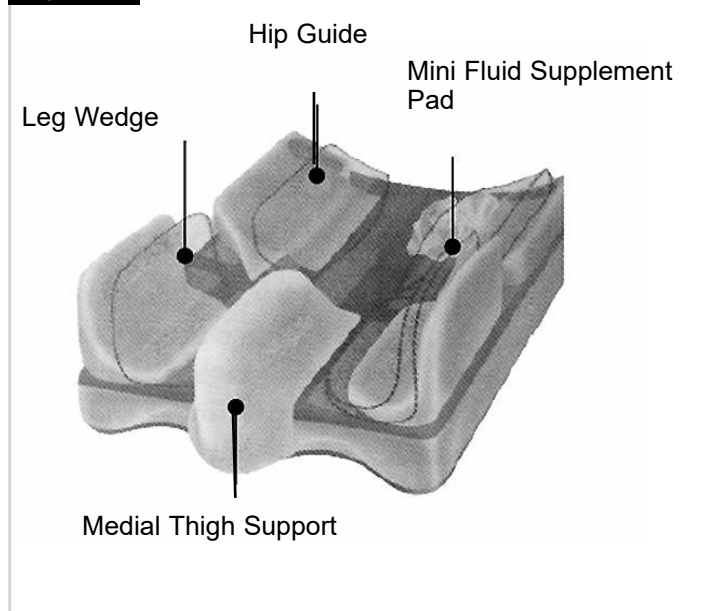Fig.2.0

The GS Cushion interfaces with a wide variety of JAY components including:

- 3 D Pressure Relief Pad (not shown) for greatly improved pressure management.
- Lateral Pelvic Supports (hip guides) for additional positioning or growth in the width of the Cushion.
- Lateral Thigh Supports (abductors) to prevent excessive abduction or spreading of the legs or to provide width adjustment for the Cushion.
- Mini Fluid Supplement Pads to compensate for a non-fixed lean or for added pressure relief and comfort.
- A variety of Medial Thigh Supports (abductors) for increased leg separation.
- A 1/2" (1 cm) Well Build-up (not shown) to decrease hip flexion for individuals with limited range of motion.

**NOTE:**

The GS Cushion with a standard fluid pad or Pressure Relief Pad will accommodate most accessories. An upsized cover is recommended when larger accessories are used.

**CAUTION:**

Fitting of accessories should take place under the supervision of a doctor or a therapist.

## CHECKING FOR BOTTOMING OUT

The standard JAY GS Cushion has less fluid than the JAY®J2 Cushion because it is designed primarily for posture, comfort, and control of abnormal tone. If the client is at high risk of skin breakdown, utilize the Pressure Relief Pad.

**CAUTION:**

Failure to check the Pressure Relief Pad for bottoming out can result in skin redness or skin breakdown.

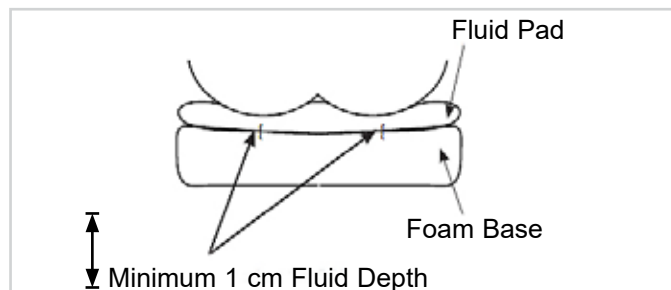
Maximum User Weight: 150 kg for all sizes

To evaluate for bottoming out, (Pressure Relief Pad only):

- Place the client on the cushion without the cover, for two minutes.
- Lift the client off the cushion, trying not to disturb the fluid. Examine the points where the ischials (seatbones) and coccyx (tailbone) were positioned.
- If there is 1/2" (1 cm) or less of fluid between the bottom of the depression and the base, extra fluid is needed. Fluid supplement pads for this purpose are available through your local supplier.
- For 10" (25 cm), 12" (30 cm), and 14" (36 cm), GS Cushions, utilize Mini Fluid Supplement Pads.
- For 16" (40 cm) and 18" (46 cm), GS Cushions, utilize Standard Fluid Supplement Pads.
- To use the Fluid Supplement Pads, place them along the side of the well, under the Pressure Relief Pad.
- Replace fluid pad over them and re-evaluate.

**CAUTION:**

- The clinical indicator for tissue breakdown is skin redness.
- Periodically check the skin for redness.
- If skin redness occurs, discontinue the use of the Cushion and consult your doctor or therapist immediately.

**EASY MAINTENANCE & CLEANING**

- Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended.
- Check the cover for tears and excessive wear and replace if ripped, torn or otherwise not fully functional. Inspect the fluid pad for punctures or any other abnormalities.
- Check the base to ensure any accessories are still in the right position.
- Check the foam inserts to ensure foam consistency.
- While checking the fluid pad if you ever feel the fluid is firmer in one area, simply knead the fluid back to its original consistency and/or call your supplier.
- An incontinence resistant cover will help reduce the frequency of cleaning.

**TO CHECK AND CLEAN THE COVER**

- Remove the cover from the foam insert and turn inside out.
- Machine wash in warm water (maximum recommended temperature is 70° for microclimatic and stretch covers, 60°C for incontinent covers) and air dry.
- No ironing required.

**⚠ CAUTION**

- Do not dry clean covers or use industrial washers and dryers to clean cover.
- Do not steam autoclave.
- Do not bleach.
- To refit cover after washing, reinstall the cover on the foam insert. Ensure the back of the cover matches up to the back of the foam insert.
- If used in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.
- Use only JAY covers on JAY cushions. The cover is an important part of allowing the immersion into the fluid insert which redistributes the pressure. Changing the cover will alter this design and its efficacy.

**CLEANING THE FOAM BASE / LEG SUPPORT WEDGES**

Remove the covers and fluid pad from the foam base. The foam base may be disinfected using immersion or spray disinfectant, (see chart opposite). Wipe off with a clean cloth. Let it completely air-dry before reassembly.

**⚠ CAUTION**

Avoid submersion of the foam base in water.

**CLEANING THE FLUID PAD**

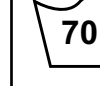
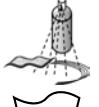
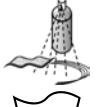
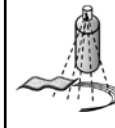
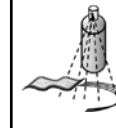
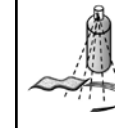
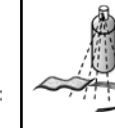
Remove the outer cover from the cushion. Remove fluid pad from the inner cover. The fluid pad may be disinfected using spray disinfectant, (see chart opposite). Rinse with a clean, damp cloth, then wipe dry with a clean cloth.

**⚠ CAUTION**

- Never submerge the fluid pad in bleach.
- Avoid harsh cleaning or rough handling as this may lead to a degradation of the fluid insert and lead to a leakage.

Should there be evidence of wear and tear on the cover, foam insert, fluid insert or other parts or if you notice that the consistency has deteriorated, please contact your dealer or Sunrise Medical for an assessment and further instructions.

LAUNDRY CARE SYMBOLS	
	Machine wash in warm water (70° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

Approved Cleaning Procedures for JAY Cushions									
Product	JAY Lite	JAY J3	J2 GS	JAY Easy Visco	JAY Soft Combi P	JAY Xtreme Active	JAY Basic	JAY Easy	JAY Balance
<b>Outer Cover (Microclimatic and Stretch)</b>									
(1). Wet Wash 70°C 									
<b>Outer Cover (Incontinent)</b>									
(2). Wet Wash 60°C 									
<b>Inner Cover</b>									
(3). Spray and Wipe disinfection  or Wet Wash 60°C 	NA	NA	NA	NA	NA	 	NA	NA	 
<b>Cushion core (foam base)</b>									
(4). Steam autoclave up to 105 °C (1) 									
Spray and wipe disinfection 									
(5). Immersion disinfection 									
<b>Cushion elements</b>									
Roho Air element	NA			NA	NA	NA	NA	NA	
Positioning elements	NA			NA	NA		NA	NA	
Fluid pad	NA			NA	NA		NA		
<p>1. 70°C is the maximum recommended temperature.                  2. 60°C is the maximum recommended temperature.                  3. The Xtreme Active and Balance inner covers can be washed at 60°C as well as spray and wipe disinfected. Possible discoloration if washed at 60°C but the functional aspect of the covers is not diminished.                  4. An autoclave is a self locking machine that sterilizes with steam under pressure.                  5. At low temperature (&lt; 30°C)</p>									
The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.									
<b>Recommended Disinfectants</b>									
All disinfectants which are listed and recommended by the Robert Koch Institute: <a href="http://www.rki.de/EN/Home/homepage_node.html">http://www.rki.de/EN/Home/homepage_node.html</a>									

## REASSEMBLING THE JAY GS CUSHION AFTER CLEANING

1. Replace any accessories on foam base (if used). Attach fluid pad to foam base along one side, on Velcro® attachment strip. Secure pad at center along Velcro® strip. Then attach the second side, same as first. The split in the front of the fluid pad should be attached evenly to the center Velcro strip. If using the (7.5 cm) Medial Thigh Support (leg abductor), the fluid pad should be attached to the sides of the accessory.
2. Check for bottoming out as described previously.
3. Pull the front of the cover over the Cushion first, then stretch the back corners down around the base.
4. Examine the top of the cover to ensure proper fit.
5. The cover should not hammock between accessories.

### CAUTION

- Use JAY covers only.
- The use of an improper cover can adversely affect the performance of the JAY GS Cushion.
- Use only JAY-designed covers in conjunction with the JAY GS Cushion. (Do not use additional covers over the JAY GS Cover).

If the client is not incontinent, we recommend the optional perspiration-reducing AirExchange™ cover for use in warm conditions.

Incontinence-resistant covers are provided standard so that the cushion does not have to be washed after accidental spills.

### NOTE:

Even a snug fitting cover will leave extra “tucks” of fabric under the client’s buttocks. This extra material will ensure total fit of the cover without tension against the buttocks

## BASE MODIFICATIONS

The JAY GS Base can be easily modified at the factory or by your authorized supplier to accommodate specialized seating needs.

These include:

- Special sizes.
- Leg length discrepancies.
- Rail cutouts for dropped or angled seats.
- Back corner cutouts to recess between the upright posts of the wheelchair.
- Specialized needs as requested.

### CAUTION

- The standard JAY GS Cushion is designed primarily to provide a high degree of postural control and comfort.
- If the client is at risk for skin breakdown and pressure relief is a concern, the GS Pressure Relief Pad should be used.
- Persons at high risk of skin breakdown should be thoroughly evaluated by a therapist prior to purchasing the GS Cushion.
- The JAY GS Cushion is no substitute for good skin care, including proper diet and cleanliness.
- Any cushion should be tried for a few hours while a therapist or doctor inspects the skin to ensure that red pressure spots are not developing.
- If the skin develops redness, discontinue the use of the cushion and consult with a doctor or therapist.
- Avoid sharp objects or exposure to excessive heat or open flame.
- If the cushion is left outside overnight at temperatures below 40° F (5° C), allow the cushion to warm to room temperature before using. Sitting on excessively hot or cold surfaces may cause skin damage.

**24 MONTH LIMITED WARRANTY**

Each JAY GS cushion is carefully inspected and tested to provide peak performance. Every JAY GS cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

**ADDENDUM TO WARRANTY USE AND CARE GUIDE**

Consult your health care professional for proper fitting and use, and read the following statements carefully:

- The fluid in this cushion may lose volume over time.
- Volume loss in this cushion may result in bottoming-out of the cushion.
- Monthly, during routine cleaning and inspection, check the cushion for bottoming-out (see instructions under "checking for bottoming-out" in the warranty use and care guide) and any fluid inconsistencies. If the fluid is firmer in one area simply knead it back to its original consistency.
- If bottoming-out occurs discontinue use of this cushion and contact your health care professional. If further assistance is necessary, contact Sunrise Medical Customer Services.

**OTHER JAY PRODUCTS****JAY Adjustable Solid Seat**

The JAY Adjustable Solid Seat offers a firm base of support. The JAY GS Cushion works even better on a solid base of support. The JAY Adjustable Solid Seat replaces the wheelchair's cloth upholstery with an easily removable seat that is fully adjustable in width, depth, height, or angle, depending on individual positioning needs.

**J2 Back**

The J2 Back is comfortable, adaptable, and lightweight. Lateral supports, angle adjustability, and simple removal and attachment make the J2 Back the most sophisticated back support available today. A back support has never been so important.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S