# USER MANUAL/GEBRAUCHSANLEITUNG MANUALE D'USO/MANUEL D'EMPLOI

(ENG - DEU - ITA - FRA)

Devices / Vorrichtungen / Dispositivi / Appareils

KLICK electric series

Dear customer,

Thank you for choosing a KCLICK product.

We sincerely hope that the KLAXON MOBILITY products can improve your day and your activities in many ways, and we remain at your disposal for any information or clarification you may require. We also encourage you to keep up to date on the latest news of our product range, on the accessories available and on our company's events and initiatives.

You can contact us by writing an email to [info@klaxon-klick.com](mailto:info@klaxon-klick.com), through our web site at [www.klaxon-klick.com](http://www.klaxon-klick.com), or on our Facebook page [www.facebook.com/klaxonklickworld](http://www.facebook.com/klaxonklickworld).

Please note also that in compliance with European regulation, KLAXON MOBILITY GMBH holds CE certification for all the products in the KCLICK range; they are classified as Class I medical devices and comply with the essential safety requirements set out in Annex I to Directive 93/42/EEC, as amended by the 2007/47/EC directive.

Thank you again for having chosen our products. We wish you a good day and enjoyable use.

Best regards,

**The Staff of KLAXON MOBILITY GMBH**

## CONTENTS

PRELIMINARY INFORMATION	PAGE 4
1. WARNINGS	PAGE 4
2. SAFETY NOTICES	PAGE 5
3. INTENDED USE OF THE DEVICE	PAGE 5
4. USAGE ENVIRONMENT	PAGE 5
5. OPERATING LIMITS	PAGE 5
6. USER REQUIREMENTS	PAGE 6
7. WARNINGS AND PRECAUTIONS	PAGE 6
8. MARKINGS	PAGE 8
9. DESCRIPTION OF THE DEVICE	PAGE 9
9.1 PRODUCT STRUCTURE	PAGE 9
9.2 KLICK® CONNECTION	PAGE 10
9.3 BATTERY CHARGER	PAGE 10
9.4 ACCESSORIES	PAGE 10
9.5 DOCUMENTATION	PAGE 11
10. INSTALLATION AND COUPLING	PAGE 11
10.1 INSTALLATION	PAGE 11
10.2 COUPLING STEPS	PAGE 11
10.3 WHEELCHAIR	PAGE 11
11. KLICK ELECTRIC STANDARD / POWER STANDARD VERSION OPERATION	PAGE 11
11.1 USER INTERFACE	PAGE 11
11.2 TURNING ON AND SETTING UP THE DEVICE	PAGE 11
11.3 USE	PAGE 12
11.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR	PAGE 12
12. KLICK ELECTRIC TETRA / KLICK POWER TETRA VERSION OPERATION	PAGE 13
12.1 USER INTERFACE	PAGE 13
12.2 TURNING ON AND SETTING UP THE DEVICE	PAGE 13
12.3 USE	PAGE 13
12.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR	PAGE 14
13. KLICK RACE STANDARD VERSION OPERATION	PAGE 14
13.1 USER INTERFACE	PAGE 14
13.2 TURNING ON AND SETTING UP THE DEVICE	PAGE 14
13.3 USE	PAGE 15
13.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR	PAGE 16
14. KLICK RACE TETRA VERSION OPERATION	PAGE 16
14.1 USER INTERFACE	PAGE 16
14.2 TURNING ON AND SETTING UP THE DEVICE	PAGE 16
14.3 USE	PAGE 17
14.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR	PAGE 17
15. KLICK MONSTER STANDARD VERSION OPERATION	PAGE 18
15.1 USER INTERFACE	PAGE 18

# KLAXON Mobility GmbH - ENG

15.2 TURNING ON AND SETTING UP THE DEVICE	PAGE 18
15.3 USE	PAGE 19
15.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR	PAGE 19
16. KLIK MINI STANDARD / KLIK CARBON STANDARD VERSION OPERATION	PAGE 20
16.1 USER INTERFACE	PAGE 20
16.2 TURNING ON AND SETTING UP THE DEVICE	PAGE 20
16.3 USE	PAGE 20
16.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR	PAGE 21
16.5 CLOSING / OPENING FOR TRANSPORTATION	PAGE 21
17. CLEANING	PAGE 21
18. SERVICING	PAGE 22
18.1 PERIODICAL MAINTENANCE	PAGE 22
18.2 MALFUNCTIONS	PAGE 22
18.3 FLY SLIM BATTERY (2.5 AH)	PAGE 22
18.4 FLY PLUS BATTERY (2.5 AH)	PAGE 23
18.5 FLY MINI BATTERY (2.5 AH)	PAGE 23
18.6 SLIM BATTERY (5.8 AH)	PAGE 23
18.7 MINI BATTERY (5.8 AH)	PAGE 24
18.8 PLUS BATTERY (11 AH)	PAGE 24
18.9 RACE BATTERY (11 AH)	PAGE 24
18.10 FLY RACE BATTERY (5.8 AH)	PAGE 25
19. TECHNICAL DATA	PAGE 26
19.1 CLASSIFICATION OF THE DEVICE	PAGE 26
19.2 PERFORMANCE	PAGE 26
19.3 SIZE INFORMATION	PAGE 26
19.4 ENVIRONMENTAL OPERATING CONDITIONS	PAGE 28
19.5 TRANSPORTATION AND STORAGE ENVIRONMENTAL CONDITIONS	PAGE 28
19.6 TECHNICAL DESCRIPTION OF BATTERY CHARGER	PAGE 28
19.7 TECHNICAL DESCRIPTION OF BATTERIES	PAGE 28
19.8 MAIN FEATURES	PAGE 29
20. DISPOSAL OF THE DEVICE	PAGE 29
21. PRODUCT WARRANTY	PAGE 29
22. ORIGINAL TEXT	PAGE 29

## PRELIMINARY INFORMATION

### INTRODUCTION

This User's Manual explains how to use the KLICK electric series devices that increase the independence of persons with disabilities. The KLICK product range described in this Manual includes the following models:

- ELECTRIC: Electric Standard (ES) and Electric Tetra (ET) versions
- POWER: Power Standard (PS) and Power Tetra (PT) versions
- RACE: Race Standard (RS) and Race Tetra (RT) versions
- MONSTER: Standard version (FS)
- MINI: Standard version (MS)
- CARBON: Standard version (CS)

### MANUFACTURER'S IDENTIFICATION DATA

#### KLAXON MOBILITY GMBH

Industriestrasse, 1  
9601 - Arnoldstein (Austria)  
Tel.: +43 (0)664 4681294  
e-mail: info@klaxon-kllick.com  
Hereinafter: **KLAXON**.

### REVISION INFORMATION

REV.	Description of the change	DATE
01	Initial release	27.07.2016
02	Revision no. 2.0	07.10.2016
03	Revision no. 3.0	27.12.2016
04	Revision no. 4.0	11.01.2017
05	Revision no. 5.0	08.06.2017
06	Revision no. 6.0	15.05.2018

All information, illustrations and specifications in this document are based on the latest product information available at the time of publication.

This document is subject to change due to technical innovations, without prior notice.

## 1. WARNINGS

- Read these instructions carefully before using the device.
- KLAXON declines all responsibility for failure to comply with the contents of this document.
- The use of KLICK devices is specific for persons with disabilities who use a manual wheelchair.
- The use of KLICK devices in public spaces (streets, sidewalks, squares, cycle paths, etc.) must comply with the specific regulations of the country in which they are used. KLAXON is not liable for use that is not in accordance with specific regulations in force.
- If any anomalies are found during use that may endanger the user, do not use the device and promptly contact KLAXON's technical support.
- Non-original spare parts and accessories have not been tested by the manufacturer. Therefore we cannot certify that these components are compliant with the needed performance and safety requirements.
- KLAXON is not responsible for any damage caused by the use of non-original spare parts or accessories.
- The product installation must be performed in accordance with the procedures defined by KLAXON, by authorised personnel. Neither the safety nor the effectiveness of the device can be guaranteed if these procedures are not followed, nor can the validity of the warranty.

The points of the manual marked with the following symbols should be read carefully:

DANGER	The word DANGER indicates a potentially hazardous situation, which, if not avoided, can result in death or serious personal injury.
CAUTION	The word CAUTION indicates a potentially hazardous situation, which, if not avoided, may cause minor personal injuries or property damage.
NOTE	The word NOTE indicates important instructions or special information.
	Generic warning sign: indicates a danger due to various causes.

# KLAXON Mobility GmbH - ENG

High voltage warning sign: indicates the presence of high voltages.

## 2. SAFETY NOTICE

	The user must read this document and have become familiar with its contents before using the device.
	If the device is not used respecting the specifications described herein, the security level specified by the manufacturer may be reduced.
	During the initial period of use the user must familiarize with operation to avoid potential risks caused by incorrect use.
	EMC - The device is compliant with ISO 7176-21:2009 CLAUSE 5.4 on electromagnetic compatibility, but to ensure the user's safety we recommend not using this device in the presence of strong environmental electromagnetic fields.

- Children, incapacitated persons, or those that are insufficiently familiar with the device or physically unsuited, must not use it.
- The device may skid on wet pavement, on gravel or on uneven terrain: always adjust speed and driving style to the conditions (weather, surface, individual capacity, etc.); never drive straight towards an object, and drive carefully towards narrow passages.
- Do not attempt to go over steps higher than 5 cm (for KLICK Electric Standard, Electric Tetra, Power Standard, Power Tetra, Race Standard, Race Tetra, Monster Standard versions) and 2.5 cm (for KLICK Mini Standard and Carbon Standard versions), both uphill and downhill, along the direction travelled. Do not drive the KLICK-equipped wheelchair on roads with slopes that exceed 10%. These obstacles must always be driven over head-on and slowly. The proper way to go over steps and to go up slopes is shown in Figure 1.
- If the pavement is wet or uneven, the maximum slope and the maximum height of the steps allowed must be considered lower to ensure the user's safety.
- In the event of a collision, the user could suffer serious *injury* to body parts.
- At high speed the user may lose control of the wheelchair and it may tip over. Never exceed 15 km/h, and avoid impacts in general.
- Always use the KLICK-equipped wheelchair suitably, always braking to avoid obstacles (steps, edges, door frames, etc.) or to descend from curbs or go down slopes.
- Driving performance of the KLICK-equipped wheelchair may be affected by electromagnetic fields. If a malfunction occurs in these conditions please contact KLAXON or one of its authorised resellers.
- Avoid contact with any liquid coming out of the battery if it is damaged.
- While using the device, the brake system components can get very hot and contact of these components contact with skin may cause scalding and burns: be extremely careful to avoid contact.
- Always observe the operating limits for the manual wheelchair on which the KLICK system is installed.

## 3. INTENDED USE OF THE DEVICE

KLICK devices can be used to add an auxiliary power assist system to most manual wheelchairs, quickly and easily.

Coupling the KLICK device to the wheelchair, the front castors rise off the ground and a single wheel, in the "traction unit", is used for steering.

KLICK devices are classified for use with wheelchairs to which they are coupled pursuant to CEI EN 12184. KLICK devices are also compliant with the requirements of class B with maximum range of about 50 km (for the version with a charged 11 Ah battery, on a flat course and 75 kg user).

In accordance with the definition of class B of CEI EN 12184, the device is for indoor use and can handle outdoor obstacles.

Any other use or misuse could result in hazardous situations

## 4. USAGE ENVIRONMENT

### Operating Environment

KLICK series devices are made for use coupled to most manual wheelchairs on the market. KLICK solutions can be used both indoors and outdoors, with the limits stated in this document, and in general with the same limitations set for the use of the manual wheelchair.

### Medical use and types of users

KLICK is not intended for specific clinical use, but as a support to the mobility of persons who use manual wheelchairs. Therefore, bearing in mind the operating precautions described in this document, there is no need for professional, technical or aptitude requirements to operate or use a KLICK device.

## 5. OPERATING LIMITS

The device must be used consistently with the intended use described in the preceding paragraphs, any other must be considered improper use.

In particular do not use KLICK devices:

- If not coupled with a manual wheelchair.
- After intake of alcohol or if the user's psycho-physical conditions are unsuitable.
- On roads with unsuitable pavement.
- In poor visibility (at night, in fog, etc.).
- If the user weighs over 120 kg or is heavier than allowed for the wheelchair used.
- In adverse weather (rain, strong wind, etc.).
- When going uphill or downhill on slopes that exceed 10% (see Figure 2).
- To go over steps higher than 5 cm, both uphill and downhill; this applies to Electric Standard, Electric Tetra, Power Standard, Power Tetra, Race Standard, Race Tetra, Monster Standard versions (see Figure 3).
- To go over steps higher than 2.5 cm, both uphill and downhill; this applies to the Mini Standard and Carbon Standard versions (see Figure 73).

## 6. USER REQUIREMENTS

KLICK series devices and its accessories can be used independently by users that:

- That are familiar and have practised proper use of the device sufficiently.
- That are aware of the risks related to the use of the device.
- That have sufficient physical and mental capacity to ensure the appropriate safe use of the device.

## 7. WARNINGS AND PRECAUTIONS

	This User's Manual is an integral part of the device and must always be available to the user. If it is lost or damaged, contact KLAXON.
	KLICK series devices and accessories can be used by users who are aware of the risks associated with the use thereof.
	KLICK series devices and accessories must be used only and exclusively by users that are familiar and have practised proper use of the device sufficiently.
	Absolutely never remove, modify or replace components of the device. If necessary, these operations must be carried out only by technical personnel authorised to do so by KLAXON.
	Never use the device after having drunk alcohol, taken drugs and in general if the user's psycho-physical conditions are unsuitable.
	Do not use KLICK devices for users weighing more than 120 kg; in total, the weight must not exceed the amount specified by the manufacturer of the wheelchair used.
	The user is responsible for monitoring and maintaining proper operation of the devices and of their accessories as indicated herein.
	Absolutely never use the device outside of the stated environmental operating conditions.
	Periodic maintenance must be performed to ensure proper operation and the safety of the device.
	Do not use a KLICK device to carry more than one person.
	Do not use a KLICK device unless it is coupled with a manual wheelchair.
	Do not use the supplied battery pack to power systems other than the KLICK device for which it was designed.

# KLAXON Mobility GmbH - ENG

	Do not leave the battery in the sun or adverse weather when not in use.
	Do not short-circuit the battery.
	Do not throw the battery in water.
	Do not attempt to open the battery pack.
	Do not allow children to play with the battery.
	If the battery has reached its end of life, it must be disposed of in the special containers at designated collection centres.
	While using the device, the brake system components can get very hot and contact of these components with skin may cause scalding and burns: be extremely careful to avoid contact.

	DANGER: The voltages inside the device can be lethal.
	DANGER: Do not connect/disconnect power cables when the device is on.
	DANGER: Maintenance or cleaning the device must always be carried out when the device is off and disconnected from the mains.
	DANGER: Do not remove any of the unit's cover panel, contact the Technical Support Service.

	The use of KLICK devices in public spaces (streets, sidewalks, squares, cycle paths, etc.) must comply with the specific regulations of the country in which they are used. KLAXON is not liable for use that is not in accordance with specific regulations in force.
	Adjust speed in relation to the characteristics of the wheelchair, the road conditions, the lateral inclination of the road and the radius of the curves to take.
	Do not operate the device in poor lighting conditions. The device is not suitable for use at night.
	Pay attention to sideways listing due to strong wind.
	Pay attention to sideways listing due to steering while moving forward.
	Pay attention to sideways listing due to the slope of the road.
	Do not use KLICK devices on roads with unsuitable pavement.
	Do not use KLICK devices with adverse weather (rain, strong wind, etc.).

# KLAXON Mobility GmbH - ENG

	Do not use KLICK devices (ES, ET, PS, PT, RS, RT, FS versions) to go up or down steps higher than 5 cm.
	Do not use KLICK devices (MS, CS versions) to go up or down steps higher than 2.5 cm.
	Do not use KLICK devices on slopes greater than 10%.
	When driving downhill, do not exceed two-thirds of the maximum speed; avoid sudden braking or accelerations on slopes.
	Never go over obstacles when driving uphill or downhill.
	Do not go up or down staircases.
	Do not attempt to go over obstacles diagonally; make sure that the rear wheels always go over the obstacle together. Never stop halfway. Do not attempt to go over obstacles over 5 cm high (ES, ET, PS, PT, RS, RT, FS versions). Do not attempt to go over obstacles over 2.5 cm high (MS, CS versions).
	When the vehicle is in motion, avoid moving the centre of gravity or abruptly moving your body.
	It is advisable to drive through narrow passages at minimum speed and with utmost caution.
	When using KLICK devices, the risk of tipping over during cornering increases: reduce speed before turning. Accelerate again only after completing the curve.
	When using KLICK devices, do not make U-turns on slopes greater than 5% and at a speed higher than level 1.
	It can be difficult to manoeuvre in front of a lift or at the entrance of a building since the wheelchair's turning radius increases when coupled with a KLICK device; this could conflict with the standards of the buildings or the actual size of the entrances.
	When entering a building and/or an elevator, always take into account the turning radius of your wheelchair equipped with the KLICK device. Avoid driving into situations from which you would not be capable of exiting because you would be unable to turn your wheelchair.
	Absolutely never use the Klick device along tramway, rail and light railway tracks. Pay the utmost attention at level crossings, keeping the Klick's and wheelchair wheels at due distance. Failure to comply with this precaution could cause serious damages to the device and the user. The same precautions apply to water drains, manholes and all kinds of hollows along the way.
	When using the KLICK MS and CS devices, pay the utmost attention to safeguard the braking system; the user must especially avoid (side and frontal) obstacles that can damage the disc.

## 8. MARKINGS

The device is uniquely identified by its serial number (SN), shown on the plate of the device, attached to the motor unit near the battery. The meaning of the symbols shown in the plate are as follows:

Symbol	Description
	Manufacturer
	Reference number
	Serial number

Symbol	Description
	Year of manufacture
	Symbol indicating: Follow the instructions for use
	CE mark
	Electrical device disposal via separated collection (special disposal, electrical components)

Picture of the CE Label, applied on the Klick devices.

## 9. DESCRIPTION OF THE DEVICE

### 9.1 PRODUCT STRUCTURE

#### A. KLICK ELECTRIC STANDARD VARIANT / KLICK POWER STANDARD VARIANT

Figure 4 shows the KLICK ELECTRIC STANDARD device coupled to a "standard" wheelchair and identifies the elements that make them up.

1. Steering wheel
2. Brake system
3. Motor unit
4. Kickstand
5. Standard Handlebar
6. Battery
7. Standard Connector

#### B. KLICK ELECTRIC TETRA VARIANT / KLICK POWER TETRA VARIANT / KLICK RACE TETRA VARIANT

Figure 5 shows the KLICK POWER TETRA device coupled to a "standard" wheelchair and identifies the elements that make them up.

1. Steering wheel
2. Brake system
3. Motor unit
4. Kickstand
5. Tetra Handlebar (with acceleration and braking functions)
6. Battery
7. Connector tetra

#### C. KLICK RACE STANDARD VARIANT

Figure 71 shows the KLICK RACE STANDARD device and identifies the elements that make it up.

1. Steering wheel and Motor unit
2. Brake system
3. Kickstand
4. Standard Handlebar
5. Battery

#### D. KLICK MONSTER STANDARD VARIANT

Figure 72 shows the KLICK MONSTER STANDARD device and identifies the elements that make it up.

1. Steering wheel
2. Brake system
3. Motor group
4. Kickstand

5. Standard Handlebar
6. Battery

## E. KLICK MINI STANDARD VARIANT / KLICK CARBON STANDARD VARIANT

Figure 70 shows the KLICK CARBON STANDARD device and identifies the elements that make it up.

1. Steering wheel and Motor group
2. Brake system
3. Kickstand
4. Standard Handlebar
5. Battery

## 9.2 KLICK® CONNECTION

See Figure 6.

### A. CLAMPING RINGS

The clamping rings are the parts of the KLICK connection that always remain attached to the wheelchair frame (see Figure 7). Together with the crossbeam, they must be installed on the wheelchair by authorised KLAXON personnel and must never be removed by the user during use. Various clamping ring sizes and shapes are available to suit the different tubing diameters and types of wheelchair frames.

### B. CROSSBEAM

Figure 8 illustrates a CROSSBEAM and identifies the elements that make it up.

1. CROSSBEAM
2. Square seats for insertion of the CONNECTOR
3. Fixing pins
4. Fastening and adjustment straps.

### C. CONNECTOR (WITH DETAIL OF PATENTED "KLICK® SYSTEM")

Figure 9 illustrates a STANDARD CONNECTOR and identifies the elements that make it up.

1. CONNECTOR lock/unlock lever
2. Mounting system in the CROSSBEAM (with ball-type lock)
3. KLICK device release button
4. KLICK® coupling (Worldwide Patented)
5. Seat of upper pin
6. Seat of lower pin.

### D. TETRA CONNECTOR (WITH DETAIL OF PATENTED "KLICK® SYSTEM")

Figure 10 illustrates a TETRA CONNECTOR and identifies the elements that make it up.

1. Velcro grip of CONNECTOR
2. Mounting system in the CROSSBEAM (with ball-type lock)
3. Easy release lever of the KLICK device
4. KLICK® coupling (Worldwide Patented)
5. Seat of upper pin
6. Seat of lower pin

## 9.3 BATTERY CHARGER

Figure 11 shows the battery charger that is supplied with the FLY SLIM (2.5 Ah), SLIM (5.8 Ah), FLY MINI (2.5 Ah), MINI (5.8 Ah), RACE (11 Ah) and FLY RACE (5,8 Ah) batteries, which are compatible with the KLICK Electric Standard, Electric Tetra, Mini Standard, Carbon Standard, Race Standard, Race Tetra and Monster Standard models.

Figure 12 shows the battery charger that is supplied with the FLY PLUS (2.5 Ah) and PLUS (11 Ah) batteries, which are compatible with the KLICK Power Standard and Power Tetra models.

## 9.4 ACCESSORIES

KLICK can be equipped with various accessories. You may need to consult the appropriate User's Manual (ask KLAXON or visit our website: [www.klaxon-klick.com](http://www.klaxon-klick.com)) for some of them.

### A. QUICK RELEASE HANDLEBAR (FIGURE 13)

The purpose of this accessory is to be able to quickly move the handlebar from a "driving" position to a "rest" position so that it is not in the way of the user's legs while coupling/releasing the device. This operation is controlled by the lever on the top of the stem.

- 1) Move the safety latch located above the release lever (Figure 14).
- 2) Raise the lever by releasing the handle as shown (Figure 15).

After doing so, you may:

- adjust the handlebar as you prefer and lock it in place by lowering the release lever (Figure 16)
- fully rotate the stem by putting the handlebar ahead of the stem seat (in this case it cannot be blocked by closing the lever - (Figure 17).

CAUTION: this accessory is not compatible with any of the "Tetra" version models. For additional information, refer to the material (written, audio/video) on [www.klaxon-klick.com](http://www.klaxon-klick.com) or ask an authorized KLAXON dealer.

## 9.5 DOCUMENTATION

This USER's MANUAL is provided with the Device in printed format and must be stored with the utmost care. If you lose the above documents, request a copy of them in writing to KLAXON.

## 10. INSTALLATION AND COUPLING

### 10.1 INSTALLATION

KLICK devices are provided by KLAXON (or by an authorised technician) only after having pre-installed the coupling set on the wheelchair.

### 10.2 COUPLING STEPS

During the entire coupling operation, stop in a flat area, as free as possible from uneven ground and away from hazards which could cause personal injury.

- Insert the CROSSBEAM in one of the two CLAMPING RINGS and secure it by inserting the specific PIN (Figure 18).
- Insert the CROSSBEAM also in the second CLAMPING RING and secure it by inserting the specific PIN (Figure 19).  
Note: The CROSSBEAM can always remain installed on the wheelchair; if necessary (for loading and unloading from vehicles and closing the folding wheelchair) it can be removed and placed in a safe place.
- Insert the CONNECTOR in the square seat of the CROSSBEAM, pressing the lever located on its bottom. Once inserted, release the lever and make sure that the CONNECTOR is securely locked (Figure 20).
- After having inserted the CONNECTOR, the device will be as shown (Figure 21).
- To secure the KLICK device to the connection, move the wheelchair into position as shown (Figures 22 and 23). Make sure the opening of the coupling is placed over the upper pin (Figure 24).
- Holding it in place, brake the wheelchair with its brakes and grip the handlebar; lift KLICK until the upper pin is fully inserted in the opening of the coupling (Figure 25).
- At this point, continue to raise the handlebar by pushing it forward until the lower pin is fully engaged with the connection; at that point the safety button is triggered (Figure 26).

### CAUTION!

Once the device is connected to the wheelchair, it is important to ensure that:

- The ground clearance of the front wheels of the wheelchair is 4 cm (with the wheelchair empty - Figure 27).
- The kickstand is located above the line of the front wheels and placed below the footboard (Figure 28).

### CAUTION

If these requirements are not met, never use the device for any reason and immediately contact an authorised KLAXON service centre.

### 10.3 WHEELCHAIR

KLICK devices can be installed only on compatible wheelchairs. KLICK products must be only be installed by authorised KLAXON technicians who may decide, under their own responsibility, if the specific wheelchair can be properly used with the KLICK device.

## 11. KLICK ELECTRIC STANDARD VERSION / KLICK POWER STANDARD VERSION OPERATION

### 11.1 USER INTERFACE

The device is equipped with a multifunction LCD display, used to switch the device on and off, select the maximum speed and display several operating parameters (Figure 29).

- The icon located top left (1) shows the current battery charge level.
- The number (2) in the centre of the display shows the speed, in km/h.
- The number (3) in the lower part of the display shows the distance travelled, in km.
- The number (4) shows the desired maximum speed.

### 11.2 TURNING ON AND SETTING UP THE DEVICE (FIGURE 29).

#### A. Turning On and Selecting the Speed

- Switch on the battery (for more information please refer to the instructions starting from section 18.3).
- Press and hold the "M" button for 5 seconds, until the display turns on.
- Press the "+" e "-" buttons to select the desired maximum speed level (viewable in the left panel of the display, number (5) in the legend); various speed levels can be selected, between 0 (motor disabled), 1 (minimum speed, 3 km/h), 2 (normal speed, 6 km/h), 3 (medium speed, 10 km/h), 4 (high speed, 13 km/h) and 5 (maximum speed, 15 km/h).

#### B. Navigating the data

- After turning on the display, the initial screen shows (in number 3) the total distance (ODO) the device has been used.
- Press the "M" button once to enter the second screen. It will show the partial distance (TRIP - number 3).

# KLAXON Mobility GmbH - ENG

- Pressing the "M" button again for two times will display the time the device has been on, in minutes (TIME – number 3).
- Pressing the "M" button again for three times will display the maximum speed (ODO MAX – number 2); pressing the "M" button again will display in the same screen the average speed (ODO AVG).
- To reset trip data, hold down the "M" and "-" buttons together for 5 seconds.

## C. Screen Backlighting

- Hold the "+" button for 5 seconds to turn on the screen backlight.
- To turn off the backlight, press the "+" button for 5 seconds again.
- When the device is turned off, the light is also automatically switched off.

## D. Speed Hold Function

- Press the "-" button for 5 seconds to turn on the speed hold function. This function accelerates the device up to a speed of 4.5 km/h and holds it as long as the button is pressed.
- When the button is released, the device instantly stops accelerating.

DANGER	Every action and/or tampering of the display to change performance and KLICK's security is strictly prohibited. The sole and only actions possible are those specified in paragraph 11.2 of this User's Manual.
--------	---

NOTE: For further information on the display, see [www.klaxon-klick.com](http://www.klaxon-klick.com) or contact an authorised KLAXON dealer.

## 11.3 USE

Figure 30 shows the Standard handlebar of the Klick device and it identifies the elements that make it up (listed below).

1. Handlebar
2. Accelerator control
3. Reverse button
4. Display
5. Brake lever

Once the device has been installed, the display has been turned on and a speed has been selected, use the accelerator control to make the device start to move (Figure 31.1). To stop the device, use the brake lever. The KLICK brake is equipped with an additional safety device which deactivates the motor when the lever is operated (Figure 31.2).

The device is equipped with reverse as standard equipment. To use reverse, press the green button once, then press the accelerator to start moving the device backward.

To return to the forward operation, release the accelerator and press the green button only once.

CAUTION	The reverse button has only luminous warning systems to show whether it is selected or not. This is shown by the button itself: when the button is fully extended (Figure 32) the reverse gear is OFF (light GREEN off). When the button is fully pressed (Figure 33) the reverse gear is ON (light GREEN on).
CAUTION	After inserting the battery in its housing and locked by turning the key, remove it and put it in a safe place. It is absolutely forbidden to use the KLICK device with the key insert in the security block.

The brake lever, the accelerator control, the reverse button and the display can be installed either on the left or on the right side of the handlebar according to the user's preference.

DANGER	For installation and positioning of controls (display, accelerator control, reverse button, brake lever), contact an authorised KLAXON service centre.
DANGER	Before using reverse to move the device, make sure that there are no persons, obstacles, stairs, slopes behind the wheelchair.
DANGER	At each restart pay maximum attention, before accelerating, to check if forward or reverse operation is currently selected, especially when close to obstacles, drop-offs and any other kind of danger.
DANGER	If obstacles, drop-offs and dangers are too close to be safe, turn off the KLICK device and manually move the wheelchair to a safe place.

## 11.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR

To perform the operation properly, carefully follow the following instructions.

- Stop the wheelchair on a flat surface.
- Apply the brake on the wheelchair.
- Turn off the display by pressing and holding the "M" button for 5 seconds.

- Push the handlebar forward with one hand grasping it in the middle and simultaneously press the release button on the CONNECTOR with the other hand. This frees the lower pin of the KLICK coupling (Figure 34).
- Reduce forward pressure on the handlebar while holding the release button on the CONNECTOR, thus lowering the wheelchair so that its front wheels make contact with the pavement (Figure 35).
- At this point the upper pin will already be pulled out the front hook of the CONNECTOR; release the wheelchair's brakes and push yourself back to move away from the KLICK device (which in the meantime will remain upright thanks to its KICKSTAND - Figure 36).
- Remove the CONNECTOR, pulling the lever located on its bottom (Figure 20).

<b>DANGER</b>	During the KLICK release operation, pay attention to not endanger the top parts of your body (fingers, hands, arms) as they could be affected by the sudden movement of the handlebar (or other parts of the device) if it is released too quickly. To avoid this danger, after pressing the release button, gently release the handlebar by lowering the wheelchair gradually.
---------------	---

## 12. KLICK ELECTRIC TETRA VERSION / KLICK POWER TETRA VERSION OPERATION

### 12.1 USER INTERFACE

The device is equipped with a multifunction LCD display, used to switch the device on and off, select the maximum speed and display several operating parameters (Figure 29).

- The icon located top left (1) shows the current battery charge level.
- The number (2) in the centre of the display shows the speed, in km/h.
- The number (3) in the lower part of the display shows the distance travelled, in km.
- The number (4) shows the desired maximum speed.

### 12.2 TURNING ON AND SETTING UP THE DEVICE (FIGURE 29).

#### A. Turning On and Selecting the Speed

- Switch on the battery (for more information please refer to the instructions starting from section 18.3).
- Press and hold the "M" button for 5 seconds, until the display turns on.
- Press the "+" e "-" buttons to select the desired maximum speed level (viewable in the left panel of the display, number (5) in the legend); various speed levels can be selected, between 0 (motor disabled), 1 (minimum speed, 3 km/h), 2 (normal speed, 6 km/h), 3 (medium speed, 10 km/h), 4 (high speed, 13 km/h) and 5 (maximum speed, 15 km/h).

#### B. Navigating the data

- After turning on the display, the initial screen shows (in number 3) the total distance (ODO) the device has been used.
- Press the "M" button once to enter the second screen. It will show the partial distance (TRIP - number 3).
- Pressing the "M" button again for two times will display the time the device has been on, in minutes (TIME - number 3).
- Pressing the "M" button again for three times will display the maximum speed (ODO MAX - number 2); pressing the "M" button again will display in the same screen the average speed (ODO AVG).
- To reset trip data, hold down the "M" and "-" buttons together for 5 seconds.

#### C. Screen Backlighting

- Hold the "+" button for 5 seconds to turn on the screen backlight.
- To turn off the backlight, press the "+" button for 5 seconds again.
- When the device is turned off, the light is also automatically switched off.

#### D. Speed Hold Function

- Press the "-" button for 5 seconds to turn on the speed hold function. This function accelerates the device up to a speed of 4.5 km/h and holds it as long as the button is pressed.
- When the button is released, the device instantly stops accelerating.

<b>DANGER</b>	Every action and/or tampering of the display to change performance and KLICK's security is strictly prohibited. The sole and only actions possible are those specified in paragraph 12.2 of this User's Manual.
---------------	---

NOTE: For further information on the display, see [www.klaxon-klick.com](http://www.klaxon-klick.com) or contact an authorised KLAXON dealer.

### 12.3 USE

Figure 37 shows the Tetra handlebar of the Klick device and it identifies the elements that make it up (listed below).

1. Tetra handlebar with PUSH AND PULL acceleration/braking system.
2. Display
3. Reverse button

# KLAXON Mobility GmbH - ENG

Once the device has been installed, the display has been turned on and a speed has been selected, use the handlebar (pulling it towards you) to make the device start to move (Figure 38). To stop the device, use the brake system by pushing the handlebar as shown (Figure 39).

The device is equipped with reverse. To use reverse, press the green button once, then use the handlebar to start moving the device backward.

To return to the forward operation, release the accelerator and press the green button only once.

CAUTION	The reverse button has only luminous warning systems to show whether it is selected or not. This is shown by the button itself: when the button is fully extended (Figure 32) the reverse gear is OFF (light GREEN off). When the button is fully pressed (Figure 33) the reverse gear is ON (light GREEN on).
CAUTION	After inserting the battery in its housing and locked by turning the key, remove it and put it in a safe place. It is absolutely forbidden to use the KLICK device with the key insert in the security block.

The reverse button and the display can be installed either on the left or on the right side of the handlebar according to user's preference.

DANGER	For installation and positioning of controls (display, reverse button), contact an authorised KLAXON service centre.
DANGER	Before using reverse to move the device, make sure that there are no persons, obstacles, stairs, slopes behind the wheelchair.
DANGER	At each restart pay maximum attention, before accelerating, to check if forward or reverse operation is currently selected, especially when close to obstacles, drop-offs and any other kind of danger.
DANGER	If obstacles, drop-offs and dangers are too close to be safe, turn off the KLICK device and manually move the wheelchair to a safe place.

## 12.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR

To perform the operation properly, carefully follow the following instructions.

- Stop the wheelchair on a flat surface.
- Apply the brake on the wheelchair.
- Turn off the display by pressing and holding the "M" button for 5 seconds.
- Firmly push the quick release lever of the CONNECTOR TETRA sideways to the right. By doing so, the lower pin is automatically released and the KLICK disengages from the CONNECTOR, lowering the wheelchair and bringing the front wheels in contact with the ground (Figures 40 and 41).
- At this point the upper pin will already be pulled out the front hook of the CONNECTOR; release the wheelchair's brakes and push yourself back to move away from the KLICK device (which in the meantime will remain upright thanks to its KICKSTAND (Figure 42).
- Remove the CONNECTOR, pulling the Velcro grip upward and forward (Figure 43).

DANGER	When releasing the KLICK from the Connector tetra, the wheelchair may be lowered abruptly until the front wheels touch the ground. Pay maximum attention to not endanger the top parts of your body (fingers, hands, arms) as they could be hit by the handlebar or by other parts of the device.
--------	---

## 13. KLICK RACE STANDARD VERSION OPERATION

### 13.1 USER INTERFACE

The device is equipped with a multifunction LCD display, used to switch the device on and off, select the maximum speed and display several operating parameters (Figure 60).

- The icon located top left (1) shows the current battery charge level.
- The number (2) in the centre of the display shows the speed, in km/h.
- The number (3) in the left part of the display shows the desired maximum speed level.
- The number (4) shows the distance travelled, in km.
- The number (5) shows the time the device has been on, in minutes.
- The number (6) shows the data regarding the device's motor (power absorption in watts and internal temperature in °C).
- The number (7) shows the external temperature, in °C.

### 13.2 TURNING ON AND SETTING UP THE DEVICE (FIGURE 60).

#### A. Turning On and Selecting the Speed

- Switch on the battery (for more information please refer to the instructions starting from section 18.3).
- Press and hold the "POWER" button for 5 seconds, until the display turns on.

- Press the **"UP ARROW"** and **"DOWN ARROW"** buttons to select the desired maximum speed level (viewable in the left panel of the display, number (5) in the legend); various speed levels can be selected, between 0 (motor disabled), 1 (minimum speed, 3 km/h), 2 (normal speed, 6 km/h), 3 (medium speed, 10 km/h), 4 (high speed, 13 km/h) and 5 (maximum speed, 15 km/h).

## B. Navigating the data

- After turning on the display, the initial screen shows the partial distance (DST – number 4), the partial time the device has been on (TM – number 5), the motor's current power absorption (MOTOR W – number 6) and the external temperature (number 7).
- Pressing the **"POWER"** button once will open the second screen, which shows the average speed (AVS – number 2), the total distance (ODO – number 4), the total time the device has been on (TTM – number 5) and the motor's internal temperature (MOTOR °C – number 6).
- Pressing the **"POWER"** button again will open the third screen, which shows the maximum speed (MXS – number 2) and the motor's electrical tension in volts (VOL. – number 4).
- To reset trip data, hold down the **"UP ARROW"** and **"DOWN ARROW"** buttons together for 5 seconds; when the data start flashing, press the **"POWER"** button to reset them.

## C. Screen Backlighting

- Hold the **"UP ARROW"** button for 5 seconds to turn on the screen backlight.
- To turn off the backlight, press the **"UP ARROW"** button for 5 seconds again.
- When the device is turned off, the light is also automatically switched off.

CAUTION: this procedure must also be used to turn on and off the front LED light connected to the RACE battery, if the device is equipped with it (listed under accessories in the order form).

## D. Speed Hold Function

- Press the **"DOWN ARROW"** button for 5 seconds to turn on the speed hold function. This function accelerates the device up to a speed of 4.5 km/h and holds it as long as the button is pressed.
- When the button is released, the device instantly stops accelerating.

## E. Cruise Control function

- To activate this function the speed has to be higher than 8 km/h.
- Keep a constant speed for at least 10 seconds and press the **"DOWN ARROW"** key for 5 seconds to automatically insert the Cruise Control function, which will run the KLICK at the actual speed; the display at number (3) will show the letter "C" as long as the function is activated.
- The Cruise Control function must be activated by one of the following modes: operating on the Brake lever (left or right), operating on the Accelerator control, pressing one of the buttons on the Display.

ATTENTION: By pressing the Reverse button the Cruise Control function will be NOT switched off.

DANGER	During the use of the Cruise Control function it is compulsory to pay the utmost attention to road hazards in order to be able to intervene on the commands promptly avoiding any kind of collision that could cause damage to people and / or things.
DANGER	Every action and/or tampering of the display to change performance and KLICK's security is strictly prohibited. The sole and only actions possible are those specified in paragraph 13.2 of this User's Manual.

NOTE: For further information on the display, see [www.klaxon-klick.com](http://www.klaxon-klick.com) or contact an authorised KLAXON dealer.

## 13.3 USE

Figure 62 shows the Standard handlebar of the Klick device and it identifies the elements that make it up (listed below).

1. Handlebar
2. Accelerator control
3. Reverse button
4. Display
5. Brake lever
6. Display controls pushbutton panel

Once the device has been installed, the display has been turned on and a speed has been selected, use the accelerator control to make the device start to move (Figure 31.1). To stop the device, use the brake lever. The KLICK brake is equipped with an additional safety device which deactivates the motor when the lever is operated (Figure 31.2).

The device is equipped with reverse as standard equipment. To use reverse, press the red button once, then press the accelerator to start moving the device backward.

To return to the forward operation, release the accelerator and push again the red button once.

CAUTION	The reverse button has only luminous warning systems to show whether it is selected or not. This can occur in two ways: VISUAL-> when button is lifted to the maximum then the reverse gear is DISENGAGED; when the button is turned to maximum then the reverse gear is ENGAGED. LUMINOUS > when the reverse gear is ENGAGED, the display flashes off and on repeatedly the backlight. When the reverse gear is DISENGAGED this function stops responding.
---------	--

<b>CAUTION</b>	After inserting the battery in its housing and locked by turning the key, remove it and put it in a safe place. It is absolutely forbidden to use the KLICK device with the key insert in the security block.
----------------	---

The brake lever, the accelerator control, the reverse button, the display and the display controls pushbutton panel can be installed either on the left or on the right side of the handlebar according to user's preference.

<b>CAUTION</b>	For installation and positioning of controls (display, accelerator control, reverse button, brake lever, display controls pushbutton panel), contact an authorised KLAXON service centre.
<b>CAUTION</b>	Before using reverse to move the device, make sure that there are no persons, obstacles, stairs, slopes behind the wheelchair.
<b>CAUTION</b>	At each restart pay maximum attention, before accelerating, to check if forward or reverse operation is currently selected, especially when close to obstacles, drop-offs and any other kind of danger.
<b>CAUTION</b>	If obstacles, drop-offs and dangers are too close to be safe, turn off the KLICK device and manually move the wheelchair to a safe place.

### 13.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR

To perform the operation properly, carefully follow the following instructions.

- Stop the wheelchair on a flat surface.
- Apply the brakes on the wheelchair.
- Turn off the display by pressing and holding the **"POWER"** button for 5 seconds.
- Push the handlebar forward with one hand grasping it in the middle and simultaneously press the release button on the CONNECTOR with the other hand. This frees the lower pin of the KLICK coupling (Figure 34).
- Reduce forward pressure on the handlebar while holding the release button on the CONNECTOR, thus lowering the wheelchair so that its front wheels make contact with the pavement (Figure 35).
- At this point the upper pin will already be pulled out the front hook of the CONNECTOR; release the wheelchair's brakes and push yourself back to move away from the KLICK device (which in the meantime will remain upright thanks to its KICKSTAND - Figure 36).
- Remove the CONNECTOR, pulling the lever located on its bottom (Figure 20).

<b>DANGER</b>	During the KLICK release operation, pay attention to not endanger the top parts of your body (fingers, hands, arms) as they could be affected by the sudden movement of the handlebar (or other parts of the device) if it is released too quickly. To avoid this danger, after pressing the release button, gently release the handlebar by lowering the wheelchair gradually.
---------------	---

## 14. KLICK RACE TETRA VERSION OPERATION

### 14.1 USER INTERFACE

The device is equipped with a multifunction LCD display, used to switch the device on and off, select the maximum speed and display several operating parameters (Figure 60).

- The icon located top left (1) shows the current battery charge level.
- The number (2) in the centre of the display shows the speed, in km/h.
- The number (3) in the left part of the display shows the desired maximum speed level.
- The number (4) shows the distance travelled, in km.
- The number (5) shows the time the device has been on, in minutes.
- The number (6) shows the data regarding the device's motor (power absorption in watts and internal temperature in °C).
- The number (7) shows the external temperature, in °C.

### 14.2 TURNING ON AND SETTING UP THE DEVICE (FIGURE 60).

#### A. Turning On and Selecting the Speed

- Switch on the battery (for more information please refer to the instructions starting from section 18.3).
- Press and hold the **"POWER"** button for 5 seconds, until the display turns on.
- Press the **"UP ARROW"** and **"DOWN ARROW"** buttons to select the desired maximum speed level (viewable in the left panel of the display, number (5) in the legend); various speed levels can be selected, between 0 (motor disabled), 1 (minimum speed, 3 km/h), 2 (normal speed, 6 km/h), 3 (medium speed, 10 km/h), 4 (high speed, 13 km/h) and 5 (maximum speed, 15 km/h).

#### B. Navigating the data

- After turning on the display, the initial screen shows the partial distance (DST – number 4), the partial time the device has been on (TM – number 5), the motor's current power absorption (MOTOR W – number 6) and the external temperature (number 7).
- Pressing the **"POWER"** button once will open the second screen, which shows the average speed (AVS – number 2), the total distance (ODO – number 4), the total time the device has been on (TTM – number 5) and the motor's internal temperature (MOTOR °C – number 6).

- Pressing the **"POWER"** button again will open the third screen, which shows the maximum speed (MXS – number 2) and the motor's electrical tension in volts (VOL – number 4).
- To reset trip data, hold down the **"UP ARROW"** and **"DOWN ARROW"** buttons together for 5 seconds; when the data start flashing, press the **"POWER"** button to reset them.

## C. Screen Backlighting

- Hold the **"UP ARROW"** button for 5 seconds to turn on the screen backlight.
- To turn off the backlight, press again the **"UP ARROW"** button for 5 seconds.
- When the device is turned off, the light is also automatically switched off.

CAUTION: this procedure must also be used to turn on and off the front LED light connected to the RACE battery, if the device is equipped with it (listed under accessories in the order form).

## D. Speed Hold Function

- Press the **"DOWN ARROW"** button for 5 seconds to turn on the speed hold function. This function accelerates the device up 4.5 km/h and holds this speed as long as the button is pressed.
- When the button is released, the device instantly stops accelerating.

DANGER	In the Klick Race Tetra version the Cruise Control function, although present, CAN NOT BE USED under any circumstances and for any reason; this function is strictly prohibited for safety reasons, safety of the user, persons and / or things that can be nearby.
DANGER	Every action and/or tampering of the display to change performance and KLICK's security is strictly prohibited. The sole and only actions possible are those specified in paragraph 14.2 of this User's Manual.

NOTE: For further information on the display, see [www.klaxon-klick.com](http://www.klaxon-klick.com) or contact an authorised KLAXON dealer.

## 14.3 USE

Figure 63 shows the Tetra handlebar of the Klick device and it identifies the elements that make it up (listed below).

1. Tetra handlebar with PUSH AND PULL acceleration/brake system
2. Display
3. Reverse button
4. Display controls pushbutton panel

Once the device has been installed, the display has been turned on and a speed has been selected, use the handlebar (pulling it toward you) to make the device start to move (Figure 38). To stop the device, activate the brake system pushing it forward (Figure 39).

The device is equipped with reverse. To use reverse, press the red button once; at this point, use the handlebar to make the device start to move backward.

To return to forward operation, release the accelerator and press again the red button once.

CAUTION	The reverse button has only luminous warning systems to show whether it is selected or not. This can occur in two ways: VISUAL-> when button is lifted to the maximum then the reverse gear is DISENGAGED; when the button is turned to maximum then the reverse gear is ENGAGED. LUMINOUS > when the reverse gear is ENGAGED, the display flashes off and on repeatedly the backlight. When the reverse gear is DISENGAGED this function stops responding.
CAUTION	After inserting the battery in its housing and locked by turning the key, remove it and put it in a safe place. It is absolutely forbidden to use the KLICK device with the key insert in the security block.

The reverse button, the display controls pushbutton panel and the display can be installed either on the left or on the right side of the handlebar according to user's preference.

DANGER	For installation and positioning of controls (display, reverse button, display controls pushbutton panel), contact an authorised KLAXON service centre.
DANGER	Before using reverse to move the device, make sure that there are no persons, obstacles, stairs, slopes behind the wheelchair.
DANGER	At each restart pay maximum attention, before accelerating, to check if forward or reverse operation is currently selected, especially when close to obstacles, drop-offs and any other kind of danger.
DANGER	If obstacles, drop-offs and dangers are too close to be safe, turn off the KLICK device and manually move the wheelchair to a safe place.

## 14.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR

To perform the operation properly, carefully follow the following instructions.

- Stop the wheelchair on a flat surface.
- Apply the brake on the wheelchair.
- Turn off the display by pressing and holding the **"POWER"** button for 5 seconds.
- Firmly push the quick release lever of the CONNECTOR TETRA sideways to the right. By doing so, the lower pin is automatically released and the KLIICK disengages from the CONNECTOR, lowering the wheelchair and bringing the front wheels in contact with the ground (Figures 40 and 41).
- At this point the upper pin will already be pulled out the front hook of the CONNECTOR; release the wheelchair's brakes and push yourself back to move away from the KLIICK device (which in the meantime will remain upright thanks to its KICKSTAND - Figure 42).
- Remove the CONNECTOR, pulling the Velcro grip upward and forward (Figure 43).

<b>DANGER</b>	During the KLIICK release operation with the Connector tetra, the wheelchair may drop suddenly until the front wheels touch the pavement. Pay attention to not endanger the top parts of your body (fingers, hands, arms) as they could be affected by the handlebar or other parts of the device.
---------------	--

## 15. KLIICK MONSTER STANDARD VERSION OPERATION

### 15.1 USER INTERFACE

The device is equipped with a multifunction LCD display, used to switch the device on and off, select the maximum speed and display several operating parameters (Figure 60).

- The icon located top left (1) shows the current battery charge level.
- The number (2) in the centre of the display shows the speed, in km/h.
- The number (3) in the left part of the display shows the desired maximum speed level.
- The number (4) shows the distance travelled, in km.
- The number (5) shows the time the device has been on, in minutes.
- The number (6) shows the data regarding the device's motor (power absorption in watts and internal temperature in °C).
- The number (7) shows the external temperature, in °C.

### 15.2 TURNING ON AND SETTING UP THE DEVICE (FIGURE 60).

#### A. Turning On and Selecting the Speed

- Switch on the battery (for more information please refer to the instructions starting from section 18.3).
- Press and hold the **"POWER"** button for 5 seconds, until the display turns on.
- Press the **"UP ARROW"** and **"DOWN ARROW"** buttons to select the desired maximum speed level (viewable in the left panel of the display, number (5) in the legend); various speed levels can be selected, between 0 (motor disabled), 1 (minimum speed, 3 km/h), 2 (normal speed, 6 km/h), 3 (medium speed, 10 km/h), 4 (high speed, 13 km/h) and 5 (maximum speed, 15 km/h).

#### B. Navigating the data

- After turning on the display, the initial screen shows the partial distance (DST – number 4), the partial time the device has been on (TM – number 5), the current motor's power absorption (MOTOR W – number 6) and the external temperature (number 7).
- Pressing the **"POWER"** button once will open the second screen, which shows the average speed (AVS – number 2), the total distance (ODO – number 4), the total time the device has been on (TTM – number 5) and the motor's internal temperature (MOTOR °C – number 6).
- Pressing the **"POWER"** button again will open the third screen, which shows the maximum speed (MXS – number 2) and the motor's electrical tension in volts (VOL – number 4).
- To reset trip data, hold down the **"UP ARROW"** and **"DOWN ARROW"** buttons together for 5 seconds; when the data start flashing, press the **"POWER"** button to reset them.

#### C. Screen Backlighting

- Hold the **"UP ARROW"** button for 5 seconds to turn on the screen backlight.
- To turn off the backlight, press the **"UP ARROW"** button for 5 seconds again.
- When the device is turned off, the light is also automatically switched off.

CAUTION: this procedure must also be used to turn on and off the front LED light connected to the RACE battery, if the device is equipped with it (listed under accessories in the order form).

#### D. Speed Hold Function

- Press the **"DOWN ARROW"** button for 5 seconds to turn on the speed hold function. This function accelerates the device up to a speed of 4,5 km/h and holds it as long as the button is pressed.
- When the button is released, the device instantly stops accelerating.

#### E. Cruise Control function

- To activate this function the speed has to be higher than 8 km/h.
- Keep a constant speed for at least 10 seconds and press the **"DOWN ARROW"** key for 5 seconds to automatically insert the Cruise Control function, which will run the KLIICK at the actual speed; the display at number (3) will show the letter "C" as long as the function is activated.
- The Cruise Control function must be activated by one of the following modes: operating on the Brake lever (left or right), operating on the Accelerator control, pressing one of the buttons on the Display.

ATTENTION: By pressing the Reverse button the Cruise Control function will be NOT switched off.

# KLAXON Mobility GmbH - ENG

DANGER	During the use of the Cruise Control function it is compulsory to pay the utmost attention to road hazards in order to be able to intervene on the commands promptly avoiding any kind of collision that could cause damage to people and / or things.
DANGER	Every action and/or tampering of the display to change performance and KLICK's security is strictly prohibited. The sole and only actions possible are those specified in paragraph 15.2 of this User's Manual.

NOTE: For further information on the display, see [www.klaxon-klick.com](http://www.klaxon-klick.com) or contact an authorised KLAXON dealer.

## 15.3 USE

Figure 62 shows the Standard handlebar of the Klick device and it identifies the elements that make it up (listed below).

1. Handlebar
2. Accelerator control
3. Reverse button
4. Display
5. Brake Lever
6. Display controls pushbutton panel

Once the device has been installed, the display has been turned on and a speed has been selected, use the accelerator control to make the device start to move (Figure 31.1). To stop the device, use the brake lever. The KLICK brake is equipped with an additional safety device which deactivates the motor when the lever is operated (Figure 31.2).

The device is equipped with reverse as standard equipment. To use reverse, press the red button once, then press the accelerator to start moving the device backward.

To return to the forward operation, release the accelerator and push again the red button once.

CAUTION	The reverse button has only luminous warning systems to show whether it is selected or not. This can occur in two ways: VISUAL-> when button is lifted to the maximum then the reverse gear is DISENGAGED; when the button is turned to maximum then the reverse gear is ENGAGED. LUMINOUS > when the reverse gear is ENGAGED, the display flashes off and on repeatedly the backlight. When the reverse gear is DISENGAGED this function stops responding.
CAUTION	After inserting the battery in its housing and locked by turning the key, remove it and put it in a safe place. It is absolutely forbidden to use the KLICK device with the key insert in the security block.

The brake lever, the accelerator control, the reverse button, the display and the display controls pushbutton panel can be installed either on the left or on the right side of the handlebar according to user's preference.

DANGER	For installation and positioning of controls (display, accelerator control, reverse button, brake lever, display controls pushbutton panel), contact an authorised KLAXON service centre.
DANGER	Before using reverse to move the device, make sure that there are no persons, obstacles, stairs, slopes behind the wheelchair.
DANGER	At each restart pay maximum attention, before accelerating, to check if forward or reverse operation is currently selected, especially when close to obstacles, drop-offs and any other kind of danger.
DANGER	If obstacles, drop-offs and dangers are too close to be safe, turn off the KLICK device and manually move the wheelchair to a safe place.

## 15.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR

To perform the operation properly, carefully follow the following instructions.

- Stop the wheelchair on a flat surface.
- Apply the brakes on the wheelchair.
- Turn off the display by pressing and holding the "POWER" button for 5 seconds.
- Push the handlebar forward with one hand grasping it in the middle and simultaneously press the release button on the CONNECTOR with the other hand. This frees the lower pin of the KLICK coupling (Figure 34).
- Reduce forward pressure on the handlebar while holding the release button on the CONNECTOR, thus lowering the wheelchair so that its front wheels make contact with the pavement (Figure 35).
- At this point the upper pin will already be pulled out the front hook of the CONNECTOR; release the wheelchair's brakes and push yourself back to move away from the KLICK device (which in the meantime will remain upright thanks to its KICKSTAND - Figure 36).
- Remove the CONNECTOR, pulling the lever located on its bottom (Figure 20).

DANGER	During the KLICK release operation, pay attention to not endanger the top parts of your body (fingers, hands, arms) as they could be affected by the sudden movement of the handlebar (or other parts of the device) if it is released too quickly. To avoid this danger, after pressing the release button, gently release the handlebar by lowering the wheelchair gradually.
--------	---

## 16. KLICK MINI STANDARD / KLICK CARBON STANDARD VERSION OPERATION

### 16.1 USER INTERFACE

The device is equipped with a multifunction LCD display, used to switch the device on and off, select the maximum speed and display several operating parameters (Figure 59).

- The number (1) shows the current battery charge level.
- The number (2) in the centre of the display shows the speed, in km/h.
- The number (3) in the lower part of the display shows the distance travelled, in km.
- The number (4) shows the desired maximum speed.

### 16.2 TURNING ON AND SETTING UP THE DEVICE (FIGURE 59).

#### A. Turning On and Selecting the Speed

- Switch on the battery (for more information please refer to the instructions starting from section 18.3).
- Press and hold the "M" button for 5 seconds, until the display turns on.
- Press the "+" e "-" buttons to select the desired maximum speed level (shown by the indicator lights in the lower part of the display, number (4) in the legend); various speed levels can be selected, between 0 (no indicator light on, motor disabled), 1 (minimum speed, 5 km/h), 2 (normal speed, 7.5 km/h), 3 (medium speed, 9 km/h), 4 (high speed, 12 km/h) and 5 (maximum speed, 15 km/h).

#### B. Navigating the data

- After turning on the display, the initial screen shows (in number 3) the partial distance (TRIP) of the device in use.
- Press the "M" button once to enter the second screen. It will show the total distance (ODO).
- Press again the "M" button once to enter the third screen. It will display the maximum speed reached (MAX).
- Press again the "M" button once to enter the fourth screen. It will show the average speed (AVG).
- Press again the "M" button once to enter the fifth screen. It will show the approximate distance that can be covered based on the current battery level (RANGE); this estimate varies according to the maximum speed level selected.
- Press again twice the "M" button to display the total time the device has been on (TIME).

#### C. Resetting data

- Once the display has been turned on, press the "+" and "-" buttons together for 5 seconds to enter the MENU. Several options will be displayed.
- Use the "+" and "-" buttons to navigate within the MENU and select DISPLAY SETTING; press the "M" button to enter the selected submenu.
- Once in the submenu, use the "+" and "-" buttons to select TRIP RESET and press the "M" button to enter.
- Press the "+" button to move from NO to YES; press the "M" button to confirm the selected option.
- Press the "+" and "-" buttons together for 5 seconds to exit the MENU and go back to the main screen.

#### D. Screen Backlighting

- Hold the "+" button for 5 seconds to turn on the screen backlight.
- To turn off the backlight, press the "+" button for 5 seconds again.
- When the device is turned off, the light is also automatically switched off.

#### E. Speed Hold Function

- Press the "-" button for 5 seconds to turn on the speed hold function. This function accelerates the device up to a speed of 4.5 km/h and holds it as long as the button is pressed.
- When the button is released, the device instantly stops accelerating.

<b>DANGER</b>	Every action and/or tampering of the display to change performance and KLICK's security is strictly prohibited. The sole and only actions possible are those specified in paragraph 16.2 of this User's Manual.
---------------	---

NOTE: For further information on the display, see [www.klaxon-klick.com](http://www.klaxon-klick.com) or contact an authorised KLAXON dealer.

### 16.3 USE

Figure 61 shows the Standard handlebar of the Klick device and it identifies the elements that make it up (listed below).

1. Handlebar
2. Acceleration control
3. Display
4. Brake lever

Once the device has been installed, the display has been turned on and a speed has been selected, use the accelerator control to make the device start to move (Figure 31.1). To stop the device, use the brake lever. The KLICK brake is equipped with an additional safety device which deactivates the motor when the lever is operated (Figure 31.2).

DANGER	At each restart pay maximum attention, before accelerating, to check if forward or reverse operation is currently selected, especially when close to obstacles, drop-offs and any other kind of danger.
DANGER	If obstacles, drop-offs and dangers are too close to be safe, turn off the KLICK device and manually move the wheelchair to a safe place.
CAUTION	After inserting the battery in its housing and locked by turning the key, remove it and put it in a safe place. It is absolutely forbidden to use the KLICK device with the key insert in the security block.

## 16.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR

To perform the operation properly, carefully follow the following instructions.

- Stop the wheelchair on a flat surface.
- Apply the brake on the wheelchair.
- Turn off the display by pressing and holding the "M" button for 5 seconds.
- Push the handlebar forward with one hand grasping it in the middle and simultaneously press the release button on the CONNECTOR with the other hand. This frees the lower pin of the KLICK coupling (Figure 34).
- Reduce forward pressure on the handlebar while holding the release button on the CONNECTOR, thus lowering the wheelchair so that its front wheels make contact with the pavement (Figure 35).
- At this point the upper pin will already be pulled out the front hook of the CONNECTOR; release the wheelchair's brakes and push yourself back to move away from the KLICK device (which in the meantime will remain upright thanks to its KICKSTAND - Figure 36).
- Remove the CONNECTOR, pulling the lever located on its bottom (Figure 20).

DANGER	During the KLICK release operation, pay attention to not endanger the top parts of your body (fingers, hands, arms) as they could be affected by the sudden movement of the handlebar (or other parts of the device) if it is released too quickly. To avoid this danger, after pressing the release button, gently release the handlebar by lowering the wheelchair gradually.
--------	---

## 16.5 CLOSING / OPENING FOR TRANSPORTATION

To perform the operation properly, carefully follow the following instructions.

- Make sure that the device is off and the safety button on the battery disabled (Figure 46).
- Loosen the safety rings on the handlebar, turning them counter-clockwise with respect to the "ON" arrow (Figure 64).
- Move the safety rings outwards, unlocking the handlebar and making it collapse downward (Figures 65 and 66).
- Push up the safety button on the release lever and pull the latter outwards until it is completely disengaged (Figures 67 and 68).
- Fold the upper part of the handlebar to the side, rotating it 180 degrees; in closed position, it shall lie alongside the lower part of the steering tube, beside the battery holder (Figure 69).
- To open the Klick device, perform the above operations in reverse order.

CAUTION	During the closing phase of KLICK MS / CS, pay the utmost attention not to damage the (electric and mechanic) wires that run inside the frame of the device. Perform the opening and closing operations slowly, making sure that the wires do not interfere with moving parts, as these could pinch or break.
DANGER	If some elements of the electric or mechanic system get damaged during the opening and closing of the KLICK MS / CS devices, do NOT absolutely use the devices and take it immediately to an authorized KLAXON service centre.
DANGER	Once the KLICK MS / CS has been opened, before using it check that all joints are properly closed and that safety buttons (side lever and handlebar rings) are fully fastened. Only use the device when it is completely open (Figure 70).

## 17. CLEANING

Regularly cleaning the entire device makes it last longer and work better.

We recommend to:

- Before cleaning, switch the device off and disconnect the battery charger.
- Do not use abrasive cleaning agents, detergents and high pressure cleaners.
- Clean the device with a soft, damp cloth.
- Thoroughly dry the device after use in the rain.
- If the device is soiled, soften the dirt and remove it as soon as possible; then dry the device carefully.
- Clean all the outside of the device at least once a month and after each ride on muddy terrain.
- Be careful not to wet the electrical parts (battery, display, motor).
- Do not wash the parts under running water
- Dry the parts immediately after washing, using a soft cloth.

- If the electrical parts get wet, let them dry and do not use the device until they are completely dry.

## 18. SERVICING

KLICK devices must be serviced regularly, to ensure safe, reliable and efficient operation.

KLAXON is not responsible for any damage caused by non-maintenance or by insufficient maintenance, and in said cases the warranty may lapse. Unscheduled maintenance can be carried out only by technicians authorised by KLAXON's Technical Support Service.

### 18.1 PERIODICAL MAINTENANCE

We recommend the following regular maintenance:

- At least once a month, check the tyre inflation pressure; a flat tyre affects the vehicle's smooth ride, its safety and its power consumption.
- At least once a month, check wear of the tyres. To replace an inner tube or a tyre, contact an authorised KLAXON technician.
- At least once every two months, check the tightness of all screws and fasteners.
- At least once a month, check the tightness of the fixing nuts of the wheels' pins with a torque wrench calibration at 30Nm (Electric Standard, Electric Tetra, Power Standard, Power Tetra, Mini Standard, Carbon Standard versions).
- At least once a month, check the tightness of the fixing nuts of the wheels' pins with a torque wrench calibration at 50Nm (Race Standard, Race Tetra, Monster Standard versions).
- Before each use, check proper operation of the accelerator and brakes: any replacement of parts subject to wear (brake pads, discs, brake system in general) should be carried out only by authorised KLAXON technicians. The vehicle should also be checked whenever you feel the braking power has decreased.
- Check the condition of the battery, before each use. To replace the battery, and any other electrical component, contact only authorised KLAXON technicians.

	Have a KLAXON qualified technician overhaul the entire device at least once a year, and/or whenever a malfunction occurs.
---	---

### 18.2 MALFUNCTIONS

If you encounter a malfunction during use of the KLICK device, contact KLAXON or of its authorised resellers.

Malfunction found	Possible cause	Corrective action
The device does not turn on	The battery is empty	Recharge the battery
	The battery is positioned improperly	Check the position of the battery by removing it and inserting it again, making sure that the electrodes are inserted in the appropriate spaces.
	General problems	Contact an authorised KLAXON service centre
It vibrates excessively during use	Loose screws	Contact an authorised KLAXON service centre
The display lights up but the device does not move	Disconnected plug	Contact an authorised KLAXON service centre
You have problems coupling the traction unit to the connector	The connector is not adjusted properly	Contact an authorised KLAXON service centre
You have problems attaching the connector	There is dirt inside the coupling seat in the crossbeam	Clean the surfaces of the seat in the crossbeam and of the mounting system in the connector
You have problems attaching the connector	The interlocking balls do not retract when pulling the lever	Contact an authorised KLAXON service centre
Wheelchair front wheels touch the ground	The Klick coupling is mounted too low	Contact an authorised KLAXON service centre
KLICK is noisy when in use	The fender touches the wheel while moving	Make sure your feet are in a position that does not interfere with the fender of the device Reposition the fender, putting it back into place.
	The brake pads touch the disc when reversing	Contact an authorised KLAXON service centre

### 18.3 FLY SLIM BATTERY (2.5 AH)

The FLY SLIM battery (36 V and 2.5 Ah) is cylindrical and secured to the connection with an elastic rubber strap (Figure 44).

On the top of the battery, there are markings along which some LEDs show the current charge level: pressing the button, the LEDs will show the approximate charge level, in four levels (20%-50%-80%-100% - Figure 45).

Another button (red) turns the battery on/off, pressing this button will power the motor and the display (Figure 46).

The rubber strap, on the side of the battery, holds the battery on the device. Remove the strap to release the battery, pulling it forward and upward (Figures 47 and 48).

To charge the battery:

- Turn the device off (by pressing and holding the "M" button);
- Remove the battery by pulling out the rubber strap on the side (Figures 47 and 48).
- Remove the protection cap located on top of the battery (Figure 49).
- Connect the battery charger's plug (Figure 50);
- Connect the battery charger's plug to the 230V 50 Hz mains (Figure 51);
- The two LED lights on the upper surface of the battery charger (Figure 52) show its status:
  - Red-red: charging;
  - Green-red: charging complete, fully charged battery.
- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery.

## 18.4 FLY PLUS BATTERY (2.5 AH)

The FLY PLUS battery (36 V and 2.5 Ah) is box-shaped and secured to the connection with a latching system (Figure 53).

On the top of the battery, there are markings along which some LEDs show the current charge level. Pressing the button, the LEDs will show the approximate charge level, in four levels (20%-50%-80%-100%) and will automatically activate the battery, powering the motor and the display (Figure 54). Use the same button to disable the battery (therefore the LEDs turn off).

The key, placed beside the battery, can be used to remove the entire unit from the device; by turning it, the battery is released and it can be removed by pulling it forward and upward (Figures 55 and 56).

To charge the battery:

- Turn the device off (by pressing and holding the "M" button);
- Remove the battery by turning the key (Figures 55 and 56).
- Remove the protection cap located under the battery (Figure 57).
- Connect the battery charger's plug (Figure 58);
- Connect the battery charger's plug to the 230V 50 Hz mains (Figure 51);
- The two LED lights on the upper surface of the battery charger (Figure 52) show its status:
  - Red-red: charging;
  - Green-red: charging complete, fully charged battery.
- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery.

## 18.5 FLY MINI BATTERY (2.5 AH)

The FLY MINI battery (36 V and 2.5 Ah) is cylindrical and secured to the connection with a latching system (figure 74).

On the top of the battery, there are markings along which some LEDs show the current charge level. Pressing the button, the LEDs will show the approximate charge level, in four levels (20%-50%-80%-100% - figure 45).

Another button (red) enables to turn the battery on and off: press this button to power the motor and the display (figure 46).

The key, placed on top of the battery, can be used to fasten it to the device; by inserting it in the relevant seat and rotating it toward the right, it will activate the lock. At this point the battery is released and can be removed by pulling it forward and upward (figures 75 and 76).

To charge the battery:

- Turn the device off (by pressing and holding the "M" button);
- Remove the battery by turning the key (Figures 75 and 76).
- Remove the protection cap located on top of the battery (Figure 49).
- Connect the battery charger's plug (Figure 50);
- Connect the battery charger's plug to the 230V 50 Hz mains (Figure 51);
- The two LED lights on the upper surface of the battery charger (Figure 52) show its status:
  - Red-red: charging;
  - Green-red: charging complete, fully charged battery.
- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery.

## 18.6 SLIM BATTERY (5.8 AH)

The SLIM battery (36 V and 5.8 Ah) is cylindrical and secured to the connection with elastic rubber strap (Figure 44).

On the top of the battery, there are markings along which some LEDs show the current charge level: pressing the button, the LEDs will show the approximate charge level, in four levels (20% -50% -80% -100% - Figure 45).

Another button (red) turns the battery on/off, pressing this button will power the motor and the display (Figure 46).

The rubber strap, on the side of the battery, holds the battery on the device. Remove the strap to release the battery, pulling it forward and upward (Figures 47 and 48).

To charge the battery:

- Turn the device off (by pressing and holding the "M" button);
- Remove the battery by pulling out the rubber strap on the side (Figures 47 and 48).
- Remove the protection cap located on top of the battery (Figure 49).
- Connect the battery charger's plug (Figure 50);

- Connect the battery charger's plug to the 230V 50 Hz mains (Figure 51);
- The two LED lights on the upper surface of the battery charger (Figure 52) show its status:
  - Red-red: charging;
  - Green-red: charging complete, fully charged battery.
- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery.

## 18.7 MINI BATTERY (5.8 AH)

The MINI battery (36 V and 5.8 Ah) is cylindrical and secured to the connection with a latching system (Figure 74).

On the top of the battery, there are markings along which some LEDs show the current charge level. Pressing the button, the LEDs will show the approximate charge level, in four levels (20%-50%-80%-100% - Figure 45).

Another button (red) enables to turn the battery on and off: press this button to power the motor and the display (Figure 46).

The key, placed on top of the battery, can be used to fasten it to the device; by inserting it in the relevant seat and rotating it toward the right, it will trigger the lock. At this point the battery is released and can be removed by pulling it forward and upward (Figures 75 and 76).

To charge the battery:

- Turn the device off (by pressing and holding the "M" button);
- Remove the battery by turning the key (Figures 75 and 76).
- Remove the protection cap located on top of the battery (Figure 49).
- Connect the battery charger's plug (Figure 50);
- Connect the battery charger's plug to the 230V 50 Hz mains (Figure 51);
- The two LED lights on the upper surface of the battery charger (Figure 52) show its status:
  - Red-red: charging;
  - Green-red: charging complete, fully charged battery.
- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery

## 18.8 PLUS BATTERY (11 AH)

The PLUS battery (36 V and 11 Ah) is box-shaped and secured to the connection with a latching system (Figure 53).

On the top of the battery, there are markings along which some LEDs show the current charge level. Pressing the button, the LEDs will show the approximate charge level, in four levels (20%-50%-80%-100%) and will automatically activate the battery, powering the motor and the display (Figure 54). Use the same button to disable the battery (therefore the LEDs turn off).

The key, placed beside the battery, can be used to remove the entire unit from the device; by turning it, the battery is released and it can be removed by pulling it forward and upward (Figures 55 and 56).

To charge the battery:

- Turn the device off (by pressing and holding the "M" button);
- Remove the battery by turning the key (Figures 55 and 56).
- Remove the protection cap located on top of the battery (Figure 57).
- Connect the battery charger's plug (Figure 58);
- Connect the battery charger's plug to the 230V 50 Hz mains (Figure 51);
- The two LED lights on the upper surface of the battery charger (Figure 52) show its status:
  - Red-red: charging;
  - Green-red: charging complete, fully charged battery.
- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery.

## 18.9 RACE BATTERY (11 AH)

The RACE battery (48 V and 11 Ah) is box-shaped and secured to the connection with a latching system (Figure 77).

On the side of the battery, there are markings along which some LEDs show the current charge level. Pressing the button, the LEDs will show the approximate charge level, in four levels (20%-50%-80%-100% - Figure 78).

The key, placed on the right side of the battery, can be used to remove the entire unit from the device; by turning it 180° anticlockwise, the battery is released and it can be removed by pulling it upward (Figures 79 and 80).

The red power button is located on the left side of the battery next to the protective cap for the charger input; pressing it will activate the battery and power the motor and the display. With the same button the battery is deactivated.

To charge the battery:

- Turn the device off (by pressing and holding the "M" button);
- Remove the battery by turning the key (Figures 79 and 80).
- Remove the protection cap located on top of the battery (Figure 81).
- Connect the battery charger's plug (Figure 82);
- Connect the battery charger's plug to the 230V 50 Hz mains (Figure 51);
- The two LED lights on the upper surface of the battery charger (Figure 52) show its status:
  - Red-red: charging;
  - Green-red: charging complete, fully charged battery.

- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery.

## 18.10 FLY RACE BATTERY (5.8 AH)

The FLY RACE battery (48 V and 5.8 Ah) is box-shaped and secured to the connection with a latching system (Figure 77).

On the side of the battery, there are markings along which some LEDs show the current charge level. Pressing the button, the LEDs will show the approximate charge level, in four levels (20%-50%-80%-100% - Figure 78).

The key, placed on the right side of the battery, can be used to remove the entire unit from the device; by turning it 180° anticlockwise, the battery is released and it can be removed by pulling it upward (Figures 79 and 80).

The red power button is located on the left side of the battery next to the protective cap for the charger input; pressing it will activate the battery and power the motor and the display. With the same button the battery is deactivated.

To charge the battery:

- Turn the device off (by pressing and holding the “M” button);
- Remove the battery by turning the key (Figures 79 and 80).
- Remove the protection cap located on top of the battery (Figure 81).
- Connect the battery charger’s plug (Figure 82);
- Connect the battery charger’s plug to the 230V 50 Hz mains (Figure 51);
- The two LED lights on the upper surface of the battery charger (Figure 52) show its status:
  - Red-red: charging;
  - Green-red: charging complete, fully charged battery.
- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery.

### NOTES:

- The batteries supplied with the Klick device are compliant with the EMC2004/108/EC regulation issued by the European Union regarding the CE Classification. It is therefore permitted to transport these batteries on a train, ship or airplane. Despite of this classification they are considered, according to the UN Recommendations on the Transport of Dangerous Goods, as dangerous material and classified as Class 9, respectively as: code UN3480 (Lithium Ion Batteries) if transported as single units and UN3481 (Lithium Ion Batteries Contained in Equipment) if transported while connected to the Klick device. This classification determines some restrictions regarding the transportation of the product by ship (IMDG Code) and by plane (IATA DGR) referring to: quantity of batteries transported, type of transport (if transported as single unit or connected to the device) and the Watt-hour value (Wh) of each single battery. In this regard, if you are planning to take your device with you, please contact directly the shipping company or the airline chosen for your travel.

Below is a list of the available battery models supplied by KLAXON:

- FLY SLIM, FLY PLUS and FLY MINI batteries: 36V 2.5Ah (90 Wh)
- SLIM and MINI batteries: 36V 5.8Ah (208 Wh)
- PLUS battery: 36V 11Ah (396 Wh)
- RACE battery: 48V 11Ah (528 Wh)
- FLY RACE battery: 48V 5.8 Ah (280 Wh)
- We recommend paying attention to the charge level indicator! Always recharge when the level is low. We always advise you to recharge the battery after extended use or overnight. Lithium-ion batteries have no memory effect, so they must be charged without having to wait until they are completely discharged.
- Store the battery charger away from heat sources, in a dry place and protected from direct sunlight.

	When not using the battery, remember to charge it at least once a month.
	If you see that the battery is damaged, its housing is broken, if it swells or leaks, absolutely do not use it and contact KLAXON's Technical Support immediately.
	Only use the supplied battery charger together with the KLICK device: any damage or malfunction due to non-compliance with these instructions or use of products that do not comply with KLAXON's guidelines will not be covered by the warranty.
	There is a risk of injury due to short-circuits and risk of electric shock if the battery charger has been damaged. Avoid using the battery charger if it has been dropped or damaged.
	Avoid using extension cords unless it is absolutely necessary. If you must use them, make sure in advance that they are undamaged and in excellent condition, to prevent fire hazards and electrical shock.
	Never recharge the batteries and operate the device at the same time, due to risk of injury to yourself and to others.
	Once the battery is fully charged, the charger automatically cuts the power supply. Do not leave the battery charger connected to the 230 V mains for too long after it has finished its charge cycle (green LED on).

Always charge the battery in a well ventilated room and keep it away from flammable material.

## 19. TECHNICAL DATA

### 19.1 CLASSIFICATIONS OF THE DEVICE

According to the Medical Devices Directive	Risk class I
Protection against electrical hazards	Class II
Degree of protection against direct and indirect contact	Type B
Use in oxygen-rich environments	No protection
Operating conditions	Device for continuous operation

### 19.2 PERFORMANCE

#	Feature	Value	Unit	Notes
1a	Available speeds	3; 6; 9; 12; 15	km/h	ES, ET, PS, PT, RS, RT, FS versions
		5; 7.5; 9; 12; 15	km/h	MS, CS versions
2a	Range (PLUS and RACE 11 Ah batteries)	45-50	km	With charged battery, moderate speed, flat course and 75 kg user.
2b	Range (SLIM, MINI and FLY RACE 5.8 Ah batteries)	20-25	km	With charged battery, moderate speed, flat course and 75 kg user.
2c	Range (FLY SLIM, FLY PLUS and FLY MINI 2.5 Ah batteries)	10-15	km	With charged battery, moderate speed, flat course and 75 kg user.
3	Maximum negotiable slope	10	%	With charged battery and 75 kg user.
4a	Maximum negotiable obstacle height	50	mm	With charged battery and 75 kg user (ES, ET, PS, PT, RS, RT, FS versions).
4b	Maximum negotiable obstacle height	25	mm	With charged battery and 75 kg user (MS, CS versions).
5a	Motor power	370	W	ES, ET, PS, PT, MS, CS versions
5b	Motor voltage	36	V	
6a	Motor power	1000	W	RS, RT, FS versions
6b	Motor voltage	48	V	
7	Maximum weight of user	120	Kg	Consistently with the technical characteristics of the wheelchair to which it is attached.

### 19.3 SIZE INFORMATION

#### 19.3 A DIMENSIONAL DATA KLICK ELECTRIC STANDARD/KLICK POWER STANDARD/KLICK RACE STANDARD

#	Feature	Value	Unit	Notes
1	Length	40	cm	
2	Width	50	cm	
3	Height	90	cm	
4	Weight	8.0	kg	ES, PS versions (weight only of drive unit without battery)
		11.4	kg	RS version (weight only of drive unit without battery)
5a	Weight of PLUS battery (11 Ah)	2.5	kg	
5b	Weight of SLIM battery (5.8 Ah)	1.3	kg	

# KLAXON Mobility GmbH - ENG

5c	Weight of FLY SLIM battery (2.5 Ah)	0.7	kg	
5d	Weight of FLY PLUS battery (2.5 Ah)	0.7	kg	
5e	Weight of RACE battery (11 Ah)	3.0	kg	
5f	Weight of FLY RACE battery (5.8 Ah)	1.5	kg	

## 19.3 B DIMENSIONAL DATA KLIK ELECTRIC TETRA/KLIK POWER TETRA/KLIK RACE TETRA

#	Feature	Value	Unit	Notes
1	Length	40	cm	
2	Width	50	cm	
3	Height	95	cm	
4	Weight	8.6	kg	ET, PT versions (weight only of drive unit without battery)
		12.00	kg	RT version (weight only of drive unit without battery)
5a	Weight of PLUS battery (11 Ah)	2.5	kg	
5b	Weight of SLIM battery (5.8 Ah)	1.3	kg	
5c	Weight of FLY SLIM battery (2.5 Ah)	0.7	kg	
5d	Weight of FLY PLUS battery (2.5 Ah)	0.7	kg	
5e	Weight of RACE battery (11 Ah)	3.0	kg	
5f	Weight of FLY RACE battery (5.8 Ah)	1.5	kg	

## 19.3 C DIMENSIONAL DATA KLIK MONSTER STANDARD

#	Feature	Value	Unit	Notes
1	Length	50	cm	
2	Width	50	cm	
3	Height	90	cm	
4	Weight	15.0	kg	Weight only of drive unit without battery
5a	Weight of RACE battery (11 Ah)	3.0	kg	
5b	Weight of FLY RACE battery (5.8 Ah)	1.5	kg	

## 19.3 D DIMENSIONAL DATA KLIK MINI STANDARD/KLIK CARBON STANDARD

#	Feature	Value	Unit	Notes
1a	Length	25	cm	Open
1b	Width	45	cm	Open
1c	Height	90	cm	Open
2a	Length	25	cm	Closed
2b	Width	25	cm	Closed
2c	Height	65	cm	Closed
3	Weight	7.7	kg	MS (Weight only of drive unit without battery)
		7.9	kg	CS (Weight only of drive unit without battery)

# KLAXON Mobility GmbH - ENG

4a	Weight of MINI battery (5.8 Ah)	1.3	kg	
4b	Weight of FLY MINI battery (2.5 Ah)	0.7	kg	

## 19.4 ENVIRONMENTAL OPERATING CONDITIONS

#	Feature	Value	Unit	Notes
1	Maximum altitude	2000	m	
2	Atmospheric pressure	700 to 1060	hPa	
3	Temperature	0 to 40	°C	
4	Relative humidity	30 to 90	%	

## 19.5 TRANSPORTATION AND STORAGE ENVIRONMENTAL CONDITIONS

#	Feature	Value	Unit	Notes
1	Temperature	10 to 50	°C	
2	Relative humidity	30 to 90	%	

## 19.6 TECHNICAL DESCRIPTION OF BATTERY CHARGER

### 19.6 A TECHNICAL DESCRIPTION OF BATTERY CHARGER (FLY MINI, FLY SLIM, FLY PLUS, MINI, SLIM, PLUS BATTERIES)

#	Feature	Value	Unit	Notes
1	Input voltage	230	V	
2	Frequency	50	Hz	
3	Output Voltage	36	V	

### 19.6 B TECHNICAL DESCRIPTION OF BATTERY CHARGER (RACE, FLY RACE BATTERIES)

#	Feature	Value	Unit	Notes
1	Input voltage	230	V	
2	Frequency	50	Hz	
3	Output Voltage	48	V	

## 19.7 TECHNICAL DESCRIPTION OF BATTERIES

### A. PLUS Battery (11 Ah)

#	Feature	Value	Unit	Notes
1	Rated voltage	36	V	
2	Capacity	11	Ah	
3	Charging time	4	h	Time to charge a completely discharged battery

### B. SLIM AND MINI BATTERIES (5.8 Ah)

#	Feature	Value	Unit	Notes
1	Rated voltage	36	V	With 100% charge, nominal 36 V voltage
2	Capacity	5.8	Ah	
3	Charging time	2.5	h	Time to charge a completely discharged battery

# KLAXON Mobility GmbH - ENG

## C. FLY SLIM, FLY MINI AND FLY PLUS BATTERIES (2.5 Ah)

#	Feature	Value	Unit	Notes
1	Rated voltage	36	V	
2	Capacity	2.5	Ah	
3	Charging time	2.5	h	Time to charge a completely discharged battery

## D. RACE BATTERY (11 Ah)

#	Feature	Value	Unit	Notes
1	Rated voltage	48	V	
2	Capacity	11	Ah	
3	Charging time	5.5	h	Time to charge a completely discharged battery

## E. FLY RACE BATTERY (5,8 Ah)

#	Feature	Value	Unit	Notes
1	Rated voltage	48	V	
2	Capacity	5.8	Ah	
3	Charging time	3.5	h	Time to charge a completely discharged battery

## 19.8 MAIN FEATURES

#	Feature	Value
1	Traction type	Electric
2	Brake	Disc (diameter 160 mm), mechanical
3	Type of use	Outdoor and indoor

## 20. DISPOSAL OF THE DEVICE

Disposal of the equipment inside the KLICK devices must absolutely comply with regulations of the country in which the devices are used. KLAXON is not liable for use that is not in accordance with specific regulations in force.

## 21. PRODUCT WARRANTY

The warranty period is 24 months for the traction device and 12 months for the batteries, starting from the delivery date of the device.

Natural wear and tear of parts is not covered by the warranty, except for improper wear due to a manufacturing defect.

If a defect occurs during the warranty period, KLAXON will, at its discretion, repair or replace the defective component.

The manufacturer assumes no liability for damages caused by negligence, careless use and maintenance, tampering or incorrect maintenance or by unauthorised persons, or for failure to follow the instructions for use set out in this Manual.

## 22. ORIGINAL TEXT

This document is a translation of the original version written in Italian. Please note that in case of discrepancies between the two versions, the Italian version will be the one considered as official and valid.

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**KLAXON MOBILITY GmbH**