User manual

Edition 8, July 2021

All-round (Light) Joystick

All-round Joystick Omni (P002-71) - All-round Joystick R-net (P002-75) - All-round Joystick Omni Light (P002-72) - All-round Joystick R-net Light (P002-76)

mo-vis
moving forward together

Contents

Other languages	8
About this manual	9
User manual	9
mo-vis bv	9
User manual (English)	10
Warning labels	10
Support, scrapping & recycling	11
Limited liability	12
Intended use	12
Features	13
Operation	15
Parts and accessories	15
Qualifications	20
First time use	20
Cleaning	21
Warnings and limitations	22
Connection options	22
Combinations	24
Dimensions	24

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EN: You can find this manual in English on our website.

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About this manual

User manual

Thank you for choosing a mo-vis product!

This manual contains **useful and important information** about your device.

Please read it carefully before use and store safely for future reference.

Our team (or your authorized dealer) will be happy to answer your questions.

mo-vis bv

Biebuyckstraat 15D . 9850 Deinze . Belgium

<http://www.mo-vis.com> . contact@mo-vis.com . +32 9 335 28 60

User manual (English)

Warning labels

Please read this manual, the safety instructions and warning texts carefully, in order to reduce the risks associated to the device. Our products are safe under normal and reasonably foreseeable operating conditions.

NOTE: This symbol indicates general notes and information.

CAUTION: This symbol indicates caution for a hazardous situation that, if not avoided, could result in minor or moderate injury.

WARNING: This symbol indicates a warning for a hazardous situation that, if not avoided, could result in death or serious injury.

Support, scrapping & recycling

Technical support

TROUBLE: In case of technical problems:

- Contact your dealer
- If your dealer is not available or unknown, please contact mo-vis: contact@mo-vis.com or +32 9 335 28 60.

Always state the product code and device serial number when contacting us. This ensures you are provided with the correct information.

Spare parts and accessories

Contact mo-vis or your dealer for more information about spare parts and accessories.

Scrapping & recycling

CAUTION: For scrapping, adhere to your local waste legislation. Dispose of obsolete electronic parts responsibly in accordance with local recycling regulations.

Limited liability

*mo-vis accepts **no liability** for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendation, warnings and instructions in this manual.*

CAUTION: This product should only be installed by a qualified service engineer.

NOTE: All mo-vis manuals can be found at <http://www.mo-vis.com> where they can be consulted in PDF format.

NOTE: In case any serious incident occurs in relation to this device, this should be reported immediately to mo-vis and the competent authority of the Member State in which the user is established.

Intended use

The All-round Joystick:

- is a joystick module which can be connected directly to the wheelchair electronics to control the wheelchair functions.

- is the compact version of a standard electronic wheelchair joystick module.
- is developed for all-round use.
- is suitable for the major part of wheelchair users.
- can be used as a standard joystick, chin joystick or as an attendant joystick.

The All-round Joystick Light has the same specifications, but with medium force (see).

Features

The All-round Joystick and All-round Joystick Light:

- is fully proportional.

	THROW	FORCE
All-round Joystick	20 mm - 0.8 in	250 gr - 8.82 oz
All-round Joystick Light	7 mm - 0.3 in	120 gr - 4.23 oz

- built in a compact housing, with two 3.5 mm jacks for switch input: (1) red = on/off power wheelchair - (2) yellow = mode of the power wheelchair.

- is fully adjustable to individual possibilities and needs with well-thought-out electronics (mini USB connection).
- is compatible with multiple types of wheelchair electronics.
- equipped with a LED light in the housing which will start to flash in case of error.
- has the possibility to insert 1 or 2 Satellite Twisters via integrated insert parts.

Operation

The movements of the joystick are translated into according movements of the wheelchair, e.g. driving or menu navigation.

Common practice to navigate the wheelchair with the joystick is as follows:

- **Direction:** point the joystick into the direction you want the wheelchair to move. The wheelchair then moves in that direction.
- **Speed:** the further you move the joystick from the default (center) position, the faster the wheelchair moves.
- **Stop:** whenever you release the joystick, the joystick moves back to the default (center) position and the wheelchair stops.

CAUTION: Avoid hitting obstacles during driving.

WARNING: When the LED light flashes and/or after every incident with the wheelchair or the mo-vis device, contact your dealer immediately to perform a functional test.

Parts and accessories

Package

The All-round Joystick (Light) package consists of:

EXAMPLE	P002-71 ALL-ROUND JOYSTICK OMNI AND P002-72 ALL-ROUND JOYSTICK OMNI LIGHT	P002-75 ALL-ROUND JOYSTICK R-NET AND P002-76 ALL-ROUND JOYSTICK R-NET LIGHT
	All-round Joystick (Light) set, with integrated SUB D9 cable	All-round Joystick (Light) set, with integrated R-net cable

D-P002-71-70-MX All-round Joystick User manual

D-P002-71-70-M1 All-round Joystick Installation manual

Accessories

For mounting and further personalization, the following accessories are available:

EXAMPLE	PRODUCT DESCRIPTION
	M002-60 All-round Joystick mounting set 8 mm

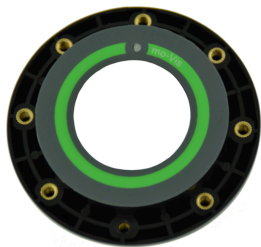
EXAMPLE**PRODUCT DESCRIPTION**

M002-61 All-round Joystick mounting set 6 mm

M002-76 All-round Joystick ball (6.35 mm)

EXAMPLE**PRODUCT DESCRIPTION**

M002-25 All-round Joystick table
mounting set

EXAMPLE**PRODUCT DESCRIPTION**

P005-40 Satellite Twister (incl. All-round Joystick housing satellite insert), short cable (300 mm), black
Available in other colors (40 = black, 41 = red, 42 = yellow, 43 = blue, 44 = green)

Unit detail

- The connections of the unit have protective covers to avoid the intrusion of dust or moist during usage.
- When in use, a green LED on the joystick indicates the proper functioning of the unit. In case of an error, the LED light will flash. See

the *Installation Manual* for more information.

Spare parts

Spare parts and accessories must be ordered by the dealer at mo-vis bv.

Always mention the product code of the desired item.

Qualifications

CAUTION: Only a qualified service engineer may install the device.

CAUTION: An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

First time use

CAUTION: Only a qualified service engineer may install the device.

CAUTION: An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

CAUTION: A qualified service engineer should assist during first time use of the device and make sure that all functions are clear to the user, including what to do in case of problems or doubt.

Cleaning

WARNING: As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

- Gently remove dust and dirt with a damp cloth.
- Use only non-aggressive disinfectant cleaning agents.

CAUTION: Do not immerse in water or use excessive amounts of liquid.

CAUTION: Do not apply additional lubricants to the moving parts.

The handles of the joystick can be removed to clean them.

- You may shortly immerse the handle in warm water with a gentle cleaning agent. Let the handle dry before putting it back on.
- If the handle is damaged, worn out or remains dirty, please contact your authorized dealer or mo-vis to replace it.

Warnings and limitations

NOTE: This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

WARNING: Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

Connection options

You can connect 1 or 2 switches to the joystick:

- **Red** = power on/off
- **Yellow** = mode

WARNING: The on/off switch must be available to the user at all times. This allows to instantly stop the wheelchair in case of problems or an emergency.

WARNING: Always power off the device when the user is no longer in the wheelchair to avoid unauthorized access or accidental movements.

Combinations

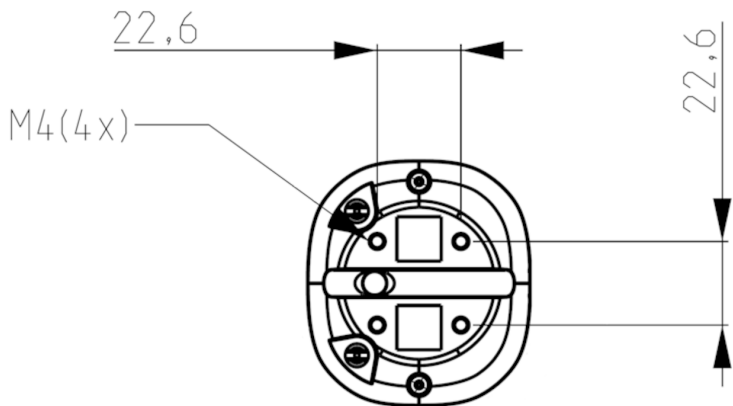
There are different versions of this device available for combination with different wheelchair electronics. Please refer to the *Installation Manual* of this device for more information.

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