User manual

Edition 1, August 2021

HID Joystick

Micro Joystick HID (P025-61) - Multi Joystick HID (P025-62)

mo-vis
moving forward together

Contents

About this manual	4
User manual	4
mo-vis bv	4
User manual (English)	5
Support, scrapping & recycling	5
Warranty	6
Warning labels	7
Quickstart guide	8
Intended use	10
Safety instructions	11
Features	13
Installation	30
Configuration	36
Supported platforms	55
First time use	68
Maintenance	69
Error codes	71
Dimensions	72
Technical data	78
Benutzerhandbuch (Deutsch)	80
Support, Entsorgung und Recycling	80

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About this manual

User manual

Thank you for choosing a mo-vis product!

This manual contains **useful and important information** about your device.

Please read it carefully before use and store safely for future reference.

Our team (or your authorized dealer) will be happy to answer your questions.

mo-vis bv

Biebuyckstraat 15D . 9850 Deinze . Belgium

<http://www.mo-vis.com> . contact@mo-vis.com . +32 9 335 28 60

User manual (English)

Support, scrapping & recycling

Technical support

TROUBLE: In case of technical problems:

- Contact your dealer
- If your dealer is not available or unknown, please contact mo-vis: contact@mo-vis.com or +32 9 335 28 60.

Always state the product code and device serial number when contacting us. This ensures you are provided with the correct information.

Spare parts and accessories

Contact mo-vis or your dealer for more information about spare parts and accessories.

Scrapping & recycling

CAUTION: For scrapping, adhere to your local waste legislation. Dispose of obsolete electronic parts responsibly in accordance with local recycling regulations.

Warranty

mo-vis bv warrants the product to be free from defects in material and workmanship for a period of 2 years under proper use, care and service. The dealer should never keep mo-vis products in stock for a period more than 6 months prior to delivery to the end-user. mo-vis' warranty will never exceed a period of 2 years and 6 months after shipment.

All warranties do not extend beyond the initial purchaser from an authorized mo-vis dealer or mo-vis itself.

Repair and replacement

For warranty service, contact your dealer (or us if bought directly). In the event of a defect in material or workmanship, the dealer or customer must obtain a Return Merchandise Authorization (RMA) number from us. The product must be shipped to a service centre designated by mo-vis. mo-vis will repair or, at mo-vis' option, replace any product covered by the warranty.

Amendments

No person is authorized to alter, extend or waive the warranties of mo-vis.

Disclaimer and limitations of remedies

The express warranties set forth in this agreement are in lieu of all other warranties of merchantability or fitness of purpose. In no event shall mo-vis be liable for any direct, indirect, incidental or consequential damages resulting from any defect in this product.

Warranty of parts subject to “normal wear and tear” (e.g. joystick handles, pads, ...) are not covered in the warranty except as it applies to defects in material or construction.

Voiding of warranties

The foregoing warranties are contingent upon the proper installation, use, maintenance and care of the product. The warranty will be void if the product has been installed or used improperly, or if it has been repaired or any part replaced by persons other than mo-vis or an authorized dealer. This product is considered as a non-serviceable part.

The addition of equipment or features that are not manufactured or recommended by mo-vis could affect the intended function of the mo-vis product and may invalidate the warranty.

Warning labels

Please read this manual, the safety instructions and warning texts carefully, in order to reduce the risks associated to the device. Our products are safe under normal and reasonably foreseeable operating conditions.

NOTE: This symbol indicates general notes and information.

CAUTION: This symbol indicates caution for a hazardous situation that, if not avoided, could result in minor or moderate injury.

WARNING: This symbol indicates a warning for a hazardous situation that, if not avoided, could result in death or serious injury.

Quickstart guide

You can find the full manual of the HID Joystick on our website <http://www.mo-vis.com>.

- 1 Connect the HID Joystick to a PC with Windows 10 to configure the device.
- 2 Download the latest version of the Configurator Software from our website <http://www.mo-vis.com>.
- 3 Open the Configurator in **Administrator Mode**.
- 4 To log in, fill in your email address and the password that you have obtained from mo-vis.
- 5 Push and hold the mode button on the HID Joystick Interface for 5 seconds to enter configuration mode (3 LED lights should flash).

- 6 A COM port is created in the Configurator. Click **Connect > Parameters**.

 **NOTE:** It can take a few minutes to load all the parameters.

- 7 Think about your set-up (modes, buttons etc...) and configure your device. For the keyboard strings, there is a Keyhelp Tool available on our website <http://www.mo-vis.com>.
- 8 Save the parameters to your device.

9 If you want to use the HID Joystick...

- on your PC: push the mode button on the HID Interface shortly to enter the mouse/keyboard/joystick mode.
- on another device, disconnect the HID Joystick from the PC and plug it into another supported device. Your device will start up in the mouse/keyboard/joystick mode.

Your HID Joystick is now ready to be used.

What to do next:

- Please find the full manual of the HID Joystick on our website.
- For more information about the Configurator Software, please find the *Configurator Software User manual* on our website too.

Intended use

The HID Joystick:

- is a low force joystick.
- enables people with (very) limited muscular force and/or movements to operate devices such as computers and game controllers.
- can be operated by hand, fingers, chin and its surrounding areas, ...
- is recommended for people with tetraplegia, (neuro)muscular diseases or other advanced diseases such as amyotrophic lateral sclerosis (ALS) or multiple sclerosis (MS).

The HID Joystick is a composite Human Interface Device, which means that this device contains three HID classes: the HID mouse, HID keyboard and HID joystick. The HID Joystick connects via USB to a PC, XBOX Game Console, Android or any other USB host device. The HID Joystick is also compatible with Bluetooth adapters.

The Configurator Software is a Windows desktop application, which can be used to program the HID Joystick to best suit the user's preferences and needs. It is available to download from <http://www.mo-vis.com>.

CAUTION: The joystick operates with high sensitivity. A good movement coordination and force control is required.

Safety instructions

Limited liability

mo-vis accepts **no liability** for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendation, warnings and instructions in this manual.

NOTE: All mo-vis manuals can be found at <http://www.mo-vis.com> where they can be consulted in PDF format.

NOTE: In case any serious incident occurs in relation to this device, this should be reported immediately to mo-vis and the competent authority of the Member State in which the user is established.

Warnings and limitations

NOTE: The HID Joystick is only intended for entertainment goals and non-critical device commands.

CAUTION: Long periods of repetitive motion using the product may be associated with nerve, tendon or muscle injury in your hands, wrists, arms, shoulders, neck or back. See a qualified health professional for pain, numbness, weakness, swelling, burning, cramping or stiffness.

NOTE: Take regular breaks when you are working with computers. Be aware of physical signs of tiredness.

CAUTION: The HID Joystick, integrated cabling and interface included, is considered as a non-serviceable part. In case of doubt about alterations or adjustments, always contact mo-vis. Do not open or alter the device yourself.

NOTE: This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in

the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

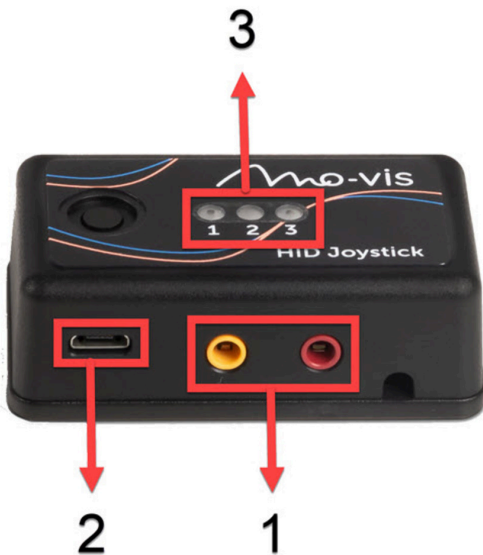
CAUTION: This package may include small parts, which can be a choking hazard to children under 3 years. Keep small parts away from children.

Features

The HID Joystick is available in 2 versions:

- **Micro Joystick HID**
 - is a very sensitive proportional joystick due to its low operating force (ca. 8 gr - 0.28 oz) and movement (3.3 mm - 0.13 in).
 - has an ergonomic shape for easy handling. For instance: you can let it rest in the palm of your hand and operate it with your thumb.
 - is supplied with two tips: small ball and a cup. Use the type which offers the best comfort and use efficiency.
- **Multi Joystick HID**
 - is a sensitive proportional joystick with reduced throw (8 mm - 0.31 in) and force (ca. 50 gr - 1.76 oz).
 - is built in a compact and discrete housing with 2 3.5 mm jacks for switch input
 - is supplied with a ball as tip.

Both joysticks have a separate interface with:

- 1 two stereo input jacks for switches. By using 2 splitter cables, you can link up to 4 mechanical switches.

- 2 a micro USB connection. Via this connection, you can link the HID Joystick with the Configurator Software to configure the joystick and buttons with mouse, keyboard and joystick functions. You also need it to link to your PC, tablet, XBOX etc...
- 3 LED indicators, which will show you the status of the device (incl. errors).

Parts and accessories

Micro Joystick HID

A Micro Joystick HID package consists of the following parts:

PICTURE	DESCRIPTION
	P002-50 - Micro Joystick sensor

PICTURE	DESCRIPTION
	M006-08 - Micro Joystick ball

PICTURE**DESCRIPTION**

PICTURE**DESCRIPTION**

M006-09 - Micro Joystick cup

PICTURE	DESCRIPTION
 The image shows four black plastic components of a micro joystick mounting set. At the top left is a circular ring with a threaded hole and a small metal pin. To its right is a small rectangular piece with a circular hole. Below these are two long, thin cylindrical rods of different lengths.	M006-20 - Micro Joystick mounting set
	M004-91 Q2M Rod 50 mm Threaded UNC 1/4x20

PICTURE	DESCRIPTION
	
	<p>P025-51 HID Joystick Interface</p>
	<p>PM CAB00031 USB connection lead A to B - micro 1m</p>

Multi Joystick HID

A Multi Joystick HID package consists of the following parts:

PICTURE	DESCRIPTION
	P002-60 - Multi Joystick Sensor

PICTURE	DESCRIPTION
	
	M002-05 - Multi Joystick ball
	M002-18 - Multi Joystick Mounting set

PICTURE	DESCRIPTION
	
	<p>M004-91 Q2M Rod 50 mm Threaded UNC 1/4x20</p>
	<p>P025-51 HID Joystick Interface</p>

PICTURE	DESCRIPTION
	
	PM CAB00031 USB connection lead A to B - micro 1m

Spare parts

Spare parts and accessories must be ordered by the dealer at mo-vis bv.
Always mention the product code of the desired item.

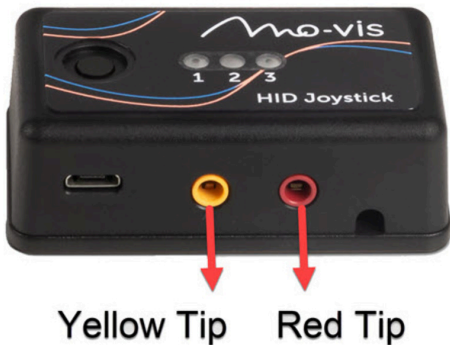
Operation

Joystick

The joystick is proportional, which means that a deflection of the joystick is directly related to a movement: the further you move the joystick away from the center position, the faster the action will be.

Buttons

If you use 1 or 2 buttons:

- You can use a button of the mo-vis Twister series, but any other button with 3.5 mm mono jack connection should be compatible too.
- You can also use a 3.5 mm stereo jack connection, but then you will have to exclude the yellow or red 'ring' in the Configurator Software.

If you want to use 3 or 4 buttons, you will need 1 or 2 splitter cables. You can then use a button of the mo-vis Twister series, or any other button with 3.5 mm mono jack connection.

Yellow Tip **Yellow Ring**

With a Multi Joystick HID it is possible to insert the jacks in the housing of the joystick (e.g. useful when this joystick is operated with the chin and buttons are placed nearby).

NOTE: The jacks in the housing are a duplication of the jacks in the interface, not additional jacks.

Mode button and LED lights

There is a mode button on the interface box to toggle between working mode and configuration mode.

- Press < 5 s: change configuration set
- Press > 5s, < 10s: change to configuration mode
- Press > 10s: software reset

DESCRIPTION	LED 1	LED 2	LED 3
Configuration set 1 is active	ON	OFF	OFF
Configuration set 2 is active	OFF	ON	OFF
Configuration set 3 is active	OFF	OFF	ON
Bootloader active	ON	ON	ON
Configuration mode	CONTINUOUS FLASHING		
Error	OFF	FLASHING	OFF

 **NOTE:** Flash count can be linked to the error codes

Installation

Preparations

CAUTION: Before you start with the installation:

- Please check the packaging and verify that all items are included.
- Make sure that you have all the necessary documentation and knowledge to install this device.

- Check the condition of the device.

Tools

Use an Allen wrench to install the device.

CAUTION: Use proper tools to install and adjust the device. The use of improper tools may cause damage to the device.

Installation plan

Set up an installation plan before beginning the installation. Based on the users' needs and capabilities, this plan should take into account:

- Where which part of the device should be placed.
- How the device will be operated.
- A robust and reliable positioning. Hard or sudden movements of the wheelchair may not disorganize the installation.

WARNING: Protect the device against bumps. Mind damaging the unit and wiring. Make sure that cabling is mounted in such a way that excessive wear and tear is avoided.

 **CAUTION:** Before inserting a connector, remove the protective cover. If the connections are not used, always put or keep the protective covers in.

 **CAUTION:** This is Class I Medical Device (*MDR 2017/745*). All accessories, including switches, must also comply with the *MDR 2017/745* regulations (e.g. the mo-vis *Twister*, tested according to *EN12184* standards).

 **CAUTION:** The HID Joystick is intended for indoor use. Protect the interface box against water.

 **CAUTION:** Incorrect use or installation may lead to risk of injury to the user or property.

Micro Joystick HID installation

- 1 Define the position of the joystick.

There is an arrow on the bottom of the Micro Joystick to indicate the default 'north' direction. This can be adjusted, however, with the Configurator Software.

- 2 Place the mounting ring around the joystick housing.
- 3 Secure the Q2M half clamp to the mounting ring.

Use an Allen wrench to mount the 20 mm bolt.

NOTE: It is not advised, but you may mount the joystick without using the Q2M half clamp.

CAUTION: Tighten the screws firmly, but not excessively. Excessive force may damage the unit.

- 4 Slide the unit over a Q2M rod or the ¼"Q2M Rod 50 mm Threaded UNC 1/4x20, or any other 6 mm rod, to position the Micro HID Joystick:
- 5 Place and secure the interface unit and all cabling on the desired location.
- 6 If required, place up to four switches (via splitter cables), secure their cabling and insert their connections.

Multi Joystick HID installation

- 1 Define the position of the joystick.

When you hold the Multi Joystick in front of you, with the jack connectors on the right-hand side as seen from your point of view, a forward movement of the joystick is the default 'north' direction. This can be adjusted, however, with the Configurator Software.

- 2 Determine which of the 3 slots you will use to guide the cable.
- 3 Secure the mounting plate with the 4 screws to the base of the joystick housing.

- 4 Place the Q2M half clamp at the bottom of the unit. Use an Allen wrench to mount the 14 mm bolt.

NOTE: It is not advised, but you may mount the joystick without using the Q2M half clamp.

- 5 Slide the unit over a Q2M rod or the ¼" Q2M Rod 50 mm Threaded UNC or any other 6 mm rod, to position the Multi Joystick HID.
- 6 Place and secure the interface unit and all cabling on the desired location.
- 7 If required, place up to four switches (via splitter cables), secure their cabling and insert their connections.

Configuration

Software download

- You can download the Configurator Software on our website <http://www.mo-vis.com>
- Software requirement: Windows version 10
- For all details on how to install and use the software, we refer you to the *Configurator Software manual*.
- To configure the parameters of the device, you need dealer level access. This level is password-protected. Contact mo-vis to obtain the password.

Keyhelp Tool

There is also a Keyhelp Tool available on our website. Download and install the tool to assist with generating keyboard strings.

Defining parameter settings

- 1 Connect the HID Joystick unit to a PC. Use a standard micro-USB - USB A cable.
- 2 Open the Configurator in the **Administrator mode**. This will shorten the time it takes to load all the parameters (30 sec - 2 min).
- 3 Press and hold the mode button on the interface for 5 seconds to enter the configuration mode of the HID Joystick. Wait until all parameters are loaded.

NOTE: The HID Joystick itself cannot be used to program the device.

- 4 Configure the parameters with the software or upload a pre-configured set of parameters into the device.
- 5 Save the configuration to the device.
- 6 Test the configuration and adjust if necessary.

Parameter settings

NOTE: For a better understanding, the modes are explained separately below. However, one configuration set can include actions from different modes.

Mouse mode

Parameters such as **Move dpi**, **Scroll dpi**, **Repeat time** and **Acceleration** are relevant for this mode. See [Global parameters on page 41](#) .

Keyboard mode

A large number of keyboard strings can be configured. Both a joystick and a button action can send out a keyboard string.

A keyboard string is valid when it contains 2 hexes, separated by a space. The first hex is the modifier, the second is the key code. When the modifier is 0x00, it will not modify the key.

NOTE: The keys are based on a QWERTY keyboard. If you use another keyboard layout, you will have to insert the equivalent key of that physical location.

Example:

- 0x00 (no modification) 0x05 (key b and B) = b
- 0x04 (left alt) 0x05 (key b and B) = B

List of modifiers:

HEX	DESCRIPTION
0x00	No modification
0x01	Left control
0x02	Left shift
0x03	Left control AND left shift
0x04	Left alt
0x05	Left control AND left ALT
0x08	Left WIN modifier
0x10	Right control
0x20	Right shift
0x40	Right alt
0x80	Right Win modifier

List of keys:

HEX	DESCRIPTION
0x04	Keyboard a and A
0x28	ENTER / RETURN
0x29	ESCAPE
0x2a	DELETE / Backspace
0x2b	TAB
0x2c	Spacebar
...	...

For easier use, you can download the Keyhelp Tool from our website or find a full list of keys and modifiers: <http://www.mo-vis.com>.

If you want to have a certain keyboard string repeated, choose **Keyboard String & Hold** as joystick nudge action or any button action.

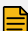
NOTE: This works like a standard keyboard. This means it is important to remember which key is pressed, as it will remain active until you switch off the **Keyboard String & Hold** (perform the same action).

Joystick mode

No additional information.

Global parameters

Global parameters

SETTING	DESCRIPTION	PARAMETERS	
Move speed	Sets the mouse movement speeds (dots per inch): the higher this value is, the faster the cursor will move.  NOTE: This is only relevant when a mouse movement action is configured.	Default	10
		Min.	1
		Max.	127
		Step	1
Scroll speed	When moving a scroll wheel on a standard mouse, you can feel a little click. The value of this parameter, represents one such click. The higher this value, the faster you will scroll horizontally or vertically.  NOTE: For a controlled way of scrolling (e.g. inventory of a game or scrolling slowly through a document), do not place it under a direct action, but rather under a joystick	Default	10
		Min.	1
		Max.	127
		Step	1

SETTING	DESCRIPTION	PARAMETERS	
	nudge or short/medium/long button press.		
Scroll repeat time	<p>This parameter is only relevant when a mouse <u>s</u>croll action is configured. Scrolling a mouse wheel on a standard mouse is a single triggered event. If the scroll action is mapped to the direct joystick action, then it becomes a repeated event. This value indicates the repeat time of the event (scroll).</p> <p> NOTE: This is only relevant when a mouse scroll action is configured.</p>	Default	50 mS
		Min.	50 mS
		Max.	10000 mS
		Step	50 mS
Acceleration	<p>This parameter is relevant when a mouse action is configured as direct AND nudge action of the joystick. The direct action will be blocked until the nudge timer has expired. After the expiration of the timer, the mouse will move suddenly. To give the user more control over this mouse movement, the speed of the</p>	Default	50%
		Min.	10%
		Max.	100%
		Step	10%

SETTING	DESCRIPTION	PARAMETERS	
	movement accelerates from zero to the max move speed . How fast this acceleration is, is determined by this parameter.		

Auditive feedback

SETTING	DESCRIPTION	PARAMETERS	
Medium timer	Beep when short timer of button action is expired	Default	Short beep
		Other	None
Long timer	Beep when medium timer of button action is expired	Default	Long beep
		Other	None
Config	Beep when changing to	Default	Number

SETTING	DESCRIPTION	PARAMETERS	
set	another config set		of beeps
		Other	None
Volume	Volume of the auditive feedback	Default	100%
		Min.	0%
		Max.	100%
		Step	1%
		 NOTE: This parameter is only relevant when the auditive feedback parameters have a beep configured.	

Buttons

NOTE: Although the maximum number of buttons is 4, it is possible to program up to 12 actions with the buttons. The buttons can either perform a **direct action** or you can distinguish between **short / medium / long press**.

SETTING	DESCRIPTION	PARAMETERS	
Short timer	Only relevant if you choose a short/medium/long press over direct action.	Default	1000 mS
		Min.	0 mS

SETTING	DESCRIPTION	PARAMETERS	
Long timer	If the button is pressed for a time smaller than the short timer, the short press action will be executed. If the button is pressed for a time longer than the short timer, but shorter than the long timer, the medium press action will be executed. If the button is pressed for a time longer than the long timer, the long press action will be executed.	Max.	10000 mS
		Step	100 mS
		Default	2000 mS
		Min.	0 mS
		Max.	10000 mS
		Step	100 mS
Debounce timer	The button has to be pressed for longer than the debounce timer, before it is seen as an input. It can be useful to increase this timer in case the user has a tremor, for instance.	Min.	20 mS
		Max.	100 mS
		Step	10 mS

Input joystick


NOTE: The joystick can distinguish between a direct action and a nudge.

SETTING	DESCRIPTION	PARAMETERS	
Threshold	The threshold is used when the joystick is configured as a switch. When the joystick is deflected further than the threshold, the system will consider this to be a key press. When the joystick is below the threshold again, the system will consider this to be a key release.	Default	50
		Min.	30
		Max.	70
		Step	1
Nudge timer	The nudge timer defines the time that will be used to detect a joystick nudge. The nudge timer starts when the joystick is deflected further than the threshold and stops when it is	Default	1000
		Min.	0
		Max.	50000

SETTING	DESCRIPTION	PARAMETERS	
	<p>below the threshold again.</p> <p> NOTE: When the nudge timer value is zero, no nudge action is possible.</p> <p> NOTE: Output of the direct action will be blocked for as long as this timer is active.</p>	Step	100
Deadband	You can adjust the deadband, or the neutral zone around the center of the joystick where movement is not seen as an input.	Default	10%
		Min.	0
		Max.	15
		Step	1

Miscellaneous

SETTING	DESCRIPTION	PARAMETERS	
Input filter	If you set this to Block diagonal axes , only movements north, south, east and west are recognised. This	Default	None
		Other	Block diagonal

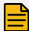SETTING	DESCRIPTION	PARAMETERS	
	can be useful for people who cannot operate the joystick very precisely.		axes
8 keyboard axes	By default, you can't combine keyboard axes. E.g. if you want to use the arrow keys UP and then LEFT, you have to go back to the center every time. However, you can activate 8 keyboard axes. This way, you can activate the UP and LEFT key at the same time.	Default	Off
		Other	On
<div style="border: 1px solid #ccc; border-radius: 10px; padding: 10px; background-color: #f0f0f0;">  NOTE: Make sure that the nudge actions in the active configuration set are all set to None and that all direct actions are set to Keyboard string. </div>			

Configuration sets

It is possible to configure 3 different configuration sets in the HID Joystick. The user can switch between the different configuration sets easily, by mapping this function to an action of the joystick or a button. The configuration sets are indicated by the LED lights on the interface (1 green LED = config set 1, ...) and optional auditive feedback.

NOTE: If the user is executing an action (e.g. left mouse click) and cycles to another configuration set at that moment, the action will be terminated.

Joystick parameters

To each direction of the joystick, different actions can be allocated. Either under a nudge or a full movement.

- North = forward
- East = right
- South = backward
- West = left

See [Micro Joystick HID installation on page 32](#) and [Multi Joystick HID installation on page 33](#) for the default north position.

SETTING	DESCRIPTION	PARAMETERS	
Action - direct	Action is activated from the moment the key is pressed until the key is released.  NOTE: Output of direct action is blocked for as long as the nudge timer is active.	Options	<ul style="list-style-type: none">• None• Left mouse click• Right mouse click

SETTING	DESCRIPTION	PARAMETERS	
			<ul style="list-style-type: none"> • Mouse scroll N - E - S - W • Mouse move N - E - S - W • Joystick button 1 - 8 • Keyboard
Action - nudge	All the different actions that can be attributed to a nudge of the joystick.	Options	<ul style="list-style-type: none"> • None • Left mouse click • Right mouse click • Middle mouse click • Left mouse click &

SETTING	DESCRIPTION	PARAMETERS
		<ul style="list-style-type: none"> hold • Double left mouse click • Mouse scroll N - E - S - W • Config set 1 - 2 - 3 • Joystick button 1 - 8 • Keyboard string • Keyboard string &
Keyboard string - direct	Only active when the Keyboard string value is selected for Action - direct .	See Keyboard mode on page 38 for more information.

SETTING	DESCRIPTION	PARAMETERS
	 NOTE: This action will send the characters for as long as the action is active.	
Keyboard string - nudge	Only active when the Keyboard string value is selected for Action - nudge .  NOTE: This action will send out a single character.	See Keyboard mode on page 38 for more information.

Button parameters

The buttons can either perform a **direct action** or you can distinguish between a **short/medium/long press**.

See [Buttons on page 26](#) for more information.

SETTING	DESCRIPTION	PARAMETERS	
Action - direct	Action is activated from the moment the button is pressed.	Options	<ul style="list-style-type: none"> • None • Left mouse click

SETTING	DESCRIPTION	PARAMETERS
		<ul style="list-style-type: none"> • Right mouse click • Middle mouse click • Left mouse click & hold • Double left mouse click • Mouse scroll N - E - S - W • Config set 1 - 2 - 3 • Joystick button 1 - 8 • Keyboard string • Keyboard

SETTING	DESCRIPTION	PARAMETERS	
Short/ medium/ long press	Action is activated from the moment the button is released.	Options	string & <ul style="list-style-type: none"> • None • Left mouse click • Right mouse click • Middle mouse click • Left mouse click & hold • Double left mouse click • Mouse scroll N - E - S - W • Config set 1 - 2 - 3

SETTING	DESCRIPTION	PARAMETERS
		<ul style="list-style-type: none"> • Joystick button 1 - 8 • Keyboard string • Keyboard string &
Keyboard string - direct + short/ medium/ long press	Enter the modifier and key code	See Keyboard mode on page 38 for more information.

Supported platforms

The HID Joystick is supported by the following platforms:

- [Windows on page 56](#)
- [OSX on page 57](#)
- [Android on page 58](#)
- [iOS on page 61](#)
- [XBOX/XAC on page 64](#)

Windows

The HID Joystick was developed for Windows 10. The USB cable of the HID Joystick can be connected to a free port of the PC.

Keyboard

See [Keyboard mode on page 38](#) .

Mouse

Some examples of mouse actions supported by Windows:

KEY	DESCRIPTION
Move north/east/south/west	Cursor moves up/down/left/right
Scroll north/east/south/west	Scroll through page or document up/down/left/right
Left mouse click	Select
Middle mouse click	Double click
Right mouse click	Right click menu
Left mouse click & hold	Select and hold
Double left mouse click	Open file

NOTE: The middle mouse button is not a supported action by default. It is supported, however, by the firefox browser: clicking on a tab in firefox with the middle mouse button closes that tab.

Joystick

The actions of the joystick can be tested in the Windows Game Controller Tester.

In Windows: **Set up USB Game Controllers > Properties > Test.**

This menu will show an axis and 8 buttons. If a button is pressed, it will highlight. If an axis is changed, the crosshairs will change.

OSX

The USB cable of the HID Joystick can be connected to a free port of the PC.

Keyboard

See [Keyboard mode on page 38](#) .

Mouse

The following mouse actions can be configured:

KEY	DESCRIPTION
Move up/down/left/right	Cursor moves up/down/left/right
Scroll up / scroll down	Scroll up / down (e.g. through website with a lot of content)
Scroll left / scroll right	Scroll horizontally (e.g. in a book)
Button click left	Select
Button click middle	Double click
Button click right	Right click menu

Joystick

Joystick actions are supported in OS.

Android

The USB cable of the HID Joystick can be connected to an Android phone or tablet via a USB OTG cable. After a couple of seconds, the phone or tablet should recognize the HID Joystick.

Keyboard

You can use the keyboard mode to navigate your Android device with keyboard shortcuts (see table below). This is a limitative list. However, there

is an Android app External Keyboard Helper, which allows you to create custom keyboard shortcuts.

DESCRIPTION	KEYBOARD STRING
Navigate Up	KEY_UP 0x00 0x52
Navigate Down	KEY_DOWN 0x00 0x51
Navigate Left	KEY_LEFT 0x00 0x50
Navigate Right	KEY_RIGHT 0x00 0x4f
Open Selection	KEY_ENTER 0x00 0x28
Go back	KEY_ESC 0x00 0x29
Remove a character	KEY_BACKSPACE 0x00 0x2a
Switch tasks	MOD_LALT + KEY_TAB 0x04 0x2b
Open browser	MOD_LMETA + KEY_B 0x08 0x05
Cut	MOD_LCTRL + KEY_X 0x01 0x1b
Copy	MOD_LCTRL + KEY_C 0x01 0x06
Paste	MOD_LCTRL + KEY_V 0x01 0x19
Next window	KEY_TAB 0x00 0x2b
Select text up	MOD_LSHIFT + KEY_UP 0x02 0x52

DESCRIPTION	KEYBOARD STRING
Select text down	MOD_LSHIFT + KEY_DOWN 0x02 0x51
Select text right	MOD_LSHIFT + KEY_RIGHT 0x02 0x50
Select text left	MOD_LSHIFT + KEY_LEFT 0x02 0x4f
Delete	KEY_DELETE 0x00 0x4c

Mouse

The following mouse actions can be configured:

KEY	DESCRIPTION
Move up/down/left/right	Cursor moves up/down/left/right
Scroll up / scroll down	Scroll up / down
Button click left	Select
Button click middle	Return to home menu
Button click right	Return to previous menu
Button double left click	Open app

Joystick

We support 8 buttons and they can be mapped per game.

NOTE: To test the function, you can install the free Android app Game controller keymapper from AnkitChowdhury. This allows you to visualize the axis and keycodes.

iOS

The USB cable of the HID Joystick can be connected to an iPad or iPhone via an Apple lightning to USB adapter cable. After a couple of seconds, the phone or tablet should recognize the HID Joystick.

Keyboard

You can use the keyboard to navigate in iOS with keyboard shortcuts:

DESCRIPTION	KEYBOARD STRING
Navigate Up	KEY_UP 0x00 0x52
Navigate Down	KEY_DOWN 0x00 0x51
Navigate Left	KEY_LEFT 0x00 0x50
Navigate Right	KEY_RIGHT 0x00 0x4f
Open selection	KEY_SPACE 0x00 0x2c

DESCRIPTION	KEYBOARD STRING
Go to home	MOD_LMETA + KEY_H 0x08 0x0b
Remove a character	KEY_BACKSPACE 0x00 0x2a
Switch tasks	MOD_LALT + KEY_TAB 0x04 0x2b
Cut	MOD_LCTRL + KEY_X 0x01 0x1b
Copy	MOD_LCTRL + KEY_C 0x01 0x06
Paste	MOD_LCTRL + KEY_V 0x01 0x19
Next window	KEY_TAB 0x00 0x2b
Select text up	MOD_LSHIFT + KEY_UP 0x02 0x52
Select text down	MOD_LSHIFT + KEY_DOWN 0x02 0x51
Select text right	MOD_LSHIFT + KEY_RIGHT 0x02 0x50
Select text left	MOD_LSHIFT + KEY_LEFT 0x02 0x4f
Delete	KEY_DELETE 0x00 0x4c

NOTE: To create custom shortcuts in iOS: **Settings > Accessibility > Keyboards > Full Keyboard Access > On**. Go to **Commands**: tap a command, then press a custom key to assign to it and click **Done**.

NOTE: To change the keyboard layout in iOS: **Settings > General > Keyboard > Keyboards**: keep finger on the keyboard and select **QWERTY/AZERTY/QWERTZ**

Mouse

Setting up the mouse buttons:

Settings > Accessibility > Touch > Assistive touch > Enable assistive touch.

Connect the HID Joystick to the iDevice with the lightning cable.

Tap **Devices > Hid Joyst. Int > Customize additional buttons**:

- Set Button 1 as **Single Tap**.
- Set Button 2 as **Secondary Click**.
- Set Button 3 as **Home**.

The following mouse actions can be configured:

KEY	DESCRIPTION
Move up/down/left/right	Cursor moves up/down/left/right
Scroll up / scroll down	Scroll up / down
Button click left	Select
Button click middle	Return to home menu
Button click right	Return to home menu

Joystick

Not supported in iOS.

XBOX/XAC

NOTE: XBOX supports the use of mouse and keyboard in some games and apps, but it doesn't work for all content. The game or app published must enable this feature for their content. Alternatively, you can use the XBOX adaptive controller to play non-supported games.

Check your XBOX manual for more information on how to connect a USB device to the XBOX.

Keyboard

You can use the keyboard to navigate the XBOX menu with keyboard shortcuts:

KEYBOARD STRING	DESCRIPTION
KEY_UP – 0x00 0x52	Navigate up
KEY_DOWN - 0x00 0x51	Navigate down
KEY_LEFT - 0x00 0x50	Navigate left
KEY_RIGHT - 0x00 0x4f	Navigate right
MOD_LMETA + KEY_LEFTMETA – 0x08 0xe3	Open the guide
KEY_ENTER 0x00 0x28	Select
KEY_SPACE 0x00 0x2x	Select
KEY_ESC 0x00 0x29	Back
MOD_LMETA + KEY_BACKSPACE 0x08 0x2a	Back
KEY_TAB 0x00 0x2b	Toggle to next tile
MOD_LSHIFT + KEY_TAB 0x02 0x2b	Toggle to previous tile

KEYBOARD STRING	DESCRIPTION
MOD_LMETA + KEY_M 0x08 0x10	Open menu
MOD_LMETA + KEY_V 0x08 0x19	Change view
KEY_Y 0x00 0x1c	Open search

Mouse

NOTE: Wired USB mice will work only in select games and apps that have enabled mouse navigation. Check your game to see if it offers mouse support.

To configure your mouse:

- 1 Press the XBOX button on your controller to open the guide.
- 2 Select **Profiles & System > Settings > Devices & Streaming**.
- 3 Select **Mouse**. You will see options for pointer speed and for swapping primary and secondary mouse buttons.

NOTE: You can't use a mouse on **Home** or within the XBOX user interface while navigating the dashboard.

NOTE: You will need to use your controller to configure your mouse. **Settings** does not support mouse interaction.

NOTE: A mouse will not work in Microsoft Edge, which already has an on-screen cursor.

XBOX Adaptive Controller (XAC)

The HID Joystick is compatible with the XAC. The XAC is designed primarily to meet the needs of gamers with limited mobility and is a unified hub for devices to help make gaming more accessible.

Connect the USB cable of the HID Joystick to the XAC and use the HID Joystick in joystick mode.

The XAC has two USB ports: the left USB port replaces the left thumbstick and the right USB port replaces the right thumbstick.

NOTE: Other HID devices such as Mouse and Keyboard are not supported by the XAC, so you have to make sure that your HID Joystick is set to the right configuration mode.

The XAC will map the joystick buttons automatically.

JOYSTICK BUTTON	LEFT USB PORT	RIGHT USB PORT
Button 1	X1	View
Button 2	X2	Menu
Button 3	ThumBtnL	ThumBtnR

JOYSTICK BUTTON	LEFT USB PORT	RIGHT USB PORT
Button 4	BumperL	BumperR
Button 5	A	X
Button 6	B	Y
Button 7	View	X1
Button 8	Menu	X2

NOTE: If the buttons are reconfigured using the XBOX accessories app, the new configurations will be applied to the USB HID Joystick buttons too.

The actions can be tested on the XBOX one.

Download and install the free Game controller tester app from the Microsoft Store. Open the app and select the XBOX adaptive controller as the controller under test. Press the buttons and move the joystick. The respective buttons will light up and the axis will move in the requested direction.

First time use

During first time use, it is advised that the user and/or his attendant thoroughly read this manual.

It is important that the user is fully aware of the installation, how to use it and what can be adjusted to optimize his/her experience.

- 1 Carefully read the manual and make sure that you understand the function of every joystick action and button.
- 2 Check whether the joystick, the housing, cabling and all connectors are intact.
- 3 Test all positions of the HID joystick.
 - Are the joystick and switches within easy reach?
 - Can the user safely operate the device with the least effort?
 - Is the placement of the joystick and the switches in all available positions optimal for the user?
 - Does the joystick return to its default position when moving and releasing the joystick forward, backward, left and right?
- 4 If needed, adjust the HID Joystick.
- 5 Make sure that you understand the possible problems and how to address them.

Maintenance

Cleaning

WARNING: As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

- Gently remove dust and dirt with a damp cloth.
- Use only non-aggressive disinfectant cleaning agents.

CAUTION: Do not immerse in water or use excessive amounts of liquid.

CAUTION: Do not apply additional lubricants to the moving parts.

The handles of the joystick can be removed to clean them.

- You may shortly immerse the handle in warm water with a gentle cleaning agent. Let the handle dry before putting it back on.
- If the handle is damaged, worn out or remains dirty, please contact your authorized dealer or mo-vis to replace it.

Monthly inspection

Monthly, or whenever needed, check whether:

- All bolts and screws are still firmly tightened.
- There is no damage to any wiring.
- There is no excessive wear to any of the parts.

Maintenance-free

The HID Joystick is maintenance-free. Under regular circumstances of use, the HID Joystick, interface unit and different parts do not require additional maintenance.

Error codes




TROUBLE: When a fault occurs, the LEDs of the HID Joystick will start to flash. A long delay is followed by a number of flashes with a short delay. Contact mo-vis or your dealer in case of error.


LED FLASH COUNT	REASON	REQUIRED ACTION
2	Msp sensor is not connected.	Contact mo-vis.
3	Connection problem with the external EEPROM.	

Dimensions

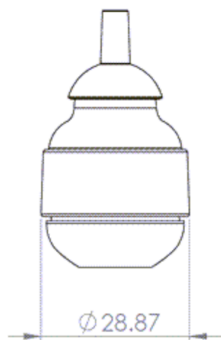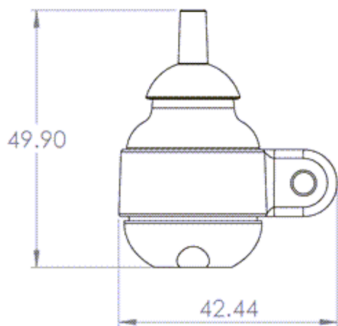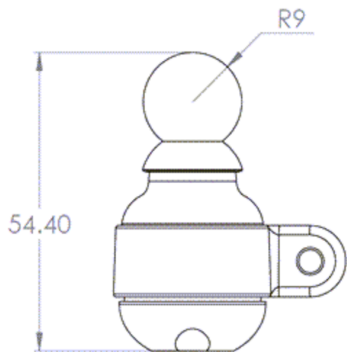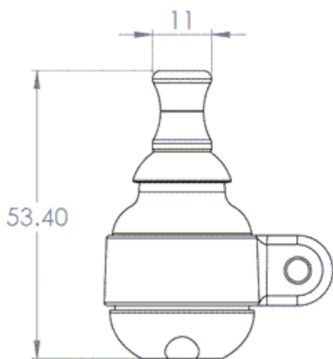Micro Joystick HID



PART	DIMENSIONS (METRIC)	DIMENSIONS (IMPERIAL)
Joystick	50 mm x 29 mm x 29 mm (HxWxD)	1.97 in x 1.41 in x 1.41 in (HxWxD)
Ball	16 mm x 18 mm x 18 mm (HxWxD)	0.63 in x 0.71 in x 0.71 in (HxWxD)
Cup	15 mm x 11 mm x 11 mm (HxWxD)	0.59 in x 0.43 in x 0.43 in (HxWxD)

Multi Joystick HID

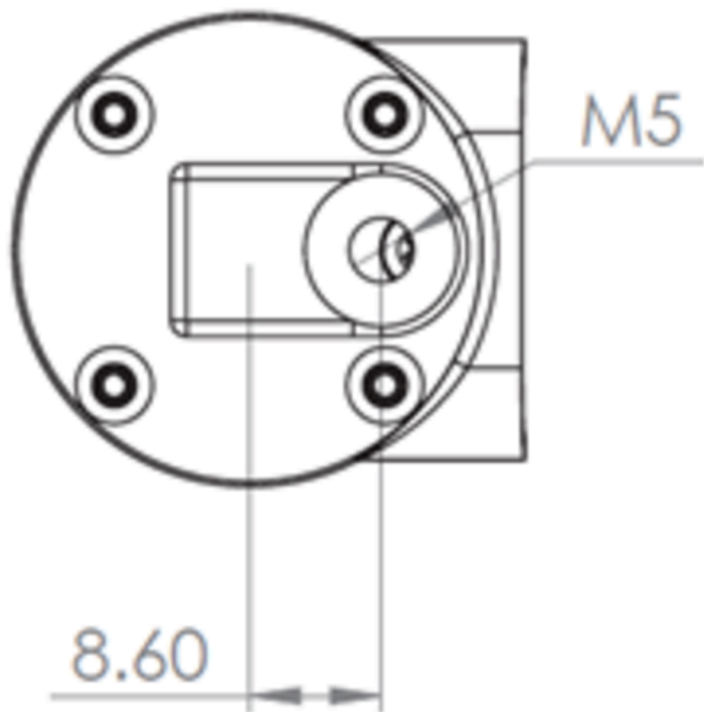Bottom view:



Side view:

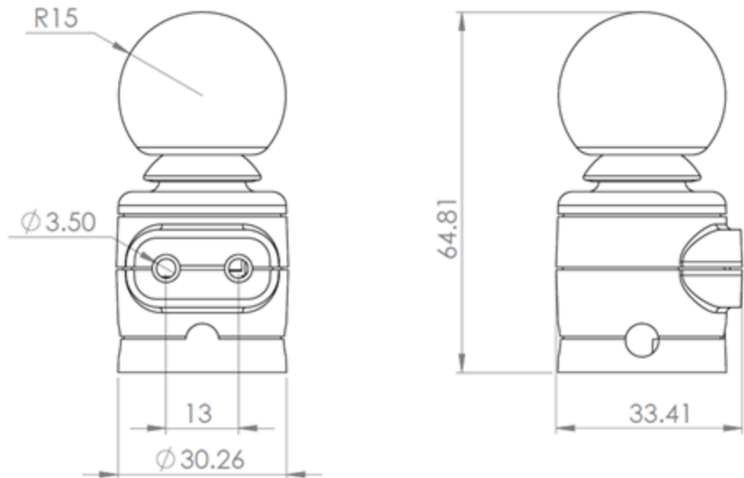PART	DIMENSIONS (METRIC)	DIMENSIONS (IMPERIAL)
Joystick	65 mm x 31 mm x 31 mm (HxWxD)	2.55 in x 1.22 in x 1.22 in
Ball	26 mm x 30 mm x 30	1.02 in x 1.18 in x 1.18

PART	DIMENSIONS (METRIC)	DIMENSIONS (IMPERIAL)
	mm	in

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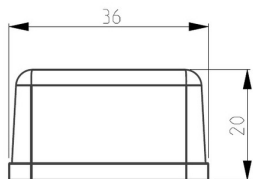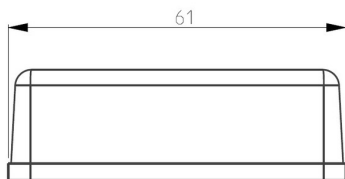IMAGE	DESCRIPTION
	P002-50 - Micro Joystick sensor

IMAGE	DESCRIPTION
 A black cable with a micro joystick ball at the end. The cable is coiled and has a small, round, black joystick ball attached to the end. The ball has a small protrusion on its side. The cable ends in a small, multi-colored connector.	M006-08 - Micro Joystick ball

IMAGE**DESCRIPTION**

IMAGE**DESCRIPTION**

M006-09 - Micro Joystick cup

IMAGE	DESCRIPTION
 The image shows four black components of a micro joystick mounting set. At the top left is a circular ring with a threaded hole and a small metal nut. To its right is a small rectangular plate with a circular hole. Below these are two long, thin cylindrical rods of different lengths.	M006-20 - Micro Joystick mounting set
	M004-91 Q2M Rod 50 mm Threaded UNC 1/4x20

IMAGE	DESCRIPTION
	
	P025-51 HID Joystick Interface
	Câble de connexion USB A vers B PM CAB00031 - micro 1 m

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IMAGE	DESCRIPTION
	P002-60 - Multi Joystick Sensor

IMAGE	DESCRIPTION
	
	M002-05 - Multi Joystick ball
	M002-18 - Multi Joystick Mounting set

IMAGE	DESCRIPTION
	
	<p>M004-91 Q2M Rod 50 mm Threaded UNC 1/4x20</p>
	<p>P025-51 HID Joystick Interface</p>

IMAGE	DESCRIPTION
	
	Câble de connexion USB A vers B PM CAB00031 - micro 1 m

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M006-08 - Micro Joystick ball

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M006-09 - Micro Joystick cup

AFBEELDING	BESCHRIJVING
 The image displays four black plastic components of a micro joystick mounting set. At the top left is a circular ring with a central threaded hole and a small metal pin protruding from the top. To its right is a small, rectangular component with a circular hole. Below these are two long, thin cylindrical rods of different lengths.	M006-20 - Micro Joystick mounting set
	M004-91 Q2M Rod 50 mm Threaded UNC 1/4x20

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