Made in Norway

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Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and manufactured in Norway. Due to advanced manufacturing by using the best equipment, we are able to offer great accuracy and maximum quality in all levels of our production. This product conforms to the Medical Device Directive 93/42/EEC. It is tested and approved according to ISO 11199-2:2005.

On receipt of the goods, please check your product. If you have any queries, please contact your mobility shop or dealer immediately. The same applies if you have general questions about the product. If you have problems reading this user manual, an electronic version is available on www.topro.co.uk

Guarantee // Expected life span of the product

TOPRO ODYSSE is guaranteed free of faults and defects for a period of 7 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g. brake block, brake cable, wheel, seat, bag and handgrip) are exempt from the 7-year guarantee. For repairs during the guarantee period, please contact your local mobility shop or dealer. The guarantee is void if unauthorised spare parts and accessories have been or are being used on the product.

The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

■ **The red numbered squares in headings refer to the numbered illustrations on the inside of the cover.**

Get to know your TOPRO PRODUCT 1

- | | | |
|---|---|---|
| A Handlebar, ergonomic | H Lock clamp | N Indicator for incorrect opening |
| B Brake handle | I Seat with strap for sideways folding | O Wheel, detachable |
| C Bell | J Lock wheel for the handle | P Brake block IBS (interior brake system on the rim) |
| D Handlebar tube | K Shopping bag with lid | Q Edge guard |
| E Brake wire | L Side frame | R Tilt function |
| F Seat bar/handle for carrying | M Releaser for downwards folding | |
| G Bar for shopping bag | | |
| S The label on the rear side of the left tube shows the product name/model and | | |
| ■ Maximum weight of user | ■ Maximum width of rollator | ■ Maximum height of rollator |
- A separate label on the right tube indicates the serial number (SN) and production year and month.

Measurements	TOPRO Odysse M	TOPRO Odysse S
Recommended user height	1500–2000 mm	1400–1700 mm
Maximum user weight	150 kg	125 kg
Weight capacity shopping bag	10 kg	10 kg
Volume shopping bag (expandable)	13–22.5 l	11–20 l
Handle height	815–1000 mm	710–860 mm
Length (in upright position)	650 mm	650 mm
Maximum width	600 mm	600 mm
Width between handles	470 mm	470 mm
Seat height	620 mm	540 mm
Turning diameter	840 mm	840 mm
Diameter wheels	200 mm	200 mm
Width wheels	36 mm	36 mm
Handles	Ergonomic – several holding positions (TOPRO Ergo Grip)	
Grip distance driving brake	65 mm	65 mm
Height when completely folded (sideways and downwards)	445 mm	445 mm
Height when only sideways folded	815 mm	740 mm
Width when folded	245 mm	245 mm
Length when completely folded (sideways and downwards)	750 mm	750 mm
Transport weight excl. wheels and shopping bag	6.0 kg	5.7 kg
Product weight incl. wheels and shopping bag	8.2 kg	7.9 kg

Intention

The TOPRO ODYSSE shall give support to users with reduced balance and/or reduced walking ability. It is developed to be pushed, not pulled. The integrated seat provides an opportunity for the user to rest if needed. The bag is designed for carrying items weighing up to a total of 10 kg. The intended user is an adult. The product is not suitable for persons with low strength in arms, with very poor balance, or with considerable cognitive disabilities. TOPRO ODYSSE is designed and approved for both indoor and outdoor use. Outdoor the rollator is intended to be an aid for walking on roads and paths. It is not suitable in rough and/or steep terrain or steep inclinations, e.g. stairs. With its two-way folding this allows the rollator to take less space and is especially developed for transport/travel.

Important to know about safety before use

- Take care when folding and unfolding, do not trap your fingers.
- Be aware not to put your fingers in open holes or between stationary parts.
- Check that the rollator is being locked into position when unfolded. A red indicator in front of the connection part shows that the rollator is incorrectly opened upright. If so the rollator must not be further unfolded.
- Check the brakes every time before use.
- Not to be used by persons heavier than the maximum user weight (see table).
- Do not use to transport heavy loads or persons.
- The rollator is not a toy and therefore should be treated in an appropriate manner.
- Make sure that the lock wheels for height adjustment for the handles are locked tightly, and that the handlebar settings are not set higher than the maximum height marked on the bars.
- Be aware that some parts of the rollator might feel hot or cold when exposed to extreme temperatures.
- Do not alter the product as this might put your safety at risk and the guarantee will not be void.

Important to know about safety when in use 2

- When stationary or parked, always use the parking brakes on both sides **A**.
- Before sitting on the seat, make sure the parking brakes on both sides are in the locked position **A**.
- Do not lean or bend sideways or over the back of the rollator whilst in the seated position. Do not tilt the rollator more than max. 5° sideways, to avoid risk of falling. Due to the same reason bags etc. must never be hung on the handlebars.
- The driving brakes can be held in a little **B** to reduce speed but must not be in locked position when the rollator is in motion.
- To give the right support and safety, make sure the rollator is kept as close to your body as possible **C**.
- Walk straight, look forward and hold your backside in **D**. That is the easiest way to keep your balance.
- Do not push the rollator too far from your body **E** as this can cause unnecessary strain and you could fall.
- Be aware that the rollator may roll faster than you walking downhill, which may put you in risk of falling.
- Do not use the rollator on stairs. Be careful when descending kerbs.
- Be careful of flared skirts and trousers as they could be caught in a wheel or other parts of the rollator. This could lead to falling and we therefore recommend not wearing flared/wide clothing.
- For safety reason it is prohibited to sit on the rollator during public transportation.
Place your rollator securely and use the parking brake.

Assembling / adjusting

TOPRO ODYSSE is delivered part assembled. The rubber band that is strapped around the front forks is to be removed and disposed of. Then front wheels should be mounted, the strap around the rollator tubes loosened and the rollator unfolded (see paragraph *Mounting / dismantling the wheels and Unfolding / folding*). Finally, the height of the handlebar tubes should be adjusted to suit the user's needs (see paragraph *Lowering and height adjusting the handles*).

Mounting / dismantling the wheels 3

Mounting the wheels: Hold the wheel, push down the red button on the hub **A** and keep it in, then slide the wheel in as far as possible on the axle **B**. Release the button and the wheel should now be in the locked position. The wheel is correctly mounted when the axle is completely visible in the middle of the hub and you cannot see any of the red plastic part. The red button should now have popped back up from the hub. To check that it is securely fastened gently pull the wheel.

Please note: If you have removed all four wheels please be aware that the front and rear wheels are different; the rear wheels have a ring for interior braking.

Before mounting the rear wheels, make sure the parking brakes are not activated. After mounting the wheels, check that the brakes are functioning properly. Adjust them if necessary (see paragraph *Adjusting / checking and maintaining the brakes*).

Dismounting the wheels: Make sure the parking brakes are not activated (applies to the rear wheels only). Then push down the red button on the hub **A** and slide the wheel off the axle **B** while still pushing the button.

Dismounting the front forks: *Please note: The forks can only be dismounted when the rollator is completely folded. Do not use force, as this may damage the parts.* Lift the lock hook for the fork bolt **C** by using a finger, a flat screwdriver or similar. Take the wheel with fork **D** off while lifting the lock hook up and outwards.

Mounting the front forks: *Please note: The forks can only be mounted when the rollator is completely folded. Do not use force; this may damage the parts.* Make sure to choose the correct wheel for each side. The forks are marked with R (right) and L (left). The fork should turn inwards and backwards and the wheel outwards. Grip the front wheel and fork **E**. Rotate the fork so that the sloping side of the bolt turns forwards in the rollator's driving direction **F**. Lift the lock hook for the fork bolt **C** up and outwards while you push the fork bolt completely into the fastening point/hole **G**. Thereafter push the lock hook for the fork bolt in and downwards **H** until you hear a "click". Tip: Lubrication of the lock hook with grease is advised to ensure smooth mounting and dismounting of the wheels.

Unfolding / folding 4

Unfolding: Loosen the clip of the strap that goes from the shopping bag around the lower rollator tubes **A**. Stand behind the rollator and grip both handlebars. Gently lift the rear part of the rollator slightly from the floor **B** and then push the rear wheels down, one by one, to release the inner locking mechanism **C**. Then lift the handles upwards **D** until you hear a clicking sound and the red indicators in front of the connection parts are not visible **E**. Loosen the lock clamp that is over the seat bars **F**. Press the seat bars down and out **G**. The side frames will slide apart. Press down the seat bars. You will hear a clicking sound. The rollator is now locked in its open position. Close the strap **A** behind the cross beams to avoid it from hooking up.

Folding: Adjust the handlebar tubes to the lowest position, so that the rollator can fit into the transport bag (see paragraph *Lowering and height adjusting the handles*). Pull the strap on the seat **H** and press the side frames together **I**. Lock the clamp **J** over the seat bars. Grip the oval shaped releaser on both sides of the rollator **K** with your thumbs at the front. Push the release handles down and lead the handlebars toward the rear wheels **L** until they lock and you can hear a "click". *Please note: The brake handles have to be situated behind the tilt function to ensure the rollator is in the locked position.*

Lowering and height adjusting the handles 5

Lowering: Unscrew the lock wheel for the brake handles **A** (approx. 4–5 turns). Push the handlebar tubes down until they stop **B**. Tighten the lock **A** again.

Height adjusting: Unscrew the lock for the brake handles **A** (approx. 4–5 turns). Adjust the handlebar tubes to a suitable height then pull them slightly up and down **C** until you hear a clicking sound. Tighten the lock **A** again. Adjust both handlebars to the same height. *Be aware of the maximum height marked on the tubes.* It is recommended that the handlebars are set to the same height as your wrists when your arms are hanging alongside your body.

Adjusting / checking and maintaining the brakes 6

Adjusting / checking: Loosen the nut **A** beside the brake. Tighten the brake wire **B** with the adjustment screw **C**. Adjust the brake block **D** as close as possible to the inner drum brake, and then tighten the nut **A**. Turn the wheel around to check that the brake block is not touching the wheel.

Maintaining: Clean the brake block **D** and the brake spring **E** frequently. The wheels and brakes can be replaced if they are worn or damaged.

Using the brakes 2

Make sure the brakes work before each trip. Each brake handle **A** works on each of the rear wheels. Wrong use can cause unsafe situations and damage to the rollator: *The rollator must not be driven or pushed with force when the parking brakes are on or when the wheels are partly locked using the brakes.* This indicates incorrect use and can cause wear and tear, which makes the tires and brake quality weaker.

Parking brakes: Push both brake handles **A** down until they lock. Both rear wheels are now locked. Squeeze both brakes handles upwards to release the parking brakes.

Driving brakes: Pull both brake handles **B** upwards to reduce speed. Do not use continuously while the rollator is in motion.

Seat 7

The parking brakes must be on before using the seat **A**. Push both brake handles **B** downwards until they lock. Both rear wheels are now locked. Sit with your back towards the bag **C**.

Shopping bag 8

The bag holds up to 10 kilos and has a volume of 13–22.5 litre (M) / 11–20 litre (S). It can be removed and is washable at 40 °C. **Removal:** Loosen the clip of the strap **A** that goes from the shopping bag around the lower rollator tubes and pull the bag off the bars **B**. **Attaching:** Attach the bag to the bars **B** and fasten the strap around the crossbeams **A**.

Tip: For easier attachment of the bag, you could slightly fold the rollator first, by pulling the strap on the seat.

Edge guard // Tilt function 9

Edge guard: The edge guards **A**, which are in front of the rear wheels prevent the rear wheels from catching on corners, door frames etc. **Tilt function:** To use the tilt function use the front of the foot and press down on the tilt pedal **B**, which is beside the rear wheel. This helps to tilt the rollator over thresholds, edges etc. Please note: Take care when descending kerbs. Do not use the rollator in stairs.

Carrying the rollator 10

The rollator must not be carried by the seat strap or the brake wires. If the rollator is only sideways folded it can be carried by the seat bars **A**. If the rollator is completely folded (sideways and downwards) you can carry the rollator by one of the upper rollator tubes **B**.

Transport // Transport bag

The rollator should be transported in the folded and locked position (see paragraph *Unfolding / Folding*). Ensure that the rollator cannot unfold unintentionally. Be careful when loading the rollator into a vehicle, giving attention to securing of the item. When using the enclosed transport bag, the front of the rollator is to be placed into the part of the bag where the TOPRO logo is. The transport bag can be washed at 40 °C.

Storage

If the rollator shall be stored outdoors, then under cover.

Material // Recycling

The rollator is made of plastic coated aluminium pipes, plastic couplings and material of plastic and polyester. Most parts can be recycled. Dispose the rollator according to the applicable regulations in your country. For information, please contact your public authorities.

Cleaning

The rollator can be steamed, hosed and washed with normal household detergents. Do not use scrub detergents, steam- or high-pressure cleaners.

Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface must be cleaned with a disinfection that contains 70–80 % ethanol. We advise against disinfection that contains chlorine and phenol. *The manufacturer cannot be held responsible for any damage or wounds that might occur by the use of harmful disinfection or disinfection carried out by unauthorized personnel.*

Inspection // Maintenance // Re-use

It is recommended that general inspection and maintenance is carried out and that the items below are checked on a regular basis, and at least once a year (how frequent is determined by how and how often the rollator is used): Tubes, frame and screws, handlebars and handlebar tubes, brakes, wheels, seat and accessories. This also applies when the rollator is to be prepared for re-use. Please refer to the detailed instructions presented in this user manual about maintenance of the brakes and cleaning and disinfection of the rollator.

If the rollator does not work properly

Make sure that the rollator is unfolded correctly and that both brake handles are in expected position.

Accessories

For pictures and TOPRO article numbers please see the inside of the cover of this user manual. Accessories mounted on the rollator may influence stability, we therefore advise you to use them carefully.

- | | |
|--|---|
| A Back support, adjustable with padding | J Umbrella with attachment (black) |
| B Tray | K Drag brake (pair) |
| C Anti slip mat for tray | L Offroad wheels for IBS, set of four |
| D Crutch holder | M Studded tyres for IBS, pair of two rear wheels |
| E Bottle holder | N Ice spurs for shoes M (shoe size 36–40) |
| F Decorative flower stickers, set of four | Ice spurs for shoes L (shoe size 41–46) |
| G Nameplate | O One-handed brake |
| H Lock | P Guiding handle for the visually impaired |
| I LED lamp | Q Rear bag with zipper |

The accessories are to be ordered separately to give the possibility to compose a TOPRO ODYSSEÉ that suits the individual users need. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage www.topro.co.uk

In case of questions or faults, please contact your mobility shop or dealer.

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