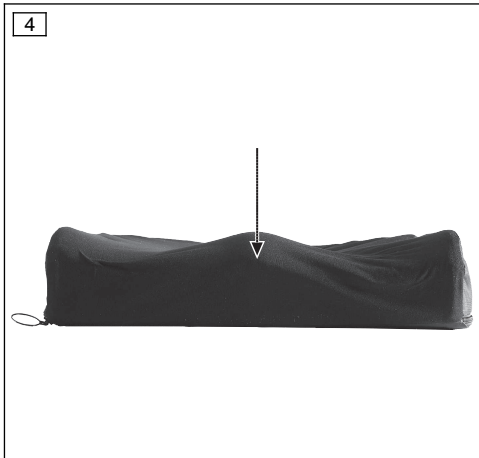
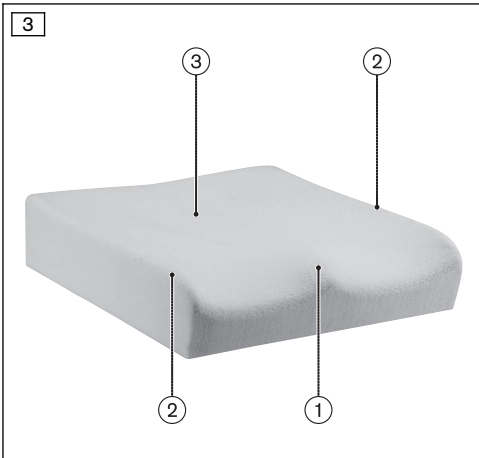
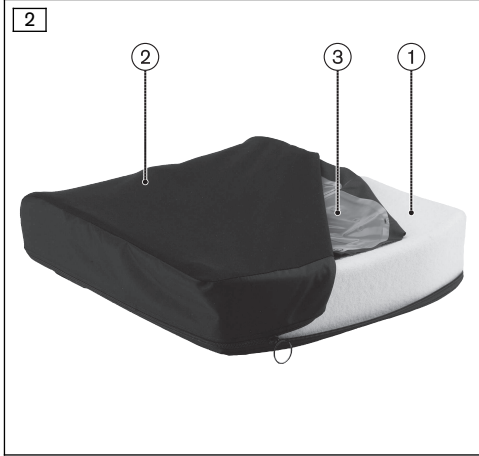
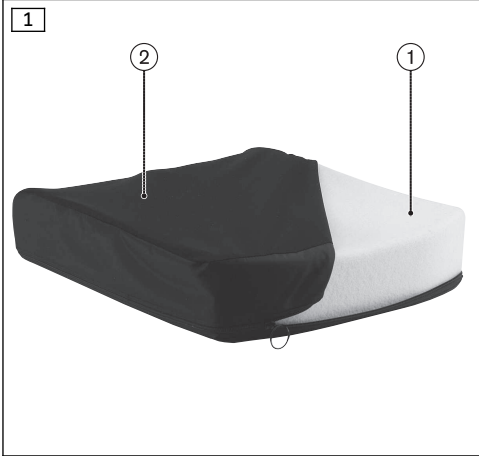
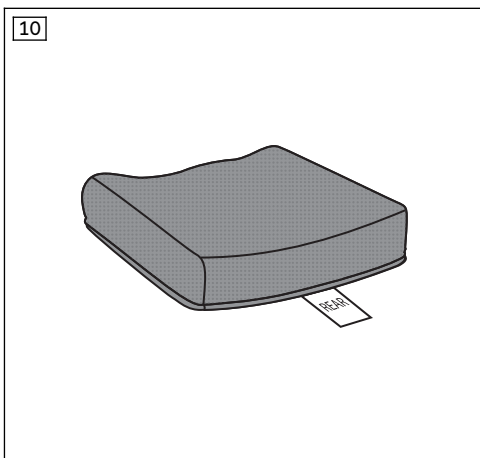
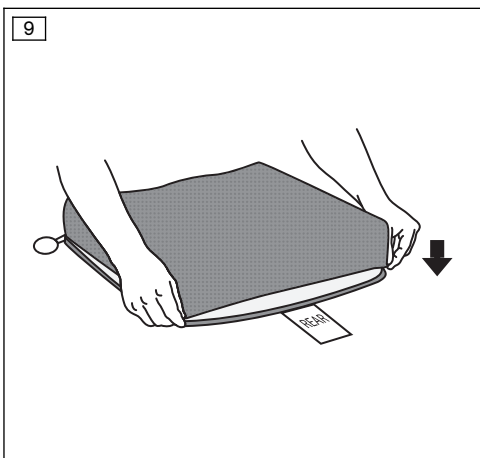
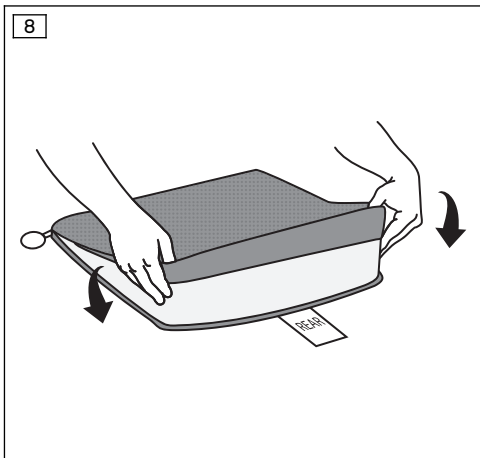
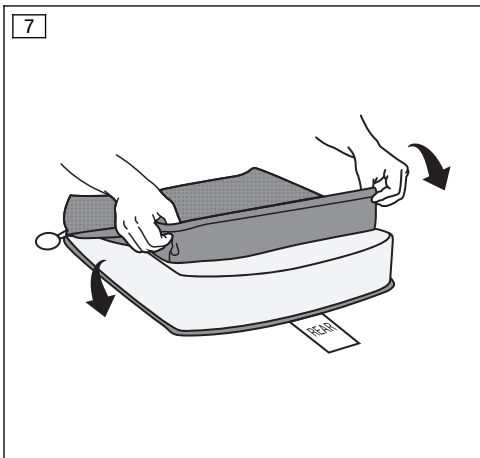
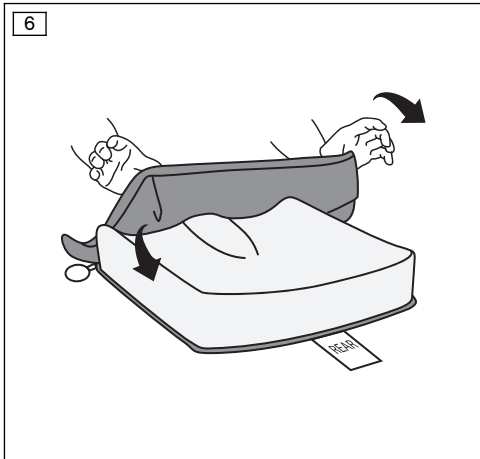
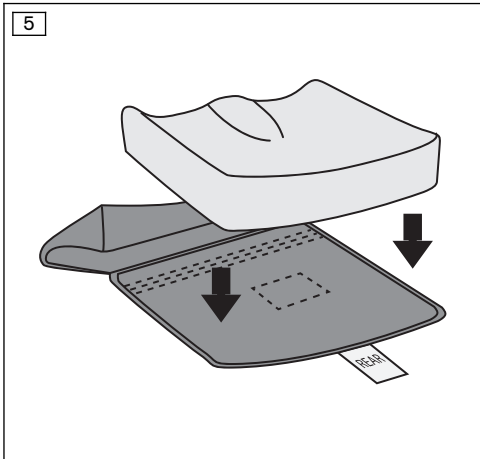
Terra, Terra Aquos

DE Gebrauchsanweisung	4
EN Instructions for use	9
FR Instructions d'utilisation	13
IT Istruzioni per l'uso	19
ES Instrucciones de uso	24
NL Gebruiksaanwijzing	29
DA Brugsanvisning	34
CS Návod k použití	39
HR Upute za uporabu	44
TR Kullanma talimatı	48

INFORMATION

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INFORMATION

Last update: 2015-02-20

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Function

The Terra (see fig. 1) and Terra Aquos (see fig. 2) wheelchair seat cushions were developed for maximum pressure relief.

The anatomically shaped foam base (see fig. 3) permits defined positioning of the user. Any pressure peaks are distributed over the largest possible area.

The special covers reduce shear forces and ensure a high level of seating comfort for the user.

The components of the seat cushions fulfil the following functions:

Component	Model	Function
Rest suspension foam base; anatomically shaped	Terra Terra Aquos	Stable leg positioning thanks to medial and lateral guides (see fig. 3, item 1/2); Pelvic recess provides relief for the pelvis and support for the trochanter (see fig. 3, item 3)
Dartex fabric cover	Terra Terra Aquos	PU-coated fabric with two-way stretch for incontinent users (Terra: see fig. 1, item 2; Terra Aquos: see fig. 2, item 2)
LiquiCell® packs	Terra Aquos	Reduces shear forces (see fig. 2, item 3)

2 Application

2.1 Intended Use

The wheelchair cushion is to be used **exclusively** as a wheelchair accessory for adult users. It provides comfort, postural guidance and pressure redistribution. An appropriately sized wheelchair cushion should be selected based on the user's size and should be used exclusively as an accessory for a correspondingly sized wheelchair.

The wheelchair cushion is particularly suitable for low to moderate positioning and preventing pressure sores in patients with paraplegia, hemiplegia, neurological deficits, and rheumatic nerve and muscle disorders as well as for geriatric patients.

Please note that the most appropriate wheelchair cushion for an individual should always be determined by a suitably qualified or experienced person based on a full risk assessment and used in conjunction with a full programme of care.

2.2 Area of Application

The wheelchair cushion can be used by people weighing up to **150 kg [330 lbs]** with no restrictions. We assume no liability if the maximum user weight is exceeded.

2.3 Contraindications

Seek professional advice before selecting an appropriate wheelchair cushion where there is an existing case or history of pressure sores/decubitus ulcers.

2.3.1 Absolute Contraindications

Not known.

2.3.2 Relative Contraindications

If redness of the skin or other problems occur during use, consult a doctor or therapist as soon as possible.

2.4 Further Usage Restrictions

The product is designed for use by **one person** only. Re-use of the product is not permitted due to hygiene considerations.

2.5 Qualification

This product may only be selected and adapted by trained qualified personnel.

3 Safety

3.1 Explanation of Warning Symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

3.2 General Safety Instructions

Altered centre of gravity resulting from use of the seat cushion

Risk of tipping over or falling due to unstable settings

- ▶ After installing any seating into a wheelchair, always check that the user's sitting position and the wheelchair's centre of gravity are appropriate for the intended use and the user's abilities. Work with the user to find the optimal sitting position.
- ▶ If required, adjust the sitting position or axle position to optimise stability, or alternatively assess the suitability of anti-tip devices.

Lack of testing prior to long-term use

Risk of skin reddening or pressure sore formation due to incorrectly selected seat cushion

- ▶ Any seat cushion should be tried out by the user under medical supervision for several hours prior to long-term use. The skin should be monitored for redness and/or damage during this time.

3.3 Information to be Provided to the Patient/Attendant

Skin damage

Risk of skin reddening during long-term use

- ▶ Do not use the wheelchair cushion until it has been adjusted by a suitably qualified or experienced person.
- ▶ Examine the skin for redness on a regular basis. Skin reddening is a clinical indicator of tissue damage.
- ▶ If redness of the skin or other problems occur during use, stop using the seat cushion immediately. Only start using the seat cushion again after consulting the prescribing doctor or therapist.

⚠ CAUTION

Failure to observe care instructions

Risk of pressure sores due to incorrect skin care

- ▶ All seat cushions were developed by the manufacturer to reduce pressure on the buttocks. A seat cushion alone, however, cannot eliminate sitting pressure or possible pressure sores entirely.
- ▶ When using the seat cushion, always pay attention to correct skin care – especially through a healthy diet, hygiene and regular pressure relief.

⚠ CAUTION

Improper handling of ignition sources

Risk of burns due to seat cushions, padding and upholstery catching on fire

- ▶ The seat cushions, padding and upholstery are not highly flammable, but can catch fire. Therefore the utmost caution must be exercised in the vicinity of an open flame.
- ▶ Keep away from all ignition sources, especially lit cigarettes.

NOTICE

Improper use

Risk of damage to the cushion surface due to user error

- ▶ Do not allow the wheelchair cushion to come into contact with sharp objects. This also applies to animals such as pet cats with sharp claws.
- ▶ Do not expose the wheelchair cushion to excessive heat or open flame.
- ▶ If the wheelchair cushion is expected to come into contact with liquid, such as spilt drinks or episodes of incontinence, always use it in conjunction with a liquid-repellent cover.
- ▶ Only use the official Ottobock incontinence barriers for this product. These barriers have been optimised for maximum pressure redistribution in the wheelchair cushion. Contact your therapist or specialist dealer to obtain a spare Ottobock cover.

4 Delivery

4.1 Scope of Delivery

The seat cushion is delivered ready for use. The scope of delivery includes:

- Seat cushion with seat base, cover and insert (see table)

Model	Insert
Terra	Without insert
Terra Aquos	LiquiCell® packs (see fig. 2, item 3)

- Instructions for Use

5 Use

5.1 Positioning

Correct positioning of the seat cushion in the wheelchair must be maintained for daily use:

- The label that says 'REAR' must be facing the back.
- The hook-and-loop strap sewn to the seat cushion serves to secure it in the wheelchair.

5.2 Installation/Removal

Removal

The foam base and foam insert may have to be removed from the cover for cleaning:

- 1) Open the zipper on the cover.
- 2) Remove the foam components as well as the inserts, if any.

Assembly (see fig. 5 – see fig. 10)

- 1) Pull the cover over the foam base with correct alignment.
- 2) Close the zip.

5.3 Cleaning

NOTICE

Improper cleaning of the wheelchair cushion

Risk of damage to cushion due to user error

- ▶ Never wash the cover at a temperature higher than the recommended as this may cause it to shrink. This could impede the cushion's ability to redistribute pressure or it could damage the fabric.
- ▶ Do not use any bleaching agents for cleaning.
- ▶ Never dry the foam components or inserts in a dryer.

INFORMATION

- ▶ Replace the cover when it shows signs of advanced wear.
- ▶ Should it be necessary to disinfect the cover, add a standard hygiene detergent during washing (e.g. Sagrotan® laundry hygiene detergent).
- ▶ In case of heavy incontinence, an additional incontinence barrier should be used. Please observe the care and cleaning instructions for each incontinence barrier used.

Cover

- 1) Unzip and remove the cover for separate washing.
- 2) Zip up the cover before washing.
- 3) The cover can be washed at 60 °C [140 °F] using a mild, environmentally friendly detergent.
Recommendation: Wash the cover using the delicate cycle at **40 °C [104 °F]** to prevent excessive wear.
- 4) Allow to air dry. When using an automatic clothes dryer, select the delicate cycle at low temperature.

6 Legal Information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

6.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as class I accessories according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

6.3 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

7 Technical data

General information	
Functional height	70 mm [3 in]
Weight (420 x 420 mm; 16 x 16 in*)	Terra: 1043 g [2.3 lbs] Terra Aquos: 1247 g [2.75 lbs]
Maximum user weight	150 kg [330 lbs]
Foam base	
Material	PUR foam
Cover	
Material	Dartex
Maximum washing temperature	60 °C
Extras	
Terra Aquos	LiquiCell® packs

* without incontinence cover

Available sizes – seat cushions

Seat width in mm [in]*	Seat depth in mm [in]*			
	355 [14]	405 [16]	455 [18]	505 [20]
355 [14]	X	X		
380 [15]		X		
405 [16]		X	X	X
430 [17]		X	X	X
455 [18]		X	X	X
505 [20]		X	X	X

* Seat width and seat depth of the wheelchair seat

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© Ottobock · 647/G780-14-1502
Otto Bock Korea HealthCare Inc.
South East Asia Co., Ltd.
Otto Bock HealthCare GmbH
Otto Bock Mobility Solutions GmbH