 Seating manual

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Sedeo Pro

Instructions for Use
Manuel d'utilisation
Gebruikershandleiding

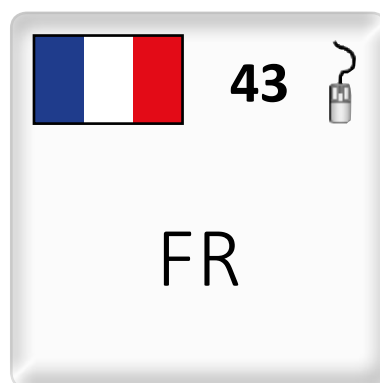
We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This products meet the requirements in accordance with EC guidelines. Options or accessories shown are available at extra cost.

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If you have any queries about the use, maintenance or safety of your seating system, please contact your local approved Sunrise Medical service agent. If you do not know of an approved dealer in your area or have any other questions please write or telephone:

Sunrise Medical

Thorns Road

Brierley Hill

West Midlands

DY5 2LD

England

Phone: 0845 605 66 88

Fax: 0845 605 66 89

www.SunriseMedical.co.uk

Dealer signature and stamp

Contents

1.0 User information.....	5
1.1 This user manual	5
1.2 For further information	5
1.3 Symbols used in this manual.....	5
2.0 Safety.....	6
2.1 Seating related symbols and labels used on the product	6
2.2 Safety: Temperature	6
2.3 Safety: Moving parts.....	6
2.4 Safety: Choking hazard	6
2.5 Safety: Trays	7
3.0 General description.....	7
4.0 Setting up the seating system.....	8
4.1 Pre-set seat depth	8
4.2 Pre-set seat width.....	9
4.3 Pre-set backrest.....	10
4.4 Pre-set Flip back armrests / Single post removable armrests.....	12
4.5 Pre-set Reclining flip back armrests	15
4.6 Side guard	16
4.7 Pre-set single post SEDEO Lite armrests	16
4.8 Pre-set center mount legrest	17
4.9 Pre-set Fixed center mount legrest.....	18
4.10 Pre-set power center mount legrest.....	19
4.11 Pre-set swing away legrests	20
4.12 Pre-set headrest	21
4.13 Pre-set fixed / swing away controller bracket.....	22
4.14 Pre-set thigh support	23
4.15 Pre-set laterals.....	24
4.16 Pre-set lateral swing away knee support	25
4.17 Pre-set pommel	25
4.18 Lap strap / seating positioning belt.....	26
5.0 Using the seating system.....	27
5.1 Getting in and out of the wheelchair (Transferring)	27
5.2 Preparing for forward transfer	27
5.3 Preparing for side transfer	28
5.4 Headrest	29
5.5 Swing-away controller bracket.....	29
5.6 Laterals (Fig 5.11)	29
5.7 Knee support (Fig 5.12).....	30
5.8 Thigh support (Fig. 5.13)	30
5.9 Hemiplegia armrest (Fig 5.14).....	30
5.10 Push handle height (Fig 5.15).....	30
5.11 Pommel (Fig 5.16)	30
6.0 Manual and Powered seating options	31
6.1 Manual seating options	31
6.2 (Powered) seating options	32
7.0 Reducing the seating system size for transportation purposes	33
7.1 Seat cushion	33
7.2 Removing the back cushion (Fig 7.3)	33
7.3 Flip down backrest quick release (Fig 7.4).....	34
7.4 Folding down backrest with reclining armrests (Fig 7.5).....	34
7.5 Folding down reclining backrest with reclining armrests (Fig 7.6).....	34
7.6 Swing away legrests (Fig 7.7).....	35
7.7 Single post removable armrest	35
8.0 Maintenance and cleaning	36
8.1 General Maintenance.....	36
8.2 Cleaning your seating system.....	36
8.3 Hygiene measures when being re-used:.....	36
8.4 General cleaning instructions:	37
8.5 Cleaning Accessories:	37
8.6 Cleaning controls:.....	37
8.7 Storage.....	37
9.0 Disposal	38
10.0 Technical specifications:	39
10.1 Applicable norms / standards	39
10.2 Technical specifications: SEDEO Pro seating system	40
11.0 Guarantee / Warranty.....	42

1.0 User information

Thank you for choosing a Sunrise Medical wheelchair. Sunrise Medical's high-quality mobility products are designed to enhance independence and make your everyday life easier.

1.1 This user manual

This user manual will help you to use and maintain your seating system safely. This seating manual is a supplement to Sunrise Medical's general wheelchair user manual:

- The general wheelchair user manual
- The user manual for the seating system (this booklet)
- The user manual for the controller.

When necessary this Seating user manual refers to other manuals as shown below:

- | | |
|---|---|
|  General manual: | Refers to the general user manual for the wheelchair. |
|  Controller: | Refers to the user manual for the controller. |
|  Battery charger: | Refers to the user manual for the battery charger. |

Do not use your wheelchair until this entire manual (and all other booklets) has been read and understood!

If one of the user manuals was not included with your wheelchair, please contact your dealer immediately. In addition to this user manual, there is also a service manual for qualified specialists.

1.2 For further information

Please contact your local, authorized Sunrise Medical dealer if you have any questions regarding the use, maintenance or safety of your wheelchair. In case there is no authorized dealer in your area or you have any questions, contact Sunrise Medical either in writing or by telephone.

For information about product safety notices and product recalls, go to www.SunriseMedical.eu

1.3 Symbols used in this manual

 **DANGER/WARNING!**


Potential risk of injury or serious injury or death

 **NOTE/CAUTION!**

Potential damage to equipment.

Not following these instructions may result in physical injury, damage to the product or damage to the environment!

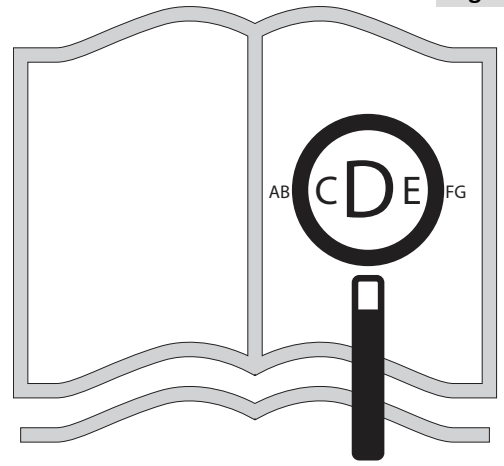Fig. 1.1

If you are visually impaired,
this document can be viewed
in PDF format at
www.SunriseMedical.com

or alternatively is available on
request in large text.

2.0 Safety

Follow the instructions carefully next to these warning symbols! Not paying careful attention to these instructions could result in physical injury or damage to the wheelchair or the environment. Wherever possible, safety information is provided in the relevant chapter.

2.1 Seating related symbols and labels used on the product

The signs, symbols and instructions affixed to the wheelchair comprise part of the safety facilities. They must never be covered or removed. They must remain present and clearly legible throughout the entire lifespan of the wheelchair.

Replace or repair all illegible or damaged signs, symbols and instructions immediately. Please contact your dealer for assistance.

Fig 2.1: Warning – Danger of finger entrapment

Fig 2.2: Warning – Ensure that the seat interface quick release is screwed in tight before using your wheelchair, especially on a slope

Fig 2.3: Warning – Do not drive your wheelchair on a slope with the backrest reclined and/or the seating in standing mode

2.2 Safety: Temperature

DANGER! / WARNING!

- Avoid physical contact with the wheelchair's motors at all times. Motors are continuously in motion during use and can reach high temperatures. After use, the motors will cool down slowly. Physical contact could cause burns. Allow the motors after using at least 30 min. to cool down.
- If you do not use the wheelchair, ensure that it is not exposed to direct sunlight for lengthy periods of time. Certain parts of the wheelchair, such as the seat, the back and the armrests can become hot if they have been exposed to full sunlight for too long. This may cause burns or allergic reactions to the skin.

2.3 Safety: Moving parts

DANGER! / WARNING!

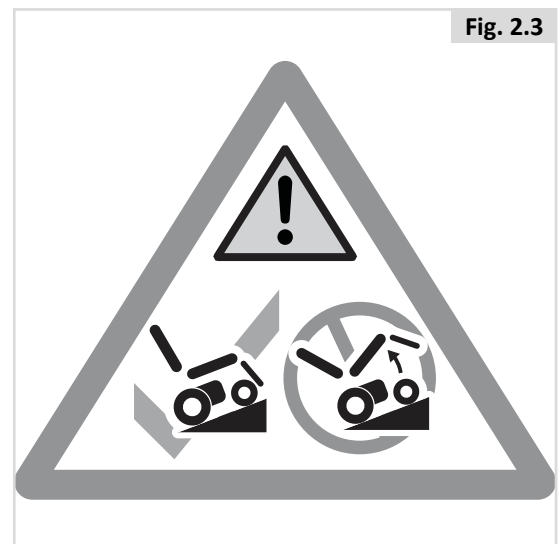
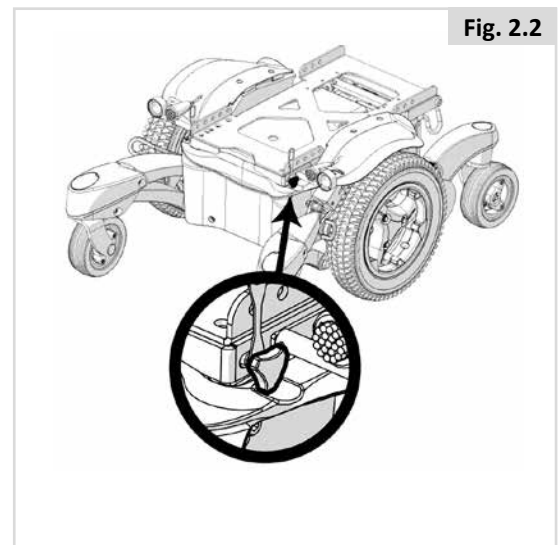
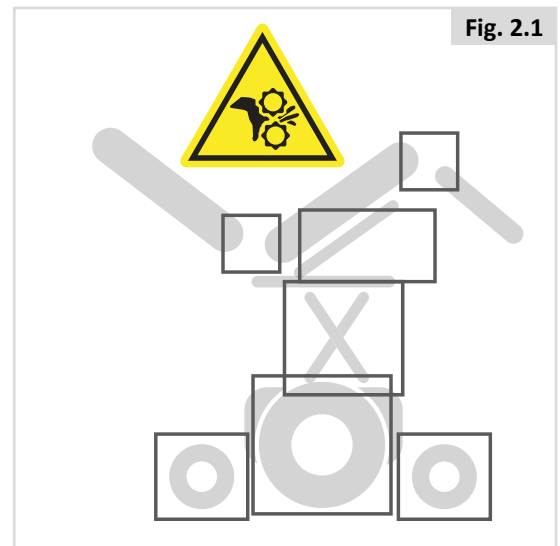
A wheelchair has moving and rotating parts. Contact with moving parts may result in serious physical injury or damage to the wheelchair. Contact with the moving parts of the wheelchair should be avoided.

- Wheels (turning and castor)
- Powered tilt in space adjustment
- Powered seat elevator option
- Powered recline adjustment
- Swing-away bracket of the controller

2.4 Safety: Choking hazard


DANGER! / WARNING!

This mobility aid uses small parts which under certain circumstances may present a choking hazard to young children.

2.5 Safety: Trays

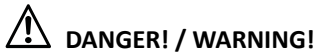Safety: Vent tray


DANGER! / WARNING!

Using a vent tray will affect the stability and overall weight of your wheelchair.

- Make sure that the batteries and the ventilator are securely fastened to the tray before use.
- Familiarise yourself with the increased size of the wheelchair before driving to prevent potential collisions.
- When activating the recline backrest make sure that the area around the vent tray is clear.
- Be aware that the vent tray assembly can cause a finger trap hazard when the backrest is reclined.
- The vent tray is not designed to hold anything other than the vent unit and its batteries, it is not designed to hold an oxygen bottle etc. Use of these items may cause your wheelchair to become unstable.
- The vent tray must be installed and maintained by a Sunrise Medical authorised dealer.

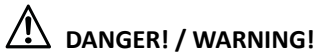Safety: Swing away tray:

DANGER! / WARNING!

- The maximum weight allowed for the tray is 2.5kg.
- Do not overload the tray, this could cause the tray to break or could cause the chair to become unstable.
- Do not leave lit cigarettes or other heat sources on the tray as this could cause the tray to deform and mark.
- Ensure that all extremities and clothing are clear when positioning the tray for use.

3.0 General description

This wheelchair is equipped with Sedeo Pro seating system. The Sedeo Pro seating system provides excellent support for the head, torso, arms and legs. Optimal body support ensures maximum independence for the wheelchair user.

The wheelchair must be adjusted by a qualified specialist. Incorrect settings may have negative consequences for the seating posture.

Main components (Fig. 3.1):

1. Headrest
2. Joystick
3. Armrest
4. Seat cushion
5. Leg rest
6. Backrest cushion

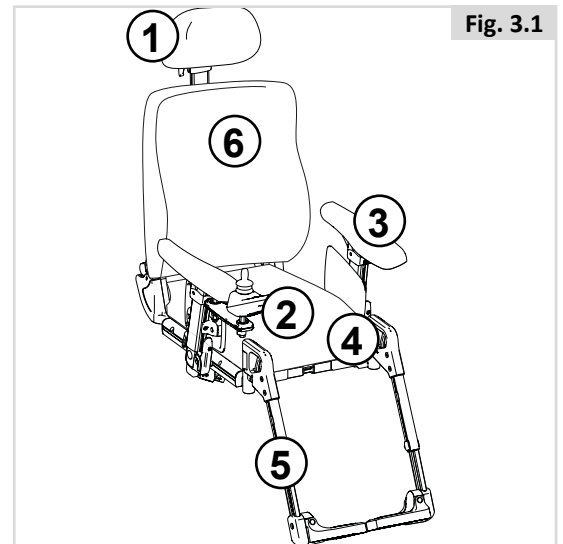
Fig. 3.1

4.0 Setting up the seating system

The average wheelchair user does not exist. This is why Sunrise Medical wheelchairs can be pre-set according to specific needs of the user.

Pre-settings are carried out once, the first time the wheelchair is going to be used. They must be executed by qualified specialist, unless explicitly stated otherwise.

Tool and calibration marks

A M5 Allen key is supplied with the seating system. This can be used by a professional for adjusting all of the seating system settings. The M5 Allen key (A) can be found on the frame at the location indicated in figure 4.1.

⚠ NOTE / CAUTION!

There are calibration marks on various parts of the seating system. The ex-factory settings are indicated.

4.1 Pre-set seat depth

Backrest frame depth (Fig 4.2):

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

⚠ NOTE / CAUTION!

- When a bigger seat depth is required ensure that the cables are lengthened before the mechanical adjustment takes place.
- There are 2 sizes seat rail/seat plates available: 400-520 and 460-560 mm. Choose the right seat rail to match the adjusted seat depth.
- Mount extra seat depth mounting plate from SD 530 mm for more stability of the long seat rail Fig 4.3.

Seat cushion depth (Fig 4.4)

For the standard seat cushion position use the holes in Fig 4.4. The cushion can be positioned one position more to the front to create extra seat depth.

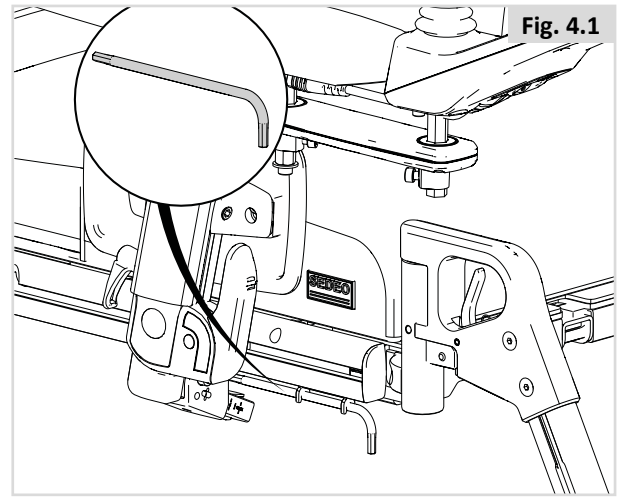
Fig. 4.1

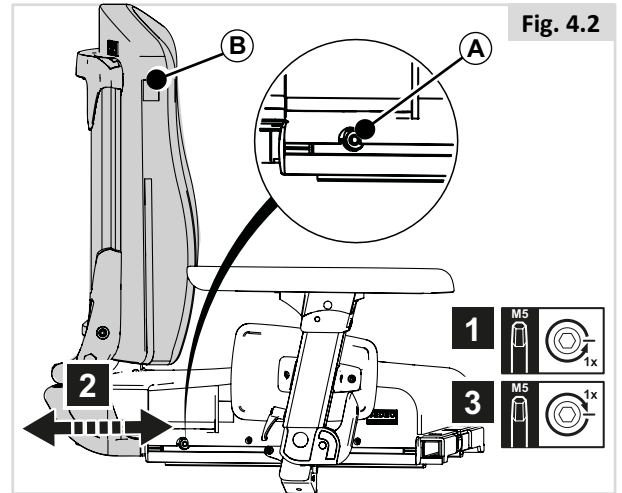
Fig. 4.2

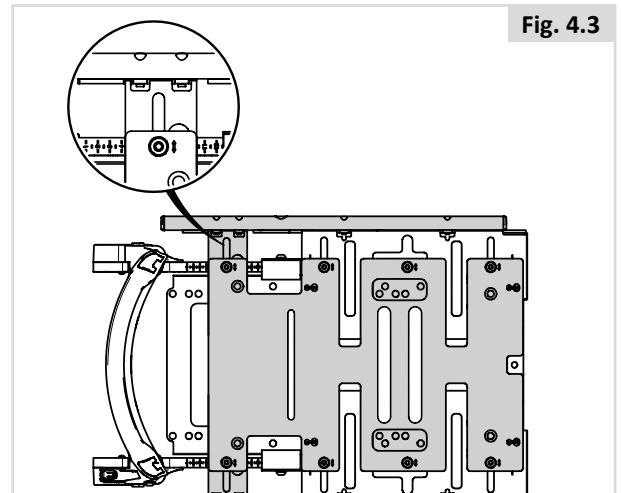
Fig. 4.3

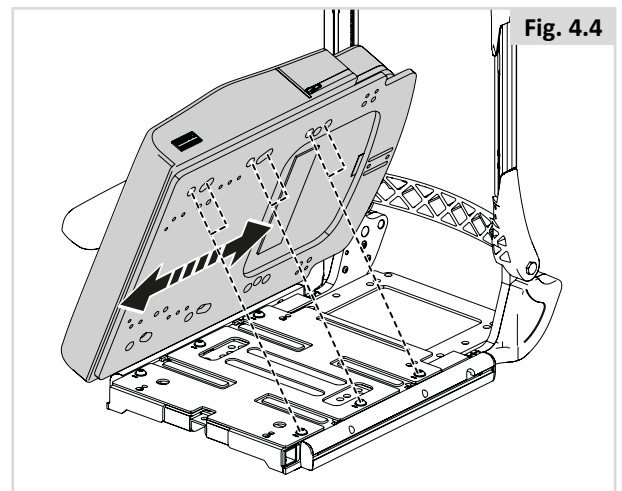
Fig. 4.4

4.2 Pre-set seat width

Flip back armrests / Single Post Removable armrests width (Fig. 4.5)

Fine adjustments can be done with the armrest only, for larger adjustments also pre-set the seat plate width.

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

NOTE / CAUTION!

- Please check that the new arm rest position does not interfere with any seat positioning.

Reclining flip back armrest width (Fig. 4.6)

1. Remove 2 (A) screws
2. Adjust (B) to desired position
3. Tighten (A)

NOTE / CAUTION!

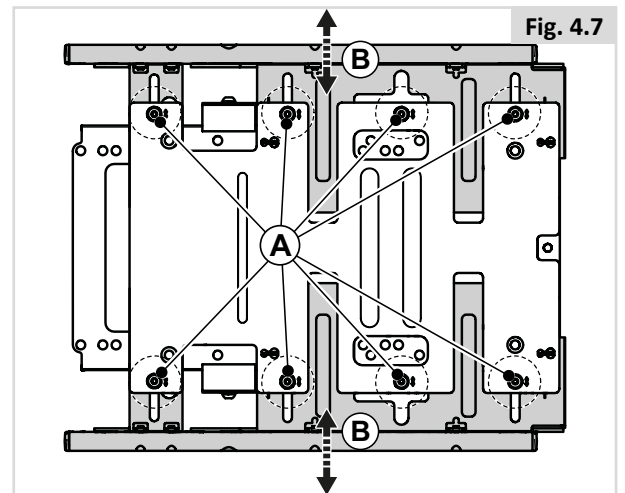
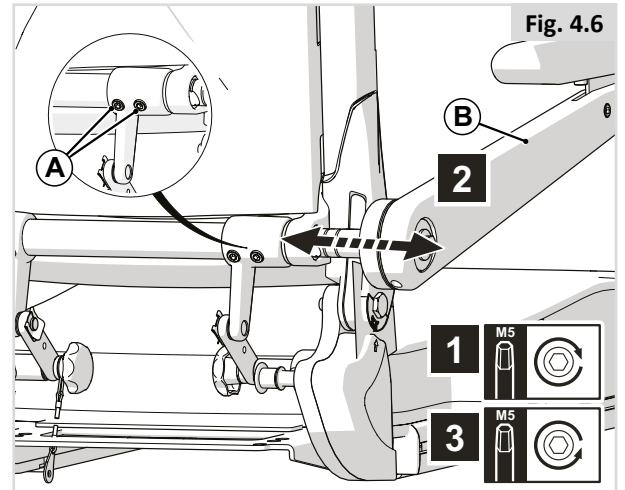
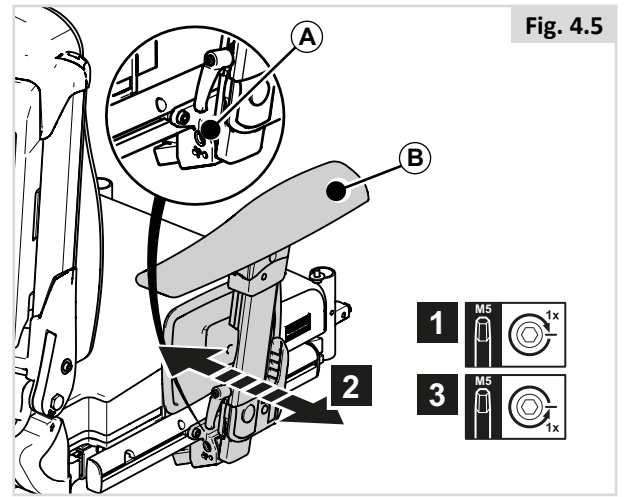
- Please check that the new armrest position does not interfere with any seat positioning. If cables are attached, check the length.

Seat plate width (Fig. 4.7)

Lift the seat cushion off the Velcro® strips to access the seat plates.

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

The seat width is indicated by the scale that runs alongside each slot.


4.3 Pre-set backrest

Backrest angle (Fig 4.8)

1. Turn (A) in order to adjust (B).

⚠ NOTE / CAUTION!

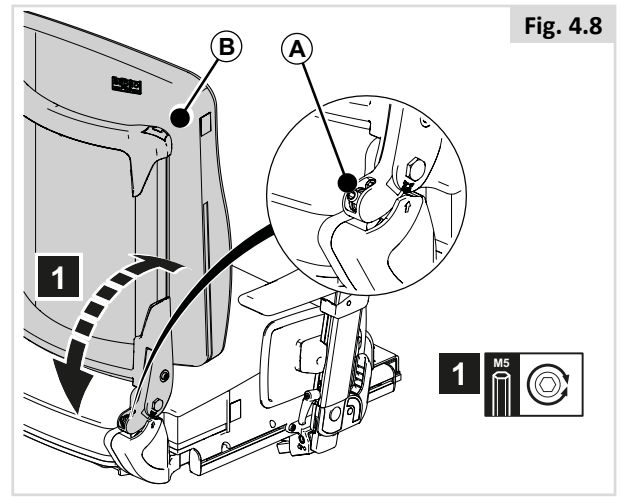Do NOT use an electric tool for this adjustment.


Backrest angle home position of reclining back (Fig.4.9 - 4.10)

1. Remove bolts (A+B) and rotate (C).
2. Remove cover (D)
3. Remove 2 screws to remove actuator cover (E).
4. Loosen (F) by one turn.
5. Adjust to the desired position.
6. Tighten (F).



⚠ NOTE / CAUTION!

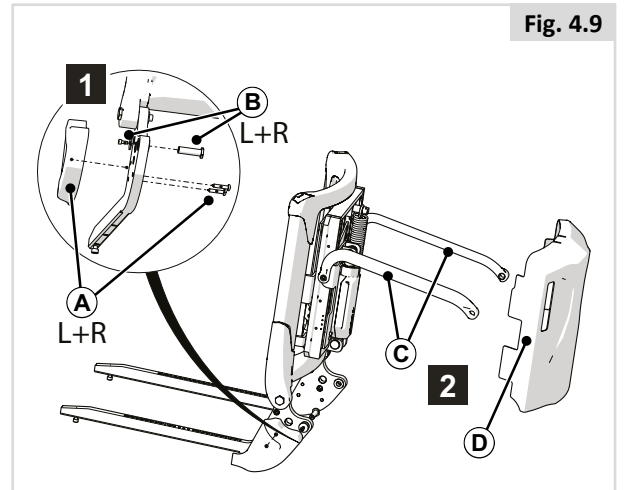After removing bolt (B) the backrest must be held, otherwise it would fall forwards or backwards.

Backrest height with fixed back (Fig 4.11)

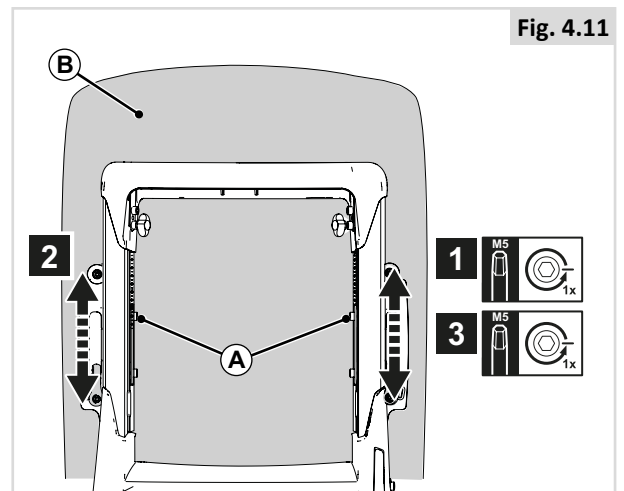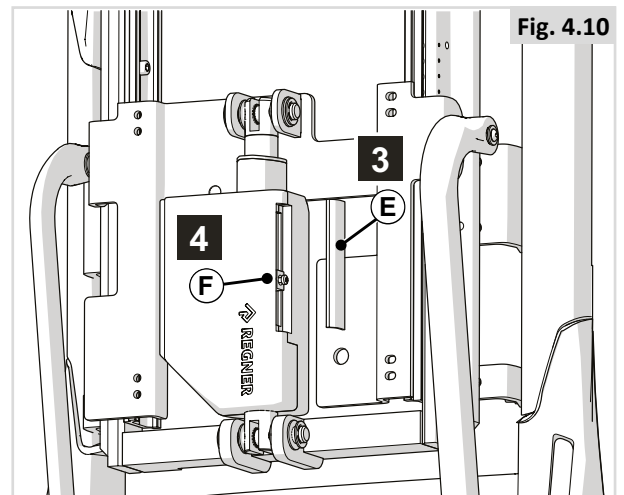1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

Backrest height with reclining back (Fig 4.12)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

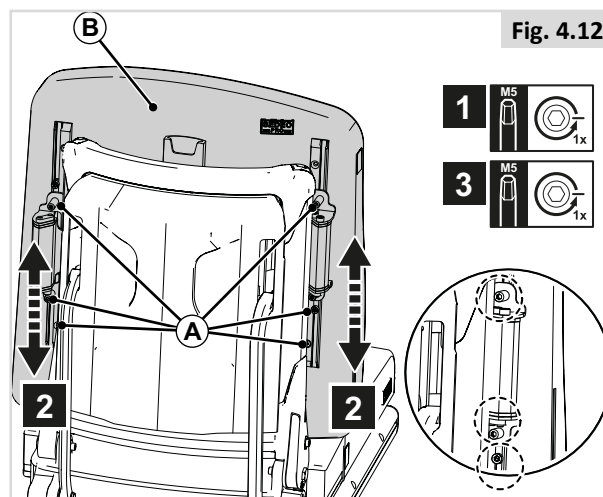
Fig. 4.12

Backrest Flexible lateral support angle (Fig 4.13)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

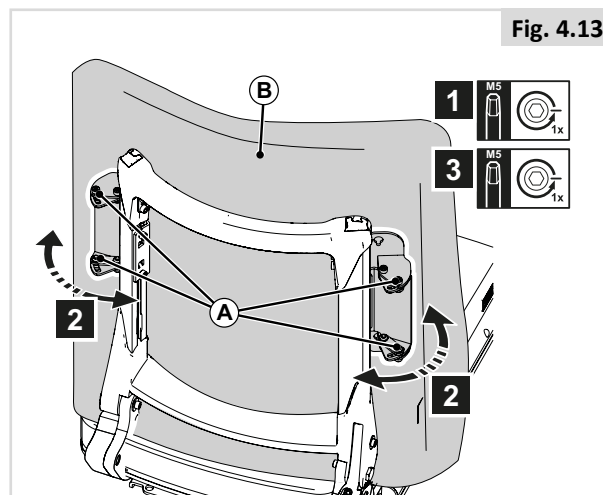
Fig. 4.13

Backrest Flexible curvature (Fig 4.14)

The wheelchair has a padded backrest that can be adjusted to the curvature of the back in order to achieve a comfortable sitting position and good stability. The user can sit in the wheelchair when adjustments are being made to the curvature of the backrest.

1. Open zipper
2. Press A to release strap
3. Release or tighten each strap individually
4. Release A to keep the setting of the strap

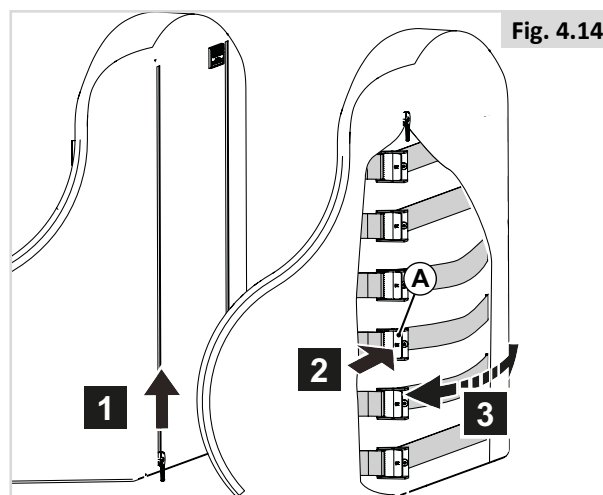
Fig. 4.14

J3 Back (Fig 4.15)

 NOTE: The Jay 3 Backrest can be used in conjunction with the Sedeo Pro seating system. Please refer to the J3 Back Owner's Manual for fitting and adjustment.

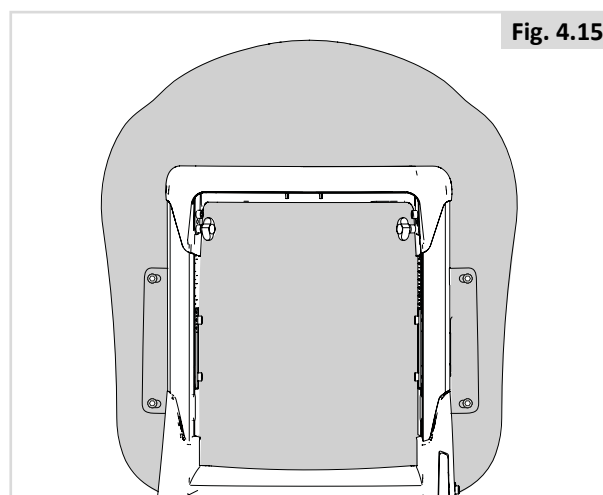
Fig. 4.15

4.4 Pre-set Flip back armrests / Single post removable armrests

Armrest height (figure 4.16)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

Ensure that the arms are supported well, without the shoulders being pushed upwards.

Armrest depth (figure 4.17)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

Position the Flip back armrest further back in order to obtain more space for side wards transfers. Position the armrest further to the front to obtain more support for forward transfers.

Armrest pad depth (figure 4.18)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

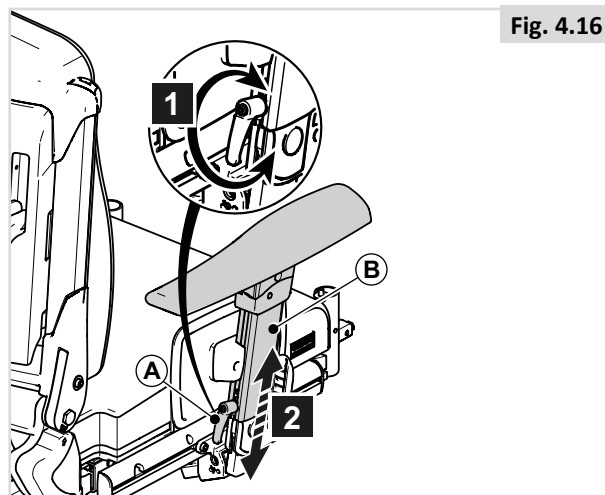
Fig. 4.16

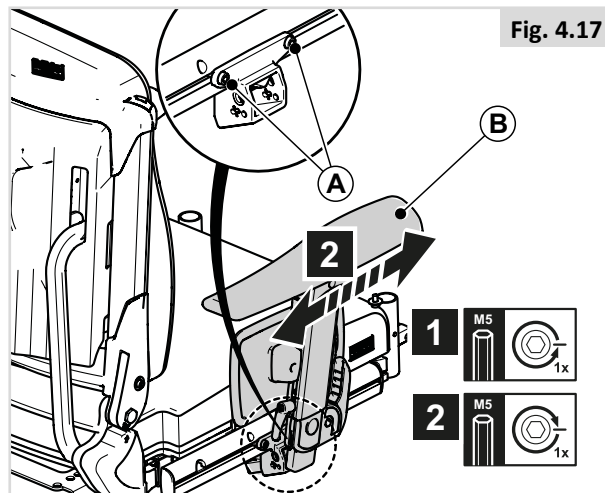
Fig. 4.17

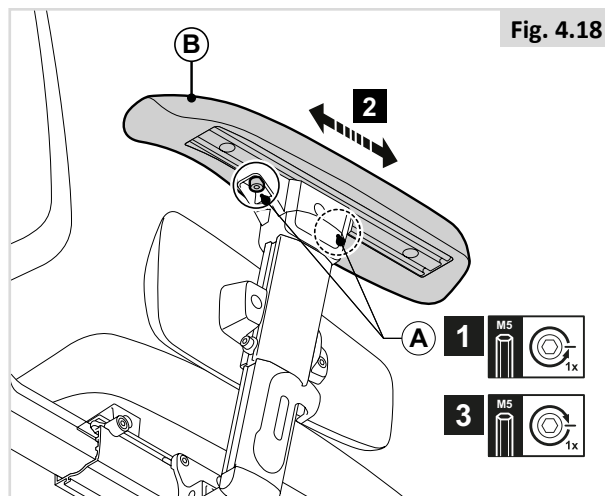
Fig. 4.18

Armrest pad width (figure 4.19-4.22)

To remove:

1. Loosen (A).
2. Remove (B).
3. Loosen (E) by one turn.
4. Remove (F).

To position:

1. Slide (A) together with B or C onto D

⚠ NOTE / CAUTION!

Ensure that the ridge on the terminal block falls into the opening

1. Adjust the depth of (A) and tighten (E).
2. Position the controller bracket.
3. Position the armrest.

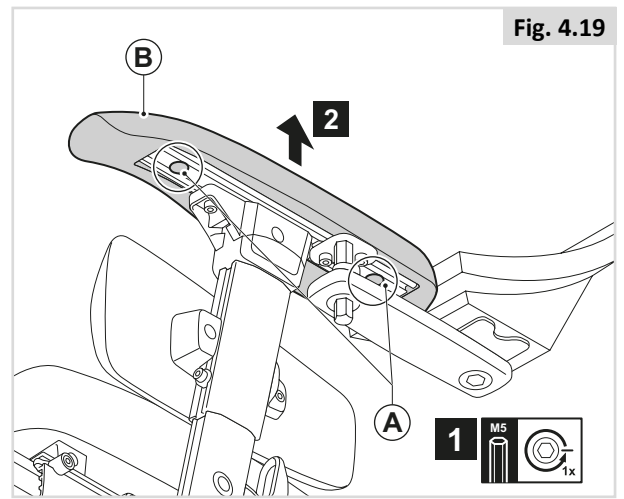
Fig. 4.19

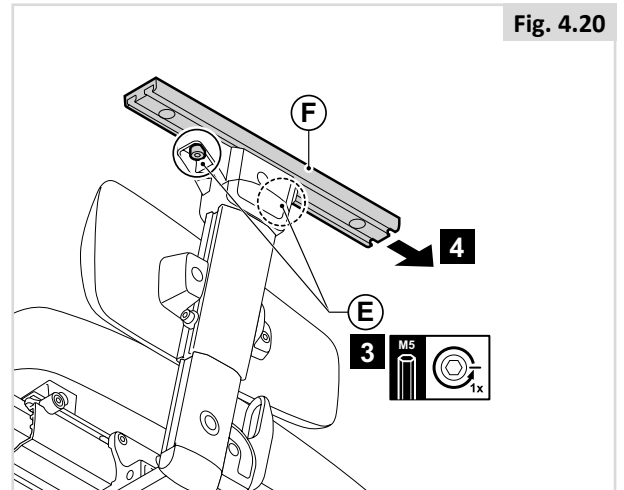
Fig. 4.20

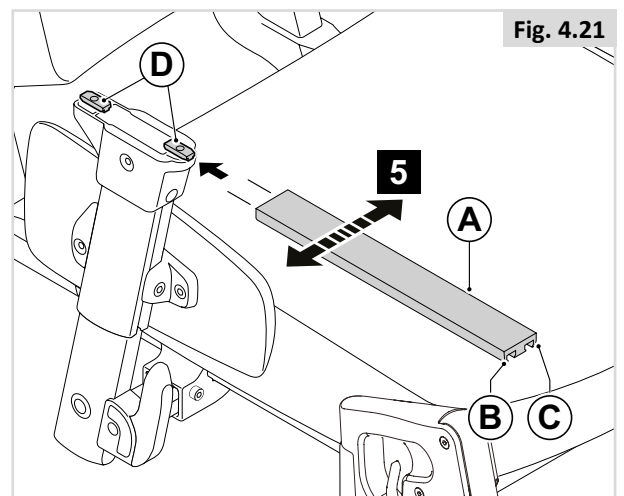
Fig. 4.21



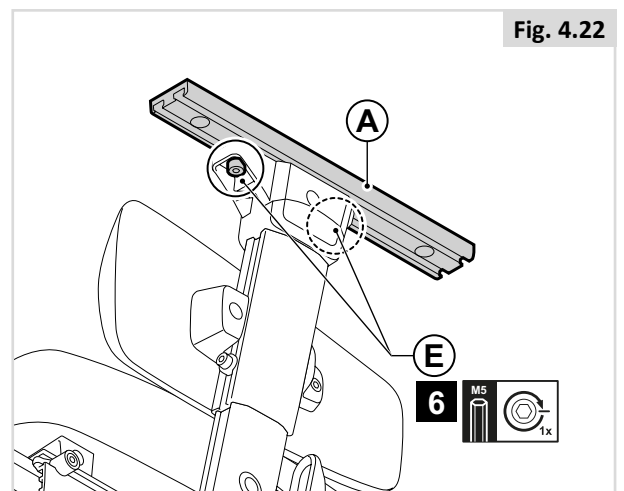
Fig. 4.22

Armrest angle (figure 4.23)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

Side guard height (figure 4.24)

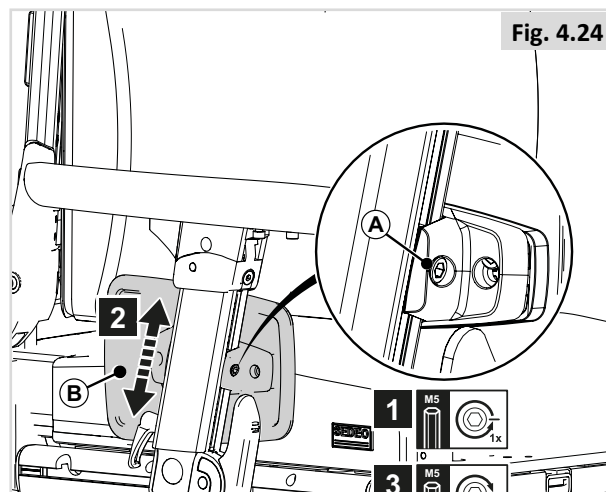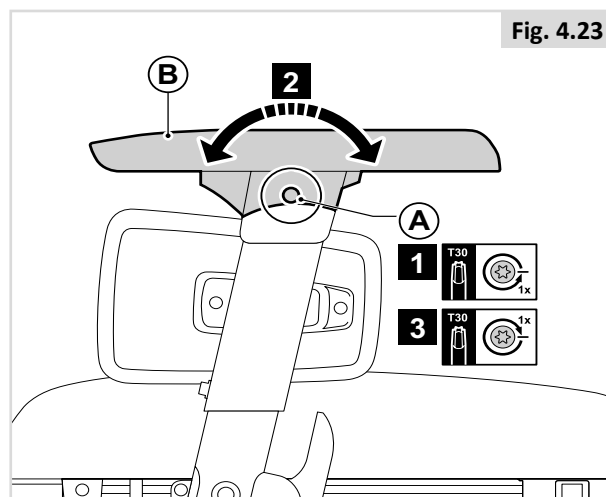1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

4.5 Pre-set Reclining flip back armrests

Armrest height

Fine or gross adjustment it is a two-part process:

PART 1 (Fig. 4.25)

1. Turn (A) in order to adjust (B).

PART 2 (Fig. 4.26)

1. Loosen (A) by one turn
2. Adjust (B) to the desired position.
3. Tighten both screws again

Armrest pad depth (Fig.4.27)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

NOTE

In order to reach the rear screw in lower armrest heights, loosen screws (A) of Fig 4.24 and adjust the angle of the pad first.

 **DANGER! / WARNING!**

All screws should be checked periodically for tightness. A loose arm pad could slip down when pressed.

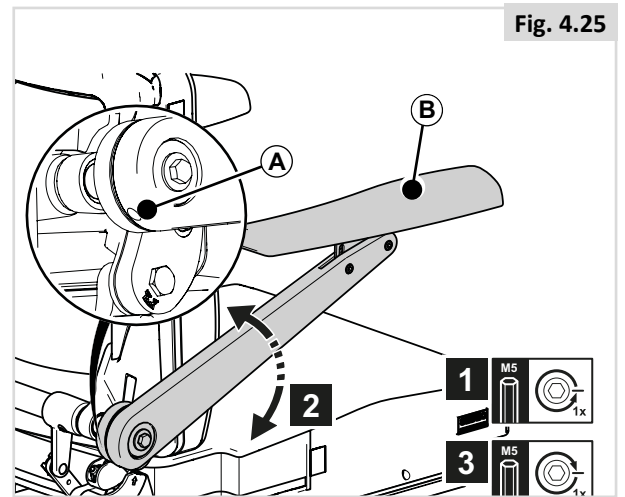
Fig. 4.25

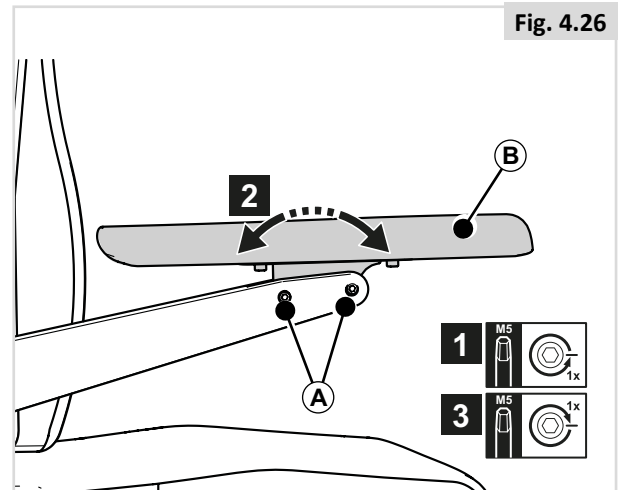
Fig. 4.26

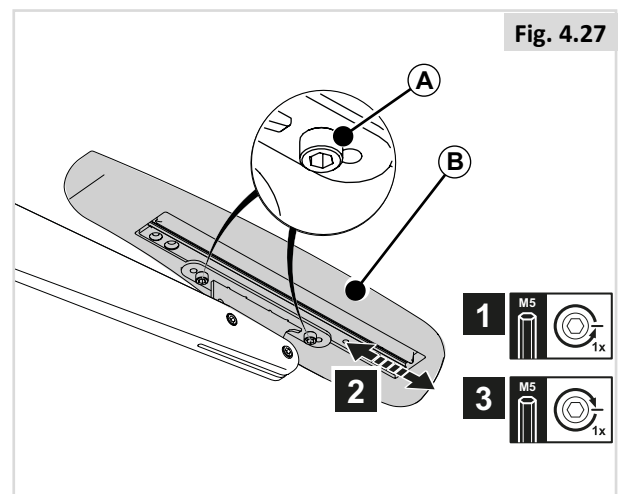
Fig. 4.27

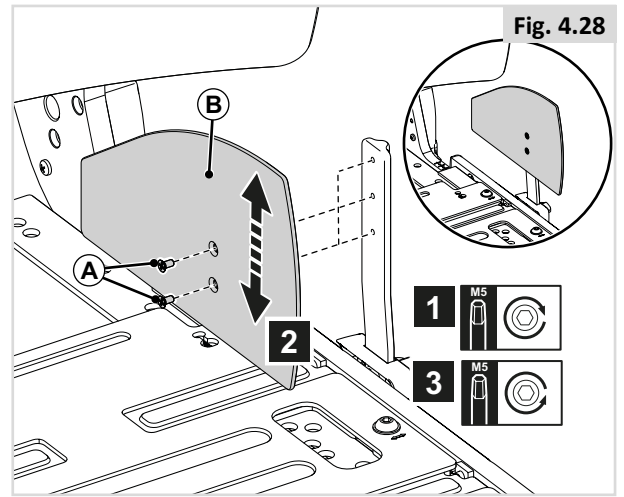
4.6 Side guard

Side guard height (Fig 4.28)

1. Unscrew (A).
2. Adjust (B) to the desired position.
3. Tighten (A).

Side guard depth (Fig 4.29)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).




4.7 Pre-set single post SEDEO Lite armrests

Armrest height (Fig 4.30)

4. Loosen (A) by one turn.
5. Adjust (B) to the desired position.
6. Tighten (A).

Armrest depth (Fig 4.31)

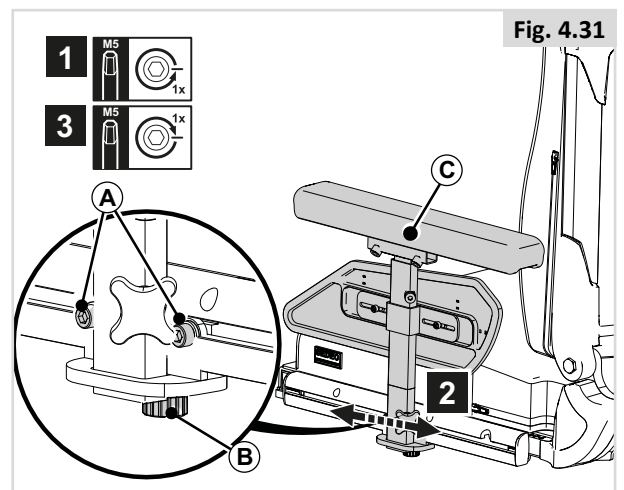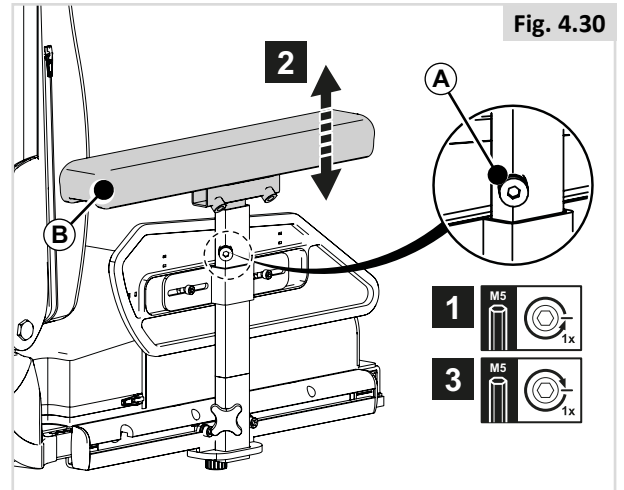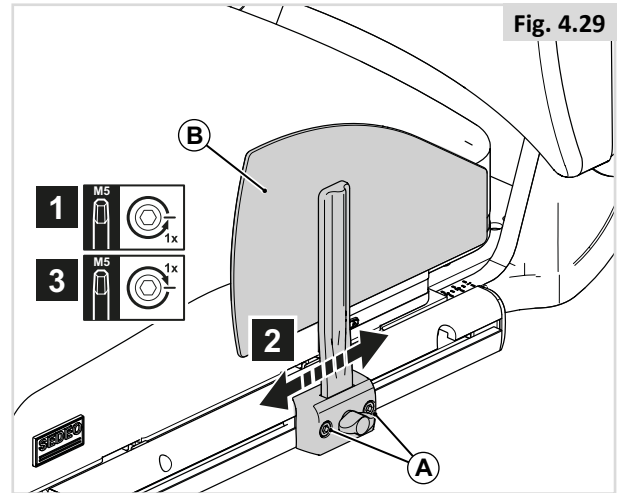1. Loosen (A) by one turn.
2. Loosen (B)
3. Adjust (C) to the desired position.
4. Tighten (A) and (B).

4.8 Pre-set center mount legrest

Lower leg length (Fig 4.32)

5. Loosen (A) by one turn.
6. Adjust (B) to the desired position.
7. Tighten (A).

 **NOTE / CAUTION!**

Ensure that there is sufficient free space under (B) for the wheelchair to drive over obstacles.

Knee angle (Fig 4.33)

1. Loosen (A) by two turns.
2. Adjust (B) to the desired position.
3. Tighten (A).

 **NOTE / CAUTION!**

Change the height or knee angle if the footplate touches the castor wheels.

Ankle angle (Fig 4.34)

1. Flip up (A).
2. Turn (B) in order to adjust the ankle angle as desired.
3. Fold (A) down.

Central calf support height (Fig 4.35)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

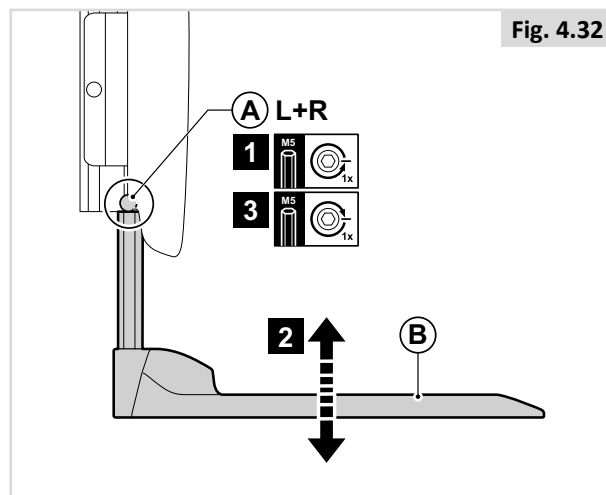
Fig. 4.32

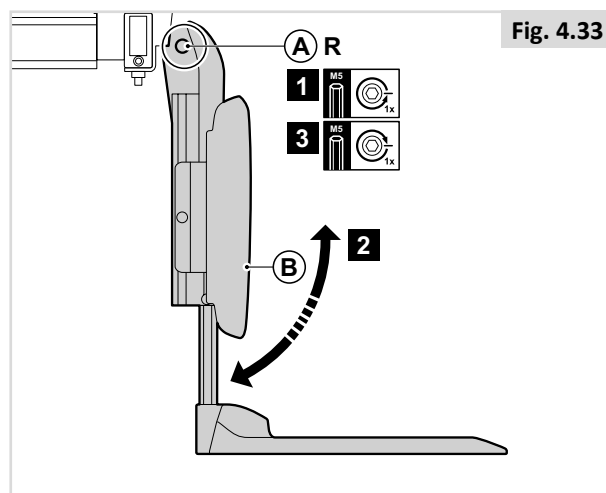
Fig. 4.33

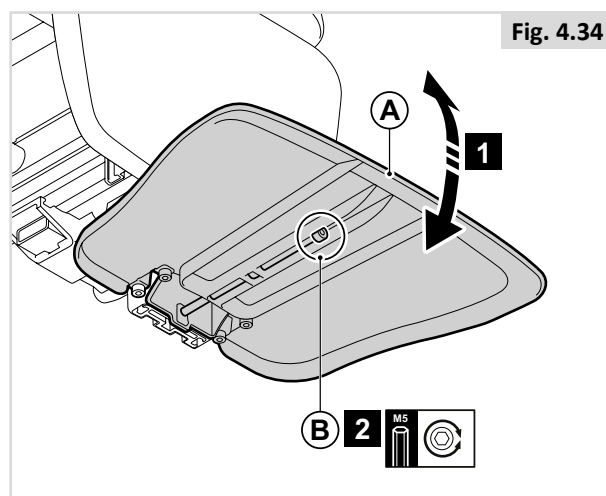
Fig. 4.34


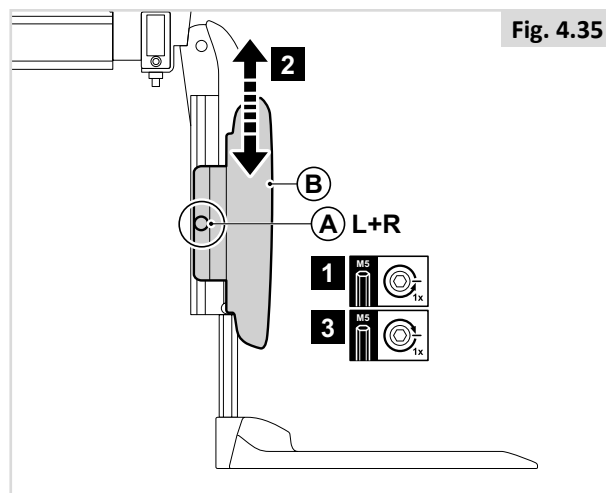
Fig. 4.35

4.9 Pre-set Fixed center mount legrest


Lower leg length (Fig 4.36)

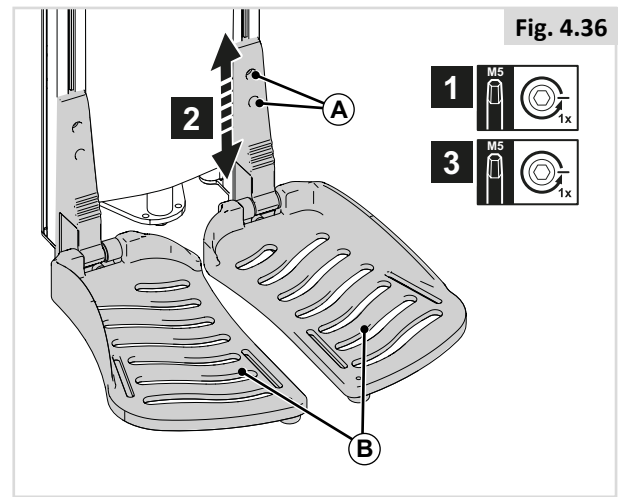1. Loosen (A) by one turn
2. Adjust (B) to the desired position
3. Tighten (A)
4. Repeat for the other side.

Ankle angle (Fig 4.37)

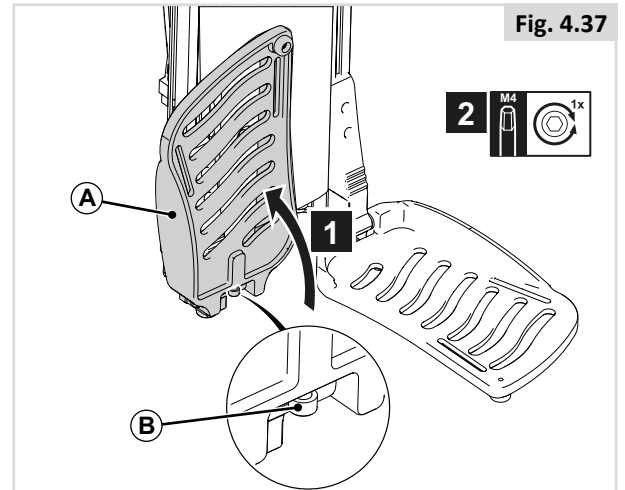1. Flip up (A).
2. Turn (B) in order to adjust the ankle angle as desired (4 mm Allen key)
3. Fold (A) down.

DANGER! / WARNING!

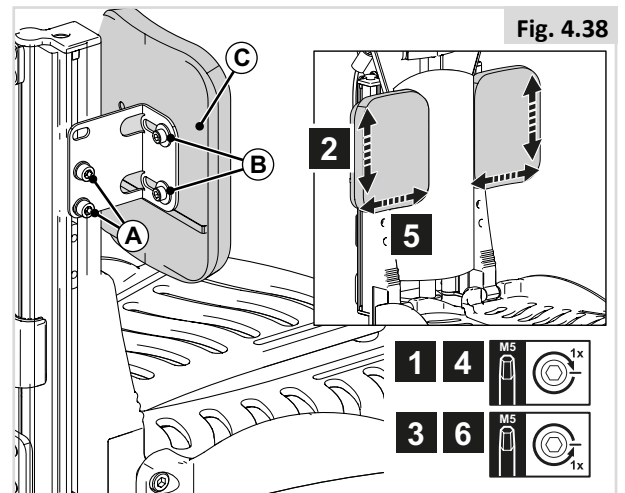
- After adjustment and with the seat in its normal drive position: Ensure there is a gap of (>25 mm.), between the floor and the footplate at all times.
- Perform this test with the user sitting in the chair loading the legrest
- Using the chair with the legrest set too low might result in personal injury or damage to the wheelchair and its surroundings.

Calf pad height (Fig 4.38)

1. Loosen (A) by one turn.
2. Adjust (C) to the desired position.
3. Tighten (A).

Calf pad width (Fig 4.38)

4. Loosen (B) by one turn.
5. Adjust (C) to the desired position.
6. Tighten (B).

4.10 Pre-set power center mount legrest

⚠ DANGER! / WARNING!

- Be aware of your environment to make sure you do not injure your legs when Leg Rests are extended.
- Always ensure that the Leg Rests or foot plates do not come into contact with the castors before driving the wheelchair.
- Leg Rests are not to be used for lifting or carrying the wheelchair under any circumstances.
- As with all moving parts be careful not to trap your fingers, (Fig.4.39).

Lower leg length (Fig 4.40)

The power centre mount legrest allows angle adjustment from 90° to 180° level to ground and a 180 mm powered length extension.

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).
4. Repeat for the other side.

⚠ NOTE / CAUTION!

In case of optional 'Footplate to the floor' function (Fig. 4.41), a switch is mounted. The switch (A) needs to be adjusted in height after adjusting the lower leg length. Make sure the ball of the switch is completely under the lowest edge of the footrest (B).

Ankle angle (Fig 4.42)

1. Flip up (A).
2. Turn (B) in order to adjust the ankle angle as desired (4 mm Allen key)
3. Fold (A) down.

⚠ DANGER! / WARNING!

- After adjustment and with the seat in its normal drive position: Ensure there is a gap of, (>25 mm.), between the floor and the footplate at all times.
- Perform this test with the user sitting in the chair loading the leg rest.

Using the chair with the leg rest set too low might result in personal injury or damage to the wheelchair and its surroundings.

Calf pad height (Fig 4.43)

1. Loosen (A) by one turn.
2. Adjust (C) to the desired position.
3. Tighten (A).

Calf pad width (Fig 4.43)

4. Loosen (B) by one turn.
5. Adjust (C) to the desired position.
6. Tighten (B).

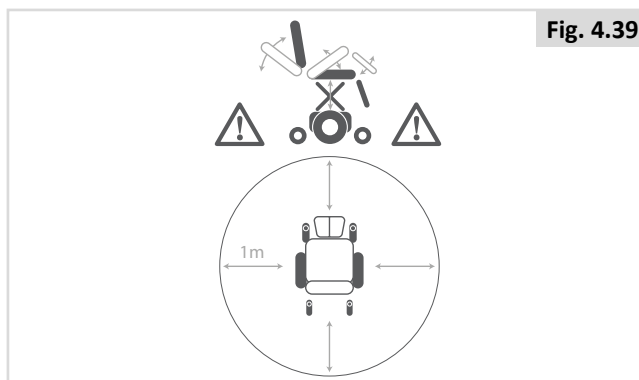
Fig. 4.39

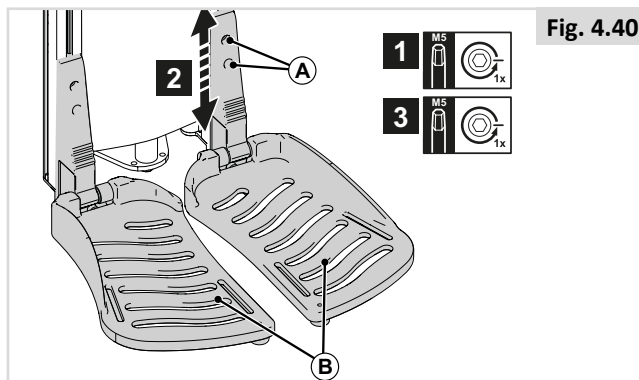
Fig. 4.40

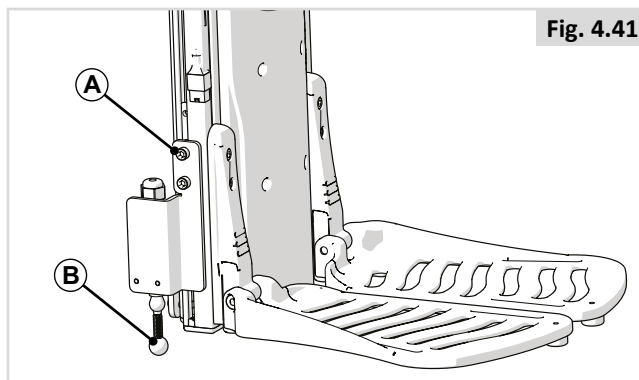
Fig. 4.41

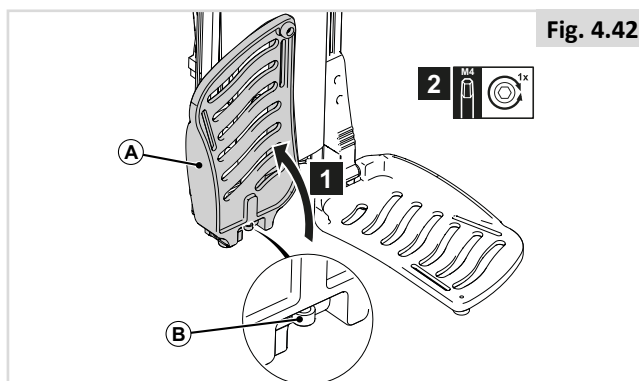
Fig. 4.42

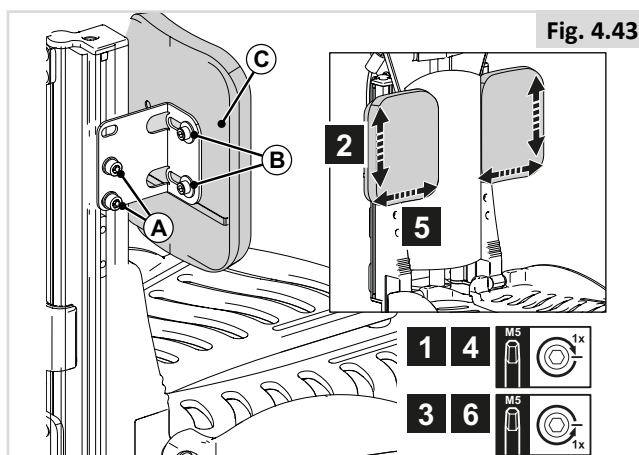
Fig. 4.43

4.11 Pre-set swing away legrests

Legrest width (Fig 4.44)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

Lower leg length (Fig 4.45)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

⚠ NOTE / CAUTION!

Ensure that there is sufficient free space under (B) for the wheelchair to drive over obstacles.

Ankle angle (Fig 4.46)

1. Loosen (A) a few turns.
2. Adjust (B) to the desired position.
3. Tighten (A).

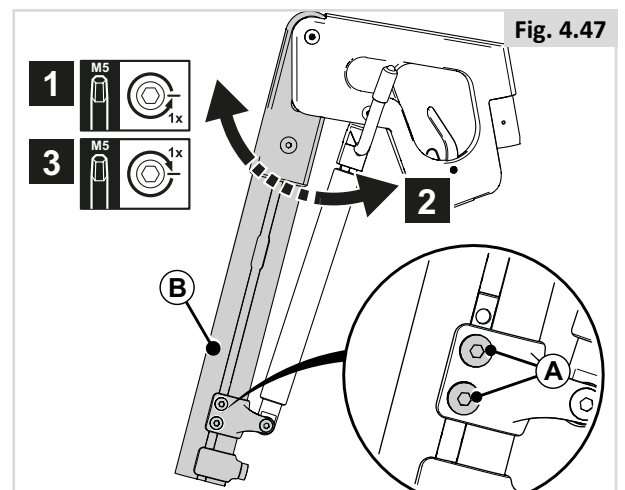
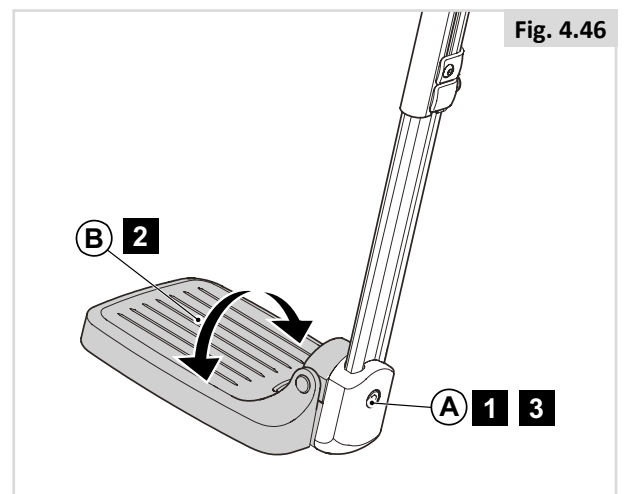
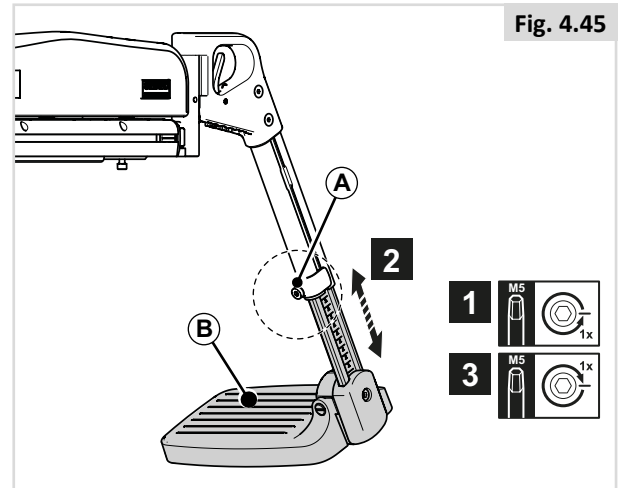
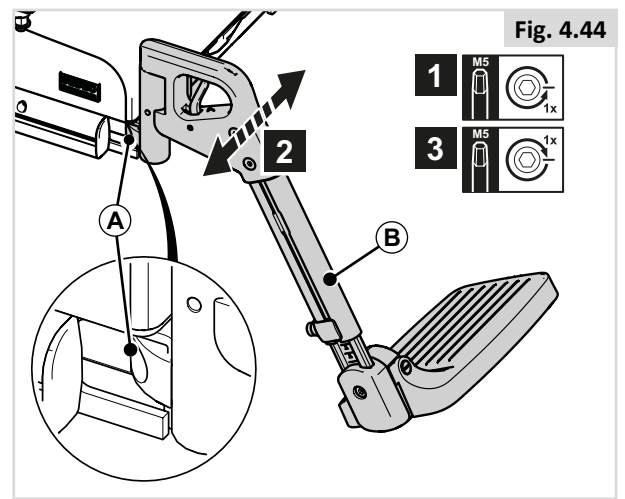
Knee angle manual/power elevating legrest (Fig 4.47)

The position of the footplates can be changed by adjusting the knee angle.

1. Loosen (A) a few turns.
2. Adjust (B) to the desired position.
3. Tighten (A).

⚠ NOTE / CAUTION!

Change the height or knee angle if the footplate touches the castor wheels.

Calf pad height (Fig 4.48)

1. Loosen (A) a few turns.
2. Adjust (D) to the desired position.
3. Tighten (A).

Calf pad depth (Fig 4.48)

4. Loosen (B) a few turns
5. Adjust (D) to the desired position.
6. Tighten (B).

Calf pad width (Fig 4.48)

7. Loosen (C) on the back of (D) a few turns.
8. Adjust (D) to the desired position.
9. Tighten (C).

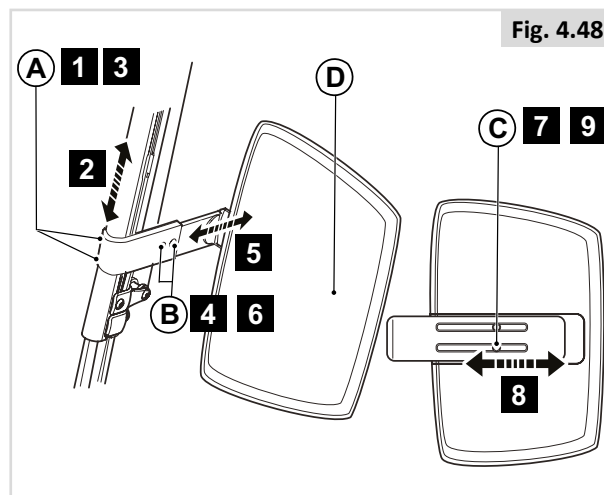
Fig. 4.48

4.12 Pre-set headrest

⚠ DANGER! / WARNING!

- If a headrest is part of the options selected for the user/chair, never drive the chair without the headrest attached.
- Never use the chair without a headrest if a person is transported with the chair in a vehicle.

Headrest height (Fig 4.49)

1. Loosen (A) a few turns.
2. Adjust (B) to the desired position.
3. Tighten (A).

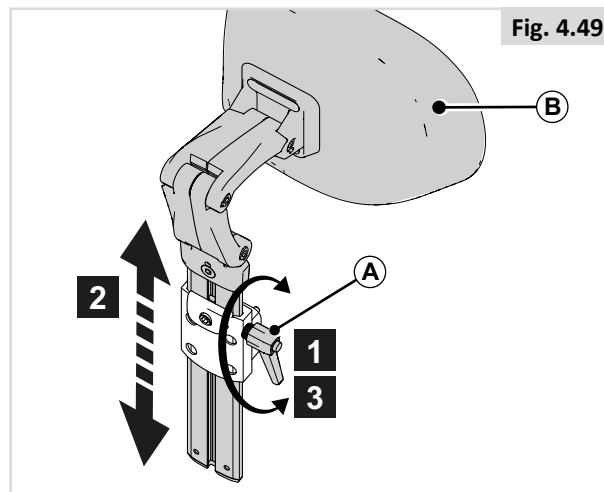
Fig. 4.49

Memory setting (Fig 4.50)

1. Loosen (C) by one turn.
2. Slide (D) as far down as possible.
3. Tighten (C).

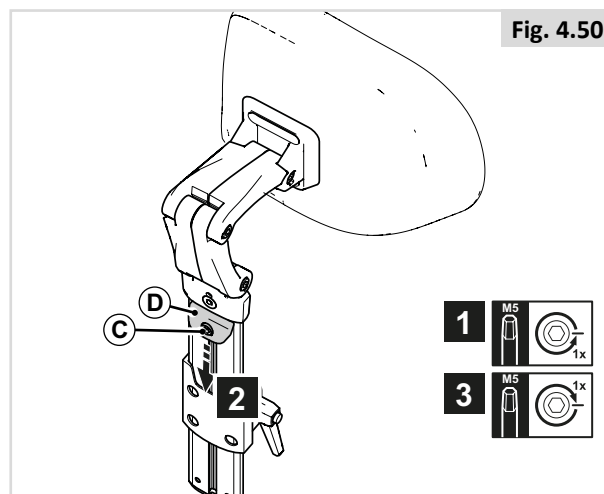
Fig. 4.50

Headrest position (Fig 4.51)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

⚠ NOTE / CAUTION!

Correct the headrest height and tilt the headrest cushion as required.


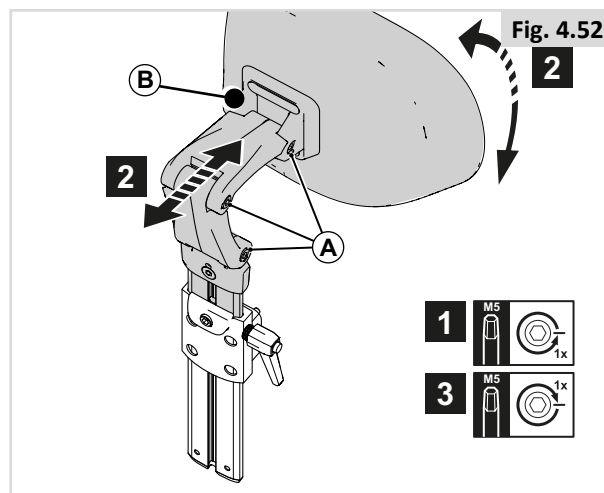
Fig. 4.52

Headrest cushion width position (Fig 4.53)

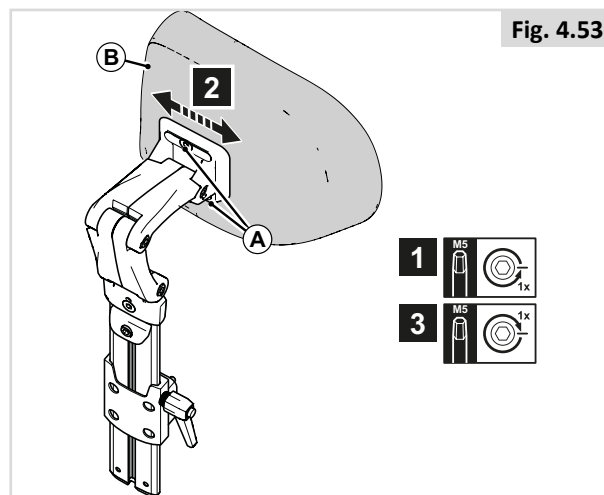
1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

4.13 Pre-set fixed / swing away controller bracket


Depth of the controller (Fig 4.54)

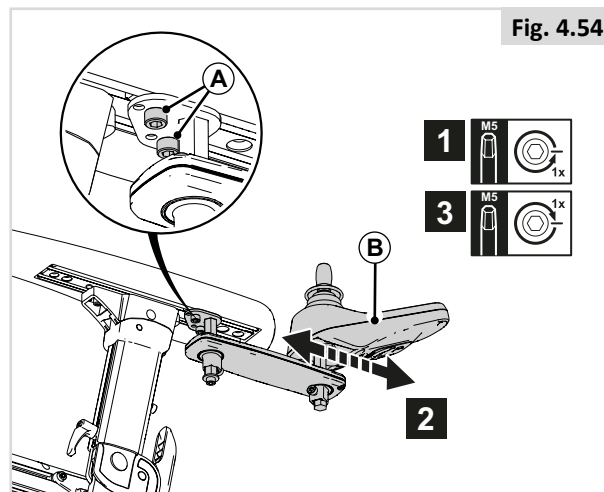1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).


Height of the controller (1) (Fig 4.55)

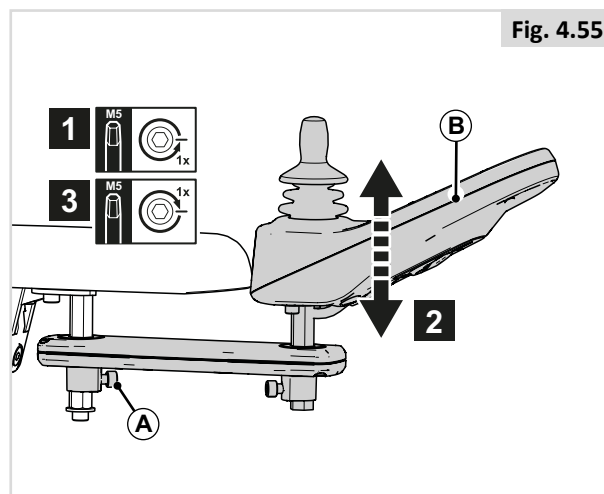1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

Height of the controller (2) (Fig 4.56)

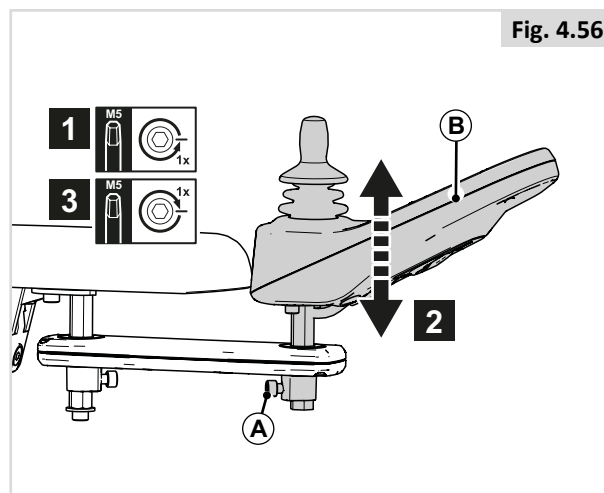1. Loosen (C) by one turn.
2. Adjust (D) to the desired position.
3. Tighten (C).

4.14 Pre-set thigh support

Thigh support height (Fig 4.57)

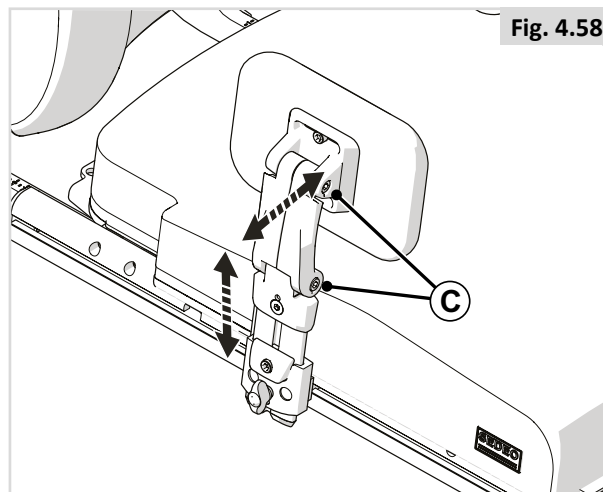
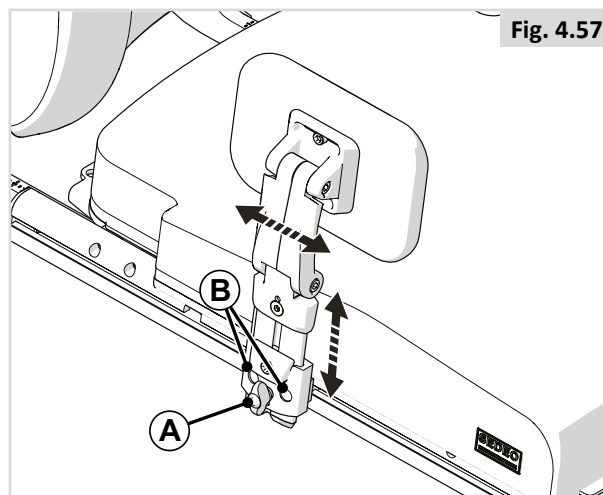
1. Loosen (A) by one turn.
2. Adjust to the desired position.
3. Tighten (A).

Thigh support depth (Fig 4.57)

1. Loosen (A) by one turn and lift thigh support.
2. Loosen 2 screws (B) by one turn.
3. Adjust to the desired position.
4. Tighten (B) then (A).

Thigh support width (Fig 4.58)

1. Loosen (C) by one turn.
2. Adjust to the desired position.
3. Tighten (C).

4.15 Pre-set laterals

Laterals width (Fig 4.59)

1. Loosen (A) completely.
2. Adjust (B) to the desired position.
3. Tighten (A).

Laterals height (Fig 4.60)

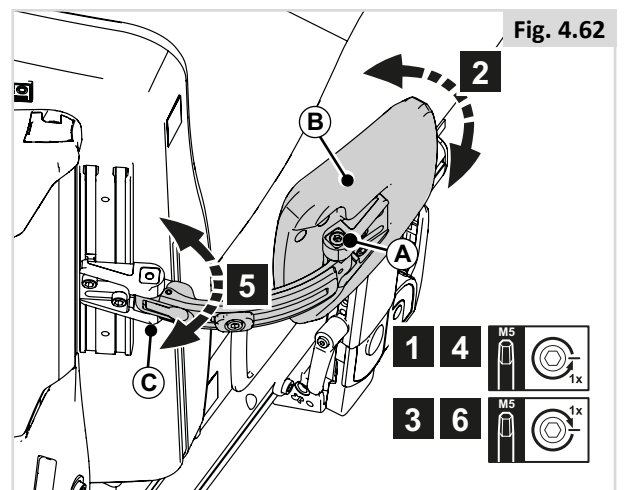
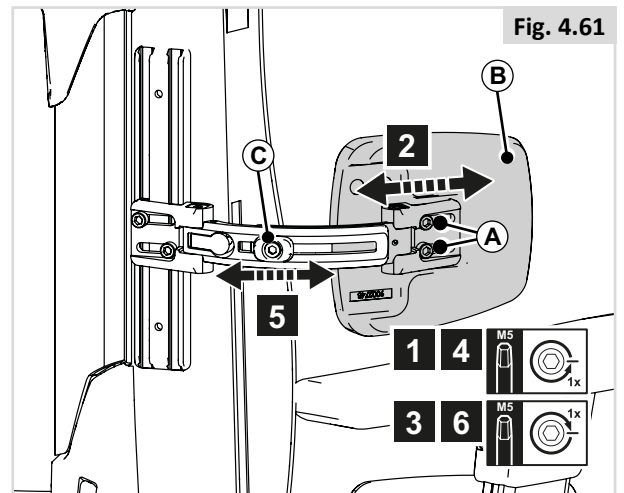
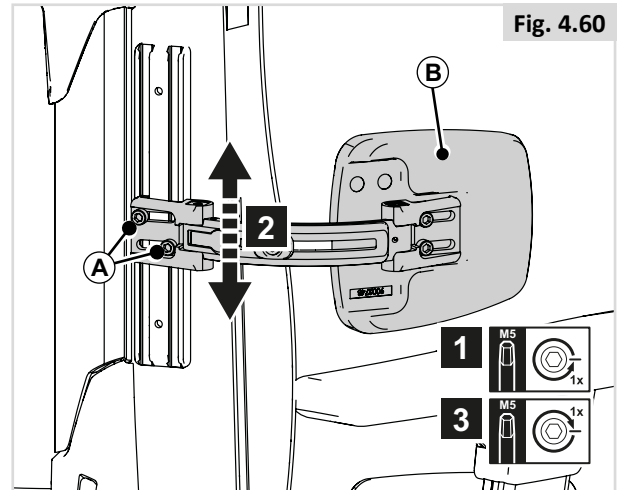
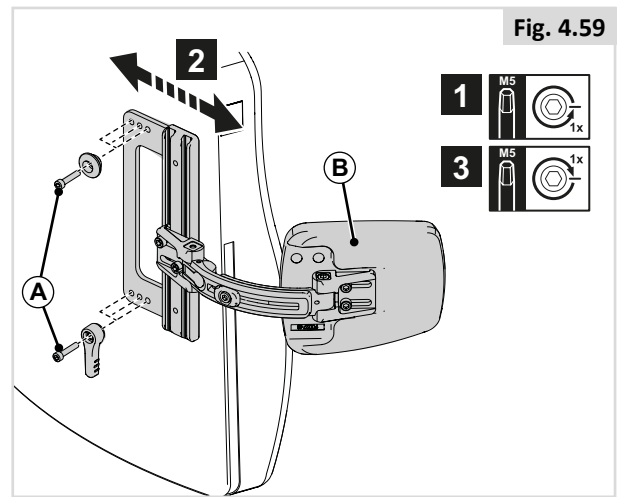
1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

Laterals depth (Fig 4.61)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).
4. Loosen (C) by one turn
5. Adjust (B) to the desired position.
6. Tighten (C).

Laterals angle (Fig 4.62)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).
4. Loosen (C) by one turn
5. Adjust (B) to the desired position.
6. Tighten (C).

4.16 Pre-set lateral swing away knee support

Knee support depth (Fig 4.63)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

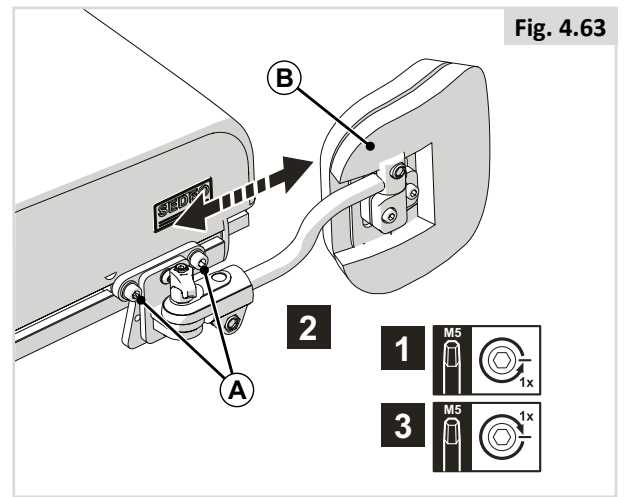
Fig. 4.63

4.17 Pre-set pommel

Pommel height (Fig. 4.64)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

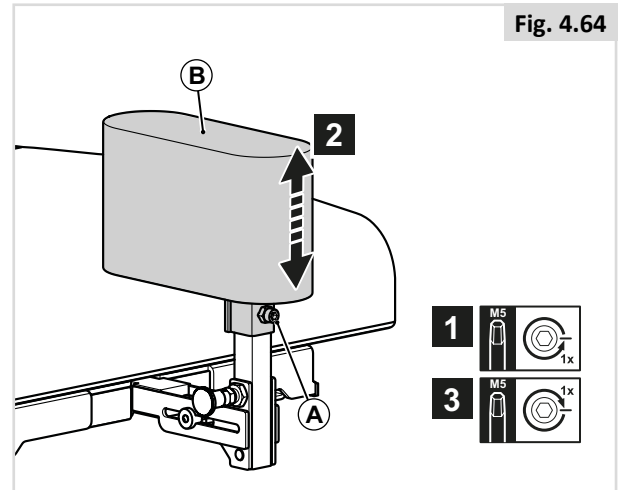
Fig. 4.64

Pommel depth (Fig. 4.65)

1. Loosen (A) by one turn.
2. Adjust (B) to the desired position.
3. Tighten (A).

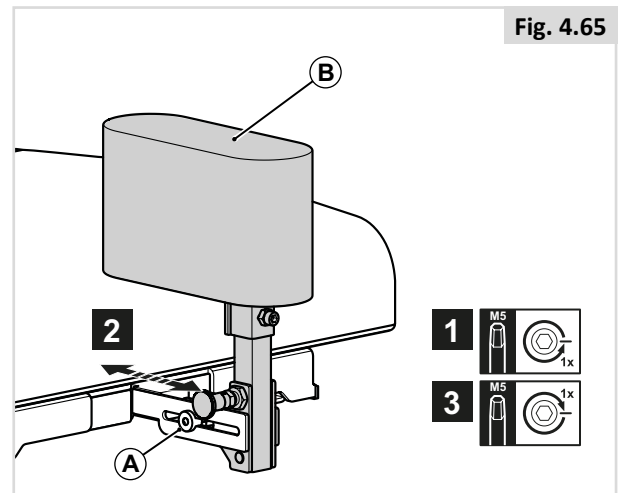
Fig. 4.65

4.18 Lap strap / seating positioning belt

DANGER! / WARNING!

- This product is only to be used to position a single person in a wheelchair.
- Lap straps are not suitable for transit purposes, approved occupant restraint systems must be used.
- Failure to heed these warnings may result in severe injury or death.
- Ensure that the carer or attendant is trained in the correct operation of the belts.
- Untrained carers or attendants may lead to delays, if an emergency situation occurs.

Mounting a lapstrap / seating positioning belt

1. Attach (A) to the seating frame using a screw on the left and on the right. (Fig 4.67)

Positioning a person with a lapstrap

- Adjust the lap strap to suit, leaving no more than a hand's width gap for comfort and safety (Fig 4.68).
- The hand clearance should be with the lap strap under normal tension and not allow large gaps or loops.
- Generally, the lap strap should be fixed so that the straps sit at an angle of approximately 45° (Fig. 4.69), and when correctly adjusted should not allow user to slip down in the seat.
- Place the strap loosely across the seat with the opening end of the buckle facing to the right for a left-handed person and to the left for a right-handed person. Pass the other ends of the strap through the gap between the backrest posts and the backrest.

DANGER! / WARNING!

- Always make sure that the lap strap is correctly secured and adjusted prior to use.
- Too loose a strap could cause the user to slip down and cause serious injury.
- Check lap strap and securing components at regular intervals for any signs of fray or damage. Replace if necessary.
- When servicing, check for correct operation of the release buckle and for any signs of wear on the material or plastic brackets.
- Regular Checks/Activities:
- As with all positioning components, there is a need for corrective adjustments as the person changes their seating position over time.
- Check the belts regularly for correct fitting, to ensure the safety and comfort of the user.

Fig. 4.66

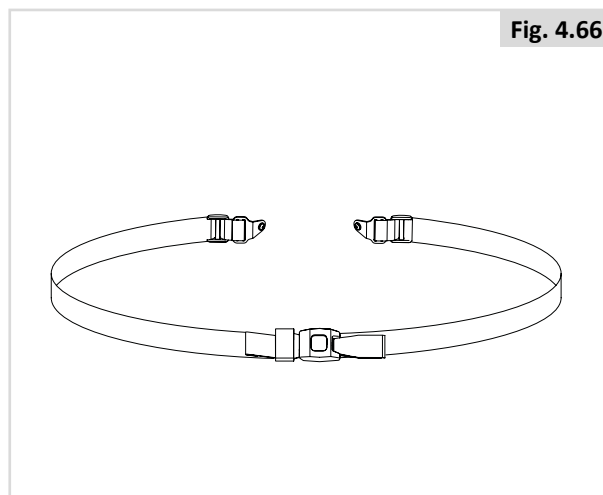
Fig. 4.67

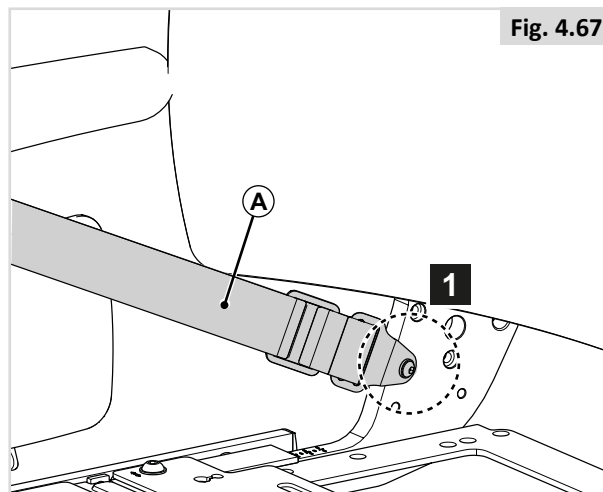
Fig. 4.68


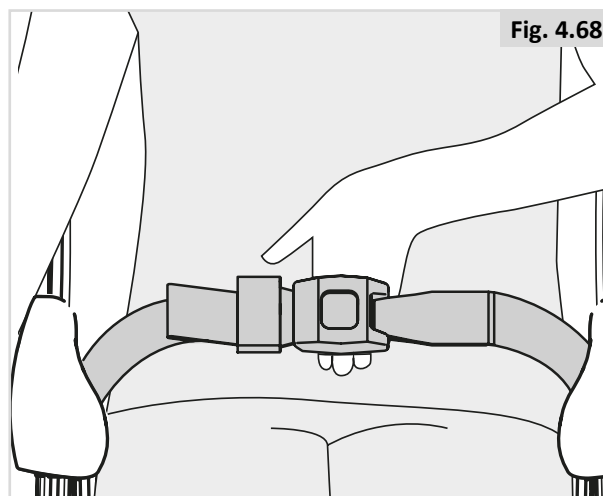
Fig. 4.69

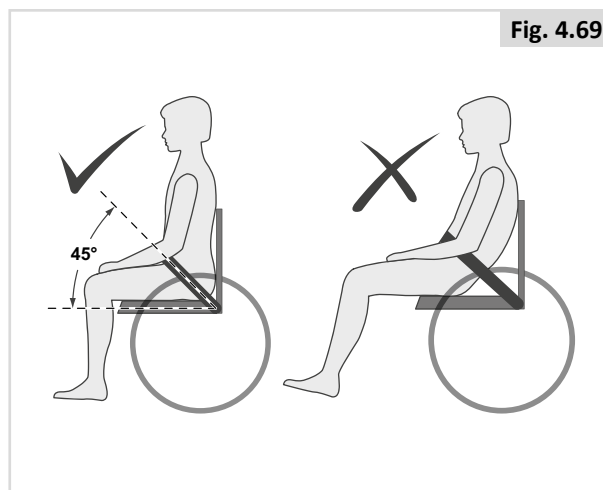5.0 Using the seating system

5.1 Getting in and out of the wheelchair (Transferring)

Sunrise Medical recommend that you consult your healthcare professional for assistance in developing your personal front or side transfer technique to best suit your needs and avoid any personal injury.

DANGER! / WARNING

To prevent a fall:

- Ensure controller is switched off during transfers to avoid unintentional movement. If you fail to do so you may touch the joystick and cause your system to move when you do not expect it.
- Ensure that the free wheel switch has been set to 'drive',  General manual.
- Rotate the front casters so that they point forward.
- Work with your health care professional to learn all safe methods for transfers.
- Make sure motor brakes are engaged. This keeps the system from moving when you transfer.
- It is dangerous to transfer on your own. It requires good balance and agility. Be aware that there is a point during every transfer when the seating system seat is not below you.
- Learn how to position your body and how to support yourself during a transfer.
- Have someone help you until you are sure you can do a safe transfer on your own.
- Do not stand on the footrest. Putting your full weight on the footrest may lead to wheelchair damage and to hazardous situations, including the wheelchair tipping up.
- Make sure your feet do not "hang up" or get caught in the space between the footrests.

5.2 Preparing for forward transfer

DANGER! / WARNING

Move your wheelchair as close as possible to the seat you intend to transfer to.

Swing-away legrests (Fig 5.1)

1. Fold up the foot supports (A)
2. Swing the calf plates (B) to the side (if applicable)
3. If possible, push (C) to the rear and swing legrests (D) to the side

Centre mount legrest (Fig 5.2)

1. Flip up (A).

Power centre mount legrest (Fig 5.3)

1. Flip up (A) 2x.

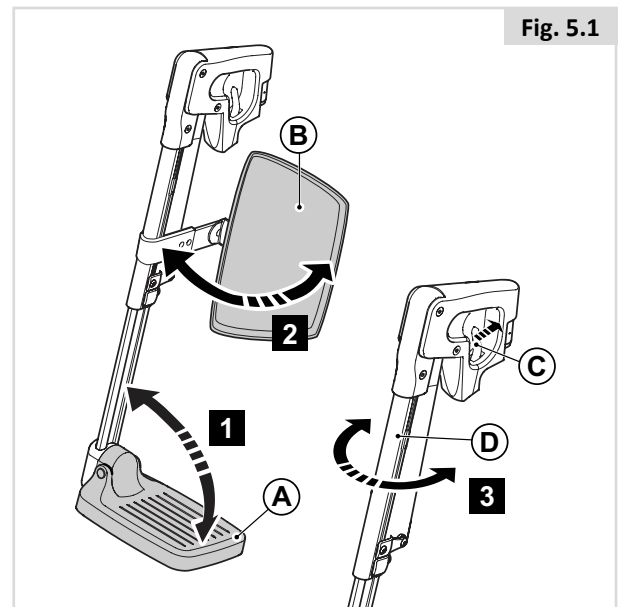
Fig. 5.2


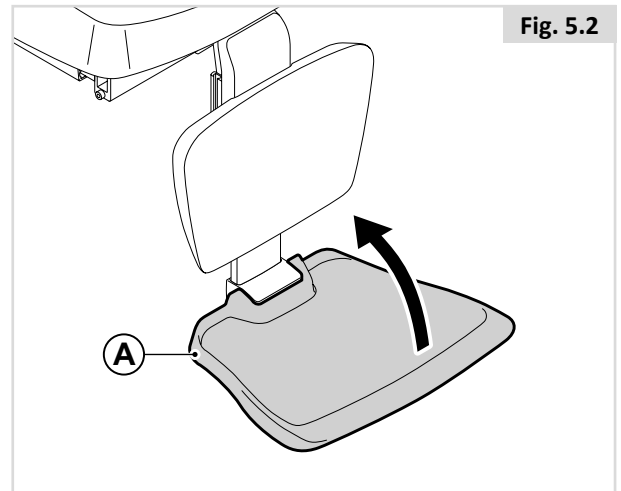
Fig. 5.3

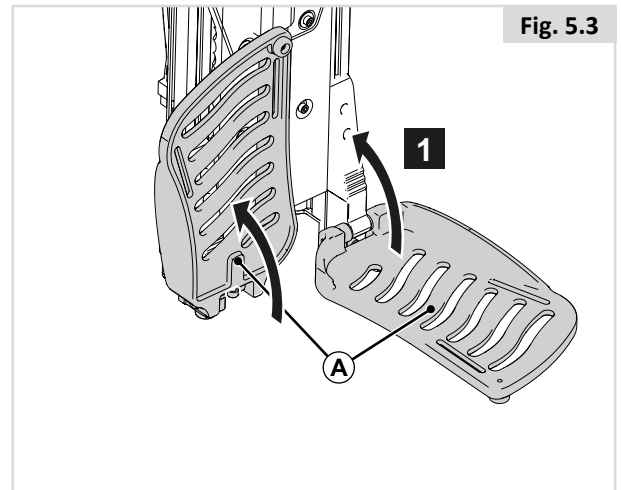5.3 Preparing for side transfer

⚠ DANGER! / WARNING

- Move your system as close as you can to the seat you are transferring to. If possible, use a transfer board.
- Transfer as far back onto the seat surface as you can. This will reduce the risk that you will miss the seat or fall.

⚠ NOTE / CAUTION!

- Some versions of the controller will need to be swung aside.
- Remove the tray before folding back the armrest.
- Do not lean on the armrest when folded back.

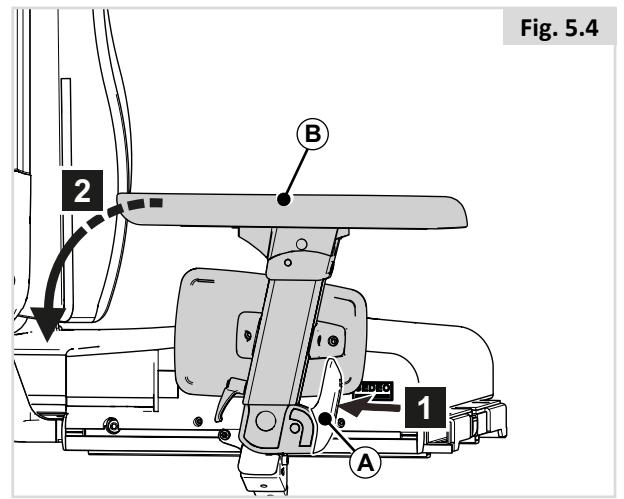
Fig. 5.4

Flip back armrest (Fig 5.4)

1. Press (A).
2. Fold (B) to the rear

Single post removable armrest (Fig 5.5)

1. Tip (A) downwards.
2. Remove (B).

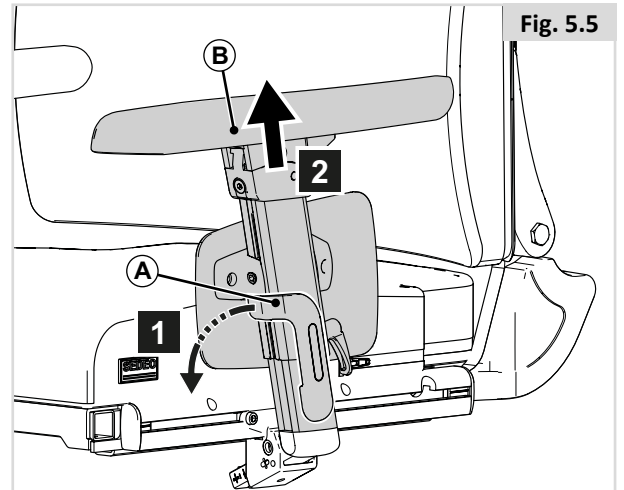
Fig. 5.5

Reclining flip back armrest (Fig 5.6)

1. Fold (A) to the rear.

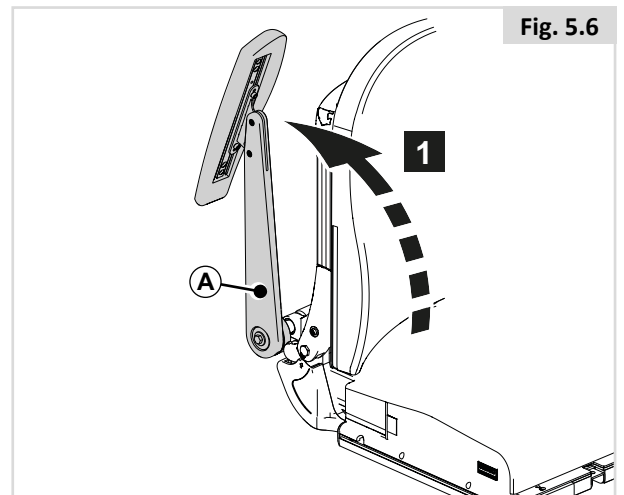
Fig. 5.6

Single post Sedeo Lite armrest (Fig 5.7)

1. Loosen A
2. Remove B


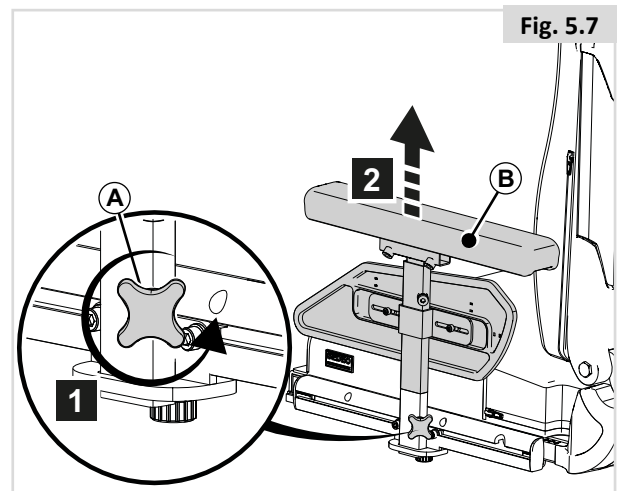
Fig. 5.7

5.4 Headrest

Removing/positioning the headrest (Fig 5.8)

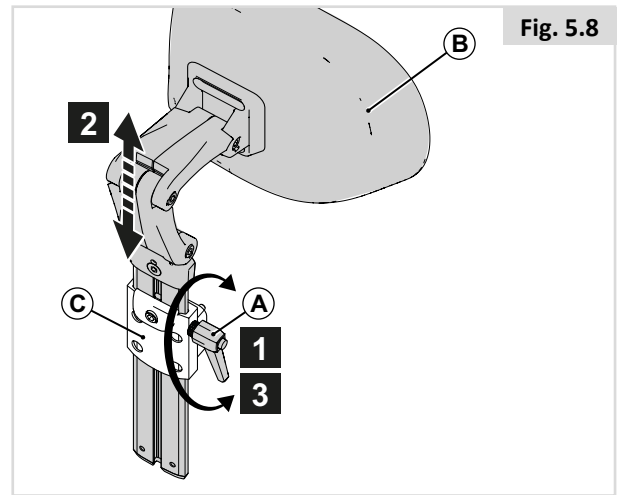1. Lift (A).
2. Slide (B) out of or into (C).
3. Push (A) down.

5.5 Swing-away controller bracket

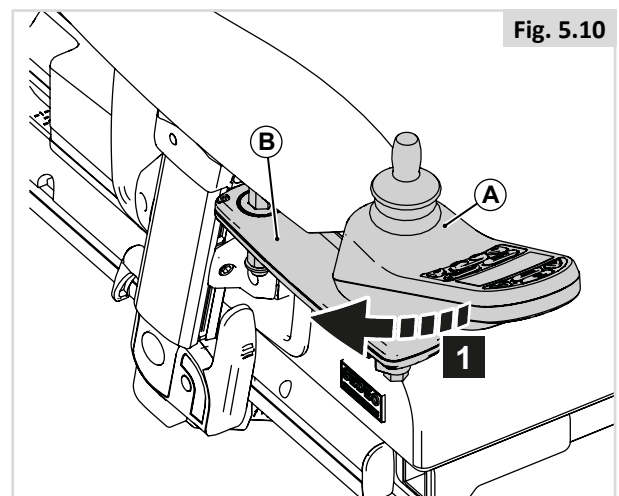
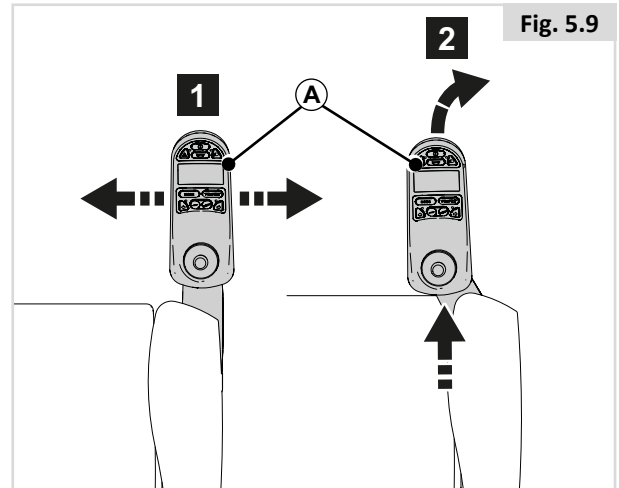
The joystick is mounted with a mechanism that allows the joystick to be locked in two forward positions. (Fig 5.9)

Swing-away the controller bracket (Fig. 5.10).

1. Push (A) outwards or inwards.
2. To return to forward position, push the back of the joystick forward until it locks into place (click)


NOTE / CAUTION!

Swinging back the controller bracket will allow you to pull up close to a table. Full operation of the controller remains possible


5.6 Laterals (Fig 5.11)

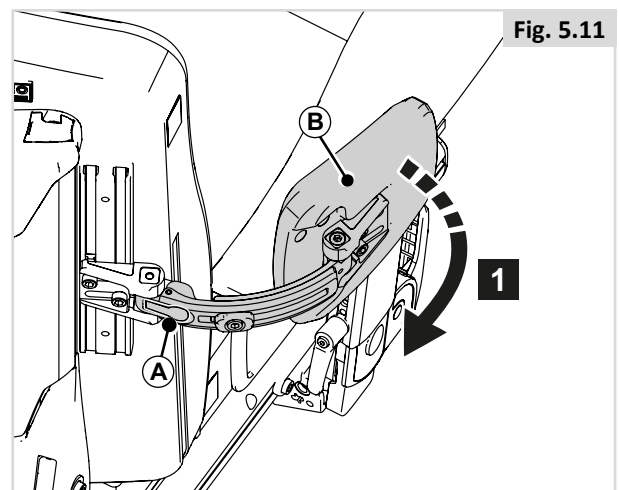1. Press A to swing away B


5.7 Knee support (Fig 5.12)

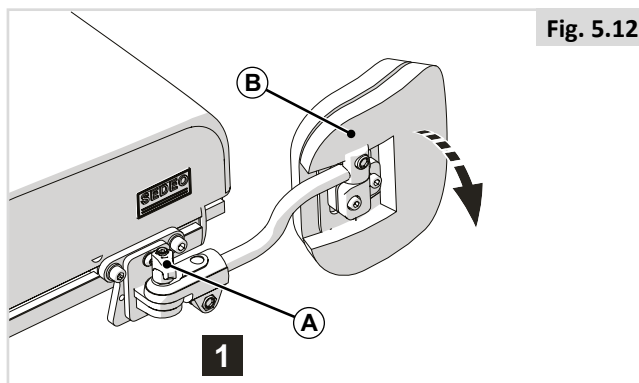1. Loosen A to swing away B


5.8 Thigh support (Fig. 5.13)

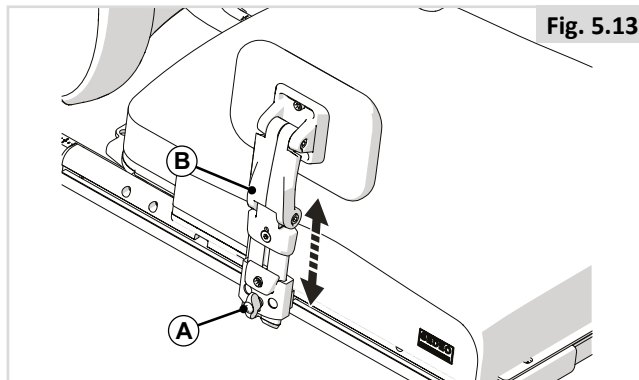1. Loosen (A) by one turn
2. Remove (B)

5.9 Hemiplegia armrest (Fig 5.14)

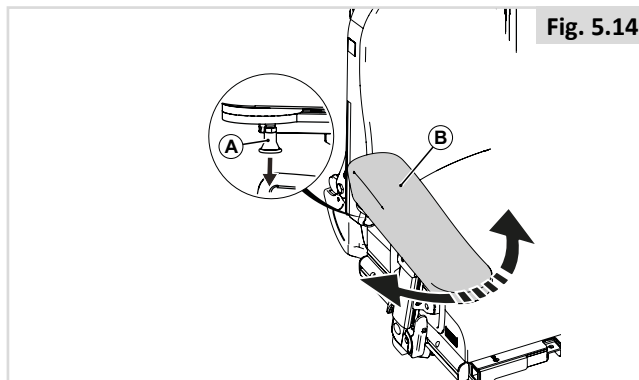1. Pull (A) out of the locked position and twist(B) into the desired position.


NOTE / CAUTION!

Ensure that (B) is properly secured!

5.10 Push handle height (Fig 5.15)

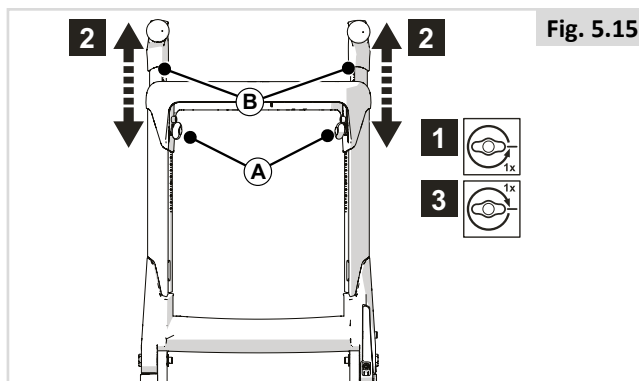1. Loosen (A) by one turn.
2. Slide (B) into the desired position.
3. Tighten (A).

DANGER! / WARNING

Do not use the push handles to tip the whole chair to the rear (in order to go up a sidewalk, for example).The push bar is not suitable for such use.

5.11 Pommel (Fig 5.16)

Flip down the pommel

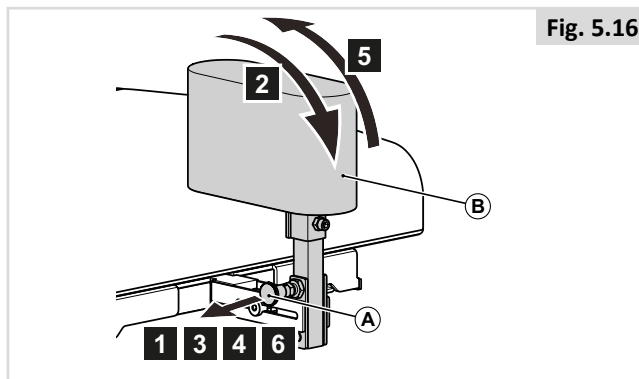
1. Pull (A) out of the locked position.
2. Lip down (B)
3. Release (A).

Flip up the pommel

1. Pull (A) out of the locked position.
2. Lip up (B).
3. Release (A) in the locked position.

NOTE / CAUTION!

Ensure that (B) is properly secured!

6.0 Manual and Powered seating options

6.1 Manual seating options

Manually reclining backrest (by user) (Fig 6.1)

1. Squeeze (A).
2. Lean forwards or backwards to adjust (B) to the desired position.

NOTE / CAUTION

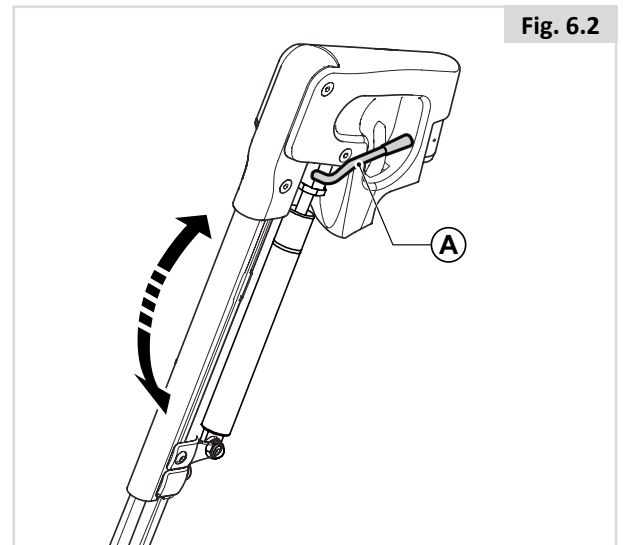
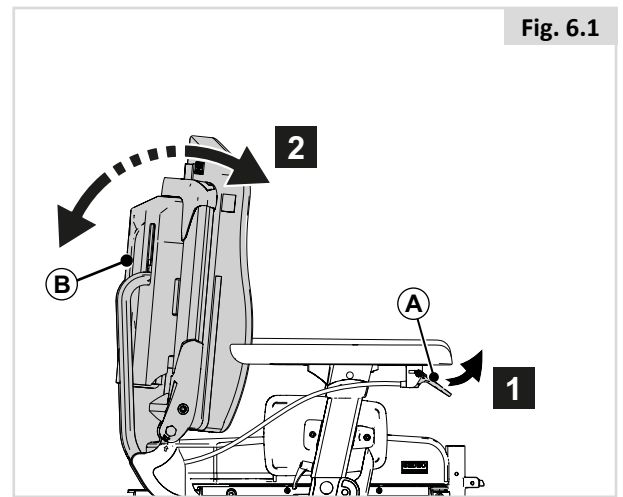
Keep squeezing (A).

3. Release (A).

Manually elevating legrests knee angle (fig 6.2)

The angle of the comfort leg rests can be adjusted independently of each other.

1. Push handle (A) forwards.
2. Lift your leg.
3. Release the handle when in the desired position.

6.2 (Powered) seating options

The wheelchair can be equipped by the following powered options to adjust your seating position. These options can be operated by the user without the use of tools.

Powered adjustable seating options (Fig. 6.3 - Fig. 6.5)

How to operate these powered seating options can be found in **CONTROLLER**.

1. Powered tilt function
2. Powered seat elevator
3. Powered backrest recline
4. Powered elevating leg rests

DANGER! / WARNING!

- Use of the powered adjustable seating options may influence the stability of the wheelchair.
- Only use the powered adjustable seating options when the wheelchair is on a flat surface! Do Not attempt to operate any power seating option while positioned on an incline.
- Please stay clear of any powered actuator while the component is in motion. Sunrise power mechanisms move over large ranges of travel. Users should be aware of their surroundings while components are in motion.
- While driving this seating system in the reclined, elevated, or tilted position use caution.
- Do Not attempt to operate any power seating option while this seating system is in motion.
- Do not attempt to operate the lift or tilt power seating options around children.
- Through programming it is possible to reverse the direction of all powered seating functions, ensure you know which direction your seat is going to move before operating.
- Please be aware that the lift/tilt modules present a trap hazard. Make sure that during operation the lift/tilt modules are free from all clothing, hands, feet and other extremities, to prevent injury.
- Do not drive on ramps or slopes with the seating in the elevated position. Before attempting to climb or descend a slope, return the wheelchair to a standard seated drive position, with the backrest upright.

This power system is designed to automatically reduce the maximum driving speed to a creep mode when certain power seating limits are reached. Depending on the configuration of the chair there might be multiple creep speeds.

Fig. 6.3

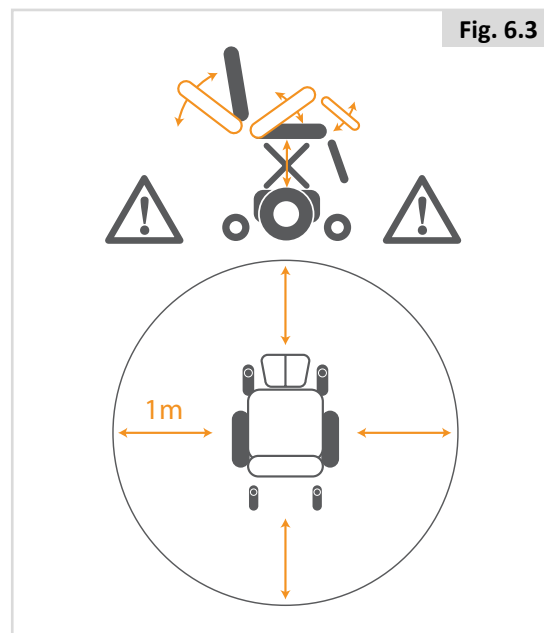
Fig. 6.4


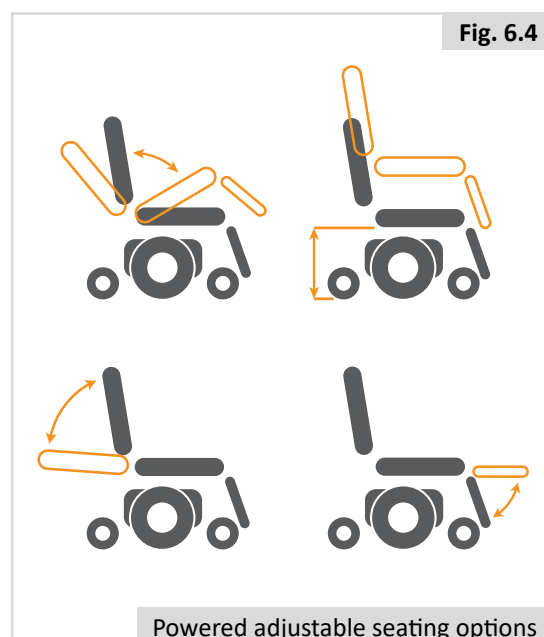
Fig. 6.5

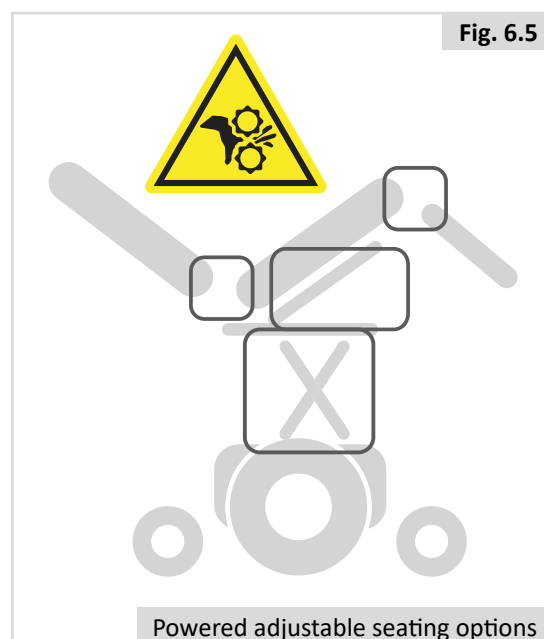7.0 Reducing the seating system size for transportation purposes

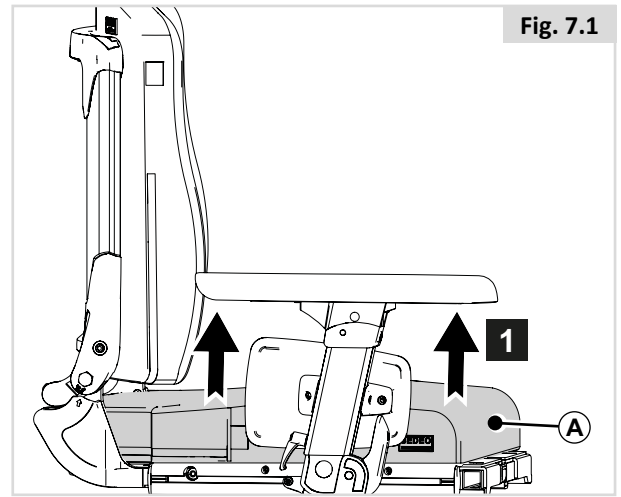
⚠ DANGER! / WARNING!

The wheelchair can be reduced in size for transportation without a passenger. Any components of the seating system that can be detached easily should be removed. Store away such components safely so that they can not cause injury or damage, or become damaged themselves.

7.1 Seat cushion


Removing the seat (Fig.7.1)

1. Pull (A) upwards.

Replacing the seat (Fig 7.2)

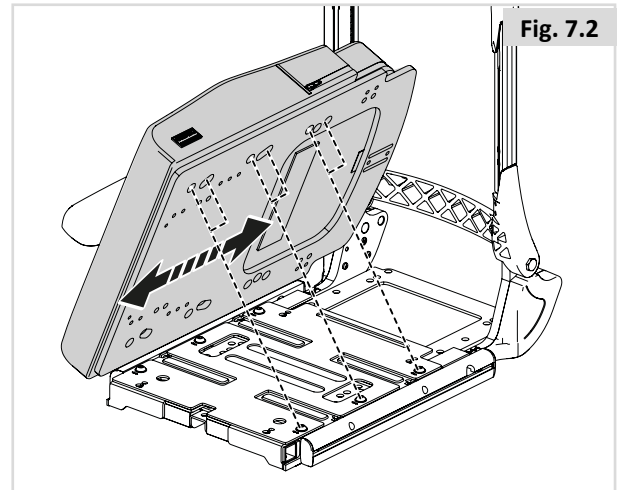1. Position (A) with the holes over the bolts

7.2 Removing the back cushion (Fig 7.3)

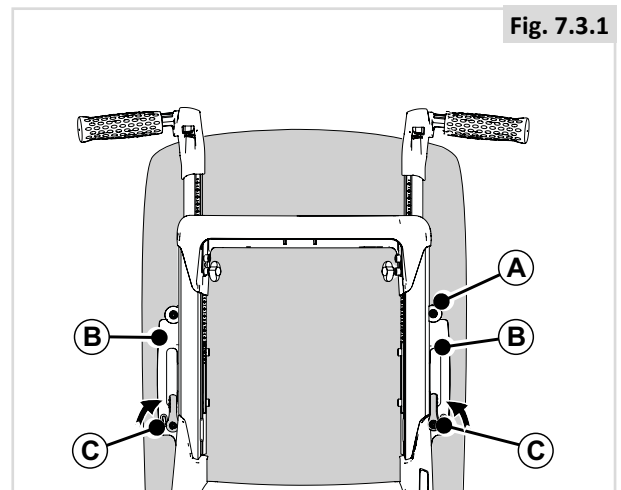
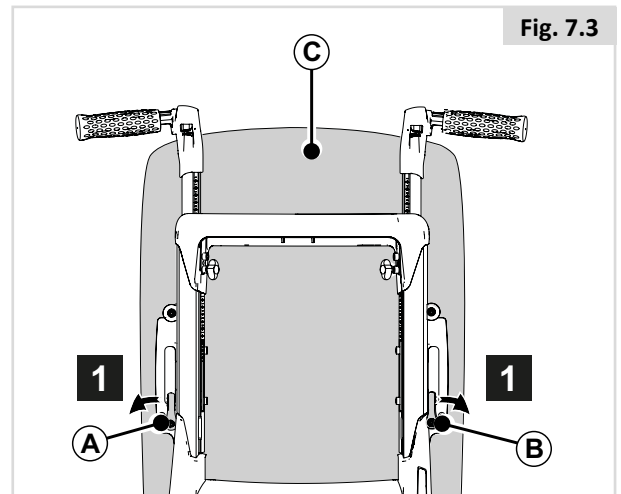
1. Turn (A) and (B).
2. From the bottom, pull (C) forwards and upwards and remove.

Positioning the back cushion (Fig 7.3.1)

1. Hook (A) behind (B).
2. Turn (C).

⚠ CAUTION!

Ensure that the back cushion is firmly in position (handles upwards).

7.3 Flip down backrest quick release (Fig 7.4)

Folding the backrest fully forward

1. Loosen (A).
2. Fold (B) fully forward.

NOTE / CAUTION

If necessary, first remove the back cushion and/or the armrest assemblies.

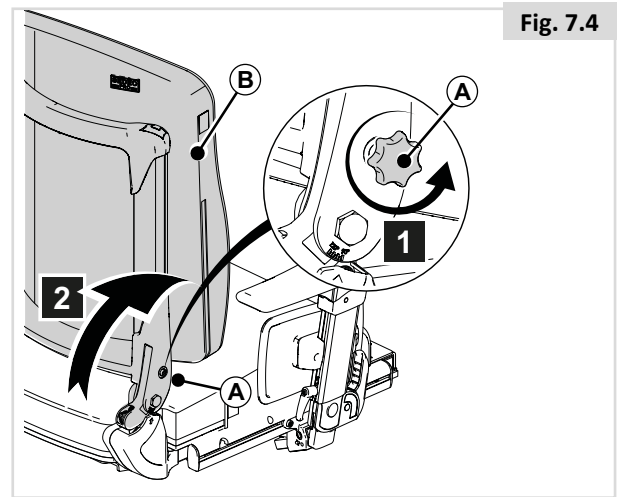
Fig. 7.4

7.4 Folding down backrest with reclining armrests (Fig 7.5)

Folding the backrest fully forward

1. Remove (A)
2. Fold (B) to the rear.
3. Loosen (C).
4. Fold (C) fully forward.

NOTE / CAUTION

If necessary, first remove the back cushion.

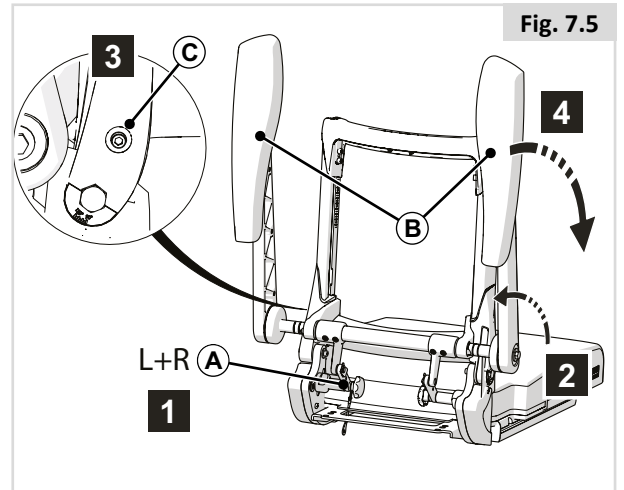
Fig. 7.5

7.5 Folding down reclining backrest with reclining armrests (Fig 7.6)

Folding the backrest fully forward

1. Remove (A).
2. Fold (B) to the rear.
3. Fold (C) fully forward.

NOTE / CAUTION

If necessary, first remove the back cushion.

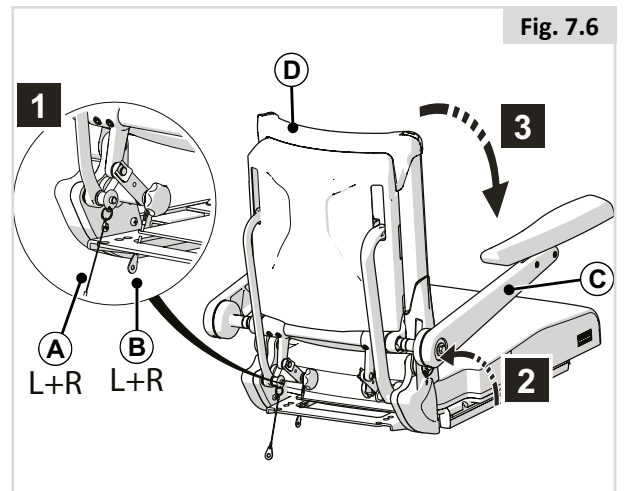
Fig. 7.6

7.6 Swing away legrests (Fig 7.7)

Removing a swing-away legrest

1. Loosen the calf band.
2. Push the catch (A) to the rear in order to unlock the leg rest.
3. Swing the leg rest outwards.
4. Remove the leg rest (B) from its bracket by lifting.

You should hear a click when replacing the leg rest. This indicates that the leg rest has locked into place.

Danger of finger entrapment (Fig.7.8)

⚠ Warning!

When the Manual Elevating swing away legrest is removed, carry the legrest at the handle intended for removing/carrying (A).

When carrying at B and at the same time releasing lever C there is a risk of finger trapping.

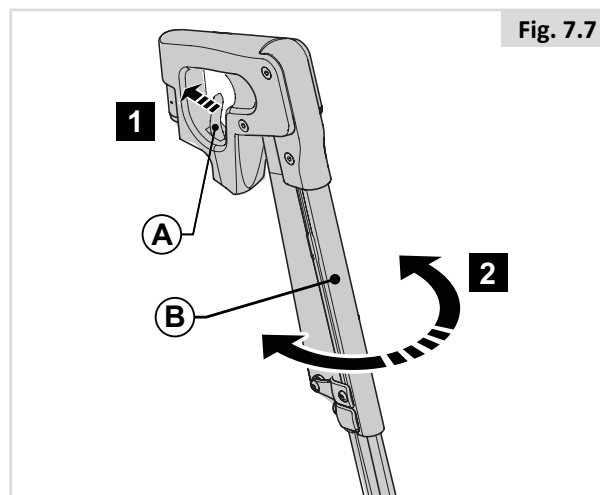
Fig. 7.8

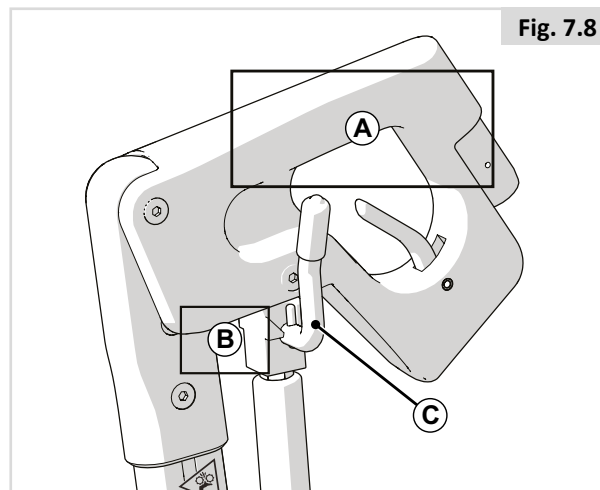
Fig. 7.8


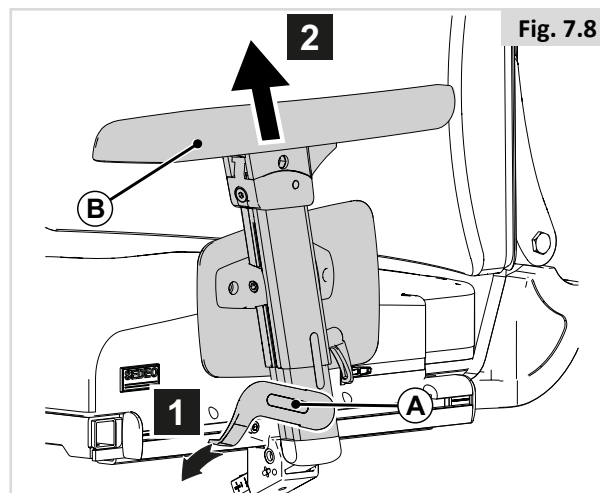
Fig. 7.9

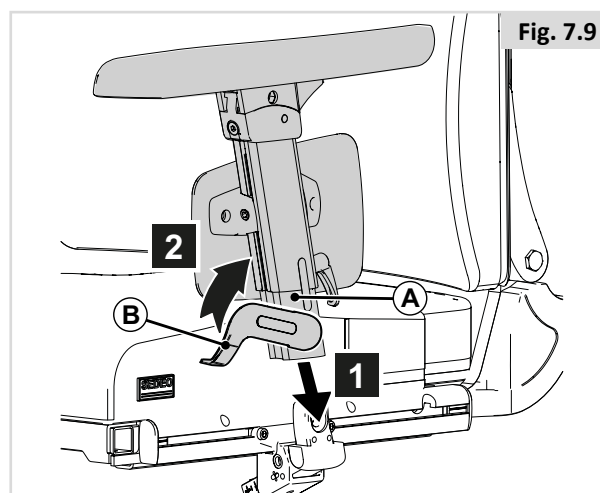7.7 Single post removable armrest

Removing the armrest assembly (Fig 7.9)

1. Tip (A) downwards.
2. Remove (B).

Positioning the armrest assembly (7.10)

1. Slide (A) onto (B).
2. Push (C) upwards.

8.0 Maintenance and cleaning

8.1 General Maintenance

- All fasteners should be checked monthly for wear, such as loose bolts or broken components.
- Loose fasteners should be re-tightened according to the installation instructions.

Please refer to the table below for needed Torques.

Fastener Matrix	Torque Setting (Nm)
M6 Bolt / Stud	9 - 10

Note: it will be necessary to use a torque wrench.

- Check all straps monthly for fraying, ripped seams or other indications of excessive wear damage. Discontinue use if damage is found.
- Chest straps should be replaced at the first indication of damage and/or excessive wear.
- If a broken or loose component is found, discontinue use immediately and contact your authorised Sunrise Medical supplier for replacement.
- Check all Velcro fastening straps for correct adhesion when pressed together.
- Ensure that any contamination, such as fluff, hair, etc is removed from the Velcro straps. Such contamination may affect adhesion.

8.2 Cleaning your seating system

The wheelchair should be wiped over once per week with a slightly damp, not wet, cloth and any fluff or dust that has accumulated around the actuators should be blown or dusted away.

NOTE / CAUTION!

Make sure that you dry all parts of your wheelchair if it becomes wet or damp after cleaning or if it is used in a wet or damp atmosphere.

DANGER! / WARNING!

It is important that should the wheelchair be used by more than one person it is cleaned thoroughly to ensure there is no cross infection.

8.2.2 Cleaning and inspection of your seating

DANGER! / WARNING!

Tears, dents, wearing or slackening of upholstery particularly near to metal could result in poor posture or lower levels of comfort and pressure relief.

Cleaning instructions for seating:

If there is a washing label inside the cushion cover, you can wash the cover. Follow the instructions on the washing label. Take out the foam inlays prior to washing and close the Velcro fasteners!

WARNING!

Do not use organic solvents such as thinner, benzene or white spirit.

Do not chemically clean the upholstery. Do not iron or spin-dry the upholstery.

Where upholstery cannot be removed, clean regularly to prevent build up or soiling. Clean with a damp soapy cloth however disinfectants may be used in dilution as specified by their manufacturer. Ensure surfaces are rinsed well with clean water and dried thoroughly.

8.3 Hygiene measures when being re-used:

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray.

To do this, you must use a suitable disinfectant for rapid alcohol-based disinfection of medical products and medical devices.

Please take into account the manufacturer's instructions for the disinfectant you are using.

NOTE / CAUTION!

- Do not use solvents, bleaches, abrasives, synthetic detergents, wax polishes or aerosols.
- Disinfectants may be used in dilution as specified by their manufacturer.
- Ensure surfaces are rinsed with clean water and dried thoroughly.

8.4 General cleaning instructions:

NOTE / CAUTION!

- You should remove stains on your seat as soon as possible.
- After a long period of use, you should clean the cover of your seat with a standard commercially available dry foam cleaning agent.
- You should always clean the whole surface and not just individual spots in order to avoid unattractive edges. The longer you wait the more difficult it will be to remove them.
- Avoid powerful rubbing with aqueous solutions. This will roughen up cover fabric.
- Dirt stains (eg. beer, blood, cola, red wine etc) are best removed with the dry foam cleaning agents or with a mild-action detergent.
- Please follow the respective instructions for use when treating the covers.
- The covers should be allowed to dry for at least 48 hours after cleaning with foam cleaning agents.
- Grease stains (eg. ballpoint pen, lipstick, chewing gum etc). Use a proprietary stain remover. Carefully rub the soiled location on the surface only with a clean cloth impregnated with the stain remover.
- Use only a very small quantity of stain remover.
- Test a small inconspicuous area first.
- The upholstery material must not be saturated, since the stain remover might then attack it and destroy it.
- Powerful rubbing should be avoided on the silk-printed scales, for example, the arrows on the seat plate and the scale for the seat width adjustment
- Do not use a hose or a pressure washer to clean your chair.

DANGER! / WARNING!

- Always read the label on any commercial or domestic cleaning substances.
- Always follow the instructions carefully.

8.5 Cleaning Accessories:

- All parts/ accessories such as swing away trays should be cleaned with a damp cloth.
- All lateral supports, headrest, armrests, side- guards, calf pads, lap belts and knee pads should be cleaned with a damp cloth.

8.6 Cleaning controls:

Should the control of your wheelchair become soiled or dirty, it can be wiped with a damp cloth with a dilute disinfectant until clean.

DANGER! / WARNING!

- This is important should the wheelchair be used by more than one person to ensure there is no cross infection.
- Ensure that wafer boards, joysticks (all variants), head arrays and switches (all variants) are cleaned with a mild disinfectant and a cleaning cloth to avoid any cross infection possibilities. Following removal from chair regularly wash the sip and puff mouth piece and tube to maintain cleanliness and functionality.
- Ensure control is switched off before cleaning.

8.7 Storage

To keep the upholstery and other parts of the wheelchair in good condition, store in a dry environment as close to room temperature as possible.

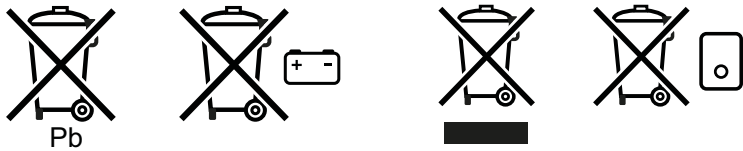
DANGER! / WARNING!

- Never store your wheelchair in direct sunlight or in a damp/ outdoor environment.
- Direct sunlight might bleach upholstery / plastic parts and cause metal parts to become hot.

9.0 Disposal

The symbols below mean that in accordance with local laws and regulations your product should be disposed of separately from household waste. When this product reaches the end of its life, take it to the local collection point designated by local authorities. The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is recycled in a manner that protects the environment.

Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations and national requirements.

In the following section, there is a description of the materials used on the wheelchair, in view of the disposal or recycling of the wheelchair and its packaging.

There may also be special local regulations in force with regard to disposal or recycling, these must be taken into account when disposing of your wheelchair. (This can include the cleaning or decontamination of the wheelchair prior to disposal).

Aluminium: Castor forks, wheels, side-guards for the chassis.

Steel: Fixing points, quick-release axle

Plastic: Handles, tube stoppers, castors and wheel/tyre

Packaging: Plastic bags made of soft polyethylene, cardboard

Disposal or recycling should be done through a licensed agent or authorised place of disposal. Alternatively your wheelchair may be returned to your dealer for disposal.

10.0 Technical specifications:

10.1 Applicable norms / standards

 This product complies with the regulations and guidelines for medical aids and carries a CE symbol.

The product meets the requirements and standards below. These are checked by independent institutions.

EU Guideline 2017/745 EEC	Applicable as mentioned in table 1
EN 12182 (2012) Class B	The requirements from EN12182 (2014) as mentioned in EN 12184 (2014). Assistive products for persons with disability – General requirements and test methods.
EN 12184 (2014) Class B	Electrically powered wheelchairs, scooters and their chargers – Requirements and test methods.
ISO 7176-8: 2014	Requirements and test methods for impact, static and fatigue strengths.
ISO 7176-9	Climate tests for electric wheelchairs.
ISO 7176-14	Requirements and test methods for control systems for electric wheelchairs.
ISO 7176-16: 2012	Requirements for resistance to ignition of upholstered parts.
ISO 7176-19 (2008)	Wheelchairs – Part 19: Wheeled mobility devices for use as seats in motor vehicles: Sedeo Pro meets the crash test requirements as described in ISO 7176-19
Dahl Docking	
ISO 7176-19 (2008)	Wheelchairs – Part 19: Wheeled mobility devices for use as seats in motor vehicles: Sedeo Pro meets the crash test requirements as described in ISO 7176-19
EN-ISO10993-5: 2009	Biological evaluation of medical devices.
EN 1021 1&2	Assessment of the ignitability of upholstered furniture. Ignition source: smouldering cigarette/ match lame equivalent.

10.2 Technical specifications: SEDEO Pro seating system

	Metric system values		Imperial system values & alternative	
	Minimum	Maximum	Minimum	Maximum
Seat angle adjustment				
Powered tilt with COG adjustment optional	0° - 30°		-5° - 25°	
Powered tilt with COG adjustment optional	-5° - 25°		0° - 30°	
Powered tilt with COG adjustment optional	0° - 50°		-5° - 45°	
Powered tilt with COG adjustment optional	-5° - 45°			
Effective seat depth	400 mm	580 mm	15.7"	23.6"
Effective seat width	380 mm	560 mm	15"	22"
Seat to floor height				
Filler Module (No powered tilt in space, no seat elevator)				
pos. 1 60 or 80 Ah batteries 0° seat angle	415 mm		16.3"	
pos. 2 60 or 80 Ah batteries 0° seat angle	440 mm		17.3"	
pos. 3 60 or 80 Ah batteries 0° seat angle	465 mm		18.3"	
pos. 4 60 or 80 Ah batteries 0° seat angle	490 mm		19.3"	
Powered tilt & seat elevator (300 mm)				
pos. 1 60 or 80 Ah batteries 0° seat angle	415 mm	715 mm	16.3"	28.1"
pos. 2 60 or 80 Ah batteries 0° seat angle	440 mm	740 mm	17.3"	29.1"
pos. 3 60 or 80 Ah batteries 0° seat angle	465 mm	765 mm	18.3"	30.1"
pos. 4 60 or 80 Ah batteries 0° seat angle	490 mm	790 mm	19.3"	31.1"
Backrest angle				
Tool angle adjustable back	90° - 120°		90° - 120°	
Powered recline	90° - 130°		90° - 130°	
Powered recline with anti shear	90° - 150°		90° - 150°	
Backrest height (seatplate to top of back cushion)	530 mm	720 mm	20.9"	28.3"
Flip back armrest / Single post removable armrest				
Armrest pad height (seatplate to top armpad)	260 mm	380 mm	10.2"	15"
Armrest width (on seatframe)	-20 mm	+20 mm	-0.8"	+0.8"
Armrest pad width (on armrest profile)	-20 mm		-0.8"	
Armrest depth (on seatframe with short seat rail)	250 mm		10.8"	
Armrest pad depth (on armrest profile)	100 mm		3.9"	
Armrest pad angle (vertical)	0° - 15°		0° - 15°	
Reclining Flip back armrest				
Armrest pad height (seatplate to top armpad)	250 mm	400 mm	9.8"	15.7"
Armrest pad width (on armrest profile)	-20 mm		-0.8"	
Armrest pad depth (on armrest profile)	140 mm		5.5"	
Armrest pad angle (vertical)	0° - 15°		0° - 15°	
Center Mount Legrest				
Lower leg length (from seatplate)	305 mm	460 mm	12"	18.1"
Knee angle	70° - 120°		70° - 120°	
Ankle angle	+20° - -40°		+20° - -40°	
Power Center Mount Legrest				
Lower leg length (from seatplate)	360 mm	540 mm	14.2"	21.2"
Knee angle	90° - 180°		90° - 180°	
Max length compensation	260 mm		10.2"	
Ankle angle	+35° - -15°		+35° - -15°	
Swing away legrests				
Lower leg length (from seatplate)	350 mm	510 mm	13.8"	20.1"
Knee angle	70°		70°	
Ankle angle	0° - 180°		0° - 180°	

	Metric system values		Imperial system values & alternative	
	Minimum	Maximum	Minimum	Maximum
Swing away legrests, manual elevating / power elevating				
Lower leg length (from seatplate)	350 mm	510 mm	13.8"	20.1"
Knee angle	0° - 70°		0° - 70°	
Ankle angle	0° - 180°		0° - 180°	
SEDEO Multi-Position headrest mount				
Headrest height adjustment range	170 mm		6.7"	
Headrest depth adjustment range	200 mm		7.9"	
Headrest angle (vertical)	-45° - +45°		-45° - +45°	
Headrest width	-20 mm	20 mm	-0.8"	+0.8"
Lateral Supports				
Lateral support width adjustment range	90 mm		3.5"	
Lateral support height adjustment range	170 mm		6.7"	
Lateral support depth adjustment range	45 mm		1.8"	
Pommel				
Pommel height (from seatplate to top of cushion)	170 mm	250mm	6.7"	9.8"
Pommel depth adjustment range	65 mm		2.5"	
Pushbar height adjustment range	140 mm		5.5"	

11.0 Guarantee / Warranty

THIS GUARANTEE DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.

Sunrise Medical* provides a guarantee, as set out in the warranty conditions, for wheelchairs to its customers covering the following.

Warranty conditions:

1. Should a part or parts of the wheelchair require repair or replacement as a result of a manufacturing and/or material fault within 24 months or for frame and cross-braces within 5 years after delivery to the customer, then the affected part or parts will be repaired or replaced free of charge. The warranty will only cover manufacturing defects
2. To enforce the warranty, please contact Sunrise Medical Customer Service with the exact details of the nature of the difficulty.
Should you be using the wheelchair outside the area covered by the Sunrise Medical customer service agent, repairs or replacement will be carried out by another agency as designated by the manufacturer. The wheelchair must be repaired by a Sunrise Medical designated Customer Service agent, (dealer).
3. For parts, which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the wheelchair in accordance with point 1).
4. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, (following the fitting), in accordance with these warranty conditions.
5. Claims from this warranty shall not arise, if a repair or replacement of a wheelchair or a part is required for the following reasons:
 - a. Normal wear and tear, which include batteries, armrest pads, upholstery, tyres, brakes shoes, etc.
 - b. Any overloading of the product, please check the EC label for maximum user weight.
 - c. The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the user instructions and/or the service instructions.
 - d. Accessories have been used which are not specified as original accessories.
 - e. The wheelchair or part having been damaged by neglect, accident or improper use.
 - f. Changes/modifications have been made to the wheelchair or parts, which deviate from the manufacturer's specifications.
 - g. Repairs have been carried out, before our Customer Service has been informed of the circumstances.

6. This guarantee is subject to the law of the country in which the product was purchased from Sunrise Medical"

* Means the Sunrise Medical facility from which the product was purchased.

Additional Notes For Australia Only:

- i. For goods provided by Sunrise medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise Medical that cannot be excluded under Australian Consumer Law.
- ii. You are entitled to a replacement or refund for a major failure and for compensation for any foreseeable loss or damage.
- iii. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.
- iv. The benefits to you given by this warranty are in addition to your other rights and remedies under a law in relation to the goods to which the warranty relates.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S