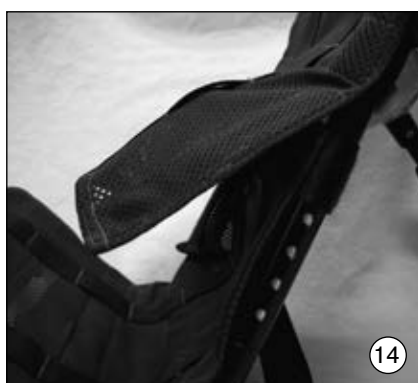
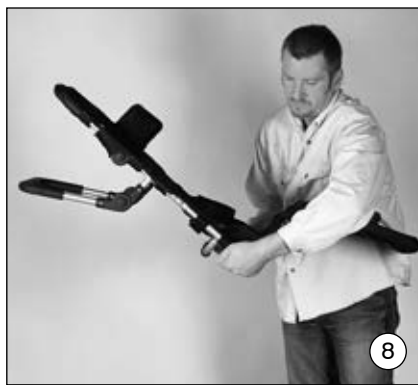
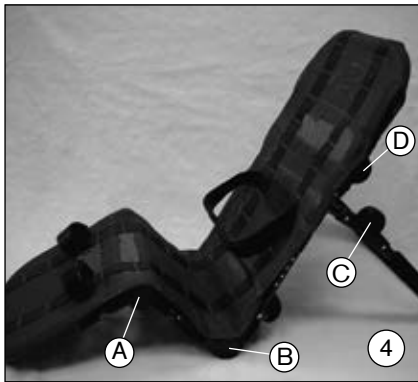
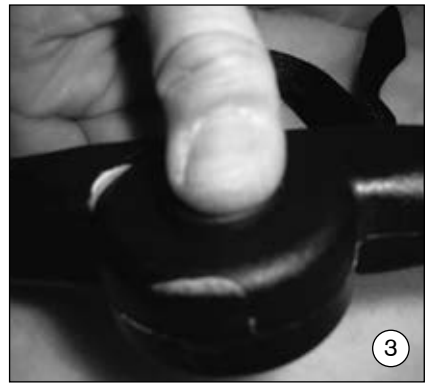
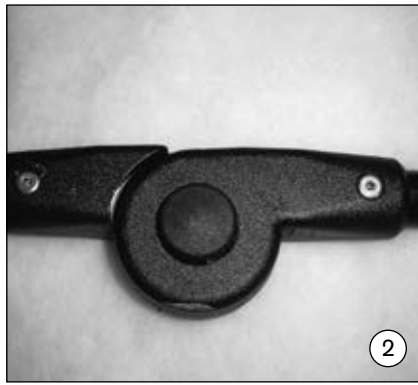
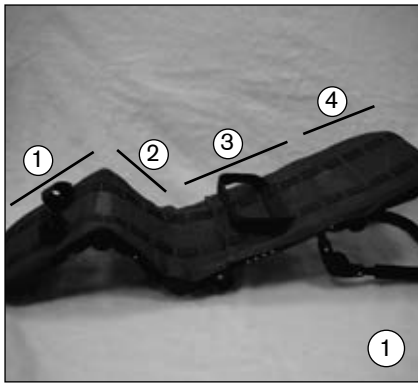
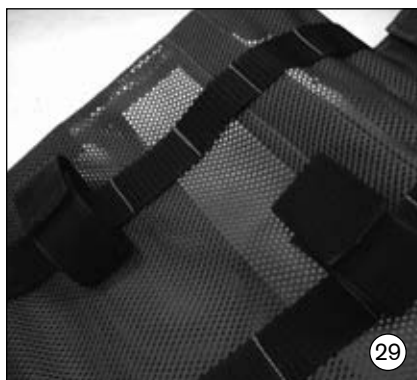
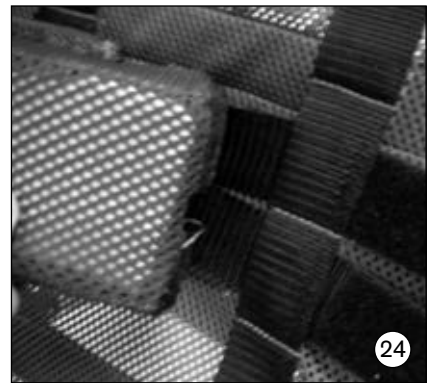
Robby

DE Gebrauchsanweisung	8
EN Instructions for use	14
FR Mode d'emploi	20
IT Istruzione d'uso.....	26
ES Instrucciones de uso.....	32
PL Instrukcja użytkowania	38
CS Návod k použití.....	44
RU Руководство по применению	50

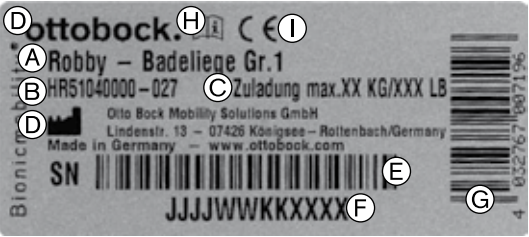
DE Therapieprodukt
EN Therapy aid
FR Produit thérapeutique
IT Prodotto terapeutico
ES Producto terapéutico
PT Produto terapêutico
NL Therapieproduct

DA Terapiproduct
SV Terapiproduct
NO Terapiproduct
PL Produkt terapeutyczny
CS Terapeutický produkt
RU Терапевтический продукт

Warn- und Typenschilder · Warning symbols and type plates ·
Plaques d'avertissement et signalétiques · Targhette di avvertimento e seriali ·
Símbolos de advertencia y placas de identificación · Tablice ostrzegawcze i identyfikacyjne ·
Varovné a typové štítky · Заводские и предупредительные таблички

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EN Instructions for use for Robby

Table of Contents	Page
Preface	15
Indications	15
ROBBY bathing and lying support	15
Construction and function of the basic version	15
Upholstery	16
Headrest Adjustment	16
Setting and adjustment options	16
Length adjustment of the back:	16
Length adjustment of the seat:	16
Accessories	16
Headrest	16
Trunk strap	17
Leg straps	17
Practical instructions on safety, maintenance and care	17
Legal information	17
Liability	17
CE conformity	17
Trademarks	18
Technical data	18
Spare parts	19

INFORMATION

Last update: 2015-04-13

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Have yourself instructed by qualified personnel in the proper and safe use of the product.
- Please keep this document in a safe place.

INFORMATION

- You can request this document as a PDF file from the Customer Care Center (CCC) at oa@ottobock.com or from the manufacturer's service department (see inside back cover or back page for addresses).
- It is possible to increase the display size of the PDF document.
- For further questions about the instructions for use, please contact the authorised personnel who issued the product to you.

Preface

Thank you for purchasing the ROBBY bathing and lying support. This brochure describes various setting and adjustment options and offers suggestions for its use.

It will show you how to use the support properly and also explain how to perform simple care and maintenance.

To prevent any damages that could occur due to improper operation and allow for the best possible use, please read the following instructions carefully.

If any questions or problems arise in connection with using this product, please contact your provider.

We hope the ROBBY meets your expectations and helps make your life more comfortable by providing you with trouble-free operation.

Indications

(Source: Catalog and list of therapy aids of German health insurance companies)

Bathing supports are designed for the use in a bath tub. This bathing support can also be used in the shower if it fits.

The ROBBY is intended especially for children from two years of age who lack trunk control or who are inclined to excessive muscular tension (for example, due to cerebral palsy).

ROBBY bathing and lying support

The Robby bathing support is a shower and bathing aid that allows you to make various adjustments to best fit the needs of your child.

The Robby bathing support supports the child in both lying and sitting positions.

The hook and loop closure tensioning straps that support the upholstery can be tightened or loosened to better accommodate children with muscular tension.

To keep soap from getting in the child's eyes during hair washing, the headrest can either be turned backwards or the fabric upholstery can be removed from this section to allow for free motion of the head.

The ROBBY has been designed to grow with your child, and has telescoping frame tubes that can easily be modified to change back length and seat depth.

The construction and functions of the Robby bathing support are explained in detail in the following section.

Construction and function of the basic version

The Robby bathing support consists of a lower leg section (1), seat (2), backrest (3) and headrest (4). (Fig. 1)

The frame is covered with two sections of upholstery that overlap each other in the pelvic area and which are connected with hook and loop straps. To accommodate any growth the child may experience, the hook and loop straps can be loosened to allow for greater or lesser upholstery depth.

The angle of the bathing support is set with ratchet-lock joints found on either side (Fig. 2). To unlock and set the ratchet-lock joint, press the push-button (Fig. 3). After setting the position, release the push-button and lock the joint by moving it to the nearest locking position.

The individual sections of the bathing support can be adjusted with the ratchet-lock joints A, B, C and D (Fig. 4).

To adjust the sections, opposing ratchet-lock joint push-buttons must be depressed simultaneously and held during adjustment (Fig. 5).

The easiest way to release the push-button is to press it in with your thumb while using your other fingers to adjust the angle of the support.

Once you've reached the desired position release the push-buttons of the ratcheting-lock joints. The joints will lock automatically.

Upholstery

The tension of the upholstery can be adjusted individually with the hook and loop straps (Fig. 6).

General Handling

If the ROBBY is dry, the easiest way to adjust the support is to butt one section up against your chest while holding it with your hands on the ratcheting-lock joints (Fig. 7). You can facilitate the whole procedure with the upper half of your body.

The setting position shown in Fig. 8 is awkward and requires too much force and is therefore not recommended.

Setting and Adjustment Options

Adjust the bathing support using the procedure described above. Both sitting positions (Fig. 9) and lying positions (Fig. 10) are set in the same way.

Headrest Adjustment

To help prevent soap from getting in the child's eyes release the ratchet-lock joint D and move the headrest backwards (Fig. 11).

Another option is to remove the upholstery from the headrest. In this situation the child's head is completely unsupported and, in case of poor head control, should be supported by hand of the attendant (Fig. 12).

To minimize excessive neck musculature tension, the headrest can also be adjusted forwards in conjunction with an increase in hip flexion,

Setting and adjustment options

Length adjustment of the back:

To change the length of the back section, loosen the bolts on either side of the back section with a hex cap socket wrench (Fig. 13).

Then loosen the hook and loop straps of both overlapping parts of the upholstery fabric in the pelvic section of the backrest (Fig. 14).

Next pull the headrest upwards and simultaneously hold the seat with the other hand.

When the required position is reached, tighten both bolts again. Make sure that each bolt has been screwed back into the hole it was removed from. (Fig. 15).

Now pull both parts of upholstery fabric tightly again one upon another and connect them with the hook and loop straps (Fig. 16).

Length adjustment of the seat:

To change the seat length, loosen the bolts on either side of the seat with a hex cap socket wrench (Fig. 17).

Then loosen the hook and loop straps on both overlapping pelvic sections of upholstery (Fig. 18).

Because the required upholstery in this case comes from the back section, it may be necessary to loosen the lowest tensioning strap.

Pull the foot section so as to set the required length and fix the new setting position with both screws.

Now pull both parts of upholstery fabric again tightly one over the other and connect them with the hook and loop closures. If necessary, connect the loosened tension strap on the backside side again.

Accessories

Headrest

The headrest (Fig. 22) consists of two foam pads with upholstery, which are located on either side of the head (Fig. 23).

Once you've determined the correct headrest height, you can fix it in place with the upholstery length strap (Fig. 24).

To set the correct width of the headrest pads, lift the outside end of the pad to be adjusted and move the pad into the required position (Fig. 25). Subsequently fix the pad by pressing it on the guide belt hook and loop fastener. Repeat this procedure with the other pad.

Trunk strap

The trunk strap (Fig. 26) offers additional restraint for children who experience involuntary, spontaneous movements, which could otherwise make their position insecure. The trunk belt can be adjusted in height just like the headrest with longitudinal straps attached to the upholstery.

Additional restraint can be provided by reducing the tension of the upholstery to create a larger space for the child.

Another possibility for providing greater trunk support is to use the headrest as a trunk guide. The headrest, with its special shape and size is well suited for this use (Fig. 27).

Leg straps

The leg straps (Fig. 28) restrain the legs and counteract extension movements, which can make the positioning and care of the child more difficult.

The straps are also height adjustable, just like the headrest and trunk straps (Fig. 29).

If necessary, these straps can also be used to support and guide the child's thighs.

All accessories, unless installed by the manufacturer, are to be installed by skilled personnel.

Practical instructions on safety, maintenance and care

- The maximal load capacity is 30 kg for size 1 and 60 kg for size 2.
- Robby is a therapeutic device and can only be used under supervision!
- Use only on a flat, even surface!
- During the installation, or when folding the bathing support, be careful not to pinch your fingers.
- The yoke for back height adjustment, the Ratchet-lock joint B (Fig. 4) and the end of the lower leg rest should always be in contact with the tub.
- Although the position shown in Figure 30 is adjustable, spontaneous movements made by the child (e.g. due to spasticity, cramping, athetoses) may result in a rocking movement of the bathing support, which can cause the child to become unstable or even slip out of the bathing support if restraints aren't used.
- Clean all frame and net parts with mild household detergents only.
- After use the bathing support can be folded to save space (Fig. 19). When the support is folded the headrest should be pointing downwards (Fig. 20). The rubber tube protectors provide protection against slipping and also release any water through the drain holes.
- The mesh material allows for free water flow and can be removed for washing (max. 60 °C).
- Prior to washing the device, it is advisable to close the straps and pack the upholstery fabric into a pillow case so that it can be easily unfolded after being washed and prevent from mixing in with other items in the wash.
- If necessary, remove any stray material that accumulates in the hook and loop closures with a nail brush.
- In any damage to the upholstery occurs, replace it immediately.
- No other maintenance is necessary.

Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.


All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

Technical data

Measurements and weights	Size 1	Size 2
Overall length of the lying surface	115 cm – 133 cm	147 cm – 173 cm
Seat depth	20 cm – 24 cm	31 cm – 43 cm
Back length	63 cm – 72 cm	89 cm – 94 cm
Width	38 cm	38 cm
Weight	3.8 kg	4.3 kg
Max. load capacity	30 kg	60 kg

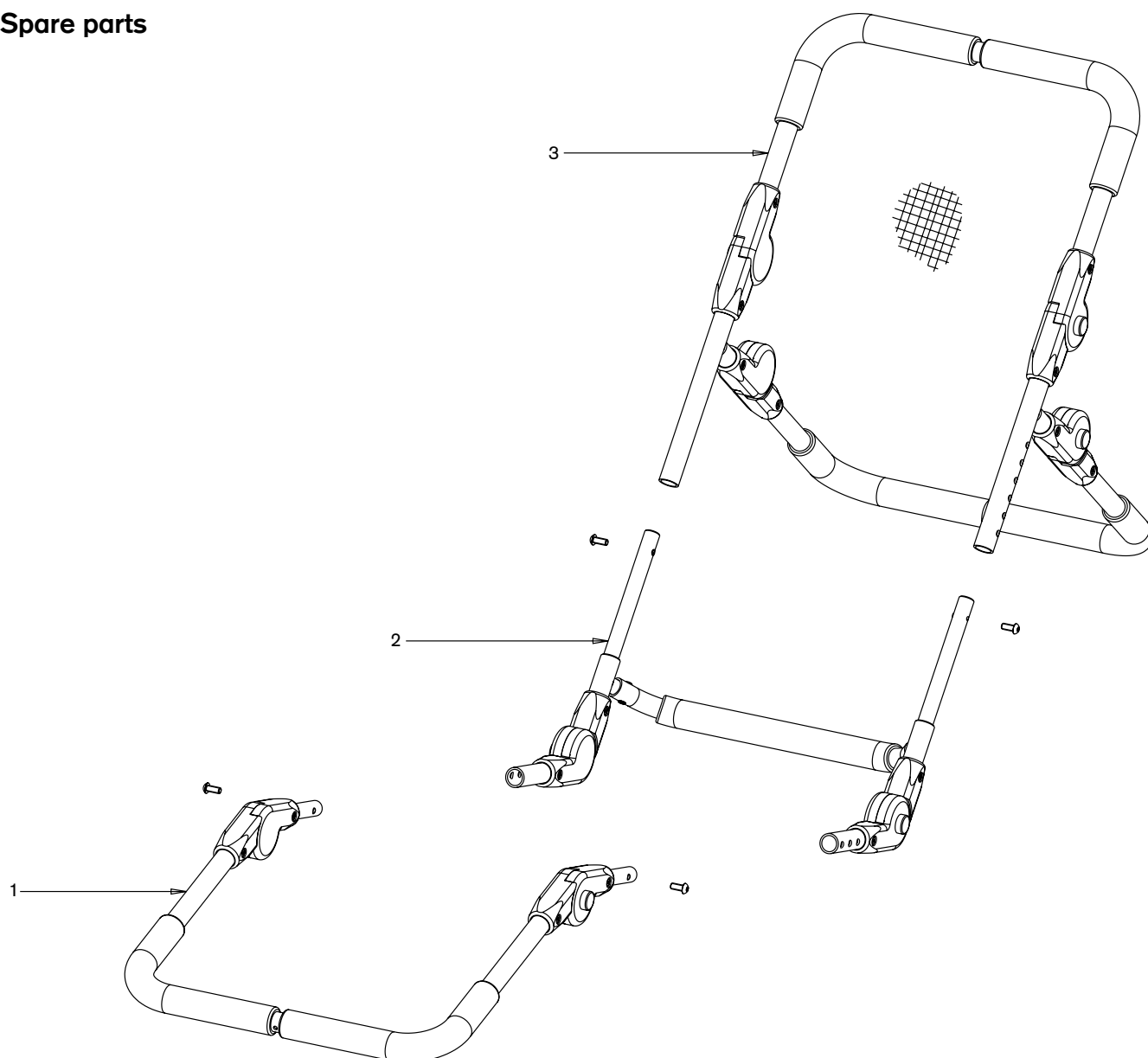Spare parts

When ordering spare parts, please always indicate the serial number of the product to ensure smooth delivery:

Item	Article number	Description	Quantity
1	HR51041300-027E	Foot frame, size 1, complete	1
	HR51051300-027E	Foot frame, size 2, complete	1
2	HR51041200-027E	Base frame, size 1, complete	1
	HR51051200-027E	Base frame, size 2, complete	1
3	HR51041100-027E	Back frame, size 1, complete	1
	HR51051100-027E	Back frame, size 2, complete	1
no illustr.	HR51045100E	Net upholstery, size 1, complete (incl. straps)	1
no illustr.	HR51055100E	Net upholstery, size 2, complete (incl. straps)	1
no illustr.	HR51045300	Trunk belt	1
no illustr.	HR51045140	Leg straps	2
no illustr.	HR51045200-103	Headrest, complete (accessory)	1

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A series of horizontal lines spanning the width of the page, providing a template for writing. The lines are evenly spaced and extend from the left margin to the right edge of the page.

A series of 25 horizontal lines spanning the width of the page, providing a template for writing.

A series of 25 horizontal lines spanning the width of the page, providing a template for writing.

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Otto Bock South Africa (Pty) Ltd
Building 3 Thornhill Office Park · 94 Bekker Road
Midrand · Johannesburg · South Africa
T +27 11 312 1255
info-southafrica@ottobock.co.za
www.ottobock.co.za

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Otto Bock HealthCare Canada
5470 Harvester Road
Burlington, Ontario, L7L 5N5, Canada
T +1 289 288-4848 · F +1 289 288-4837
infocanada@ottobock.com · www.ottobock.ca

Otto Bock HealthCare Andina Ltda.
Calle 138 No 53-38 · Bogotá · Colombia
T +57 1 8619988 · F +57 1 8619977
info@ottobock.com.co · www.ottobock.com.co

Otto Bock de Mexico S.A. de C.V.
Prolongación Calle 18 No. 178-A
Col. San Pedro de los Pinos
C.P. 01180 México, D.F. · Mexico
T +52 55 5575 0290 · F +52 55 5575 0234
info@ottobock.com.mx · www.ottobock.com.mx

Otto Bock HealthCare
Two Carlson Parkway North, Suite 100
Minneapolis, MN 55447 · USA
T +1 763 553 9464 · F +1 763 519 6153
usa.customerservice@ottobockus.com
www.ottobockus.com

Asia/Pacific

Otto Bock Australia Pty. Ltd.
Suite 1.01, Century Corporate Centre
62 Norwest Boulevard
Baulkham Hills NSW 2153 · Australia
T +61 2 8818 2800 · F +61 2 8814 4500
healthcare@ottobock.com.au · www.ottobock.com.au

Beijing Otto Bock Orthopaedic Industries Co., Ltd.
B12E, Universal Business Park
10 Jiuxianqiao Road, Chao Yang District
Beijing, 100015, P.R. China
T +8610 8598 6880 · F +8610 8598 0040
news-service@ottobock.com.cn
www.ottobock.com.cn

Otto Bock Asia Pacific Ltd.
Unit 1004, 10/F, Greenfield Tower, Concordia Plaza
1 Science Museum Road, Tsim Sha Tsui
Kowloon, Hong Kong · China
T +852 2598 9772 · F +852 2598 7886
info@ottobock.com.hk · www.ottobock.com

Otto Bock HealthCare India Pvt. Ltd.
20th Floor, Express Towers
Nariman Point, Mumbai 400 021 · India
T +91 22 2274 5500 / 5501 / 5502
information@indiaottobock.com · www.ottobock.in

Otto Bock Japan K. K.
Yokogawa Building 8F, 4-4-44 Shibaura
Minato-ku, Tokyo, 108-0023 · Japan
T +81 3 3798-2111 · F +81 3 3798-2112
ottobock@ottobock.co.jp · www.ottobock.co.jp

Otto Bock Korea HealthCare Inc.
4F Agaworld Building · 1357-74, Seocho-dong
Seocho-ku, 137-070 Seoul · Korea
T +82 2 577-3831 · F +82 2 577-3828
info@ottobockkorea.com · www.ottobockkorea.com

Otto Bock South East Asia Co., Ltd.
1741 Phaholyothin Road
Kwaeng Chatuchark · Khet Chatuchark
Bangkok 10900 · Thailand
T +66 2 930 3030 · F +66 2 930 3311
obsea@ottobock.co.th · www.ottobock.co.th

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Ottobock has a certified Quality Management System in accordance with ISO 13485.

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Otto Bock HealthCare GmbH
Otto Bock Mobility Solutions GmbH