


Accessories for the Rollz Motion Rhythm

Umbrella

This umbrella offers protection against both rain and sun. It is firmly attached to the frame so the user can keep both hands on the rollator all the time.

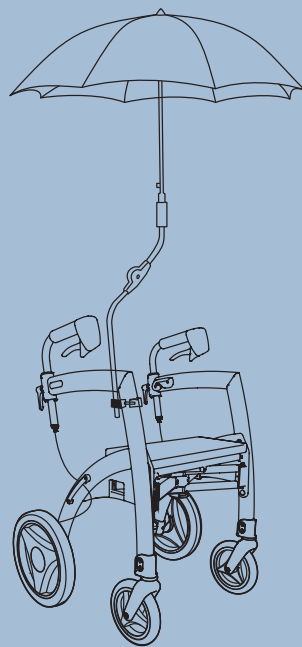

Wheelchair package

The wheelchair package offers the possibility to quickly transform the Rollz Motion Rhythm into a transport chair. This accessory makes it possible to go further and rest along the way.



Slow down brake

The Rollz Motion Rhythm can be equipped with slow down brakes, that give continuous resistance while walking. The desired brake intensity is easy to adjust.




rollz motion
rhythm

www.rollz.com

Stable rollator with three customizable cues

The Rollz Motion Rhythm addresses people with Parkinson's or a disrupted gait pattern

CUES
visual
tactile
audible

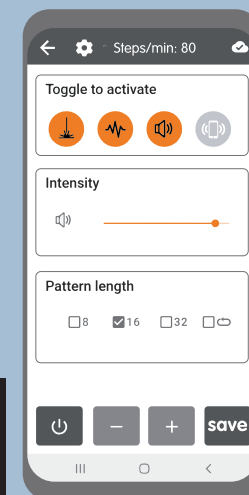

rollz®
takes you further



Helps regain the walking rhythm

The Rollz Motion Rhythm is developed for people with a disrupted gait pattern or who suffer from freezing episodes, like those with Parkinson's. This very stable rollator encourages the start of a rhythmic gait pattern with the help of three cues.

Three cues

This Parkinson rollator gives multiple stimuli to step out of a freeze: a laser line on the ground, a metronome with two different sound tones, and vibration in the handlebars. These three cues can be combined as desired.



Start the rhythm

The laser, vibration and sound indicate the desired walking rhythm. Cues combination and tempo can be set in the module on the rollator. The rhythmic pattern starts by pressing the button on the handlebar.




With laser light, vibrations and sound signals

CUES | see
feel
hear

Speed up or slow down

The pace of the cues can be lightly accelerated to stimulate the user to walk faster. Or the pace can be set slower, for smoother movements.

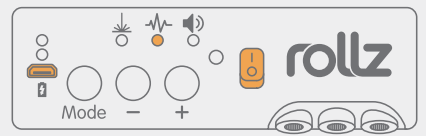

Two tones

Variation in pitch of the metronome prevents habituation, this way the user stays focused. These tones can also be played via the smartphone.

	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	All		
Laser	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	—
Vibration	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	—
High note	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	—
Low note	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	■	—

Smartphone app

The vibration, pitch and sound intensity can be adjusted in the corresponding app on the smartphone. Via the app, the pattern length of the cues can be set to: 8x, 16x, 32x or non-stop.

