

www.steets-lab.com

Quick and easy installation

Thanks to the included mounting kit, the existing crutch can be retrofitted quickly and effortlessly with the STEETS Steady Stand, and then parked in an open space using just one hand movement. Once the crutch is supporting body weight again, the STEETS Steady Stand system closes automatically, ensuring full usability of the crutch.



Scan the QR code to get more information.

steets.[®]

Intelligent folding mechanism that closes automatically when loaded.

Statement by Prof. Dr. Klaus M. Peters: Head Physician of Orthopaedics and Osteology at the Dr. Becker Rhein-Sieg-Klinik

“The carefree parking of the crutch in any open space not only helps to make everyday life easier for those who are affected, but the STEETS Steady Stand also enables users to grasp the crutch safely and quickly if they lose their balance. This innovative approach by STEETS has been not only very well received by patients in our clinic but also by medical and nursing staff. This invention truly supports fall prevention through its mobile solution.”

Crutches falling over?

Not with STEETS!



steets.[®]
STEADY STAND
The crutch upgrade.



MADE IN GERMANY






Everyday challenges with crutches

Placing crutches on the floor quickly and easily often poses a challenge, as they will frequently fall down when leaned against walls or corners. However, it's often vital to drop the crutch for a short period of time while keeping it within easy reach, meaning it can be picked up again quickly and safely, if necessary.


This supposedly simple problem – the falling over of crutches – causes discomfort and uncertainty for users, carers, and medical staff alike. It impairs independence, slows the course of rehabilitation, and can even lead to re-injury. Unfortunately, the relief offered by stationary holders is limited, as they can only be used in defined areas.




steets.

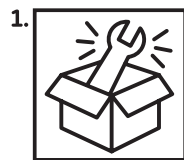

Use the assembly kit included in the packaging.

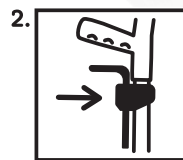


Attach the STEETS Steady Stand in a few simple steps.

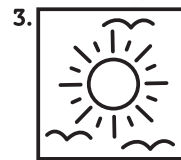

Discover greater lightness in your daily life.

Greater lightness with STEETS

The STEETS Steady Stand is not only an upgrade for greater lightness, but it also ensures greater safety when using crutches. With this aid, the crutch can be placed upright within easy reach, allowing for quick, safe, and easy access as and when needed. This function serves as a crucial contribution to fall prevention and was developed in close collaboration with specialists, therapists, and users. In addition to ensuring greater ease and safety, the STEETS Steady Stand is characterised by its high quality and use of recycled plastic, minimising the impact on our environment.

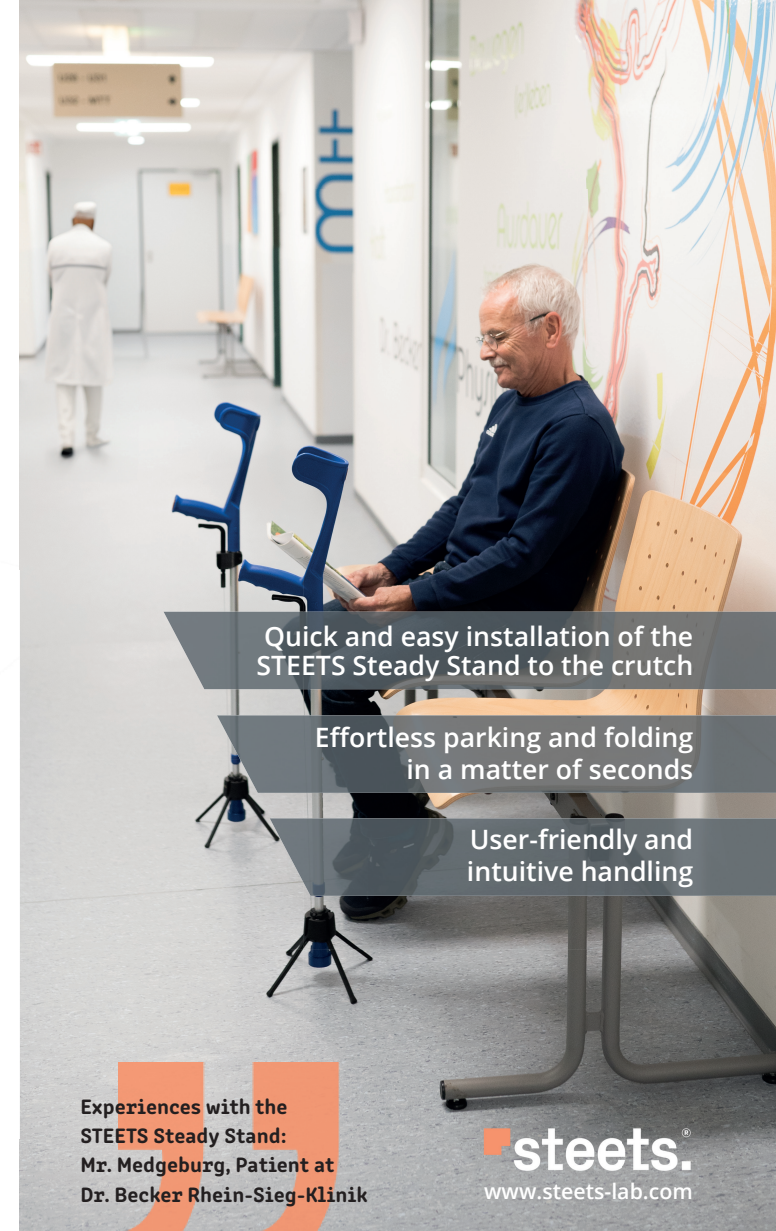

Quick and easy installation of the STEETS Steady Stand to the crutch

Effortless parking and folding in a matter of seconds

User-friendly and intuitive handling

Experiences with the STEETS Steady Stand:
Mr. Medgeburg, Patient at
Dr. Becker Rhein-Sieg-Klinik

steets.
www.steets-lab.com

“The STEETS Steady Stand is a truly beneficial invention. It has not only greatly supported and facilitated my daily clinical routine as a user, but it has also been of great value to my fellow human beings and the entire clinical staff. This innovative solution has proven indispensable, and I am grateful to have been able to act as a tester.“