zippie®

VOYAGE

Zippie® VOYAGE

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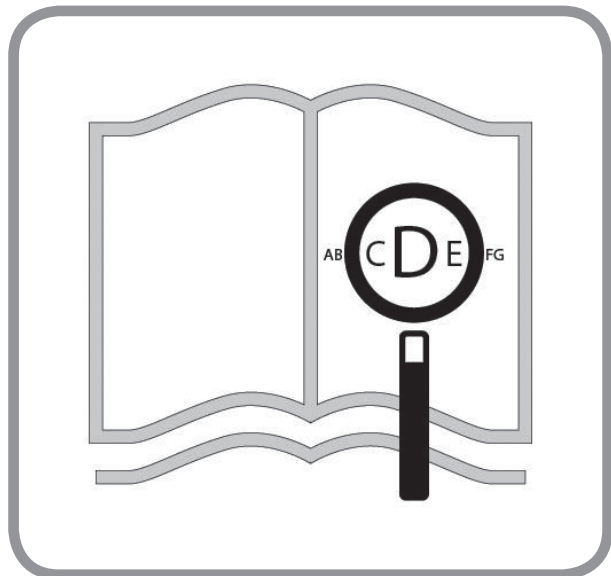
Handleiding





If you are visually impaired, this document can be viewed in pdf format at www.Sunrisemedical.co.uk

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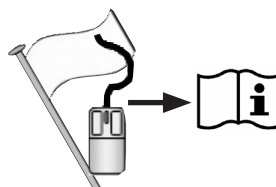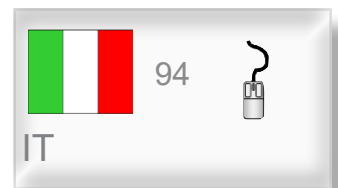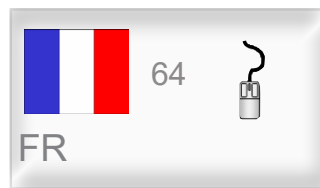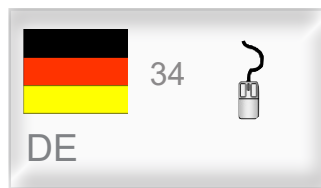SUNRISE LISTENS

Thank you for choosing a Zippie product. We want to hear your questions or comments about this manual, the safety and reliability of your device, and the service you receive from your supplier. Please feel free to write or call us at the address and telephone number below:

SUNRISE MEDICAL
THORNS ROAD
BRIERLEY HILL
WEST MIDLANDS
DY5 2LD
ENGLAND
PHONE:0845 605 66 88
FAX:0845 605 66 89
WWW.SUNRISEMEDICAL.CO.UK

Let us know your address. This will allow us to keep you up to date with information about safety, new products and options to increase your use and enjoyment of this Dependent Mobility Device.

FOR ANSWERS TO YOUR QUESTIONS

Your authorized supplier knows your Zippie product best, and can answer most of your questions about device safety, use and maintenance.

For future reference, please fill in the following:

Supplier: _____

Address: _____

Telephone: _____

Serial #: _____ Date/Purchased: _____

ADDITIONAL INFORMATION YOU SHOULD KNOW

No component of this chair was made with Natural Rubber Latex.

II. TABLE OF CONTENTS

I. INTRODUCTION	4	VIII. SET-UP & USE	15
II. TABLE OF CONTENTS	5	A. UNFOLDING AND FOLDING BASE FRAME	15
III. YOUR DEVICE AND ITS PARTS	6	B. CASTER WHEELS	16
YOUR DEVICE AND ITS PARTS	6	C. REAR WHEELS	16
ZIPPIE VOYAGE WITH COMFORT SEATING	6	D. PARKING BRAKE OPERATION	16
IV. NOTICE- READ BEFORE USE	7	E. SEATING SHELL INSTALLATION	17
A. CHOOSE THE RIGHT DEVICE & SAFETY OPTIONS	7	F. SEATING SHELL REMOVAL	17
B. REVIEW THIS MANUAL OFTEN	7	G. TILT IN SPACE ANGLE ADJUSTMENT	18
C. WARNINGS	7	H. RECLINE ADJUSTMENT	19
V. GENERAL WARNINGS	8	I. KNEE ANGLE ADJUSTMENT	19
A. WEIGHT LIMIT	8	J. STEP TUBE	19
B. GETTING TO KNOW YOUR DEVICE	8	K. CANOPY INSTALLATION AND ADJUSTMENT	20
C. TO REDUCE THE RISK OF AN ACCIDENT	8	L. TELESCOPING STROLLER HANDLE EXTENSION	20
D. SAFETY CHECKLIST	8	IX. DEALER SERVICE & ADJUSTMENT	21
E. CHANGES AND ADJUSTMENTS	8	A. POSITIONING AND GROWTH ADJUSTMENTS	21
F. ENVIRONMENTAL CONDITIONS	9	B. COMFORT SEAT ADJUSTMENT	21
G. STREET USE	9	C. SHOULDER STRAP HEIGHT ADJUSTMENT:	22
H. MOTOR VEHICLE SAFETY	9	D. FOOTREST ADJUSTMENTS:	23
I. FOR CAREGIVERS	9	E. FLOOR TO SEAT HEIGHT ADJUSTMENT	23
VI. WARNINGS: FALLS & TIP-OVERS	10	F. PELVIC POSITIONING BELT ADJUSTMENTS:	24
A. CENTER OF BALANCE	10	G. ANTERIOR TRUNK BUTTERFLY SUPPORT	26
B. DRESSING OR CHANGING CLOTHES	10	H. LATERAL TRUNK AND HEAD SUPPORTS (OPTIONAL)	26
C. OBSTACLES	10	I. ANTERIOR SUPPORTS	27
D. REACHING OR LEANING	10	J. HIP GUIDES (OPTIONAL):	27
E. RAMPS, SLOPES AND SIDEHILLS	10	K. TRAY INSTALLATION AND ADJUSTMENT:	28
F. TRANSFERS	10	L. FOOT STRAP ADJUSTMENT (OPTIONAL):	28
G. NEGOTIATING A CURB OR SINGLE STEP	11	M. SHOE HOLDER STRAP ADJUSTMENT (OPTIONAL):	28
H. CLIMBING STAIRS	11	N. VENT/BATTERY TRAYS (OPTIONAL):	29
I. DESCENDING STAIRS	11	O. FRONT OR REAR ADD-ON PLATFORM (OPTIONAL):	29
J. ESCALATORS	11	P. IV SUPPORT POLE / O ₂ TANK HOLDER (OPTIONAL):	29
VII. WARNINGS- COMPONENTS & OPTIONS	12	Q. CUP HOLDER, UTILITY HOOK (OPTIONAL):	29
A. BASE FRAME	12	X. MAINTENANCE	30
B. UPPER EXTREMITY SUPPORT TRAY	12	A. NOTES	30
C. FASTENERS	12	B. CLEANING	30
D. FOOTREST	12	C. STORAGE TIPS	31
E. TYRES AND WHEELS	12	D. RE-USE / DISPOSAL / RECYCLING OF MATERIALS	31
F. PARKING BRAKE	13	E. ORDERING PARTS	32
G. UNDERCARRIAGE STORAGE	13	XI. WARRANTY	33
H. ACCESSORY TRAYS	13	XI. WARRANTY	33
I. CANOPY	13		
J. SEATING SYSTEMS	13		
K. PELVIC POSITIONING BELTS	13		
L. CUSHIONS	13		
M. TILT AND RECLINE	13		
N. TRANSIT OPTION AND USE Fig.1.0 - Fig.3.0	13		

YOUR DEVICE AND ITS PARTS

1. Folding Base Frame
2. Tray Attachment
3. Rear Transit Hardware (optional)
4. Folding trickers
5. Rear Tyre
6. Quick-release Wheel
7. Undercarriage storage basket
8. Front transit hardware (optional)
9. Anti-swivel caster lock
10. 7" Front castor wheel
11. Adjustable Footplate
12. Footrest adjustment lever
13. Adjustable hip support
14. Grab handle
15. Adjustable Laterals
16. Adjustable Head Support
17. Comfort seating shell
18. Canopy attachment
19. Push handle angle adjustment
20. Canopy

ZIPPIE VOYAGE WITH COMFORT SEATING

Adjustment Ranges:

Tilt	+5° to - 30°	
Recline	90° to 180°	
Push Handle Height	39- ¹ / ₄ " to 47- ³ / ₄ "	(820 mm to 1210 mm)
Seat Depth	8" - 13"	(200 mm to 330 mm)
Seat Width (at hip)	9.5" - 12"	(240 mm to 300 mm)
Backrest Height	16.1" - 24"	(410 mm to 610 mm)

Lower Leg / Footrest Adjustment Range

Lower leg/foot	6.1" - 12.2"	(155 mm-310 mm)
Footrest angle Adjustment	50°	

NOTE:

- All features may not be available with some device setups or in conjunction with other device features. Please consult your supplier for more information. Your authorized supplier can also provide you with information on accessories.
- Individual components should not exceed the weight capacities listed opposite.
- The overall combined Maximum load should not exceed 77 kg, (170 lbs).
- The Zippie Voyage is considered to be ISO 7176-8 compliant.

Dimensions and Weights

Overall Width	24- ¹ / ₈ "	(615 mm)
Floor-to-seat height	25"-23.5"-22.5"	(640 mm-600 mm-575 mm)
Overall Length	42.5"	(1080 mm)
Overall Height	32" - 48"	(810-1220 mm)
Folded length	35- ¹ / ₈ "	(800 mm)
Folded Height	12.25"	(310 mm)
Folded Width	23.6"	(625 mm)
Turning Circle	43.3"	(1100 mm)
Max. Slope for Parking Brake	9°	9°
Max. User Weight	75 lbs	(34 kg)
Overall Weight	42.5 lbs	(22.5 kg)
Weight of Base	28 lbs	(12.7 kg)
Weight of Seat	21.5 lbs	(9.8 kg)

Maximum Load Capacities

Under Frame Tray	50 lbs	(22.5 kg)
Front Mount Platform	50 lbs	(22.5 kg)
Rear Riding Platform	45 lbs	(20 kg)
Oxygen Tank Holder	10 lbs	(4.5 kg)
Cup Holder:	1 lb	(0.5 kg)
Undercarriage Storage	15 lbs	(7 kg)
Vertical (Rear Mount) Vent	15 lbs	(7 kg)

A. CHOOSE THE RIGHT DEVICE & SAFETY OPTIONS

Sunrise provides a choice of many wheelchair styles to meet the needs of your child. However, final selection of the type of device, options, and adjustments rests solely with you and your health care advisor. Choosing the best early intervention device and set-up for safety depends on such things as:

1. The child’s disability, strength, balance, coordination, and the limits of their abilities.
2. The types of hazards that must be overcome in daily use (where you live and work), and other places you are likely to use the device.
3. The dependents need of options for their safety and comfort.

INTENDED USE: The Zippie Voyage is intended for medical purposes to provide mobility by a carer (not self-propelling) to persons under 75lbs, (34kg), restricted to a sitting position. This product is not suitable for jogging, skating or similar activities.

LIFETIME: The intended lifetime of this product is 5 Years

B. REVIEW THIS MANUAL OFTEN

Before using this device you, and each person who may assist you, should read this entire manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you.

As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the Medical Device Regulation (2017/745).

C. WARNINGS

The word “WARNING” refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The “Warnings” are in three main sections, as follows:

- 1. V — GENERAL WARNINGS**
Here you will find a safety checklist and a summary of risks you need to be aware of before you ride this device.
- 2. VI — WARNINGS — FALLS & TIP-OVERS**
Here you will learn how to avoid a fall or tip-over while performing daily activities with the device.
- 3. VII — WARNINGS — FOR SAFE USE**
Here you will learn about practices for the safe use of the device.
- 4. VIII — WARNINGS — COMPONENTS & OPTIONS**
Here you will learn about the components of the device and options you can select for safety. Consult your authorized supplier and your health care advisor to help you choose the best set-up and options for safe use.

NOTE: Where they apply, you will also find other safety notes in various sections of this manual. A list of these notes and their meanings is shown in the table below. Heed all of these safety notes. If you fail to do so, a fall, tip-over or loss of control may occur and cause severe injury to the rider and/or others.

Word	Definition
 DANGER!	Advice to the user of Potential Risk of serious injury or death if the advice is not followed
 WARNING!	Advice to the user of a potential risk of injury if the advice is not followed
 CAUTION!	Advice to user that potential damage to equipment may occur if the advice is not followed
NOTE:	General advice or best practice
	Reference To Additional Documentation

V. GENERAL WARNINGS

A. WEIGHT LIMIT

WARNING

1. NEVER exceed the weight limit of 34 kg, (75 lbs), for the weight of the occupant. If the limit is exceeded, damage to your device, a fall, tip-over or loss of control may occur and cause severe injury to the rider and/or others.
2. Never hang items greater than 5lbs (2.3kg) from stroller handle or seat back handle. A fall, tip-over, or loss of control may occur.

B. GETTING TO KNOW YOUR DEVICE

WARNING

Read all instructions before using this product. In particular, you should be trained in the safe use of this product by your health care professional.

C. TO REDUCE THE RISK OF AN ACCIDENT

WARNING

1. BEFORE using this device, you should be trained in its safe use by your health care professional.
2. Practice transfers until you know your limits and the extent of your child's abilities. Always ask if you need to have someone help you to avoid a fall or tip-over.
3. Be aware that you must develop your own methods for safe use best suited to your level of function and ability.
4. NEVER try a new maneuver on your own until you are sure you can do it safely.
5. Get to know the areas where you plan to use your device, look for hazards and learn how to avoid them.

D. SAFETY CHECKLIST

WARNING

Before each use of the Early Intervention Device.

1. Parking brake must be adjusted to maintain proper performance. Always engage the parking brake before transferring a dependent child.
2. Check that the device rolls easily, and that all parts work smoothly. Check for noise, vibration, or a change in ease of use, (they may indicate loose fasteners, or other damage).
3. Repair any problem you may encounter. Your authorized supplier can help you find and correct any problems.
4. Make sure the tire/axle on both sides are fully engaged and locked. If they are not locked, the wheel may come off and cause a fall.
5. Never seat your child in the mobility device until it is fully unfolded and locked.
6. After adjusting handle length check to make sure that it is locked at the new length.
7. Always use a pelvic positioning belt when child is seated.
8. Never leave your child unattended.
9. CHOKING HAZARD – This mobility aid uses small parts which under certain circumstances may present a choking hazard to young children.

E. CHANGES AND ADJUSTMENTS

WARNING

1. See your health care professional and have them adjust seating components and accessories as your child grows.
2. Unauthorized modifications or use of parts not supplied or approved by Sunrise Medical may change the product's structure. This will void any warranty and may cause a safety hazard.

V. GENERAL WARNINGS

F. ENVIRONMENTAL CONDITIONS

⚠ WARNING

1. Your mobility device is designed for use on firm, even surfaces such as concrete, asphalt, indoor floors and carpeting.
2. Use extra care if you must use this mobility device on a wet or slick surface. If you are in doubt, ask for help.
3. Do not operate your mobility device in sand, loose soil or over rough terrain.
4. If the stroller is subject to direct sunlight for a long period of time, then parts of the wheelchair (e.g. frame, legrests, wheel locks) may become hot (>41°C).
5. Be careful with fire, in particular with burning cigarettes. Keep away from sources of ignition. Seat and back slings can be set alight.

G. STREET USE

⚠ WARNING

1. When using this device, avoid street use whenever possible.
2. This product should stay in, and obey all legal pedestrian pathways, and laws that apply to pedestrians.
3. Be alert to the danger of motor vehicles in parking lots.
4. At night, or when lighting is poor, use reflective tape or clothing for maximum visibility.
5. IF you have to cross a road, make eye contact with drivers before you go forward. When in doubt, yield until you are sure it is safe.
6. Based on ANSI/RESNA testing, Sunrise Medical recommends the use of a caster wheel with a minimum diameter of 5" (127mm), if the wheelchair will be overcoming obstacles up to 1/2" (12.7mm) on a regular basis.
7. Your chair is designed for use on firm, even surfaces such as concrete, asphalt, indoor flooring, and carpets.
8. Do not operate your chair in sand, loose soil, or over rocky terrain.
9. If you use your chair on terrain that is rougher than described above there is a danger that screws and bolts will loosen prematurely, and that damage to wheels or axles could put the rider at risk of a fall, tip-over, or loss of control.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

H. MOTOR VEHICLE SAFETY

⚠ WARNING

As indicated on the front cover of this user manual, identify whether your device has been manufactured with the transit option installed. If your device is NOT equipped with the transit option:

1. NEVER use the mobility device seating system as a car seat.
2. ALWAYS move the rider to an approved vehicle seat.
3. ALWAYS secure the rider with proper motor vehicle restraints.
4. In an accident or sudden stop the rider may be thrown from the mobility device. Mobility device pelvic positioning belts will not prevent this and further injury may result from these belts or straps.
5. NEVER transport this mobility device in the front seat of a vehicle. This action could risk serious injury or death to the dependent rider and driver.
6. ALWAYS secure the unoccupied mobility device so that it cannot roll or shift when being transported in a vehicle.
7. Do not use this product if it has been involved in a motor vehicle accident. If your device is equipped with the transit option, refer to Section VII. "Transit Option and Use".

I. FOR CAREGIVERS

⚠ WARNING

1. Work with the child's health care professional to learn safety methods best suited to your abilities and those of your child.
2. To prevent injury to your back, use good posture and proper body mechanics when lifting, or tilting your child.
3. Remind the dependent to lean back when you tilt the mobility device backward.
4. When you descend a curb or single step, slowly lower the mobility device in one easy movement.
5. NEVER leave the dependent child alone.
6. When making adjustments to the mobility device, always engage the parking brake and lock caster wheels to help minimize unintended movement.
7. Make sure that the attendant only holds the wheelchair using securely mounted parts (e.g. not on the footrests).
8. Never lift the chair with child sitting in the chair.

VI. WARNINGS: FALLS & TIP-OVERS

A. CENTER OF BALANCE

WARNING

Dependent mobility devices are designed for stability and performance. The point where this device will tip forward, back, or to the side depends on its center of balance and stability. The center of balance is also affected by:

- The seat height and seat angle.
- Backrest angle.
- Changes in your child's body position, postures or weight distribution.
- Pushing the mobility device on a ramp or slope.
- The use of a backpack or other options, and the amount of added weight.
- The use of a vents, batteries, oxygen and other options, and the amount of added weight that these options add to the device.

To maintain the proper center of gravity and good balance:

- Consult your healthcare advisor and/or supplier to help you in setting up the mobility device correctly.
- Consult your authorized supplier or healthcare advisor BEFORE you modify or adjust the mobility device.

B. DRESSING OR CHANGING CLOTHES

WARNING

Your child's weight may shift if you dress your child or change your child's clothes while they are seated in the mobility device.

To reduce the risk of a fall or tip over while changing clothes or diapers:

- Rotate the front casters until they point forward.
- Engage the parking brake.

C. OBSTACLES

WARNING

Obstacles and road hazards (such as potholes and broken pavement), can damage your mobility device and may cause a fall, tip over, or loss of control.

To avoid these risks:

- Scan the area well ahead for danger as you travel with your mobility device.
- Make sure floor areas ahead are level and free of obstacles.
- Install a ramp at entry or exit doors. Make sure there is not a drop off at the bottom of the ramp.

D. REACHING OR LEANING

WARNING

If your child reaches or leans it will affect the center of balance of the mobility device. This may cause a fall or tip over.

- NEVER allow your child to reach or lean if they must rise up off their seat for the action.
- If your child must reach, move the mobility device as close as you can to the object, and rotate the front casters until they are as far forward as possible.

NOTE: To do this: Move your mobility device past the object that the child might want to reach, then back up alongside it.

E. RAMPS, SLOPES AND SIDEHILLS

WARNING

Pushing the mobility device on a slope, including ramps or hills, will change the center of balance. Some safety reminders include:

1. Do not push this product on a slope steeper than 10%. (A 10% slope means: one meter in elevation for every ten meters of slope length).
2. Do not turn or change direction on a slope or ramp.
3. For your child's safety, ramps at home and work must meet all legal requirements for your location.
4. Always stay in the CENTER of the ramp.
5. Use a ramp with guardrails (or raised borders at least 76 mm, (3"), high.
6. Always go straight up and straight down on hills or ramps.
7. Do not stop on a steep slope.
8. Watch for a drop-off at the bottom of a slope or ramp.
9. Do not engage the parking brake to slow or stop your mobility device.
10. Beware of wet or slippery surfaces, especially on slopes.
11. Do not recline or tilt the shell while going up hills.
12. Ask for help any time you are in doubt of stability.

F. TRANSFERS

WARNING

1. Work with your health care advisor to learn safe transfer and lifting methods.
2. Have someone help you until you know how to do a safe transfer of the dependent on your own.
3. Move your child's mobility device as close as you can to the location you are transferring to.
4. Rotate the front casters until they point forward.
5. Engage the parking brake before you transfer. This keeps the device stable during the transfer.
6. When transferring a child into the device, make sure they are placed as far back onto the seat surface as possible. This will reduce the risk that the mobility device will tip over and/or move away from you.

VI. WARNINGS: FALLS & TIP-OVERS

G. NEGOTIATING A CURB OR SINGLE STEP

 **WARNING**

Follow the instructions below for descending or climbing a stair or curb. Do not try to climb a high curb or step more than 100 mm, (4"), high, if the accessory tray, or rear glider platform is installed. Doing so can risk injury to the caregiver, may cause the device to exceed its balance point and tip over, and may cause damage to your mobility device.

To descend a curb or single step going BACKWARD:

1. Stay at the rear of the mobility device.
2. Several feet before you reach the edge of the curb or step, turn the device around and pull it backward.
3. While looking over your shoulder, carefully step back until you are off the curb or stair and standing on the lower level.
4. Pull your mobility device toward you until the rear wheels reach the edge of the curb or step. Then allow the rear wheels to slowly roll down onto the lower level.
5. When the rear wheels are safely on the lower level, tilt the mobility device back to its balance point. This will lift the front casters off the curb or step.
6. Keep the mobility device in balance and take small steps backward. Turn the mobility device around and gently lower front casters to the ground.

To climb a step or curb going FORWARD:

1. Stay behind the mobility device.
2. Face the curb and tilt the mobility device up on the rear wheels so that the front casters clear the curb or step.
3. Move forward, placing the front casters on the upper level as soon as you are sure they are past the edge.
4. Continue forward until the rear wheels contact the face of the curb or step. Lift and roll the rear wheels to the upper level.

H. CLIMBING STAIRS

 **WARNING**

1. Do not climb stairs if your mobility device has an accessory tray installed.
2. Use at least two attendants when moving the occupied mobility device up stairs.
3. The rider and mobility device should be facing BACKWARD when moving up the stairs.
4. ALWAYS carry the mobility device by holding the FRAME, being careful to avoid grabbing the footrest, wheels, tray parts, and other areas that are meant to detach from the mobility device.
5. The person at the rear should be in control. He or she should tilt the mobility device back to its balance point.
6. A second attendant at the front firmly grasps a non-detachable part of the front frame and lifts the mobility device up and over the stair.
7. The attendants should move up one stair at a time until the landing is reached.

I. DESCENDING STAIRS

 **WARNING**

1. Do not descend stairs if your mobility device has an accessory tray installed.
2. Use at least two attendants to move an occupied mobility device down stairs.
3. Move the mobility device with both the device and rider facing FORWARD down the stairs.
4. ALWAYS carry the mobility device by holding the FRAME, being careful to avoid grabbing the footrest, wheels, tray parts, and other areas that are meant to detach from the mobility device.
5. The person at the rear is in control. He or she tilts the mobility device back to the balance point of the rear wheels and rolls it to the edge of the top step.
6. A second attendant stands on the third step from the top and grasps the frame. He or she lowers the device one step at a time by letting the rear wheels roll over the stair edge.
7. The attendants should move down one stair at a time until the landing is reached.

J. ESCALATORS

 **WARNING**

NEVER use this mobility device on an escalator, even with an attendant.

 **WARNING**

This early intervention device has been tested and approved using Sunrise accessories. Any accessory that has not been expressly approved by Sunrise Medical will void your warranty, and may add a safety risk to attendant and child.

VII. WARNINGS— COMPONENTS & OPTIONS

A. BASE FRAME

⚠ WARNING

Inspect cables and engage/release buttons before every use to make sure they are in good working condition. Be sure that your base frame is locked before seating your child.

B. UPPER EXTREMITY SUPPORT TRAY

⚠ WARNING

Before adding or removing a tray to the mobility device be sure to point front casters to the front of the device and engage parking brake. This keeps the wheels from rolling when attaching the tray. The support tray has a weight capacity of 23 kg, (50 lbs).

1. Do not lift this product by its tray support arms.
2. ALWAYS make certain that the tray is fully locked into place after adding it to the dependent mobility device.

C. FASTENERS

⚠ WARNING

Many of the screws, and nuts on this product are considered to be special high-strength fasteners. They require periodic checks and maintenance to ensure the safety of both the user and attendant.

1. Most adjustments that require removing hardware should be done by your Service provider.
2. Only use fasteners provided by an authorized supplier (or ones of the same type and strength), as indicated by the markings on the heads.
3. Do not over- or under-tighten fasteners. If screws or nuts become loose, tighten them as soon as possible.

D. FOOTREST

⚠ WARNING

1. Adjustment of the footrest may cause interference with the front lower frame in some positions. This interference can affect the comfort and proper support for the seated dependent.
2. Avoid putting extra weight on the footrest. The mobility device may tip forward (or backward if seating system has been reversed).
3. Never lift this mobility device by the footrests.
4. Do not stand on the footplate during transfer. The chair is likely to tip to the front when standing on the footplate.
5. Always place your feet securely on the ground for transferring into or out of the chair.

E. TYRES AND WHEELS

⚠ WARNING

1. Yearly tire inspection for excessive wear is suggested to ensure good traction in all types of environments.
2. When installing front or rear wheel/tire assembly, slide the wheels into the axle opening until the quick release button pops out, and you hear a click. Your wheel is now fully engaged and locked.
3. In case of tyre damage contact your health care professional to get a replacement.

VII. WARNINGS— COMPONENTS & OPTIONS

F. PARKING BRAKE

⚠ WARNING

Never engage the parking brake to slow or stop this product when it is moving. To keep the rear wheels from rolling, always engage the parking brake fully when you transfer your dependent or the dependent seating system.

G. UNDERCARRIAGE STORAGE

⚠ WARNING

This mobility device's storage basket was designed for the addition of up to 15lbs, (7 kg), of additional weight. Excessive weight may cause an unstable or hazardous condition to exist.

H. ACCESSORY TRAYS

⚠ WARNING

The accessory trays should not be used to store excessive weight, or a piece of equipment that has not been approved for use by Sunrise Medical. This may cause an unstable or hazardous condition for the rider or attendant.

I. CANOPY

⚠ WARNING

When using the canopy, make sure that it is secure by checking the clamps that hold it to the base.

J. SEATING SYSTEMS

⚠ WARNING

DO NOT change the seating system, or individual components of your dependent mobility device unless you have first consulted your health care professional.

K. PELVIC POSITIONING BELTS

⚠ WARNING

Use positioning belts ONLY to help support the child's posture.

1. Make sure the rider does not slide down in the mobility device seat. The belts must be snug, but must not be so tight that they interfere with breathing. You should be able to slide your open, flat hand, between the belt and the rider.
2. Use positioning belts with a cooperative rider only. Make sure that you can remove the belts in an emergency.
3. NEVER use the positioning belts:
 - As a restraint.
 - On a child who is comatose, agitated or is prone to seizures.
 - As a motor vehicle restraint— except those that are included with the transit option for the purpose of occupant restraint.

L. CUSHIONS

⚠ WARNING

This mobility device's seating was designed for comfort and postural support. If seating component modifications are necessary, consult your healthcare advisor.

M. TILT AND RECLINE

⚠ WARNING

Using the tilt or recline adjustments triggers before taking the necessary precautions may effect the balance and stability of your device. BEFORE making these adjustments you must be aware of the following:

1. The child must be secure in the device.
2. You must rotate the front casters forward, and engage the parking brake.
3. ALWAYS make certain that the dependent mobility device is level and stable before and after adjusting the tilt and/or recline.
4. ALWAYS support seating shell when releasing the tilt trigger.

N. TRANSIT OPTION AND USE Fig.1.0 - Fig.3.0

NOTE: The Zippie Voyage has passed transit testing and is considered to be ISO 7176-19 compliant for a dependent child weighing at least 22 kg and up to 34 kg. In transit, the occupant must be secured within the vehicle and anchored with a 3 point harness, with transit straps secured to the front and rear transit loops(A).

⚠ WARNING**USER WEIGHT LESS THAN 22 kg**

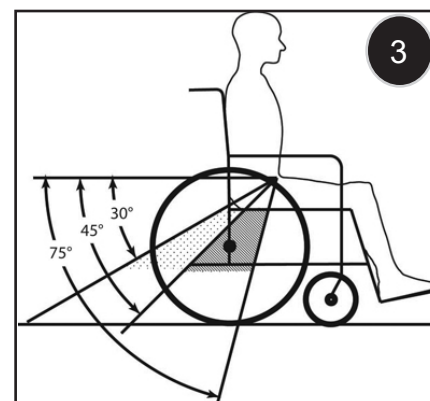
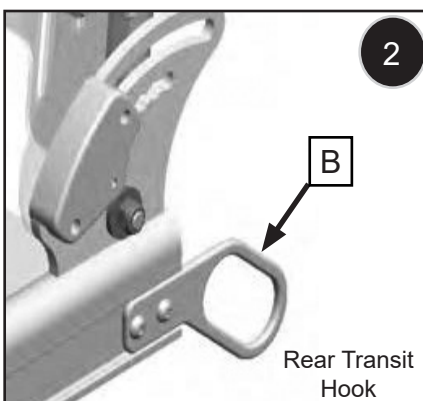
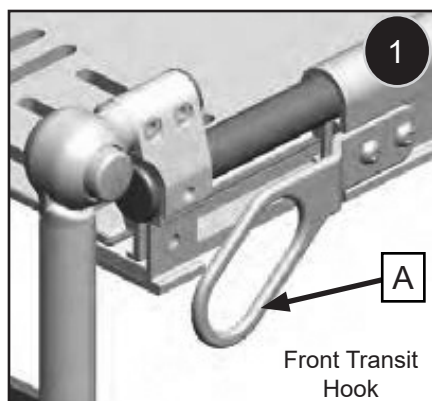
When the user being transported is a child, less than 22 kg mass and the vehicle involved has less than eight (8) seated passengers it is recommended that they be transferred to an UNCE Regulation 44 compliant child restraint system (CRS). This type of restraint system provides a more effective occupant restraint system than the conventional 3 point occupant restraint system and some CRS systems also include additional postural supports to assist in maintaining the position of the child when seated.

Parents or care providers may consider the option, in some circumstances, for their child is to remain in their wheelchair whilst in transport due to the level of posture control and comfort provided by the set up in the wheelchair.

We would recommend in such circumstances that a risk assessment be carried out by your attendant and relevant competent persons.

VII. WARNINGS– COMPONENTS & OPTIONS

1. The occupant must be facing the front of the vehicle during transport.
2. Wheelchair Tiedown and Occupant Restraint Systems (WTORS) must be used.
3. A Wheelchair Tiedown that has been tested in accordance with ISO 10542 must also be used in any transit situation.
4. The Occupant must be restrained with a vehicle anchored 3-point belt.
5. For all transit situations, you must use the Transit Strap provided with your device (1). It secures the seating shell to the rear frame. It is critical that the strap be threaded thru the 2 "D" clips as shown in (figure 1) for the safety of the occupant.
6. Mobility device mounted accessories, such as trays and respiratory equipment, and batteries, should be removed, and secured separately. The optional Voyage vent tray, oxygen holder, and IV pole must also be removed and secured separately before using as a seat in a motor vehicle.
7. The occupant must not weigh more than 75 lbs. (34 kg).
8. The location(s) of the front and rear securement points (A) are shown in (figure 2 and marked with a transit point label for clear identification.
9. Use only with WTORS that have been installed in accordance with the manufacturer's instructions and ISO 10542. Attach WTORS to the securement points (A) in accordance with the manufacture's instructions and SAE J224
10. The angle of the pelvic belt should be within the preferred zone of 45 to 75 degrees to the horizontal, or within the optional zone of 30 to 45 degrees to the horizontal (figure 3)
11. The pelvic belt should be worn low across the front of the pelvis. Position upper torso belts over the shoulders. The belt(s) should not be held away from the body by wheelchair components. Ensure that the belts are not twisted.
12. If a recline angle is required during transport, the chair seat/back should be positioned at an angle of no more than 30° to the vertical, or the occupant is at risk of injury in the event of an accident or collision.
13. Attach all occupant restraints in accordance with the manufacturer's instructions and ISO 10542.
14. Remove canopy, belly bar, tray, underframe tray and buggy board for transit.
15. Remove any content out of the basket.


Look for this symbol on your wheelchair. It indicates wheelchair securement points which conform to ISO 7176-19 (2008).

A. UNFOLDING AND FOLDING BASE FRAME

⚠ WARNING

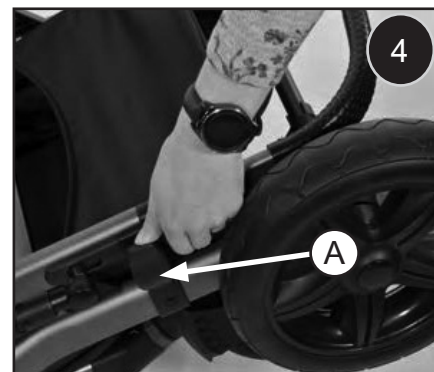
1. Zippie dependent mobility devices are designed for stability and performance. When unfolded, the device must be fully open and locked to avoid injury to the dependent rider.
2. Follow the instructions for unfolding the device carefully, and double-check that the device is fully locked before adding the seating shell and rider.
3. Be careful not to trap or pinch your fingers.

Unfolding:

4 5 6

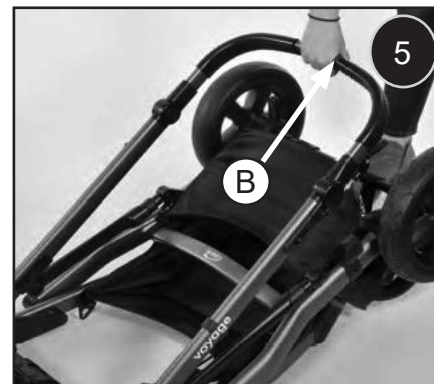1. Unsnap the storage lock (A) from around the folded frame. It should remain attached to the frame for future use.
2. Pull up on the upper frame (B) until it looks into place. It may be necessary to put your foot on the bottom bar while pulling up on the upper frame (Fig. 6).
3. Rear wheels and front casters can now be attached if they are not attached already.

Folding:

7 6 4

⚠ WARNING

The Voyage is not intended to be folded with the seating system installed. ALWAYS remove the seating system prior to folding the base. Failure to do this may damage the product.

1. Remove the child from the seating shell, then remove the seating shell (refer to subsection (F))
2. Unlock the front caster wheels.
3. Return handle to the straight position as show in Fig. 7.
4. Set parking brake.
5. Pull up on both triggers (D) at the same time. This action will release the frame lock. Continue pulling up on the triggers while pushing down on the upper frame.
6. Push handle down to begin folding of the base.
7. Lift on the center tube of the Slide-N-Lock (C). Fig. 6.
8. Snap the storage lock (A) around the upper frame to prevent the frame from opening when transporting, or storing the frame.

NOTE: Rear wheels, or front casters do not have to be removed for this procedure, but may be removed to save space for storage.

If you decide to leave the front casters on the device, it may not lock correctly due to interference between the caster wheels and the push handle.

In most cases you should fully retract the push handle before folding, then you will be able to snap the storage lock easily. Do not force the base open or closed if it sticks during unfolding or folding, damage could occur.

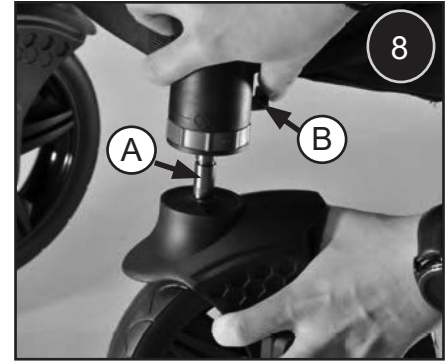
VIII. SET-UP & USE

B. CASTER WHEELS**Installing and Removing the Caster Wheels:**

8

1. Locate the caster stem guide hole location on the bottom of the Voyage frame. Line up the caster stem (A) with the hole, while pushing in tab (B) insert the caster stem until it clicks into place. Check the secure engagement to the base by pulling on the caster wheel. If it does not move the caster wheel has been installed properly. Repeat this procedure for other side.
2. To remove the caster wheel, press in tab (B) and pull the caster wheel out.

**Caster Wheel Locks:**


9

1. To lock the caster wheel, locate the lock ring (C). Turn the lock ring clockwise until it clicks into place. Do this with both of the front caster wheels, and check to insure that the lock has been engaged.
2. To unlock the caster wheel turn the lock ring counter clockwise until the wheel freely rotates.

**C. REAR WHEELS****Installing and Removing the Rear Wheels:**

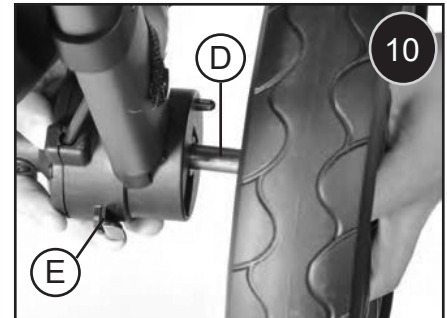10


1. To install rear wheel, gently slide the rear wheel's axle (D) into the hole in the frame until the wheel clicks into place.
2. To remove the rear wheel, simply press the wheel button (E) while gently sliding the wheel away from the frame.

**D. PARKING BRAKE OPERATION****Engage or Disengage the Parking Brake:**

11

12

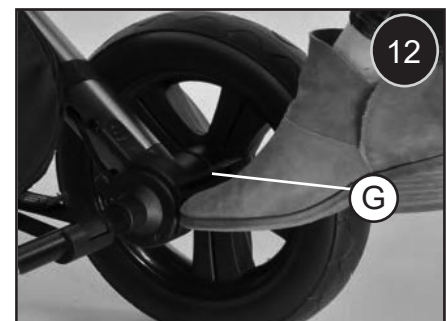1. Engage the parking brake by gently pushing down with your foot on the brake foot pedal (F).
2. To release the parking brake, push forward on vertical lever (G).

E. SEATING SHELL INSTALLATION

⚠ WARNING

BEFORE installing the seating shell, always check that the mobility device is completely unfolded and in the locked position, and that all four wheels are locked in place. The seating shell should never be installed with a seated occupant. Always remember to stabilize the mobility device before performing this action.

- Rotate the front casters until they point forward.
- Engage the parking brake.

The seating shell has a latch mounted underneath the seat plate. The interface on the base is the counterpart to the latch and mates with the latch. The seating shell may be installed so that the child can face either frontwards or rearwards. Installation is the same in either direction.

Using the latch lock:

13

14

NOTE – The latch under the seat plate has a lock that can be used to provide additional securement after the seat is installed.

1. BEFORE attaching or removing the seat to or from the base, the latch lock (K) MUST be slid to the right (unlocked position) as shown in Fig 14.
2. To prevent inadvertent activation of the seat latch, slide the latch lock to the left as shown in Fig 15 AFTER the seat is fully secured to the base (you cannot attach seat to base while lock is engaged).

Install the seating shell:

15

1. Make sure the seat is in the fully upright position (no recline or tilt).
2. Ensure all positioning straps and accessories are out of the way and will not interfere with the seat from fully engaging the receiver.
3. Line up the alignment tag (D) with the support cross bar (E) on the base (fig. 15)
4. Lower the seating shell so that the seating shell and mounting plate are in contact.
5. Slide the seat backwards evenly. You should hear an audible click. Check the secure engagement to the base by pulling up on both sides. If it does not move or tilt, the seating shell is now locked and has been installed properly.
6. The seating shell is now ready to accept an occupant for transportation and/or adjustment.

F. SEATING SHELL REMOVAL

⚠ WARNING

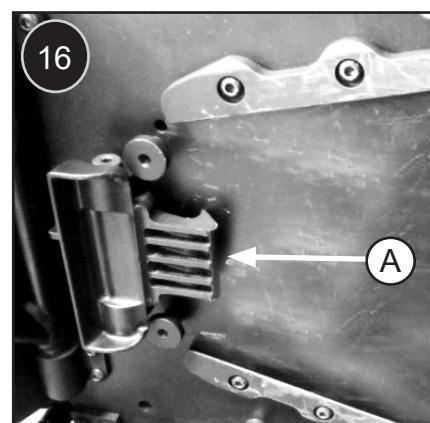
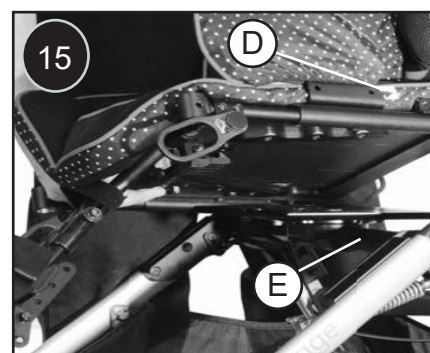
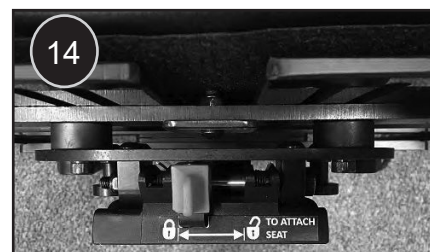
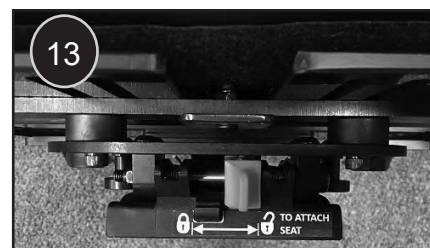
BEFORE removing the seating shell, always remove the occupant to a safe location. Failure to do so may result in injury to the dependent, and/or caregiver. Always remember to stabilize the mobility device before performing this action.

- Rotate the front casters until they point forward.
- Engage the parking brake.

Removal of the seating shell:




16

1. Make sure the seat is in the fully upright position (no recline or tilt)
2. Pull and hold release lever so that it disengages (A) from the receiver, slide the seating shell forwards and lift off once the seating shell has been disengaged.
3. The seating shell is now ready to be inspected and/or stored.

Rotating the seat shell direction: 17 18 19

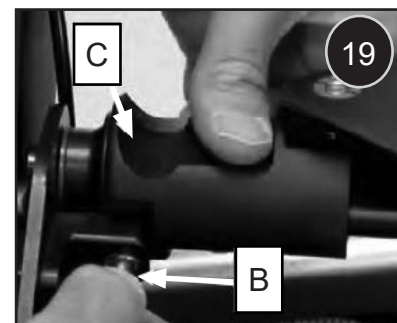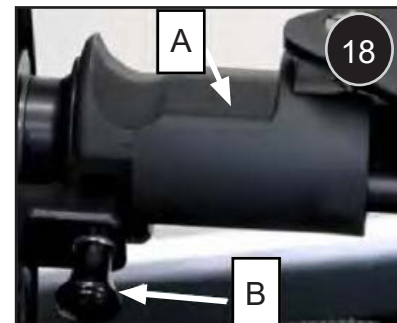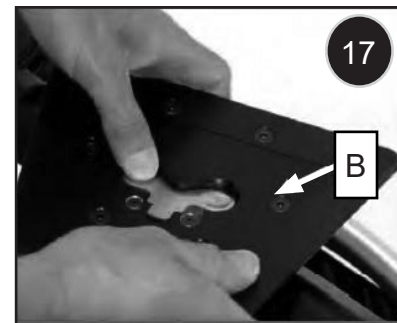1. Lock the wheels, remove the seat.
2. Be sure the Tilt setting is at the "home" or 0° position. Do this by squeezing the Tilt Release lever and letting the "Slide-n-Lock" (Receiver) plate spring flat.
3. STEP 1: Press the gold button, on the Slide-n-Lock plate and rotate the plate until it stops (Fig 18).
4. STEP 2: Pull the Pin (B) with one hand and rotate the Tilt Limiter Sleeve (C) with your other hand* (Fig 19).
5. -NOTE: Only rotate the Tilt Limiter Sleeve in the process of rotating the Plate.
6. *When the seat is forward facing the Tilt Limiter Sleeve should be rotated rearward, when the seat is rearward facing the Sleeve should be rotated Forward.
7. STEP 3: Finish rotating the Plate 180° until it locks into place.

**Insert or remove the Grab Handle:** 20

Always stabilize the mobility device before performing the action.

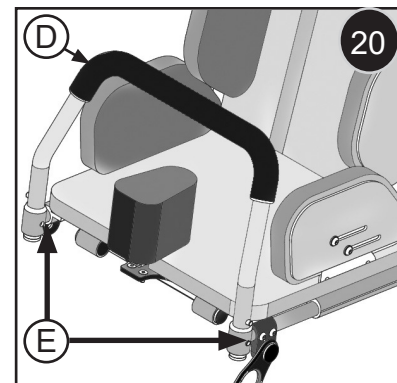
- Rotate the front casters until they point forward.
- Engage the parking brake.

8. Be careful to avoid pulling or crimping any life support tubing that may be attached to the child. This adjustment can create a pinch point, so be careful to remove any obstruction, and/or clothing that could get stuck in this area.
9. The adjustment is made from the front of the device. Depress both detent pins (E) simultaneously, and either insert or remove the Grab handle (D).
10. If you are inserting the Grab Handle (D) push the handle in far enough so that the detent pins pop out of the locking hole (E).
11. Pull up on the handle to make sure that it is locked into place.
12. Depending on preference, the grab bar is reversible.

G. TILT IN SPACE ANGLE ADJUSTMENT**⚠ WARNING**


BEFORE using the tilt adjustment, always check that the mobility device is completely unfolded and in the locked position, and that all four wheels are locked in place. Be careful to avoid pulling or crimping any life support tubing that may be attached to the child. Always remember to stabilize the mobility device before performing this action.

- Rotate the front casters until they point forward.
- Engage the parking brake.

Tilt adjustment: 21

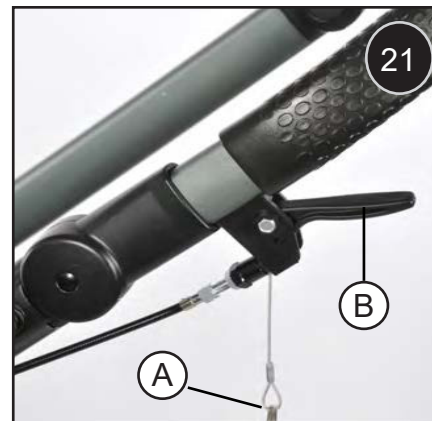1. Remove the safety pin to allow tilt. Grasp the tilt lever (A) on the left side of the frame, and squeeze, and then pull back on top of the seating shell to achieve desired angle adjustment. The back support and seating support components, including the footrest should move together.
2. Release the handle and gently pull on the seating shell to make sure it is locked in the new position.
3. This adjustment can change the balance of the device slightly so it is good to check that there is no danger of tipping with the new setting


**H. RECLINE ADJUSTMENT****⚠ WARNING**

BEFORE using the recline adjustment, always check that the mobility device is completely unfolded and in the locked position, and that all four wheels are locked in place. Be careful to avoid pulling or crimping any life support tubing that may be attached to the child. Always remember to stabilize the mobility device before performing this action.

- Rotate the front casters until they point forward.
- Engage the parking brake.
- If equipped with shoulder harness, loosen the shoulder straps to allow room for the recline.

Recline adjustment: 22

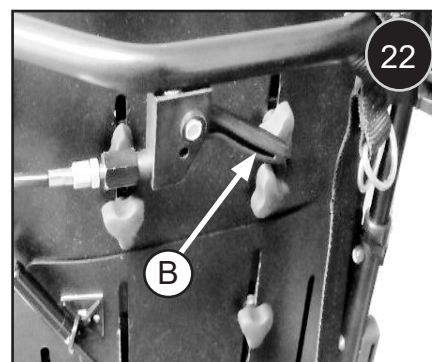1. Remove the safety pin from the recline lever. From behind your mobility device, grasp the recline lever (B). Squeeze the lever then pull on the back support area. (The back should move independently of the seating support and footrest). When desired angle adjustment has been reached, release the handle and gently pull on the seating shell to make sure it is locked in the new position.
2. This adjustment can change the balance of the device slightly so it is good to check that there is no danger of tipping with the new setting.
3. If equipped with shoulder harness, re-adjust the straps for proper fit.


**I. KNEE ANGLE ADJUSTMENT****⚠ WARNING**

Always stabilize the mobility device before performing this action.

- Rotate the front casters until they point forward.
- Engage the parking brake.

Knee angle adjustment: 23

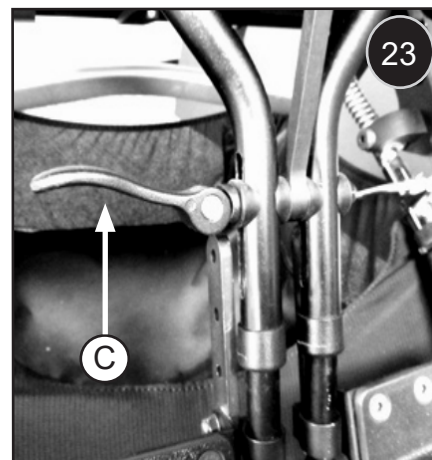1. Be careful to avoid pulling or crimping any life support tubing that may be attached to the child. This adjustment can create a pinch point, so be careful to remove any obstruction, and/or clothing that could get stuck in this area.
2. Release the quick release lever, (C) and adjust the knee angle to the proper angle.
3. Lift up the quick release lever to lock it properly after every adjustment.

**J. STEP TUBE**

The Step Tube is used by attendants to push the chair over an obstacle. Simply step on the tube to push a wheelchair, for example, over a kerb or step.

VIII. SET-UP & USE

K. CANOPY INSTALLATION AND ADJUSTMENT

⚠ WARNING

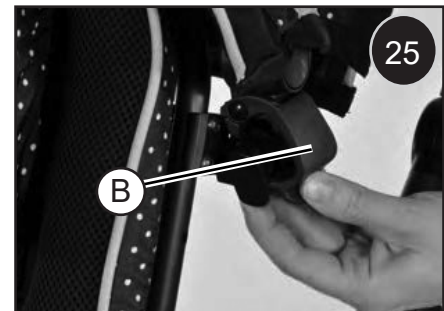
It is always best to make sure the dependent is not seated in the mobility device during the installation of accessories.

Canopy installation:

24

25

1. From behind your mobility device, attach the canopy to the seating shell receivers (A).
2. Check that the canopy has been installed correctly by visual inspection and checking for looseness, or an incorrect attachment. The plastic tabs (B) must be locked into place for safe operation.

**Canopy adjustment:**

26

1. Once installed, you can adjust the canopy angle by pushing it forward or backward until desired protection is obtained.
2. You can also loosen the button-head screws (C) and rotate the Canopy Brackets (D) for more clearance or coverage for the occupant. Make sure to adjust both sides evenly when adjusting the rotating canopy brackets.
3. Lift the front of the canopy all the way up for clearance and access to the child.
4. When canopy is being used with the tilt and recline adjustments, it may be necessary to angle the canopy either forward or rearward for better clearance with the push handle.

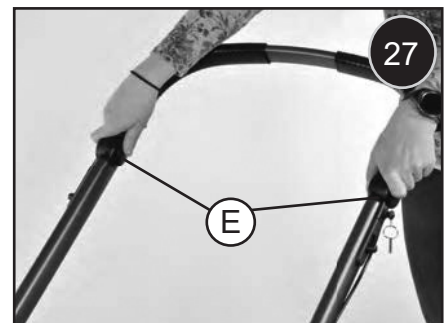
L. TELESCOPING STROLLER HANDLE EXTENSION

To adjust the Handle:

27

The handle height is adjustable for pushing comfort.

1. The adjustment is made from the back of the device. Depress both stroller handle pivot buttons (E) simultaneously, and adjust to desired angle.
2. Release the buttons and allow the stroller handle to lock into place.

A. POSITIONING AND GROWTH ADJUSTMENTS

⚠ WARNING

If a change in adjustment is necessary for growth or positioning, a healthcare professional must be consulted before any changes are made. These adjustments are very important to the orthopedic and neurological needs of the child. Care should be taken to make sure the changes are physiologically and medically appropriate. Sunrise Medical suggests that any positioning or growth adjustments be made by your service provider. Do not allow any adjustments while the child is seated in the mobility device.

Torque Settings:

A torque setting is the optimum tightening which should be made on a particular fastener. It is important to use proper torque settings where specified. When not specified, torque settings should be: M5 fasteners = 44 in-lbs (5 Nm), M6 fasteners = 62 in-lbs (7 Nm), M8 fasteners = 88 in-lbs (10 Nm). For all others torque setting should be 60 in-lbs (6.8 Nm).

Tools you will need: **28**

1. Imperial Hex key set
2. Metric Hex key set
3. Imperial Open-end Wrench set
4. Metric Open-end Wrench set
5. Torque wrench
6. Imperial Hex bit Socket set
7. Metric Hex bit Socket set
8. Phillips and Flat screwdrivers

NOTE: The torque wrench must measure inch-pounds. You may buy a torque wrench and proper metric sockets from any local hardware store.

In general, positioning and growth adjustments should be done in the following order:

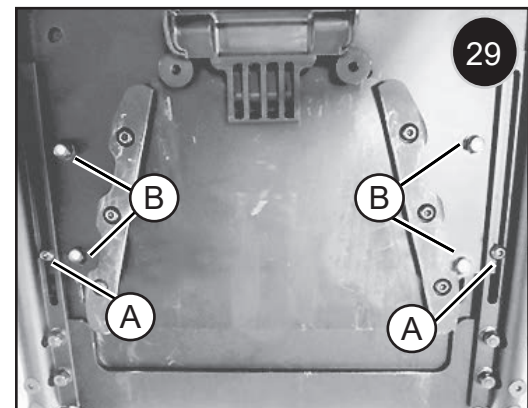
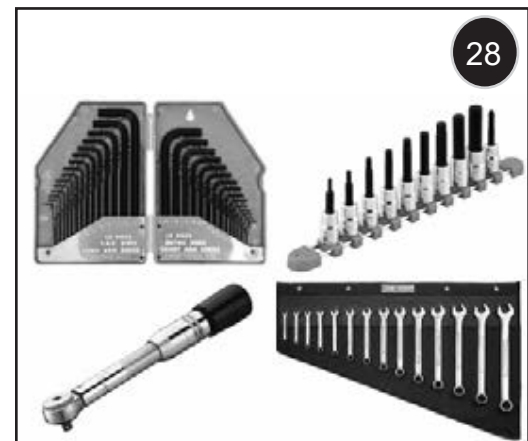
1. Adjust seated area
2. Adjust back support area
3. Adjust pelvic positioning belt
4. Adjust footrest
5. Adjust head support(s)
6. Make other standard and optional adjustments that are needed
7. Check to be sure that all adjustment screws are torqued to specifications

B. COMFORT SEAT ADJUSTMENT

Comfort seat depth adjustment: **29**

NOTE: Seat Plate viewed from underneath.

1. Remove the seat cushion. It is attached with Velcro®.
2. Loosen the screws (A) on both sides (do not remove).
3. Loosen four nuts (B); hardware may need to be removed for certain adjustments.
4. Adjust the depth by sliding front seat pan forwards or backwards.
5. Re-tighten fasteners (A) & (B) to (75-85 in-lbs / 8,5-9,5 Nm).
6. Place seat cushion back on the seat.


⚠ WARNING


- For ease of adjustment and the safety of the dependant, remove the occupant before any adjustment is made.
- Always stabilise the mobility device before performing these actions.
- Adjusting the back height can alter the overall centre of gravity. Check that the mobility device is stable and won't tip at the new setting.

Comfort back height adjustment: 30

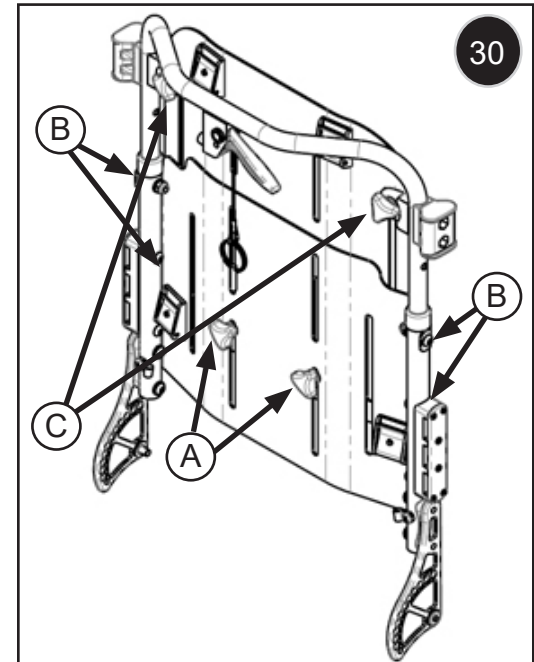
1. Rotate the front castors until they point forwards.
2. Engage the parking brake.
3. Remove backrest cushion.
4. For quick adjustment, 100 mm, (4"), loosen, or remove if necessary, four knobs (A) and (C) on the back.
5. If laterals are present, loosen those attachment knobs as well.
6. Adjust the height by sliding upper backrest shell up or down.
7. Re-tighten the knobs (A) and (C).
8. For additional height, (more than 100 mm), adjustment beyond the range of slots, remove the screws on the back canes, (B) on both sides, and loosen bottom two knobs (A).
9. If laterals are present, loosen, or remove if necessary, their attachment knobs as well, or completely remove laterals.
10. Slide the back shell assembly and handle up or down to the next hole. The spacing is 76 mm, (3").
11. Re-tighten fasteners (A) & (B).
12. Replace backrest cushion.

**Comfort medial installation: 31 32**

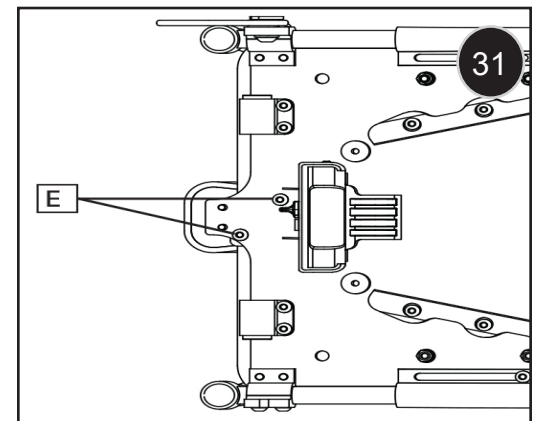1. Position footrest up horizontally, parallel to the floor.
2. Find two holes (E) in seat pan, located on both sides of footrest tube.
3. Seat cushion must be attached in place.
4. Slide metal bracket through slot in the cushion and align holes in the bracket with holes in the seat pan.
5. Attach medial pad (F) to seat pan with screws (G).

**C. SHOULDER STRAP HEIGHT ADJUSTMENT:**

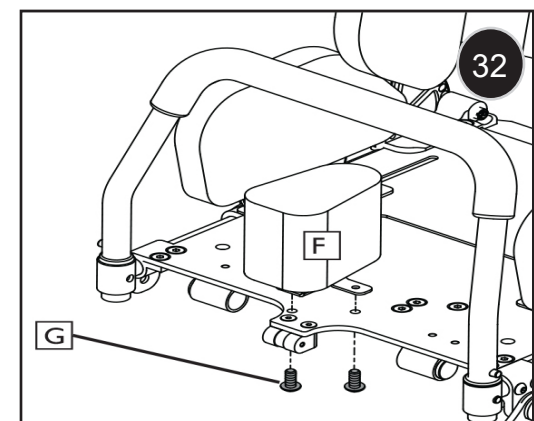1. Loosen, do not remove, the hand wheels on each side on the back plate.
2. Adjust shoulder strap to proper height.
3. Tighten all of the hand wheels (do not overtighten) and then re-adjust the shoulder straps for a comfortable fit.

D. FOOTREST ADJUSTMENTS:

33 34

Height Adjustment:

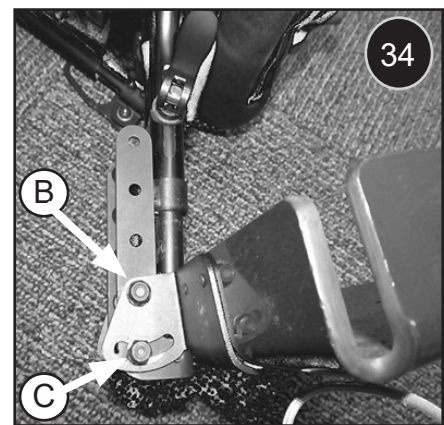
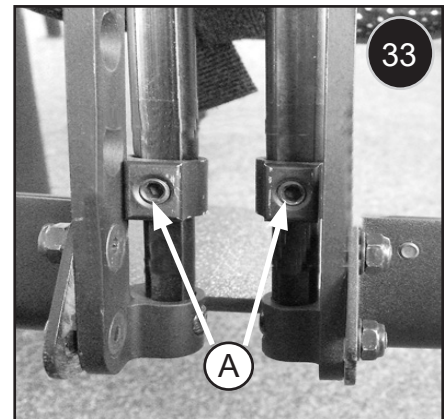
The height adjustment for the footrest consists of one telescoping tube sections which locks the footrest height.

1. Identify adjustment screw locations (A) on the rear side of the footrest hanger.
2. First loosen the upper screws on each side of the footrest hanger.
3. Slide the footrest up or down to proper height.
4. Tighten the upper screw to 1.7 Nm - 2.26 Nm, (15 - 20 in-lbs), do not overtighten.
5. Repeat procedure using lower screw if additional range is required.

NOTE: Footrest is shown inverted for shortest possible leg length. It can be attached either way by flipping the footrest for more height or more secure foot placement.

Angle Adjustment

1. The footrest has two screws (B) and (C) that control footrest rotation. Loosen both screws on each side of the footrest as shown in (figure 30).
2. Adjust the footrest to proper angle.
3. Tighten the adjustment screws to 4.5 Nm - 5.1 Nm, (40 - 45 in-lbs), to maintain proper footrest angle. (Do not overtighten this hardware against plastic parts).

E. FLOOR TO SEAT HEIGHT ADJUSTMENT Table 1

The Voyage Stroller has three floor to seat height positions:
 Low: 23" (590 mm), Med: 24.5" (625 mm), and High: 26" (660 mm) See Table 1.0

Floor to Seat Adjustment:

1. Remove all four screws (A), two on each side of stroller.
2. Position the cross bar tilt assembly to the desired height.
3. Re-install the four screws in the proper position indicated in the images to the right.
4. Tighten the bolts to a torque of 75-85 in-lbs (8.5-9.5 Nm).

LOW: 23" (590 MM)	MED: 24.5" (625 MM)	HIGH: 26" (660 MM)

Table 1

F. PELVIC POSITIONING BELT ADJUSTMENTS: 35**Length Adjustment:**

1. Remove any belt twists and place the belts outside of the seating area.
2. Engage wheel locks, seat the child, and snap the pelvic positioning belt ends together.
3. Adjust belt length to provide a snug fit.
4. The adjuster is to the child's left, (D-Fig.35).

Position Adjustment:

The Belt can be mounted to the Pivot bracket in the rear, or the Hip bracket when needed forward.

⚠ WARNING

- Before using the wheelchair make sure that the pelvic restraint belt is fitted.
- The pelvic restraint belt must be checked on a daily basis to ensure it is free from any obstruction or adverse wear.
- Always make sure that the pelvic restraint belt is correctly secured and adjusted prior to use.

⚠ DANGER

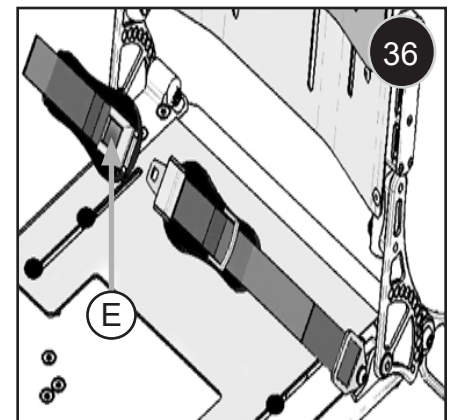
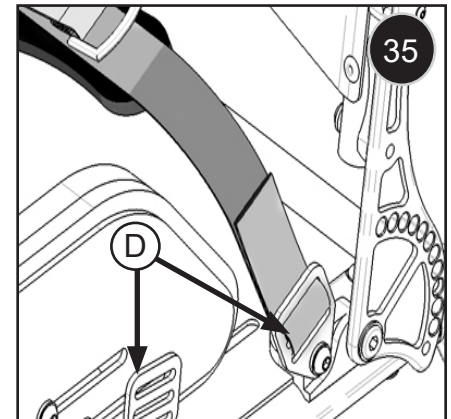
If the strap is too loose, this could cause the user of the wheelchair to slip down and risk suffocation or serious injury.

Adjust the pelvic restraint belt to suit the user's needs as follows: 36 36b

- Engage wheel locks, seat the child and snap the pelvic positioning belt ends together.
- Adjust the belt position so buckles are in the centre of the seat.
- Feed free belt back through male buckle and slide adjusters.
- Ensure belt is not looped at male buckle.
- Feed free belt through slide adjusters and male buckle to provide more belt length.
- When fastened, check space between the pelvic restraint belt and user.
- When correctly adjusted it should not be possible to insert more than the flat of the hand between the pelvic restraint belt and the user.

The pelvic restraint belt should be fixed so that the belt sits at an angle of 45 degrees across the user's pelvis.

The user should be upright and be as far back as possible in the seat when correctly adjusted. The pelvic restraint belt should not allow the user to slip down in the seat.

To fasten buckle:

Firmly push male buckle into female buckle.

To release buckle:

Press the RED button, "E", in the centre of the buckle then pull the two halves gently apart.

NOTE: If you have any questions on the use and operation of the pelvic restraint belt, the please contact your wheelchair dealer, carer or attendant.
If you want to purchase a pelvic restraint belt subsequently, please contact your wheelchair dealer.

⚠ WARNING

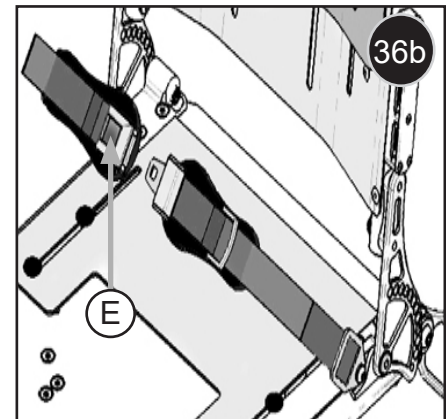
- The pelvic restraint belt must only be fitted or adjusted by an approved Sunrise Medical dealer / agent.
- The pelvic restraint belt must be checked on a daily basis to ensure it is adjusted correctly and free from any obstruction or adverse wear.
- Sunrise Medical does not allow the transportation of a person in a vehicle with the pelvic restraint belt as a restraint system.

Maintenance:

- Check pelvic restraint belt, and securing components, at regular intervals for any sign of frays or damage. Depending on usage, it may need to be replaced.
- If you want to fit a belt subsequently, please contact your wheelchair dealer.

⚠ WARNING

The pelvic restraint belt should be adjusted to suit the end user as detailed above Sunrise Medical also recommends that the length and fit of the belt is checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length.


G. ANTERIOR TRUNK BUTTERFLY SUPPORT

⚠ WARNING

- To avoid serious injury or issues with eating or breathing, the pelvic positioning belt is required to be used in combination with an anterior trunk support. Double-check all adjustments to ensure there is no possibility of obstructing the airway or any other undesired constrictions. Do not use the anterior trunk support without the pelvic positioning belt.
- Sunrise Medical does not allow the transportation of a person in a vehicle with the anterior trunk support as a restraint system.

Harness Adjustment: 37 38

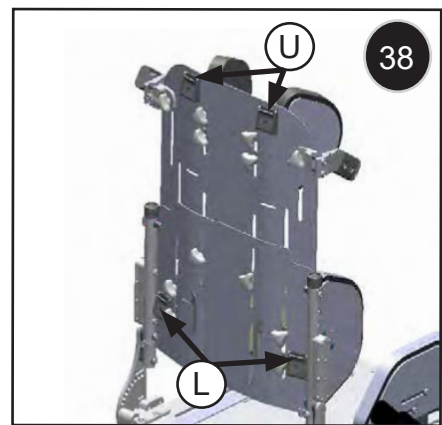- Place the child in the seating system and fasten the pelvic positioning belt.
- Position the child's hips and tighten the belt to the proper position.
- Unlock the two lower cam locks (L) and the two upper cam locks (U).
- First, adjust the lower straps to keep the harness from riding up on the child's torso and lock lower cams (L).
- Then, adjust the upper straps making sure there is enough room around the child's neck for easy breathing and eating, and lock upper cams (U).



H. LATERAL TRUNK AND HEAD SUPPORTS (OPTIONAL)

Lateral and head support height and width adjustments: 39 40

NOTE: The following adjustments may be able to be accomplished with the child present in the seating system.

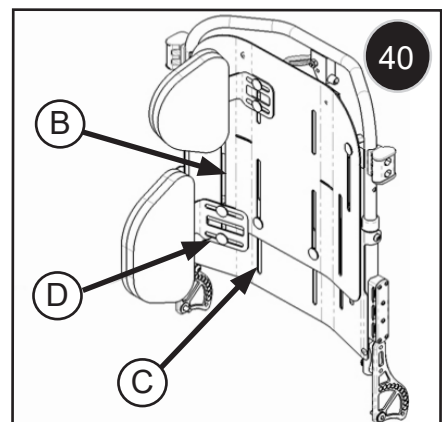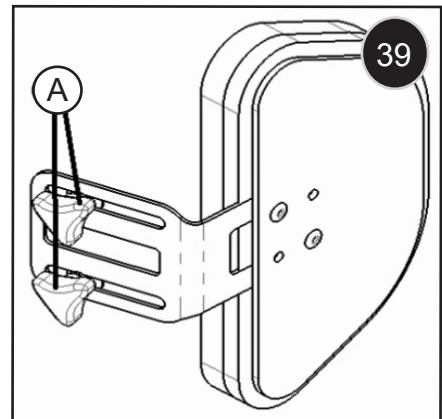- Loosen or remove if necessary, the two knobs (A) from the rear of the shell enough to gain adjustment of the laterals.
- Use shell slots, (outer, "B", or inner, "C"), for height adjustment. Move the laterals up or down to the proper position and tighten the knobs.
- Use lateral bracket slots, "D", for width adjustment or move from outer to inner shell slots. Adjust the lateral brackets for width bringing the lateral assembly to the proper position and tighten the knobs.

I. ANTERIOR SUPPORTS

When deciding if you should use Anterior supports, always consult your healthcare advisor.

Some children, due to their disability level or physical condition, may need anterior supports to help maintain proper posture or body position. Such devices include head straps, shoulder straps, thoracic strap systems, pelvic supports, and belts.

⚠ WARNING

Risks of injury with head supports and how to avoid them.

Excess movement or migration of the pelvis- This may cause the supports to displace. If this occurs, severe injury to the child could result.

To avoid risks:

1. It is VITAL to keep the child in a stable pelvic position.
2. Anterior supports should ONLY be used with seat, back, and pelvic supports or belts that keep the child from sliding down in the seat.
3. Anterior supports must be fitted by your authorized supplier and your healthcare professional.
4. After fitting, anterior supports, they should ONLY be placed and adjusted by trained professionals.
5. DO NOT leave the child unattended when using anterior supports.

J. HIP GUIDES (OPTIONAL): 41

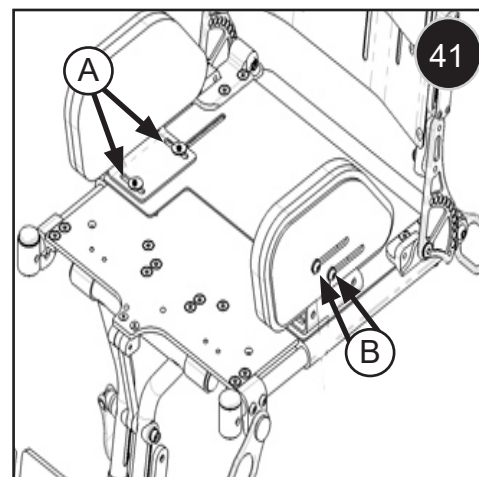
Width Adjustment, (A):

1. Determine the proper location for each hip pad.
2. Remove the seat cushion to expose hip pad adjustment screws.
3. Loosen two adjustment screws for each hip pad just enough to allow adjustment.
4. Adjust hip pads to the desired position.
5. Tighten the adjustment screws and replace the seat cushion.

Forward/Rear Adjustment, (B):

1. Determine the proper location for each hip pad.
2. Remove the cushion of the hip guide to expose hip pad adjustment screws.
3. Loosen two adjustment screws for each hip pad just enough to allow adjustment.
4. Adjust hip pads to the desired position.
5. Tighten the adjustment screws and replace the hip guide cover.

NOTE: Hip pad hardware must be removed for hip widths greater than 10" (254 mm). The foam hip pad attaches beneath fabric at back pivot point to accommodate an 11" (280 mm) hip width.

K. TRAY INSTALLATION AND ADJUSTMENT: 42

1. Slide the tray over the tray supports, adjust depth and click into place.
2. Check that nut (C) is just loose enough so that the tray can be adjusted.
3. Knob (D) is loosened to adjust the depth of the tray and tightened on both sides to hold the adjustment.
4. Adjust the angle of the tray by holding in both knuckle buttons (E) on the support arms and raising the tray to desired angle, the tray should click in place for a positive lock.

⚠ WARNING

Using a pelvic positioning belt is critical for child safety when foot straps or shoe holders must be used for therapeutic positioning needs.

L. FOOT STRAP ADJUSTMENT (OPTIONAL):

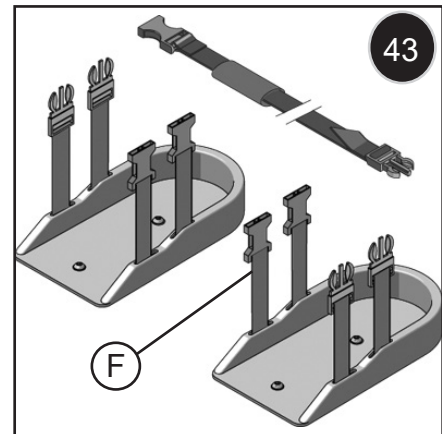
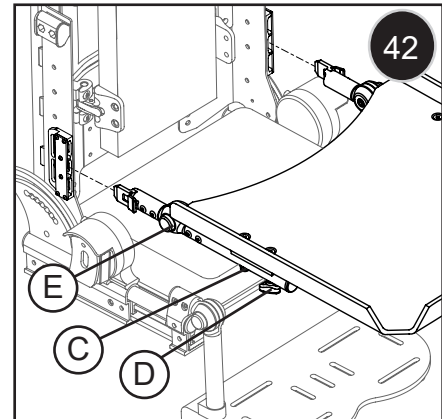
NOTE: Optional foot straps are intended for therapeutic positioning needs only.

1. Engage wheel locks, then seat the child.
2. Unlatch and loosen the two foot straps (F).
3. Place child's foot into position and snap the foot strap into place.
4. Keep the straps as loose as possible and only tighten enough to maintain therapeutic position.

M. SHOE HOLDER STRAP ADJUSTMENT (OPTIONAL): 43


NOTE: Shoe holders are intended only for therapeutic positioning needs.

1. Engage wheel locks and seat the child.
2. Loosen the shoe holder straps (F).
3. Child should be wearing shoes for this step. Place child's foot into shoe holder and snap the two shoe holder straps into place.
4. Keep the straps as loose as possible and only tighten enough to maintain a therapeutic position.

N. VENT/BATTERY TRAYS (OPTIONAL): 44


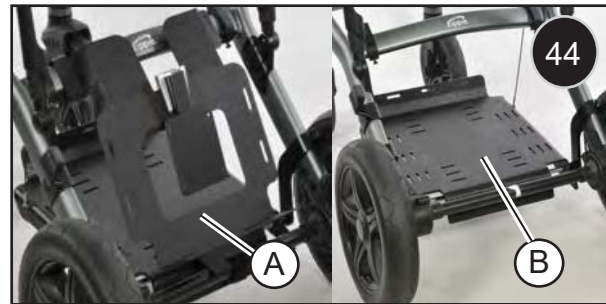
1. There are two types of Vent platforms. A rear mount or dovetail vent mount (A) which has a weight capacity of 15lbs, and an underframe vent tray (B) which has a weight capacity of 50lbs (22.7 kg).
2. The vent platforms attach easily to the Voyage base frame.
3. Tie down straps are included with this option to secure the equipment.
4. The Catch-all (I) is included and folded forward of tray for a smaller storage space.

**O. FRONT OR REAR ADD-ON PLATFORM (OPTIONAL):** 45 46

1. The rear platform (C) can be used for support of a vent, or other equipment that might be needed.
2. The front mount utility platform (D) has a weight limit of 50lbs (22.7 kg).
3. Tie down straps are included with this option to secure the equipment.

**P. IV SUPPORT POLE / O2 TANK HOLDER (OPTIONAL):** 47a 47b

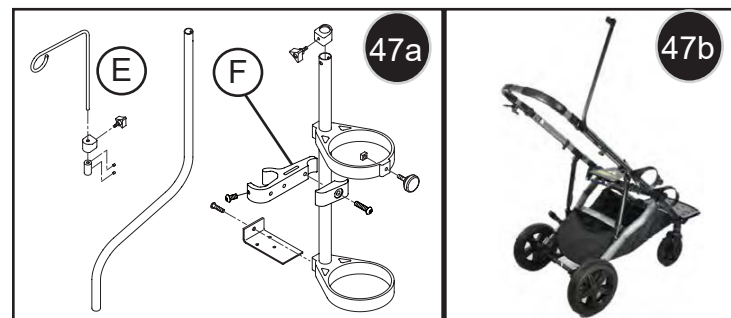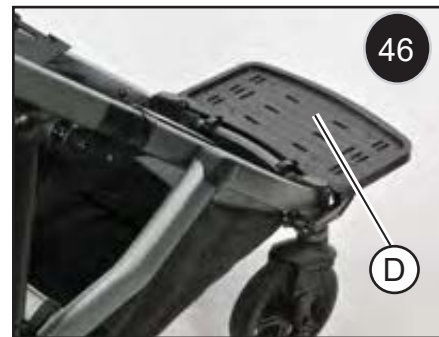1. IV pole (E) and Oxygen holder (F) are both fully adjustable and attach directly to your Voyage base frame using the hardware shown in (figure 47a).
2. The location of the clamp is shown in (Figure 47b).

**Q. CUP HOLDER, UTILITY HOOK (OPTIONAL):** 48

1. Cup holder (G) is easy to attach and lock into place.
2. The utility hook (H) uses a Velcro® attachment, and is usually used to route oxygen hoses, but can also be used for hanging other light items.

A. NOTES

1. Proper maintenance will improve performance and extend the useful life of your mobility device.
2. Clean the device regularly. This will help you find loose or worn parts and will make your device easier to use.
3. To protect your investment, have all major service and repair work done by your supplier.
4. At least every four months, check all screws and knobs for tightness. If parts are missing, broken, damaged or worn, then discontinue use of the product until repairs are made, using factory-authorized parts.
5. Wipe down all plastic components periodically with a solution of mild soapy water to maintain cleanliness.
6. Hand wash and air-dry the fabric components as needed.
7. Lubricate all moving metal parts with a teflon lubricant only.

NOTE: Using lubricant other than Teflon-based may damage moving parts

8. Check the parking brake lock pin weekly to make sure it engages smoothly into the wheel hub. This simple check will insure the safety of the rider. Lubricate with a Teflon Lubricant when needed.
9. Check and clean the wheel attachment points frequently, this is a main area that can collect dirt and grime. Debris in this area can effect the performance and safety of the device.

⚠ WARNING

1. Your device needs regular maintenance for peak performance and to avoid injury from a device failure, damage or premature wear.
2. Inspect and maintain this device strictly using this Owner's manual as a reference.
3. If you detect a problem, make sure to service or repair the device before putting it back into use.
4. At least once a year, have a complete safety check and service of your device performed by your supplier.
5. Sand and sea water (or salt in the winter) can damage the bearings of the front and rear wheels. Clean the wheelchair thoroughly after exposure.

B. CLEANING**Base Frame Parts:**

- Clean with mild soapy water monthly.

Upholstery:

- Hand-wash only as needed.
- Machine wash as needed.
- Drip-dry only. Heat from a dryer may damage fabric.
- Flammability resistance of upholstered parts in accordance with ISO 7176-16 (EN 1021-1/2).

NOTE: Washing the fabric may decrease fire retardant properties.

C. STORAGE TIPS

1. Store your device in a clean, dry area. If you fail to do so, parts may rust or corrode.
2. Before using your mobility device, make sure it is in proper working order. Inspect and service all items included in this Manual.
3. If you store this device for more than three months, have it inspected by a supplier before use.

D. RE-USE / DISPOSAL / RECYCLING OF MATERIALS**⚠ WARNING**

Choking Hazard - Keep packing materials out of the reach of children.

Hygiene measures when being re-used:

Prior to the wheelchair being re-used, it must be carefully prepared.

⚠ WARNING

- All surfaces which come into contact with the user must be treated with a disinfection spray.
- To do this, you must use a disinfectant from the DGHM list, e.g. Antifect Liquid (Schülke & Mayr) for rapid alcoholbased disinfection for medical products and medical devices, which must be disinfected quickly.
- Please take into account the manufacturer's instructions for the disinfectant you are using.
- In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of upholstery to avoid microbacterial contamination with active agents according to § 6 infection protection law.

Weekly	Quarterly	6 Monthly	Annually	Maintenance Tasks
○				Tire inspection (excessive wear, cracks)
○				Parking brake holds firm (no slipping)
	○			Visually check for loose hardware
		○		Securement and positioning belts
		○		Axle and axle sleeves (check for debris)
	○			Quick-release axles (Remove clean)
○				Castor tires: (Excessive wear, cracks)
	○			Castors: (level, and move smoothly)
	○			Seating interface: (check for debris)
		○		Frame, Folding Mechanism
		○		Upholstery: (clean and dry)
			○	Service by Authorized Dealer

X. MAINTENANCE

NOTE: If the product has been made available to you as part of a charity or medical loans scheme, then it may not belong to you. If it is no longer required, then follow the instructions to return it as given by the organisation that made the product available to you.

In the following section, there is a description of the materials used on the product, in view of the disposal or recycling of the product and its packaging.

There may also be special local regulations in force with regard to disposal or recycling, these must be taken into account when disposing of your wheelchair. (This can include the cleaning or decontamination of the product prior to disposal).

Aluminium:	Stroller frame, seat frame
Steel:	Fixing points, axles, screws and nuts
Plastic:	Pushbar, caster forks, wheels, footrests, tyres, release Hook
Packaging:	Plastic bags made of soft polyethylene, cardboard
Upholstery:	Polyester fabric, polyurethane foam

Disposal or recycling should be carried out by a disposal company or at a public disposal point. You can also return your wheelchair to your dealer for disposal

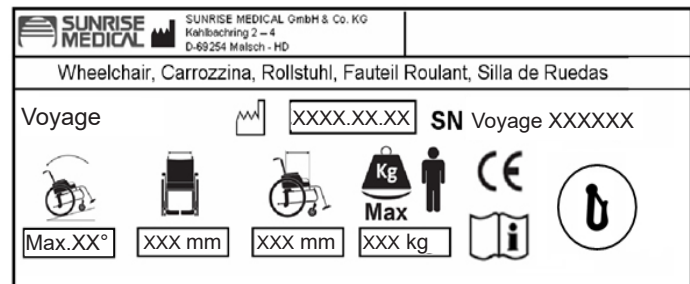
E. ORDERING PARTS

Name Plate

The nameplate is located underneath the seat, as well as on a label in the owner's manual. The nameplate indicates the exact model designation and other technical specifications. Please provide the following pieces of information whenever you have to order replacement parts or to file a claim:

- Model of device.
- Serial number of device.
- Part number, description and quantity of parts you need.
- State the reason for the replacement.

SAMPLE

Voyage
XXXXX-XXX Product Name/SKU Number.

Maximum safe slope with anti-tip tubes fitted, Depends on wheelchair setting, posture and physical capabilities of the user.

Seat width.

Depth (maximum).

Load Maximum.

CE Mark

User's Guide.

Crash Tested (Option dependant)

Not Crash Tested (Option dependant)

Date of manufacture.

Serial number.

XI. WARRANTY

THIS WARRANTY DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.

Sunrise Medical*, Kahlbachring 2-4, 69254 Malsch / Heidelberg provides a worldwide warranty for wheelchairs to its customers covering the following

Warranty conditions:

1. Should a part or parts of the Zippie Voyage require repair or replacement as a result of a manufacturing and/or material fault within 24 months after delivery to the customer, then the affected part or parts will be repaired or replaced free of charge.
2. To enforce the warranty, please contact Sunrise Medical Customer Service with the exact details of the nature of the difficulty. Should you be using the Zippie Voyage outside the area covered by the Sunrise Medical customer service agent, repairs or replacement will be carried out by another agency as designated by the manufacturer. The Zippie Voyage must be repaired by a Sunrise Medical designated Customer Service agent, (dealer).
3. For parts, which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the Zippie Voyage in accordance with point 1).
4. For spare parts which have been fitted at the customer's expense, these will have a 12 month warranty, (following the fitting), in accordance with these warranty conditions.
5. Consumable parts such as pads, tyres, inner tubes and similar parts are excluded from the warranty, except in the case that premature wear of the part is the direct result of a manufacturing fault.
6. Claims from this warranty shall not arise, if a repair or replacement of Zippie Voyage or a part is required for the following reasons:
 - a. The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the User Instructions and/or the Service Instructions.
 - b. Accessories have been used which are not specified as original accessories.
 - c. The Zippie Voyage or part having been damaged by neglect, accident or improper use.
 - d. Changes/modifications have been made to the Zippie Voyage or parts, which deviate from the manufacturer's specifications.
 - e. Repairs have been carried out, before our Customer Service has been informed of the circumstances.
7. This warranty is subject to the law of the country in which the product was purchased from Sunrise Medical“

* Means the Sunrise Medical facility from which the product was purchased.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S