Tarta Kid

User Manual


Italiano

English

Deutsch

Español

Download the
technical data sheet
from our website
www.tartadesign.it

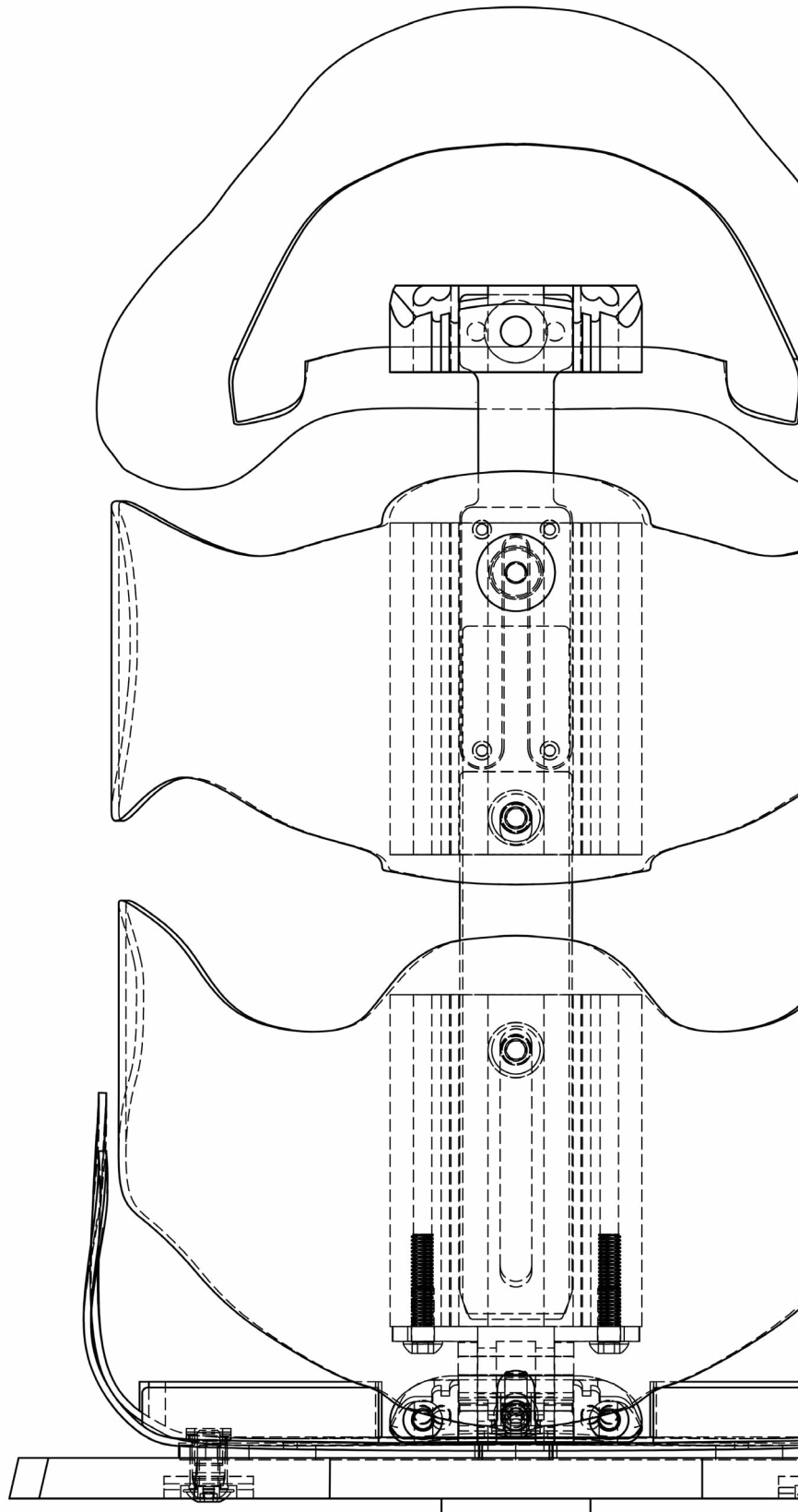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Index

Adjustable seat depth	p.	3
Adjustable backrest height	p.	4
Adjustable backrest and seat width	p.	5
Adjustable backrest slope	p.	6
Adjustable headrest height	p.	7
Adjustable headrest position	p.	8
How to attach the headrest	p.	8
Securing and removing the Tarta Kid from the base	p.	9
Positioning the seat wedge	p.	10
Positioning the lumbar wedge	p.	11
Hip strap	p.	11
Chest harness	p.	12
Fitting the leg separator	p.	13
Warnings	p.	17
Contraindications	p.	18
Cleaning and maintenance	p.	18
Key to symbols on label	p.	18
Disposal methods	p.	19
Limited guarantee for 24 months	p.	19

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T +39 0432 722 533

F +39 0432 542 920

www.tartadesign.it
info@tartadesign.it

Warnings

- The device must be prescribed exclusively by a specialist doctor.
- The specialist doctor is responsible for:
 1. prescribing the device;
 2. indicating the most appropriate setting and adjustments for the user's needs;
 3. prescribing the time period for continual use of the device.
- Tartar KID should be regulated and assembled only by accredited healthcare workers, orthopaedic technicians and staff authorised by Tartar Design Srl.
- Read the instructions carefully before use. Keep them for future reference. Incorrect use of the product could cause serious damage to the user.
- Never leave the user unattended on the product. An adult must be constantly present.
- In the event that the device is incomplete or damaged, do not use it and contact the supplier immediately.
- All adjustments to the positioning of the user on the product and assembly of the accessories should be carried out in the certainty that all screws, handles, straps and the system for securing Tartar KID to the base are solidly fixed before use.
- Always use the restraint system.
- In case of doubts regarding the device's use, assembly or safety systems, stop using it immediately and contact an authorised dealer or Tartar Design Srl.
- Having installed the Tartar KID ergonomic backrest on its base, check the stability of the base and ensure that the safety lock fixing the backrest to the base is in the correct position and that there are no evident defects or signs of wear.
- The device should be used solely on perfectly flat and smooth surfaces. Never use it on sloping surfaces.
- Do not use the Tartar KID postural system to push or lift the base from inside or outside. Incorrect use may cause it to become accidentally detached from the base or the buggy.
- Do not move the base from inside when the child is sitting on the device.
- Never lift the Tartar KID by holding the headrest, as this could become detached, causing the device to fall and risking injury.
- Having adjusted the device as necessary, ensure that the supports are tight enough and that the system is ready for use.
- Check regularly that the mechanical parts of the device have not become loose or misaligned from their original positions; if this is the case, have the device checked by an authorised maintenance technician.
- It is advisable (if possible) to regularly change the user's position so as to avoid excessive pressure on one particular part of the body and to make sitting more comfortable.
- Avoid contact with open flames and heat sources: these could damage the device and cause injury to the user.
- The device should not be exposed to direct sunlight.
- Avoid contact with sharp and/or pointed objects which could cause physical damage to the device, particularly padded parts and straps.
- Tartar KID should not be used in unsuitable circumstances (in situations with high humidity or salinity; with abrasive and/or corrosive substances etc.).
- Modifications must not be made to the device's structure and components except by a qualified technician.
- Always use components and spare parts designed and produced by Tartar Design Srl.
- The Stroller base, Wooden base and Height-adjustable base are designed to be used exclusively in conjunction with the Tartar Kid system.

Contraindications

- Do not use in direct contact with the skin; the user should wear clothes to avoid possible allergies and/or skin irritations.
- Do not use in contact with injuries;
- Do not use in the event of intolerance to the materials used in the device.

Cleaning and maintenance

- Individual parts of the device can be cleaned with a soft cloth soaked in water and well wrung.
- Do not immerse the device in water;
- The cushioning can be removed and washed at 40°;
- Do not carry out maintenance on the device: if damaged, consult your local orthopaedics department.

Key to symbols on label

	EC mark - compliance with Directive 93/42/CEE and subsequent amendments.
	This symbol identifies the manufacturer and should be accompanied by the manufacturer's name and address.
	Device code
	Production batch
	Caution! It is important to read the instructions before use
	Latex-free

Disposal methods

Some components and/or materials can be recycled; however, the device should be disposed on in line with solid urban waste requirements.

Limited guarantee for 24 months

Every Tarta Kid is carefully checked and tested to ensure maximum performance. Every Tarta Kid is guaranteed free of defects of manufacture and material for a period of 24 months from the date of purchase, when subject to normal use. In the event of a defect of production or material found within 24 months of the original date of purchase, Tarta Design Srl will repair or replace the device at no cost. This guarantee does not cover holes, tears or burns.

For any claims or repairs, contact the nearest authorised dealer.

With the exception of the guarantees expressly indicated, all other guarantees are excluded (including those relating to sale and suitability for particular uses).

Guarantees falling outside the description described above are excluded. The application of the guarantee is limited to the repair or replacement of the product.

In no circumstances can damage caused by the breakage of a product under guarantee be the motive for claims for damages or exceed the retail value of the product.

Questions?
Info?
Curiosity?

Visit our website:
www.tartadesign.it

or write to:
info@tartadesign.it

Tarta Design srl

Via Cottonificio, 56
33037 Pasian di Prato
Udine - Italy
T +39 0432 722 533
F +39 0432 542 920

www.tartadesign.it
info@tartadesign.it

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TARTA DESIGN S.r.l.
TARTA DESIGN S.r.l. a socio unico